

How to Clean a ROHO DRY FLOATATION® Cushion & Cover

Clean your ROHO cushion regularly to maintain hygiene and ensure your needs are met throughout the lifetime of the cushion. Be sure to follow manufacturer safety guidelines for all chemicals.

Important! Cleaning and disinfecting are separate processes. Be sure to *clean first, then disinfect*.



1 Cleaning the ROHO Cushion Cover



1.1 Remove the cushion from the cover. If you have the optional hook fasteners attached, remove them from the cover.



1.2 Machine wash on the gentle cycle in cold water with mild detergent, or wash by hand with neutral detergent and warm water.



1.3 Tumble dry on low or no heat, or hang to air dry.

2 Disinfecting the ROHO Cushion Cover



2.1 Machine wash on the gentle cycle in hot water with mild detergent, or hand wash in warm water with a mixture of 9 parts water: 1 part bleach.




2.2 Rinse with fresh water.




2.3 Tumble dry on low or no heat, or hang to air dry.

3 Cleaning the ROHO Cushion

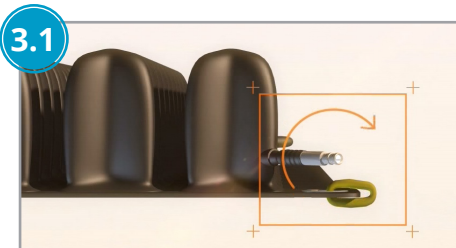


3.1 Close the valve.

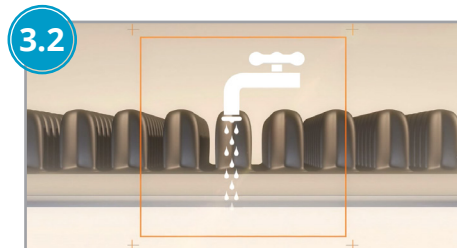


3.2 Begin by rinsing and removing anything between the cushion cells.




3.3 Take care while cleaning. If a substance has dried on the cushion and sticks cells together, do not attempt to separate the cells when they are dry. This may damage the cushion.

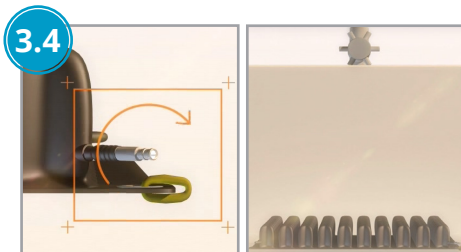


Check the valve to make sure it is still closed. Place in a large sink. A clean shower or a tub is also acceptable. Do not allow water or cleaning solution to enter the product.

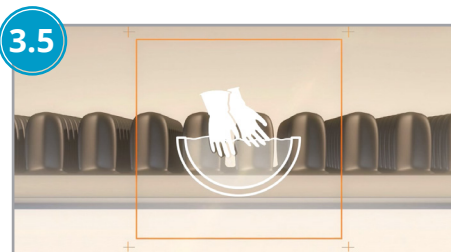


Hand wash the cushion using a mild dishwashing liquid or laundry detergent.

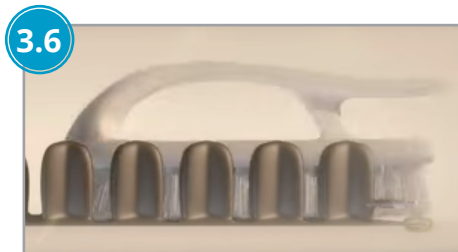

Use a soft plastic bristle brush, sponge or washcloth to gently scrub all cushion surfaces.

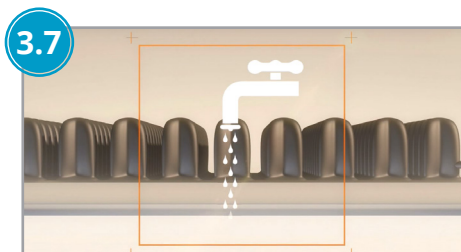


Rinse with fresh water.

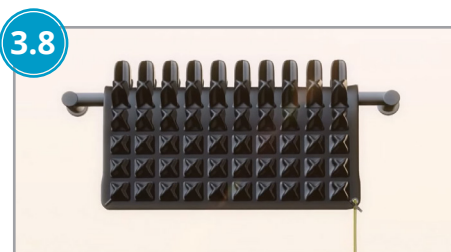


Hang to air dry.


4 Disinfecting the ROHO Cushion

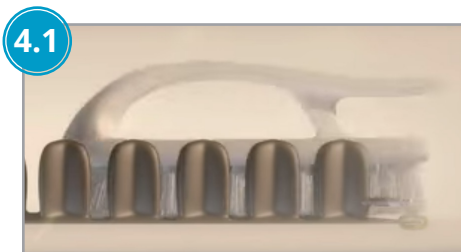


Repeat previous cleaning instructions using a solution of 9 parts warm water: 1 part household bleach. Most germicidal disinfectants are safe if the manufacturer's dilution directions are being followed.

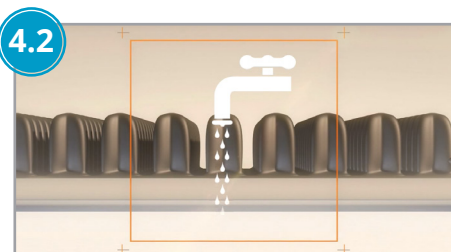


Keep the clean product wet with bleach solution for 10 minutes, then rinse thoroughly with fresh water.

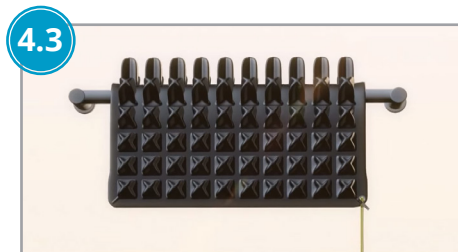

Hang to air dry.

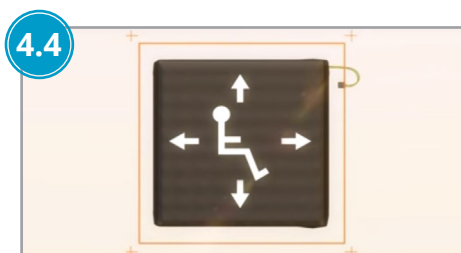

Once the cushion and cover are both dry, replace the cushion inside the cover. Use the directional labels on the cushion and cover to be sure the cushion is inserted correctly.

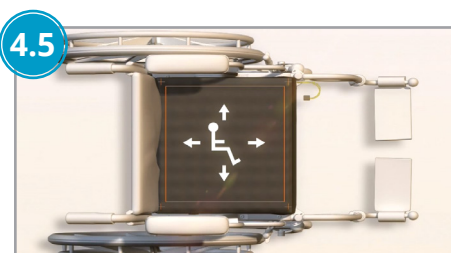

Zip the cover and place the cushion on the wheelchair. Again, refer to the directional labels to be sure the cushion is placed properly on the wheelchair.

For more detailed information on cleaning and disinfecting a ROHO cushion, refer to the product operation manual provided with your cushion. Refer to the product operation manual for additional instructions, cautions and information regarding sterilization.

All ROHO product operation manuals and 'how-to' videos are available at www.permobil.ca/support/