SLING MAINTENANCE INSTRUCTIONS

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CAUTION: Your laundry staff must be made aware of care instructions as the maintenance of this sling will have a direct impact on its condition.

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WARNING: To ensure safe patient transfers, use only BHM manufactured slings with your BHM lift.

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NOTE: A detailed *Sling Operating and Product Care Instructions* is available upon request, contact your BHM Medical local representative to order (part number 001.03680).

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This care label can be found on all BHM Medical slings

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It is imperative that a patient transfer sling be inspected prior to each use. A documented monthly inspection program should be established to formally inspect all slings to ensure the safest possible transfer of a patient. It is BHM Medical's position that the number of factors influencing the life span of a patient transfer sling are so varied that a sling should be taken out of service after 2 years. This 2 year span is a guideline for the useful life of a sling and, in fact, it may be shorter or longer depending on how the slings condition is affected by the number of washings, washing temperature, detergents, disinfectants, frequency of use, patient weight and/or numerous other factors. Currently there is no method to measure the strength of a sling once it is put into service and has been laundered multiple times, without damaging the sling itself. BHM Medical manufactures all of its sling models to the highest standards and under ideal circumstances it will provide many years of service. With this in mind, BHM Medical has developed a set of visual guidelines to assess the safety of a sling currently in use. Any visual inspection, as these guidelines are, is a subjective evaluation and therefore can never be considered a guarantee of a slings safety. It will however dramatically reduce the risk of failure.

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WARNING: If there is ever any question concerning the safe condition of a sling (e.g.: exposure to bleach), take it out of service.

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Chemical Damage/*Dommages chimiques*

Oil, grease or paint stains are generally harmless but other forms of chemical attack of a sufficient degree may lead to deterioration or extreme softening of the webbing, which would lead to the fibres being rubbed off (almost as powder in extreme cases). It is best to avoid fumes, spray or mists of acid and alkalis or organic solvents. If contamination is suspected, wash out well in warm water. Avoid contact with excessive heat which is likely to affect the product.

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Cleaning/*Nettoyage*

Certain chemical substances mentioned above may be removed by diluted heavy duty cleaner and warm water. Ensure that any cleaning agents are removed by rinsing thoroughly in warm water, maximum 80°C (176°F), and allowed to dry naturally away from any source of direct heat. For normal washing and cleaning, follow the care instructions on the label of the product.

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Storage/*Entreposage*

Slings should be stored away from direct sunlight, stress, pressure, excessive heat or humidity. The sling should be kept away from contact with sharp objects, corrosives or other possible damaging material.

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Important Notice/*Avis important*

BHM Slings are designed to be used only with BHM Patient Lifts. We will not accept any responsibility for use of this sling with any other patient lift or transport equipment. This sling is designed and manufactured to the highest possible performance standards and is constructed of synthetic fabrics offering durable service when handled according to the stated instructions. Due to the potential exposure to harsh cleaners, disinfectants or other chemicals as well as washing and drying temperatures, frequency of use, number of washings, patient weight, incontinence etc. BHM is unable to guarantee the continued integrity of this product under all conditions.

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It is possible that a washing machine or dryers spinning and tumbling action may cause damage to a sling, if the straps are caught-up in the mechanisms of the laundry machine. To prevent this possibility BHM Medical suggests the use of our mesh washing bag. Sling may also be damaged by dragging along the floor and being caught on doors, furniture or the patient lift itself. BHM suggests our sling storage bag to prevent such damage. This bag Velcro mounts on the lift, effectively and safely contains the sling.

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BHM Medical Inc.