



**ZIPPIE® TS**

# PARTS MANUAL

**Zippie TS (Z1A Z1SE)**  
**JANUARY 2014**

Sunrise Medical appreciates your business. For your ordering convenience, our Customer Service Representatives can be contacted at:

**To place orders:  
(800)333-4000**

**Quickie Technical Assistance:**

You can reach our Technical Support Team by choosing prompt 1 followed by the # sign, then prompt 2 for Technical Support. Choose option 1 for Quickie Technical Support.

**For Assistance with a Web Order,  
or to sign up for [sunrisemedical.com](http://sunrisemedical.com)  
(800) 333-4000 Option 5**

**FOR YOUR CONVENIENCE YOU MAY PLACE ORDERS THROUGH  
OUR WEBSITE OR BY FAX 24 HOURS A DAY:**

**[www.sunrisemedical.com](http://www.sunrisemedical.com)**

**Fax: (800)300-7502**

For easy reference please fill in your Customer Number and Sunrise Account Manager's name.


Customer Account Number \_\_\_\_\_  
Ship to Account Number \_\_\_\_\_  
Sunrise Account Manager \_\_\_\_\_

\*Part prices and configurations or specifications of products included in this Parts Manual are subject to change without prior notice.

ZIPPIE TS / ZIPPIE TS SE (Chairs w/ S/N Prefix Z1A & Z1SE) .....	6
BASE FRAME .....	6
FRAME .....	6
FRAME (RIGID).....	6
FRAME (FOLDING).....	8
FOOT RELEASE.....	10
TRANSIT KIT ZIPPIE TS.....	12
TILT CABLES.....	12
TILT COMPONENTS DUAL TRIGGER CABLE W/ ANGLE ADJ PUSH HANDLE BACKPOST .....	13
TILT COMPONENTS DUAL TRIGGER CABLE W/ STROLLER HANDLE BACKPOST .....	15
AXLES & AXLE PLATES .....	16
AXLE BRACKET .....	17
AXLE BRACKET 12" & 16" .....	18
QUICK RELEASE AXLES.....	19
CASTERS & FORKS.....	19
CASTER STEM & FORK ASSMS.....	20
CASTER PIN LOCK .....	22
CASTER PLATE.....	23
CASTERS .....	25
MICRO CASTERS.....	25
3 SPOKE CASTERS .....	26
SOFT ROLL CASTERS.....	28
8" x 2" CASTERS .....	29
ANTI-TIPS .....	29
ANTI TIPS .....	30
FRONT ANTI-TIP .....	32
WHEEL LOCKS .....	32
HUB LOCK for use w/ 12" AXLE BRACKETS.....	33
HUB LOCK for use w/ LONG AXLE BRACKET .....	35
HIGH MOUNT PUSH WHEEL LOCK.....	36
HIGH MOUNT PULL WHEEL LOCK.....	37
FOOT LOCK.....	38
WHEEL LOCK PUSH TO LOCK (QUICKIE STYLE).....	39
WHEEL LOCK PULL TO LOCK (QUICKIE STYLE).....	41
WHEELS & TIRES .....	42
WHEELS .....	42
12" MAG .....	43
16" MAG .....	44
MAG .....	45
LITE SPOKE.....	47
SPINERGY SPOX (18 SPOKE) .....	48
TIRES.....	48
PNEUMATIC & PNEU. AIRLESS TIRE.....	49
FULL POLY TIRE .....	50
LOW POLY TIRE.....	51
HIGH PRESSURE CLINCHER TIRE .....	52
KEVLAR TIRE .....	53
TUBES .....	53
TUBES AND AIRLESS INSERTS .....	54
SEATING & POSITIONING .....	55
SEAT .....	55
GROWING SEAT PAN.....	56
SOLID SEAT & SOLID BACK .....	58
BACK.....	61
TILT CABLES .....	61
TILT COMPONENTS DUAL TRIGGER CABLE W/ STROLLER HANDLE BACKPOST .....	61
TILT COMPONENTS DUAL TRIGGER CABLE W/ ANGLE ADJ PUSH HANDLE BACKPOST .....	63
SOLID SEAT & SOLID BACK .....	65
HEIGHT ADJ (SWING-AWAY) STROLLER HANDLE BACKPOST .....	67
FIXED HEIGHT STROLLER HANDLE BACKPOST .....	69
ANGLE ADJ PUSH HANDLE BACKPOST .....	71
STROLLER HANDLE EXTENSION-REMOVABLE.....	73
RIGID-IZER .....	75
DYNAMIC BACK .....	76
HANGERS & FOOTPLATES .....	77
EXTENSION MOUNT HANGERS .....	77
EXTENSION MOUNT HANGERS .....	77
60 DEG HANGER SWING IN-OUT .....	78
70 DEG HANGER SWING IN-OUT .....	79
80 DEG HANGER SWING IN-OUT .....	80

EXTENSION MOUNT FOOTPLATES & EXTENSIONS .....	80
COMPOSITE FOOTPLATE.....	81
COMPOSITE ANGLE ADJUSTABLE FOOT PLATE .....	85
KIDS ANGLE ADJ FOOTPLATE EXTENSION MOUNT.....	89
ADULT ANGLE-ADJ FOOTPLATE EXT MOUNT .....	92
PLATFORM FLIP-UP FOOTPLATE.....	95
HIGH MOUNT EXTENSION TUBE .....	97
FRONT MOUNT HANGERS .....	97
FRONT MOUNT HANGERS .....	97
70 DEG FRONT MOUNT HANGER SWING IN-OUT .....	98
80 DEG FRONT MOUNT HANGER SWING IN-OUT .....	100
90 DEG FRONT MOUNT HANGER SWING IN-OUT .....	102
70 DEG LIFT-OFF HANGER.....	104
80 DEG LIFT-OFF HANGER.....	105
90 DEG LIFT-OFF HANGER.....	106
FRONT MOUNT EXTENSIONS.....	106
FRONT MOUNT HANGER EXT.....	107
FRONT MOUNT FOOTPLATES .....	107
KIDS ANGLE ADJ FOOTPLATE FRONT MOUNT .....	108
ADULT ANGLE-ADJ FOOTPLATE FRONT MOUNT.....	110
90 DEG FOOTBOARD .....	112
ELR (ELEVATING LEGREST) .....	113
FRONT MOUNT ELR .....	113
FRONT MOUNT ELR HANGER.....	113
90 DEG ELEVATING HANGER .....	114
90 DEG ELEVATING CALF PAD .....	116
FRONT MOUNT ELR EXTENSION .....	116
FRONT MOUNT HANGER EXT.....	117
FRONT MOUNT ELR FOOTPLATE.....	117
KIDS ANGLE ADJ FOOTPLATE FRONT MOUNT .....	118
ADULT ANGLE-ADJ FOOTPLATE FRONT MOUNT.....	120
EXTENSION MOUNT ELR.....	121
EXTENSION MOUNT ELR HANGER .....	121
ELEVATING HANGER .....	122
ELEVATING HANGER CALF PAD 14 -15" .....	124
ELEVATING HANGER CALF PAD 16-20" .....	125
EXTENSION MOUNT ELR EXTENSION .....	125
DOUBLE D FOOTREST EXTENSION TUBE "OLD STYLE" .....	126
EXTENSION MOUNT ELR FOOTPLATE .....	127
COMPOSITE FOOTPLATE.....	128
KIDS ANGLE ADJUSTABLE FOOTPLATE EXT MOUNT .....	130
ADULT ANGLE-ADJUSTABLE FOOTPLATE EXT MOUNT.....	132
ELR ALUMINUM FOOTPLATE .....	134
ALR (ARTICULATING LEGREST) .....	134
EXTENSION MOUNT ALR HANGER .....	134
ARTICULATING HANGER.....	135
ARTICULATING HANGER CALF PAD .....	137
EXTENSION MOUNT ALR EXTENSION.....	137
ROUND EXTENSION TUBE .....	138
EXTENSION MOUNT ALR FOOTPLATE .....	139
COMPOSITE FOOTPLATE.....	140
KIDS ANGLE ADJUSTABLE FOOTPLATE EXT MOUNT .....	142
ADULT ANGLE-ADJUSTABLE FOOTPLATE EXT MOUNT.....	144
MULTI-POSITION LATCH STUD SUPPORT .....	146
ARMRESTS & RECEIVERS .....	146
ARMRESTS.....	146
SINGLE POST HEIGHT ADJUSTABLE ARMREST .....	147
T POST ARMREST .....	149
FLIP-UP ARMREST .....	151
ADJUSTABLE LOCKING FLIP-UP ARMREST.....	152
LENGTH ADJUSTABLE LOCKING FLIP-UP ARMREST .....	153
RECEIVERS.....	154
SINGLE POST ARMREST RECEIVER - STD .....	155
SINGLE POST ARMREST RECEIVER - LOW .....	157
ACCESSORIES .....	158
POSITIONING BELTS .....	158
AUTO BUCKLE POSITIONING BELTS .....	159
AIRCRAFT BUCKLE POSITIONING BELT.....	160
PADDED DOUBLE PULL HIP BELT.....	161
POSITIONING BELT ATTACHMENT KIT.....	162

BACKPACKS & SEAT POUCHES .....	162
BACKPACKS AND SEAT POUCHES.....	163
CASE LOGIC BAGS.....	164
CELL PHONE PACK.....	165
TRAYS .....	165
TUFFAK TRAY (MANUAL & PEDIATRIC CHAIRS) .....	166
TRAY HARDWARE.....	168
SIDE GUARDS (ZTS &ZGS).....	169
SPOKE GUARDS.....	170
IMPACT GUARD NEOPRENE & GEL VELCRO PADS.....	171
SOFT FEEL LEG STRAP.....	172
ADJUSTABLE LEG STRAP .....	173
TOOL KIT .....	174
ONE-ARM DRIVE RETRO KIT .....	175
TIRE PUMP .....	177
CANOPY .....	178

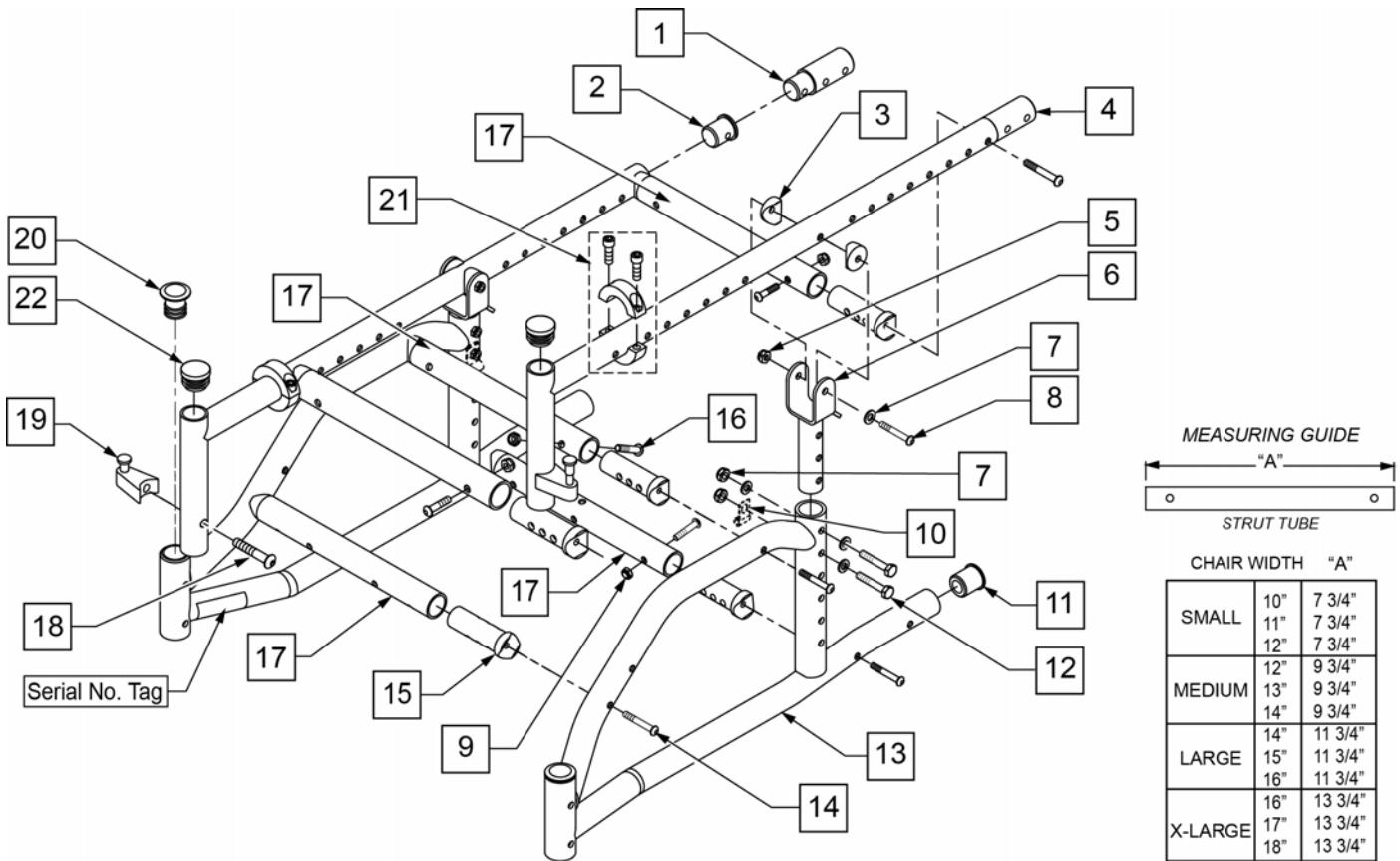


Pos.	Item Number	Description	Remarks
1	483088	REAR FRAME EXT .065 15 -16" ZP	
2	483089	SEAT TUBE PLUG S/A TS/P5	
3	561004	1" SADDLE 1/4" HOLE	
5	106934	NUT NYLOCK 1/4 -28 ROHS	
6	505251	TILT BRACKET ZP TS 45°	
7	106943	WASER FLAT AN 1/4 ROHS	
8	106965	BHCS 1/4 -28 x 2 SHLDR ROHS	
9	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
10	112388	P-CLIP 3/16" VINYL COAT	
11	561018	1" TUBE END BUSHING THICK	
12	106972	HHCS 1/4 -28 x 1 1/2 ROHS	
14	107028	BHCS 1/4 -28 x 1 1/2 ROHS	
15	483184	STRUT SADDLE ZGS	
16	106961	BHCS 1/4 -28 x 1 1/4 ROHS	
18	106982	BHCS 5/16 -18 x 1 3/4 SHLD ROHS	
19	105787	S/A LATCH STUD SUPPORT NUT	
20	561062	1" INSERT PLUG .065 RIBS	
21	342048	ADJ SEAT STOP 1"	
22	560735	7/8" ROUND TUBE PLUG	Use in place of SA Hanger
1,14	920309	TS S/A 2" SEAT EXT W/HDWR	
1,4,16,18,19	115713S	SEAT TUBE TS 19-20D W/ LTCH	WORKS W/ 19-20 SEAT DEPTHS
2,4,16,18,19	115714S	SEAT TUBE TS 13-18D W/ LTCH	WORKS W/ 13-18 SEAT DEPTHS
9,14-17	115717	STRUT TUBE SML ASSM	SMALL - WORKS W/ 10" -12" SEAT WIDTHS
9,14-17	115718	STRUT TUBE MED ASSM	MEDIUM - WORKS W/ 12" -14" SEAT WIDTHS
9,14-17	115719	STRUT TUBE LRG ASSM	LARGE - WORKS W/ 14" -16" SEAT WIDTHS

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
9,14-17	115720	STRUT TUBE X-L ASSM	X-LARGE - WORKS W/ 16' -18" SEAT WIDTHS
11,13,20	115715S	RIGID ZTS S/F ASSM	RIGHT
11,13,20	115716S	RIGID ZTS S/F ASSM	LEFT
14,15,16	920461	STRUT TUBE HDWR KT ZP TS GRW	
18,19	107491	LATCH STUD KIT	


Note: When replacing a Zippie TS SE Side Frame, a New Zippie TS Side Frame must be purchased. The additional functionality of a third width adjustment hole is the only difference.

Note: Prior to serial number Z1A-034388 & Z1SE-008579, please replace X-hinges in pairs.

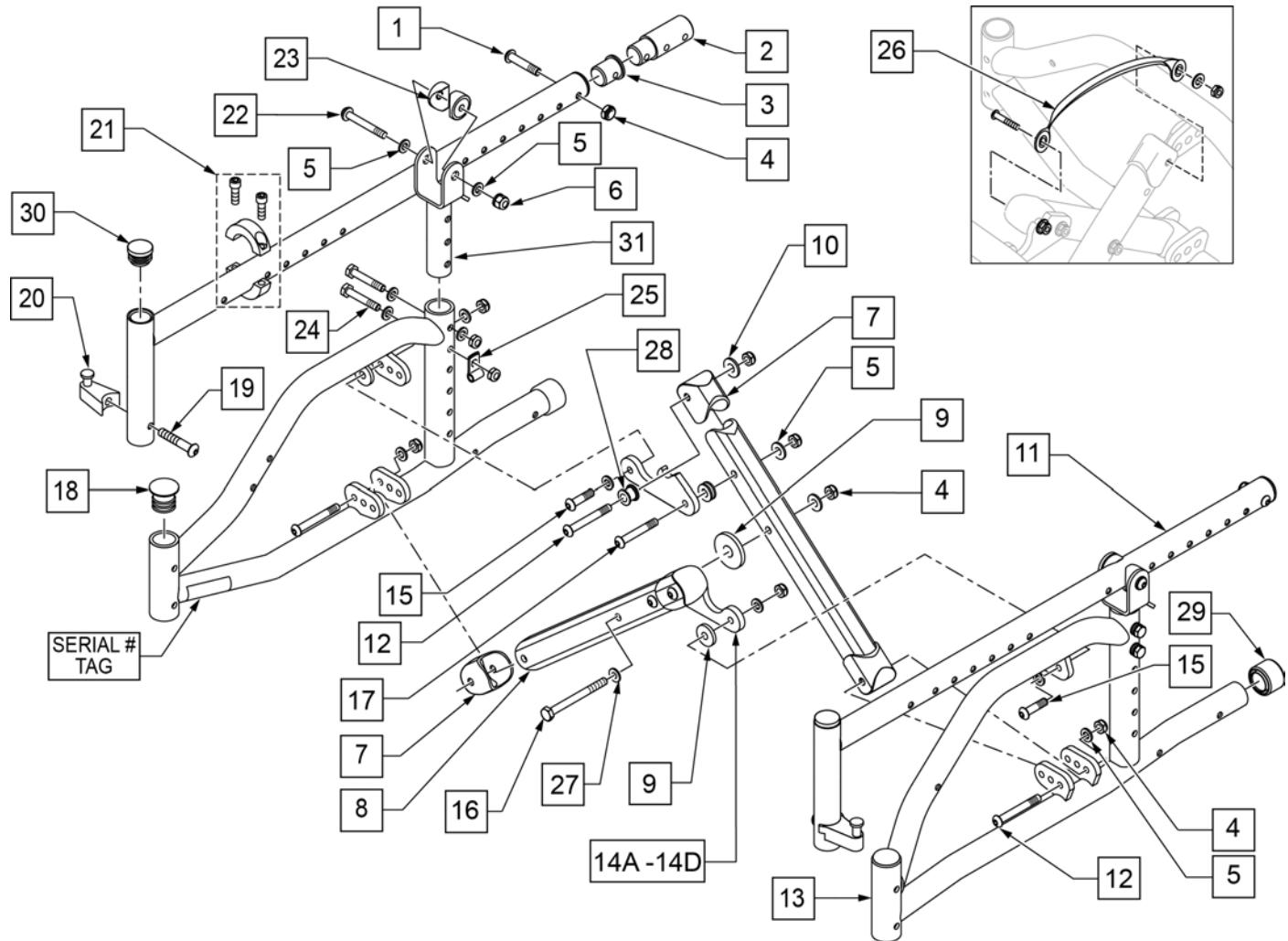

Pos.	Item Number	Description	Remarks
1	106961	BHCS 1/4 -28 x 1 1/4 ROHS	
2	483088	REAR FRAME EXT .065 15 -16" ZP	
3	483089	SEAT TUBE PLUG S/A TS/P5	
4	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
5	106943	WASHER FLAT AN 1/4 ROHS	
6	106934	NUT NYLOCK 1/4 -28 ROHS	
7	562845	END CAP OVAL TUBE ZGS	
9	561016	1" WASHER 5/16 HOLE	
10	106945	WASHER FLAT SAE 1/4 ROHS	
12	106964	BHCS 1/4 -28 x 2 ROHS	
14A	120996	X-HINGE 10-12IN ZGS	SMALL - WORKS W/ 10-12 SEAT WIDTHS

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
14B	120997	X-HINGE 12-14IN ZGS	MEDIUM - WORKS W/ 12-14 SEAT WIDTHS
14C	120998	X-HINGE 14-16IN ZGS	LARGE - WORKS W/ 14-16 SEAT WIDTHS
14D	120999	X-HINGE 16-18IN ZGS	X-LARGE - WORKS W/ 16-18 SEAT WIDTHS
15	107355	BHCS 1/4 -28 x 1" ROHS	
16	109265	HHCS 1/4 -28 x 2 1/2 ROHS	
17	106962	BHCS 1/4 -28 x 1 3/4 ROHS	
18	561062	1" INSERT PLUG .065 RIBS	
19	106982	BHCS 5/16 -18 x 1 3/4 SHLD ROHS	
21	342048	ADJ SEAT STOP 1"	
22	106965	BHCS 1/4 -28 x 2 SHLDR ROHS	
23	561004	1" SADDLE 1/4" HOLE	
24	106972	HHCS 1/4 -28 x 1 1/2 ROHS	
25	112388	P-CLIP 3/16" VINYL COAT	
26	750552	ZGS FOLDING STRAP	
27	561028	.25 NYLON WASHER	
28	481089	Z2 X-HINGE SPACER	
29	561018	1" TUBE END BUSHING THICK	
30	560735	7/8" ROUND TUBE PLUG	Use in place of SA Hanger
31	505251	TILT BRACKET	
1,2,4,11,19,20	115713S	SEAT TUBE TS 19-20D W/ LTCH	WORKS W/ 19-20 SEAT DEPTHS
1,3,4,11,19,20	115714S	SEAT TUBE TS 13-18D W/ LTCH	WORKS W/ 13-18 SEAT DEPTHS
13,18,29	115721S	FOLDING ZTS S/F ASSM R	RIGHT
13,18,29	115722S	FOLDING ZTS S/F ASSM L	LEFT
19,20	107491	LATCH STUD KIT	
4-5,7-10,12,14A,15-17,27,28	673100	X-TUBE ASSM 10-12" ZGS	SMALL - WORKS W/ 10-12 SEAT WIDTHS
4-5,7-10,12,14B,15-17,27,28	673101	X-TUBE ASSM 12-14" ZGS	MEDIUM - WORKS W/ 12-14 SEAT WIDTHS
4-5,7-10,12,14B,15-17,27,28	673102	X-TUBE ASSM 14-16" ZGS	LARGE - WORKS W/ 14-16 SEAT WIDTHS
4-5,7-10,12,14B,15-17,27,28	673103	X-TUBE ASSM 16-18" ZGS	X-LARGE - WORKS W/ 16-18 SEAT WIDTHS

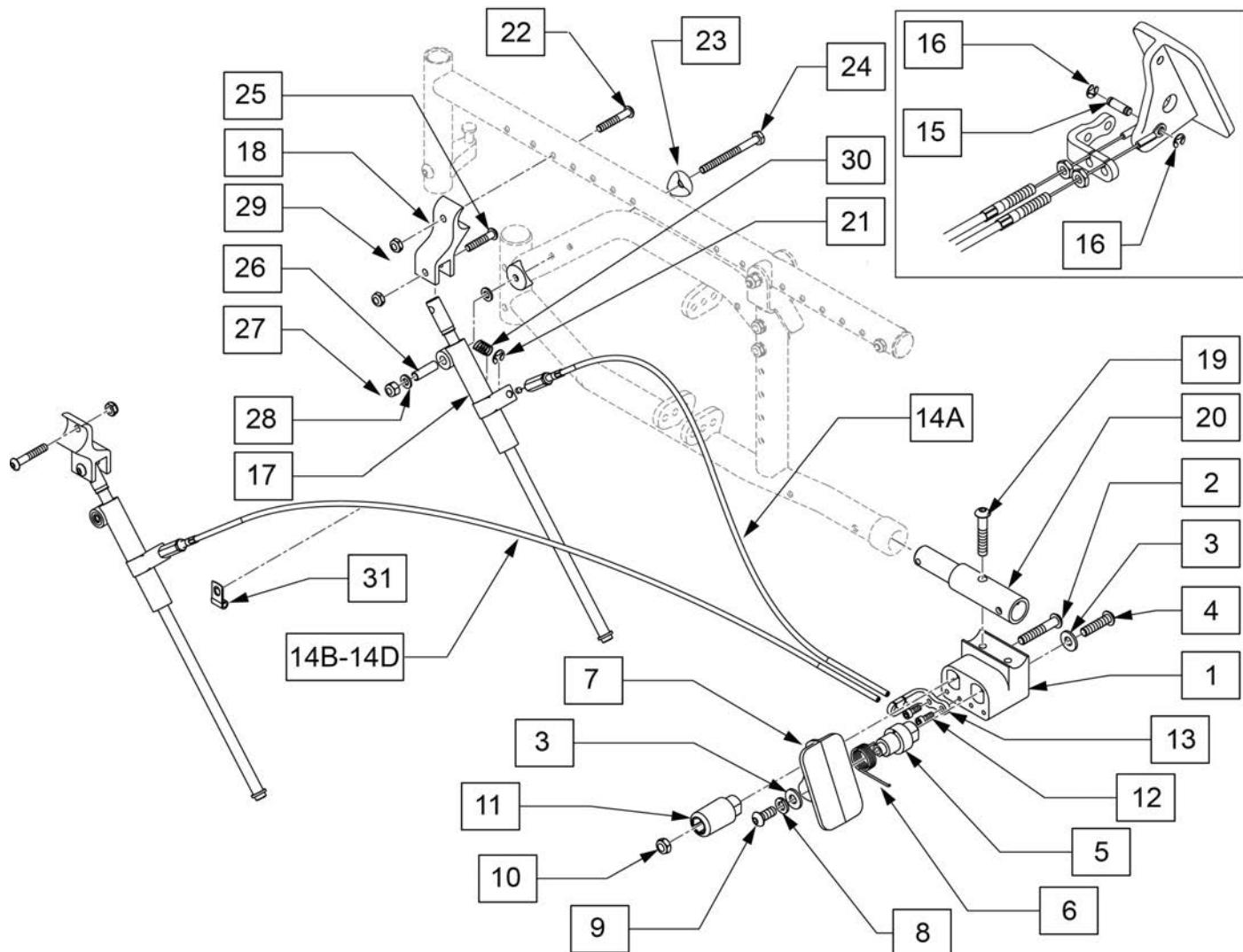


Pos.	Item Number	Description	Remarks
1	111842	BLOCK MOUNT FOOT TILT REL	
2	106982	BHCS 5/16 -18 x 1 3/4 SHLD ROHS	
3	106951	WASHER SPLIT LOCK 5/16 ROHS	
4	106980	BHCS 5/16 -18 x 1 1/4 ROHS	
5	106377	SHAFT PIVOT PEDAL	
6	106668	TORSION SPRING RH	
7	107165	PEDAL FOOT TILT RELEASE	
8	263109	5/16 FENDER WASHER SS 3/4 OD	
9	106979	BHCS 5/16 -18 x 3/4 ROHS	
10	202117	5/16-18 NYLOCK NUT THICK BLK	
11	107167	POST STOP FOOT RELEASE	
12	107181	SHCS 10 -32 x 1/2 ROHS	
13	109737	BRACKET CABLE FOOT TILT RELEASE	
14A	109733-001	CABLE FOOT TILT REL STD	RIGHT SIDE ONLY ON ALL QTY 1 ALL WIDTHS
14B	109733-002	CABLE FOOT TILT REL 10 -14"	LEFT SIDE FITS 10-14" W
14C	109733-003	CABLE FOOT TILT REL 12 -16"	LEFT SIDE FITS 12-16" W


<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
14D	109733-004	CABLE FOOT TILT REL 14 -18"	LEFT SIDE FITS 14-18" W
15	109738	PIN CABLE PEDAL	
16	106383	E-CLIP 3/16"	
17	120625	MECH CYLINDER LOCK R BR RCLINR	RIGHT
17	120626	MECH CYLINDER LOCK L BR RCLINR	LEFT
18	500609	FOLDING ZTS MECH LOCK MNT SEAT FRM	
19	106981	BHCS 5/16 -18 x 1 1/2 SHLD ROHS	
20	109734	TUBE EXTENSION FRAME	
21	342000	1/4" E-CLIP	
1-11	115770	ZTS FOOT PADAL ASSEMBLY	
1-14A,14B-16,19-21	115723	ZTS TLT FT RLSE 10-14W KIT	WORKS W/ 10-14 WIDE CHAIRS
1-14A,14C-16,19-21	115724	ZTS TLT FT RLSE 12-16W KIT	WORKS W/ 12-16 WIDE CHAIRS
1-14A,14D-16,19-21	115725	ZTS TLT FT RLSE 14-18W KIT	WORKS W/ 14-18 WIDE CHAIRS
22	107028	BHCS 1/4 -28 x 1 1/2 ROHS	
23	561004	1" SADDLE 1/4" HOLE	
24	107264	HHCS 5/16 -24 x 2 3/4 PL ROHS	
25	107027	BHCS 1/4 -28 x 1 1/4 SHLDR ROHS	
26	520916	FOLDING ZTS MECH LOCK PVT SPACER FRM	
27	106934	NUT NYLOCK 1/4 -28 ROHS	
28	106943	WASHER FLAT AN 1/4 ROHS	
29	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
30	321141	.650L X 6 COIL SPRING	
31	112388	P-CLIP 3/16" VINYL COAT	

NOTE: Fits Zippie TS chairs with s/n prefix ZP, ZT, ZTS, ZTE, Z1A, & Z1SE.

RIGHT-SIDE REAR

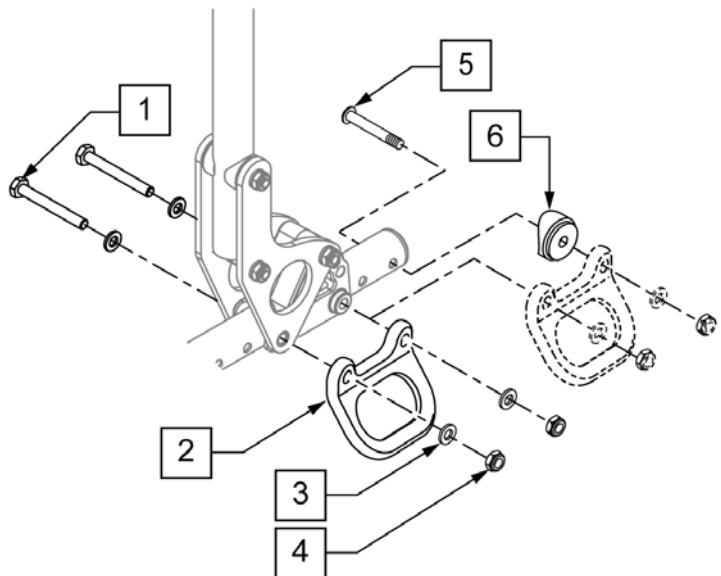

RIGHT-SIDE FRONT

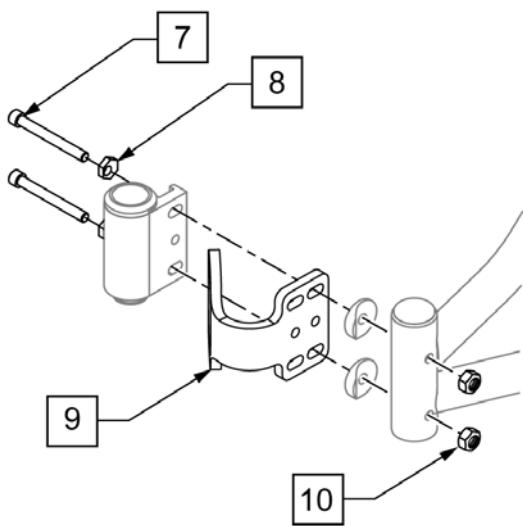


Pos.	Item Number	Description
1-9	922970S	TRANSIT KIT ZIPPIE
1	107351	HHCS 1/4-28 X 2 1/4 ROHS
3	106943	WASHER FLAT AN 1/4 ROHS
4	106935	NUT NYLOCK THIN 1/4-28 ROHS
5	226150	1/4-28 X 2 BH C/S
6	560518	SADDLE BACKREST Q2 DAB RIGHT
6	560519	SADDLE BACKREST Q2 DAB LEFT
7	221202	1/4-28 X 2-1/8" C/S CLR ZN
8	481430	ECCENTRIC WASHER TRANSPORT
10	201120	1/4-28 AUTOMATION NUT
1-10	105169	TRANSIT KIT ZIPPIE TS
1-10	105175S	TRANSIT KIT NEW ZIPPIE TS

**TILT COMPONENTS DUAL TRIGGER CABLE W/ ANGLE ADJ PUSH HANDLE BACKPOST** (03/2010)

**NOTE:** Cables can be routed in a CROSSED Configuration (Left trigger controls right mech lock) and a NON-CROSSED configuration (Left trigger controls left mech lock).

See Attached TRIGGER CABLE ORDERING GUIDE for identification & use.

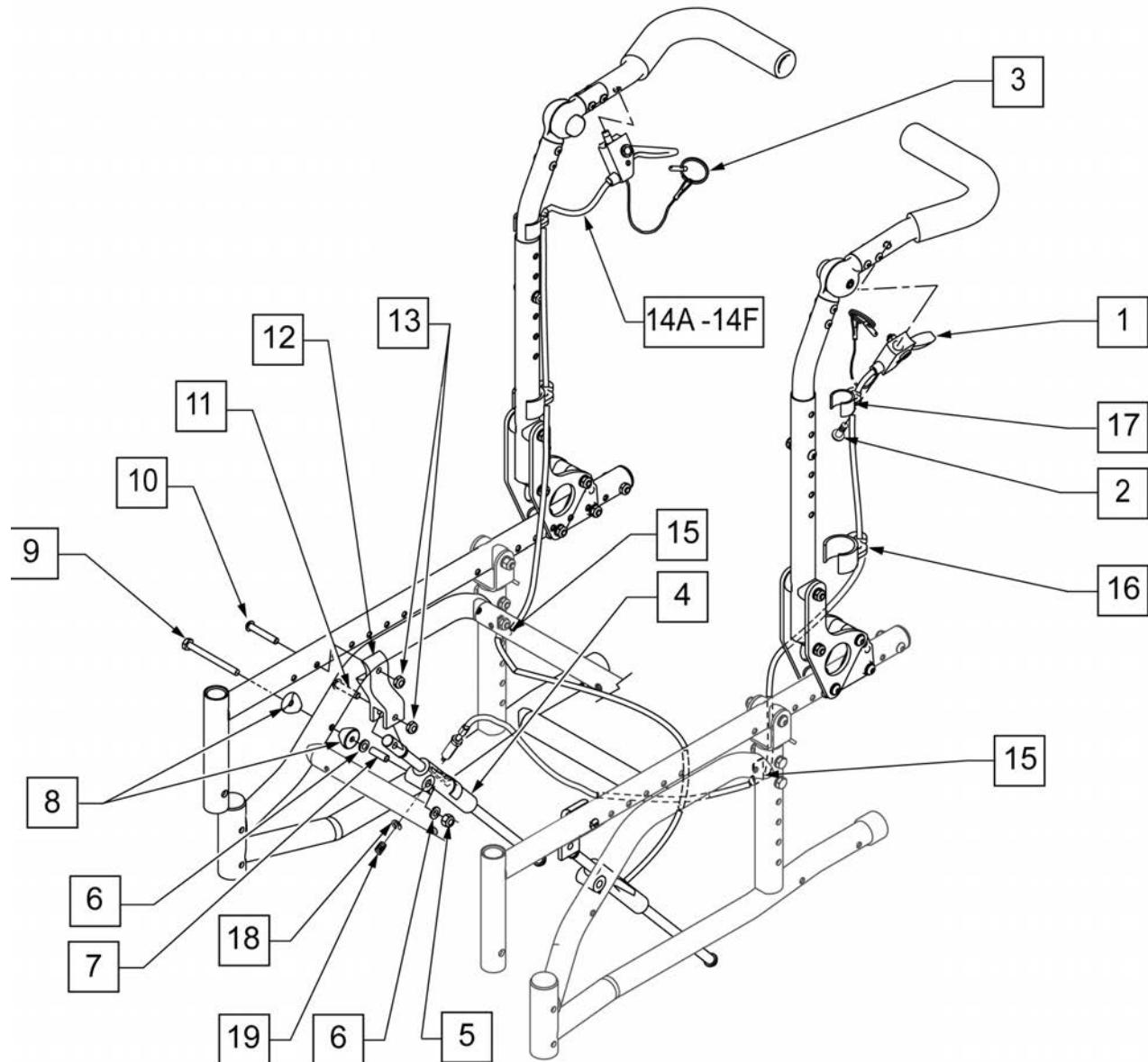

Pos.	Item Number	Description	Remarks
1,3	580109	PLUNGER PIN LEVER ASSM	
2	341497	10 -32 X 5/8 PAN HD PHIL	
3	349021	TRIGGER RELEASE PIN TS/QT/P5	
4	120625	MECH CYLINDER LOCK R BR RCLINR	RIGHT
4	120626	MECH CYLINDER LOCK L BR RCLINR	LEFT
5	106934	NUT NYLOCK 1/4 -28 ROHS	
6	106943	WASHER FLAT AN 1/4 ROHS	
7	520916	FOLDING ZTS MECH LOCK PVT SPACER FRM	


Pos.	Item Number	Description	Remarks
8	561004	1" SADDLE 1/4" HOLE	
9	107264	HHCS 5/16 -24 x 2 3/4 PL ROHS	
10	107028	BHCS 1/4 -28 x 1 1/2 ROHS	
11	107027	BHCS 1/4 -28 x 1 1/4 SHLDR ROHS	
12	500609	FOLDING ZTS MECH LOCK MNT SEAT FRM	
13	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
14A	112700-001	CABLE TILT ZTS SHORT	950 mm/ 37.40 Inches long. See TRIGGER CABLE ORDERING GUIDE.
14B	112700-002	CABLE TILT ZTS MEDIUM	1025 mm/ 40.35 Inches long. See TRIGGER CABLE ORDERING GUIDE.
14C	112700-003	CABLE TILT ZTS LONG	1100 mm/ 43.31 Inches long. See TRIGGER CABLE ORDERING GUIDE.
14D	112700-004	CABLE TILT ZTS X-LONG	1175 mm/ 46.26 Inches long. See TRIGGER CABLE ORDERING GUIDE.
14E	112700-005	CABLE TILT ZTS 2X-LONG	1250mm/ 49.21 inches long. See TRIGGER CABLE ORDERING GUIDE.
14F	112700-006	CABLE TILT ZTS 3X-LONG	1325mm/ 52.17 inches long. See TRIGGER CABLE ORDERING GUIDE.
15	112388	P-CLIP 3/16" VINYL COAT	
16	108134	CLIP CABLE 1" PACK (10)	PACK OF 10
17	108135	CLIP CABLE 7/8" PACK (10)	PACK OF 10
18	342000	1/4" E-CLIP	
19	321141	.650L X 6 COIL SPRING	
4,18	922055	ZTS MECH LOCK REPAIR KIT	PAIR

**TILT COMPONENTS DUAL TRIGGER CABLE W/ STROLLER HANDLE BACKPOST**

(03/2010)

**NOTE:** Cables can be routed in a CROSSED Configuration (Left trigger controls right mech lock) and a NON-CROSSED configuration (Left trigger controls left mech lock).


See Attached TRIGGER CABLE ORDERING GUIDE for identification & use.

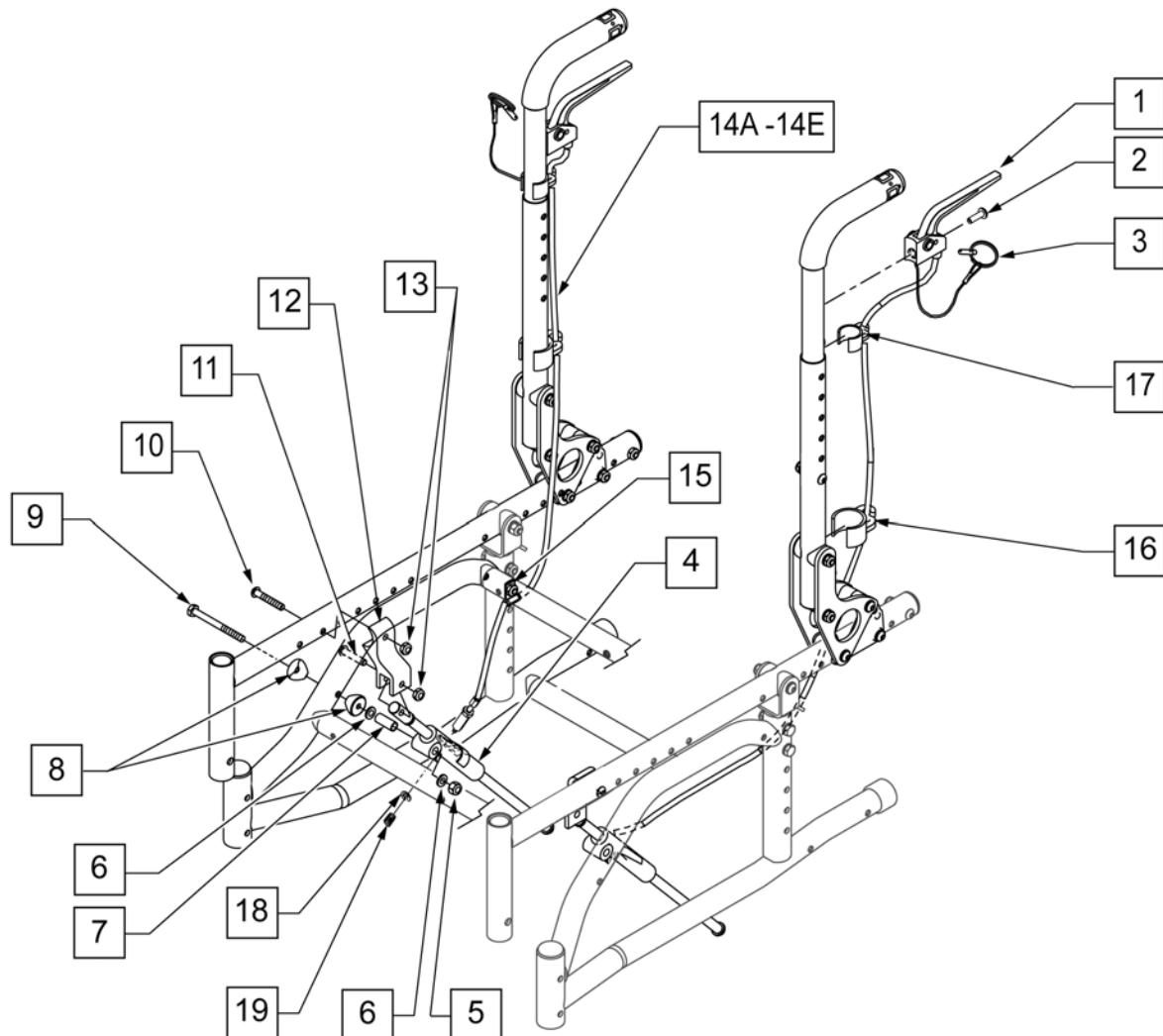



Pos.	Item Number	Description	Remarks
1,3	580110	PLUNGER PIN LEVER ASSM	
2	281120	10 -32 X 5/8 PAN HD PHIL	
3	349021	TRIGGER RELEASE PIN TS/QT/P5	
4	120625	MECH CYLINDER LOCK R BR RCLINR	RIGHT
4	120626	MECH CYLINDER LOCK L BR RCLINR	LEFT
5	106934	NUT NYLOCK 1/4 -28 ROHS	
6	106943	WASHER FLAT AN 1/4 ROHS	
7	520916	FOLDING ZTS MECH LOCK PVT SPACER FRM	
8	561004	1" SADDLE 1/4" HOLE	
9	107264	HHCS 5/16 -24 x 2 3/4 PL ROHS	
10	107028	BHCS 1/4 -28 x 1 1/2 ROHS	

Pos.	Item Number	Description	Remarks
11	107027	BHCS 1/4 -28 x 1 1/4 SHLDR ROHS	
12	500609	FOLDING ZTS MECH LOCK MNT SEAT FRM	
13	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
14A	112700-001	CABLE TILT ZTS SHORT	950 mm/ 37.40 Inches long. See TRIGGER CABLE ORDERING GUIDE.
14B	112700-002	CABLE TILT ZTS MEDIUM	1025 mm/ 40.35 Inches long. See TRIGGER CABLE ORDERING GUIDE.
14C	112700-003	CABLE TILT ZTS LONG	1100 mm/ 43.31 Inches long. See TRIGGER CABLE ORDERING GUIDE.
14D	112700-004	CABLE TILT ZTS X-LONG	1175 mm/ 46.26 Inches long. See TRIGGER CABLE ORDERING GUIDE.
14E	112700-005	CABLE TILT ZTS 2X-LONG	1250mm/ 49.21 inches long. See TRIGGER CABLE ORDERING GUIDE.
14F	112700-006	CABLE TILT ZTS 3X-LONG	1325mm/ 52.17 inches long. See TRIGGER CABLE ORDERING GUIDE.
15	112388	P-CLIP 3/16" VINYL COAT	
16	108134	CLIP CABLE 1" PACK (10)	PACK OF 10
17	108135	CLIP CABLE 7/8" PACK (10)	PACK OF 10
18	342000	1/4" E-CLIP	
19	321141	.650L X 6 COIL SPRING	
4,18	922055	ZTS MECH LOCK REPAIR KIT	PAIR

## LONG ADJUSTABLE

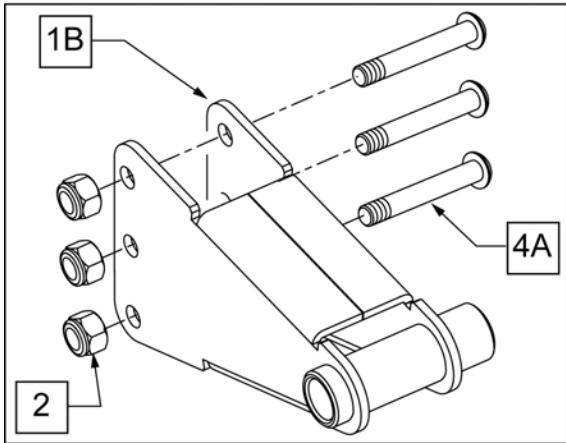


## SHORT ADJUSTABLE

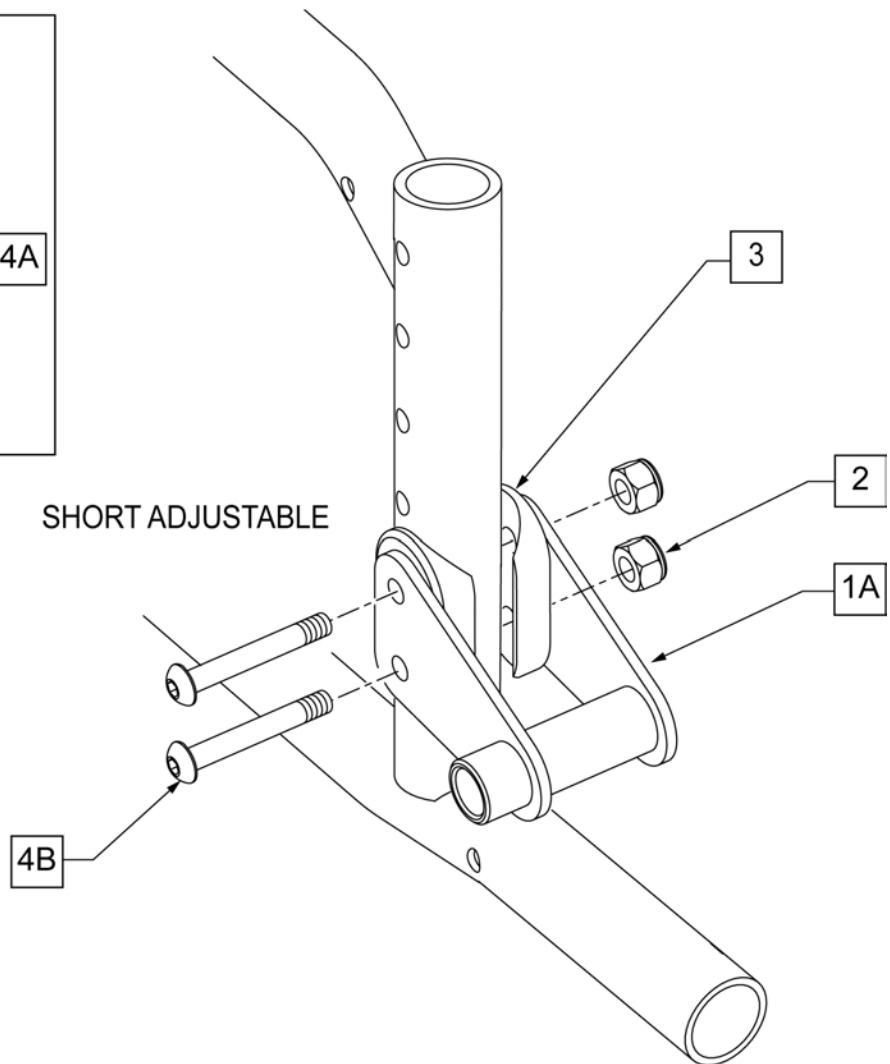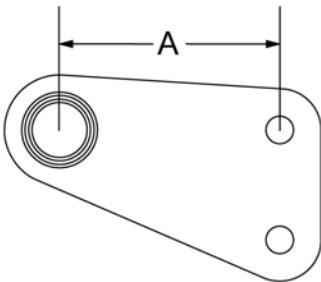

AXLE	DIM "A"
SHORT	2.0"
LONG	4.0"

Pos.	Item Number	Description	Remarks
1A	505320	QR AXLE PLATE ZP TS R	SHORT-RIGHT
1A	505321	QR AXLE PLATE ZP TS L	SHORT-LEFT
1B	506120	AXLE BRACKET RIGHT	LONG-RIGHT
1B	506121	AXLE BRACKET LEFT	LONG-LEFT
2	106934	NUT NYLOCK 1/4 -28 ROHS	
3	561085	1 1/8" HALF SADDLE 2 HOLE QTS	
4A	106962	BHCS 1/4 -28 x 1 3/4 ROHS	USE W/ LONG AXLE PLATE
4B	106964	BHCS 1/4 -28 x 2 ROHS	USE W/ SHORT AXLE PLATE
1A-4B	115726	AXLE BRACKET SHT R KIT	RIGHT - SHORT AXLE PLATE KIT
1A-4B	115727	AXLE BRACKET SHT L KIT	LEFT - SHORT AXLE PLATE KIT
1B-4A	115728	AXLE BRACKET LNG R KIT	RIGHT - LONG AXLE PLATE KIT
1B-4A	115729	AXLE BRACKET LNG L KIT	LEFT - LONG AXLE PLATE KIT






## AXLE BRACKET 12" &amp; 16"

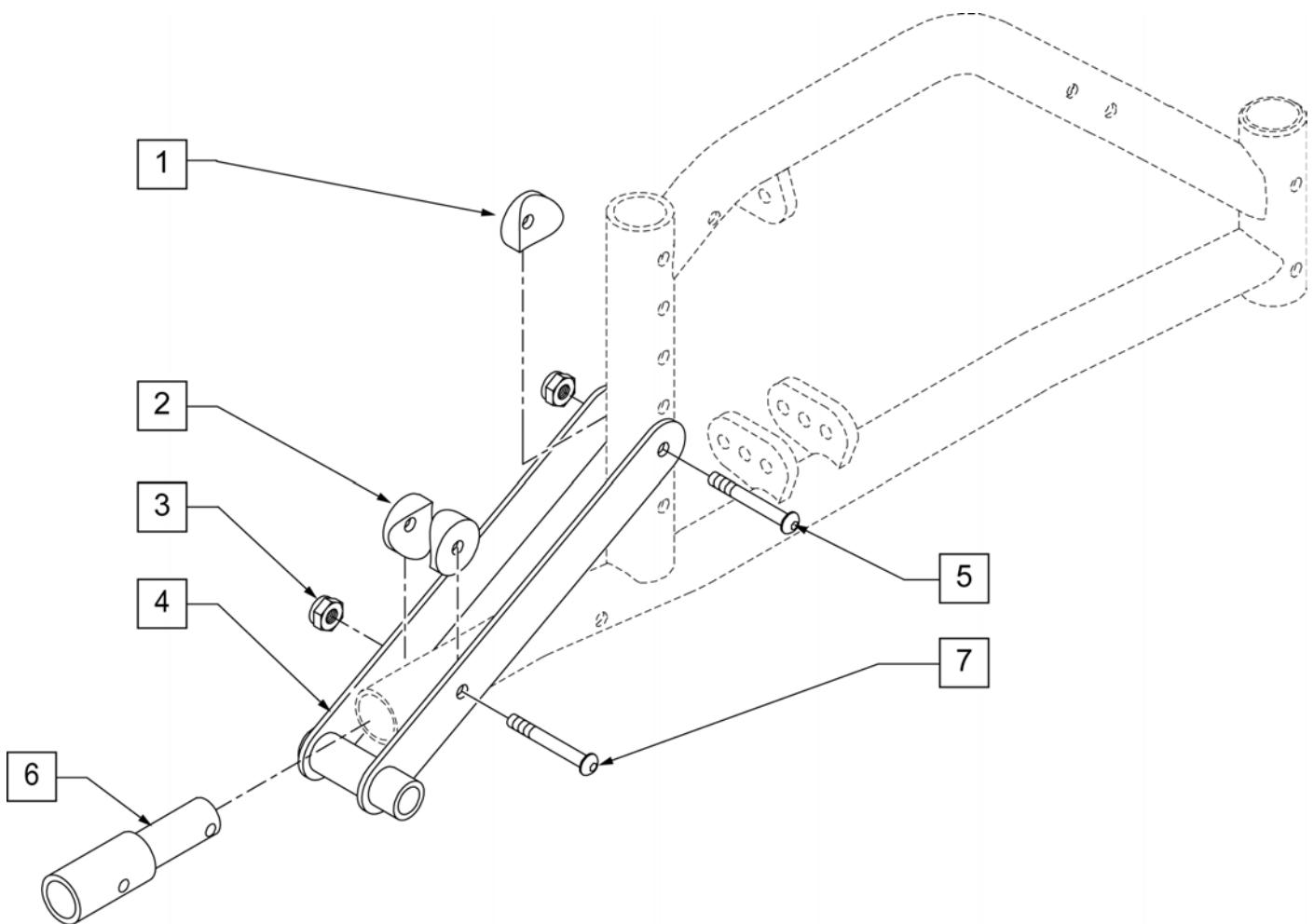


Pos.	Item Number	Description
1	561001	1 1/8" SADDLE 1/4" HOLE
2	561004	1" SADDLE 1/4" HOLE
3	106935	NYLOCK NUT THIN 1/4 -28 ROHS
5	106962	BHCS 1/4-28 x 1 3/4 ROHS
6	109734	TUBE EXTENSION FRAME
7	112390	BHCS 1/4-28 X 1-3/4 FULL ROHS
1-7	920161	QR AXLE PLATE TS 12" WHEEL W/ AT REC

## QUICK RELEASE AXLES

[03/2010]

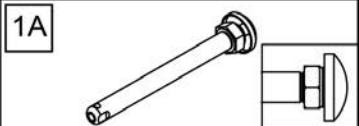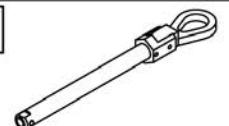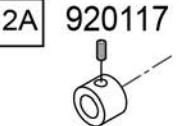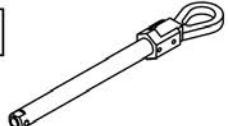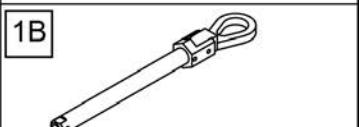Wheel Type	Axle Type	Axle P/N	Spacer P/N
Mag Wheels 12" Axle Plate	Quick Release 	102065 A=4 3/4	N/A
	Quad Quick Release 	116390 A=4 3/4	N/A
Mag Wheels STD & LONG Axe Plates	Quick Release 	102063 A= 4 1/4	N/A
	Quick Release 	102065 A= 4 3/4	2A 
	Quad Quick Release 	116390 A= 4 3/4	2A 
Lite Spoke Wheels (20 -24")	Quick Release 	107268 A=4 1/4	N/A
18" Spoke	Quick Release 	102064 A=4 1/2	N/A
	Quad Quick Release 	922170 A=4 1/2	N/A



Quick Release Measuring Guide

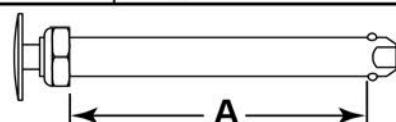

1/2" Axle Diameter


Pos.	Item Number	Description
1A	102065	QR AXLE MAG W/M-12 NUT
1B	116390	QR AXLE W/ QUAD NUT PUSH PUTTON
1C	107268	QR AXLE AND NUT SHORT BIG BUTTON
1D	102064	QR AXLE M12 NUT CLR BIG BUTTON
1E	102063	QR AXLE MAG W/M-12 NUT
2A	920117	MAG SPACER 6S W/ SET SCREW

## MEASURING GUIDE

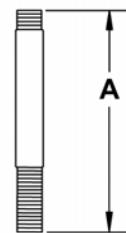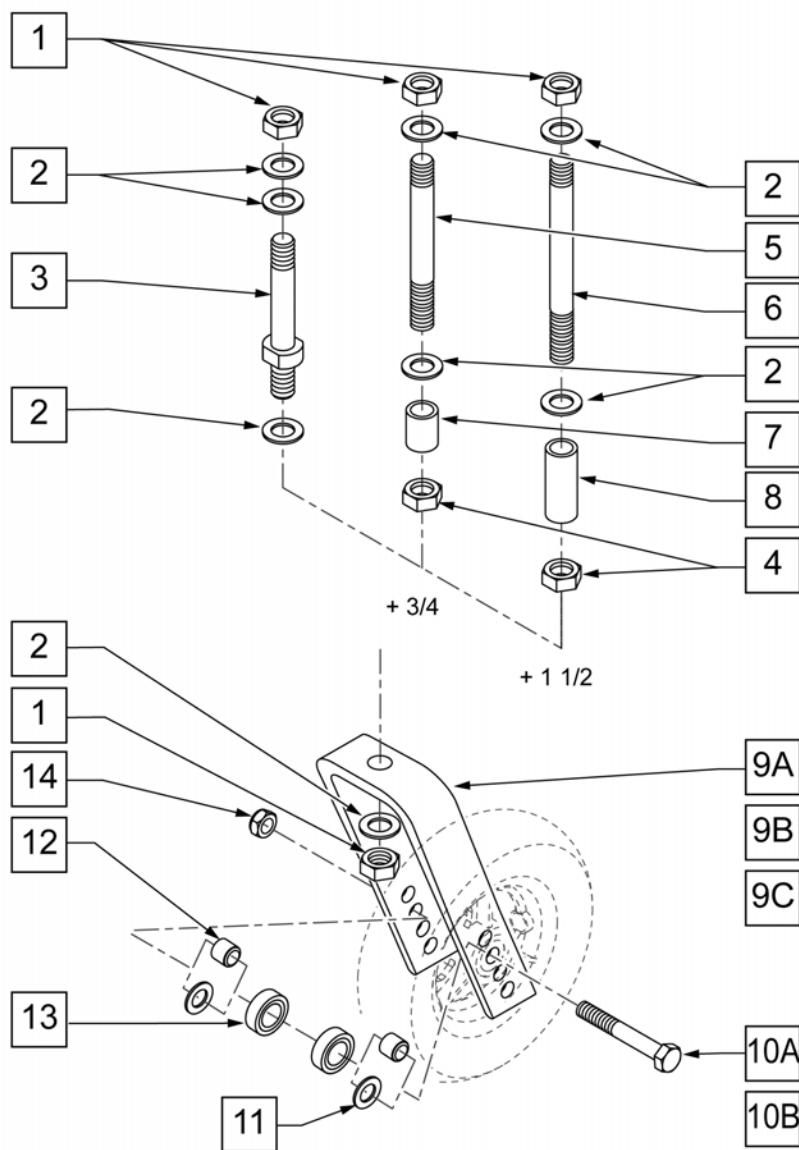


	PART NUMBER	"A"
STD	920050	3 5/8"
+ 3/4	920100	4 3/8"
+1 1/2	920110	5 1/8"

NOTE: CASTER STEM  
DIAMETER IS 1/2".

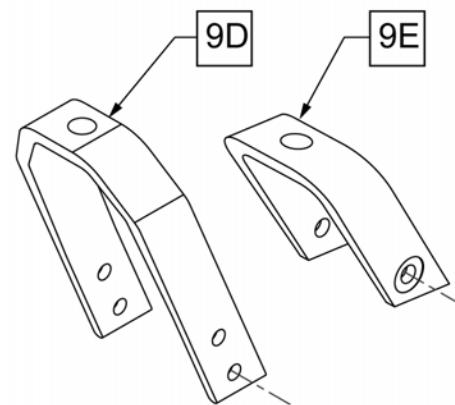

Pos.	Item Number	Description	Remarks
1	106940	NUT NYLON THIN 1/2 -20 ROHS	
2	106949	WASHER FLAT AN 1/2 ROHS	
4	203110	1/2 - 20 JAM NUT	
1-3	920050	REG CASTER STEM W/ HARDWARE	STD STEM
7	521560	+ 3/4 CASTER STEM SPACER BLK	+3/4" STEM
8	521562	+ 1 1/2 CASTER STEM SPACER BLK	+1 1/2" STEM
9A	105839-001	FORK CASTER 5 1/4"	WORKS WITH 4", 5", 6", & 8" CASTERS
9B	105839-002	FORK CASTER 6"	WORKS WITH 4", 5", 6", & 8" CASTERS
9C	105839-003	FORK CASTER 7"	WORKS WITH 4", 5", 6", & 8" CASTERS
9D	550730	8" FORK P100 ONLY	WORKS WITH 8" x 2" CASTER ONLY
9E	106810	FORK CASTER 3"	WORKS WITH 3" CASTERS ONLY
10A	106987	HHCS 5/16 -24 x 2 1/2 ROHS	
10B	106988	HHCS 5/16 -24 x 3 1/4 ROHS	Used w/ 8x2 fork only.
11	106946	WASHER FLAT AN 5/16 ROHS	Used w/ 8x2 fork only.


Pos.	Item Number	Description	Remarks
12	481113	BEARING SPACER	
13	349502	7/8 x 5/16 BEARING	
14	106939	NUT NYLOCK THIN 5/16 -24 ROHS	
1-3	920050	REG CASTER STEM W/ HARDWARE	STD STEM
1, 2, 4, 5, 7	920100	+ 3/4" CASTER STEM W/ HARDWARE BLK	+3/4" STEM
1, 2, 4, 6, 8	920110	+ 1 1/2" CASTER STEM W/ HARDWARE BLK	+1 1/2" STEM
1-3,9E,10A,12	109334	FORK CASTER 3" REG STEM	STD Stem - 3" Fork - Works w/ 3" Casters only
1-3,9A,10A,12	109335	FORK CASTER 5 1/4" REG STEM	STD Stem - 5 1/4" Fork - Works w/ 4", 5", 6", & 8" Casters
1,2,4,5,7,9A,10A,12	109336	FORK CASTER 5 1/4" + 3/4" STEM	+3/4" Stem - 5 1/4" Fork - Works w/ 4", 5", 6", & 8" Casters
1,2,4,6,8,9A,10A	109337	FORK CASTER 5 1/4" + 1 1/2" STEM	+1 1/2" Stem - 5 1/4" Fork - Works w/ 4", 5", 6", & 8" Casters
1-3,9B,10A,12	109338	FORK CASTER 6" REG STEM	STD Stem - 6" Fork - Works w/ 4", 5", 6", & 8" Casters
1,2,4,5,7,9B,10A	109339	FORK CASTER 6"+ 3/4" STEM	+ 3/4" Stem - 6" Fork - Works w/ 4", 5", 6", & 8" Casters
1,2,4,6,8,9B,10A	109340	FORK CASTER 6" + 1 1/2" STEM	+1 1/2" Stem - 6" Fork - Works w/ 4", 5", 6", & 8" Casters
1-3,9C,10A,12	109341	FORK CASTER 7" REG STEM	STD Stem - 7" Fork - Works w/ 4", 5", 6", & 8" Casters
1,2,4,5,7,9C,10A,12	109342	FORK CASTER 7" + 3/4" STEM	+3/4" Stem - 7" Fork - Works w/ 4", 5", 6", & 8" Casters
1,2,4,6,8,9C,10A,12	109343	FORK CASTER 7" + 1/1/2" STEM	+1 1/2" Stem - 7" Fork - Works w/ 4", 5", 6", & 8" Casters
1-3,9D,10B,11	109344	FORK CASTER 8" REG STEM	STD Stem - 8" Fork - Works w/ 8" X 2" Casters only
1,2,4,5,7,9D,10B,11	109345	FORK CASTER 8 + 3/4" STEM	+3/4" Stem - 8" Fork - Works w/ 8" X 2" Casters only
1,2,4,6,8,9D,10B,11	109346	FORK CASTER 8 + 1 1/2" STEM	+1 1/2" Stem - 8" Fork - Works w/ 8" X 2" Casters only

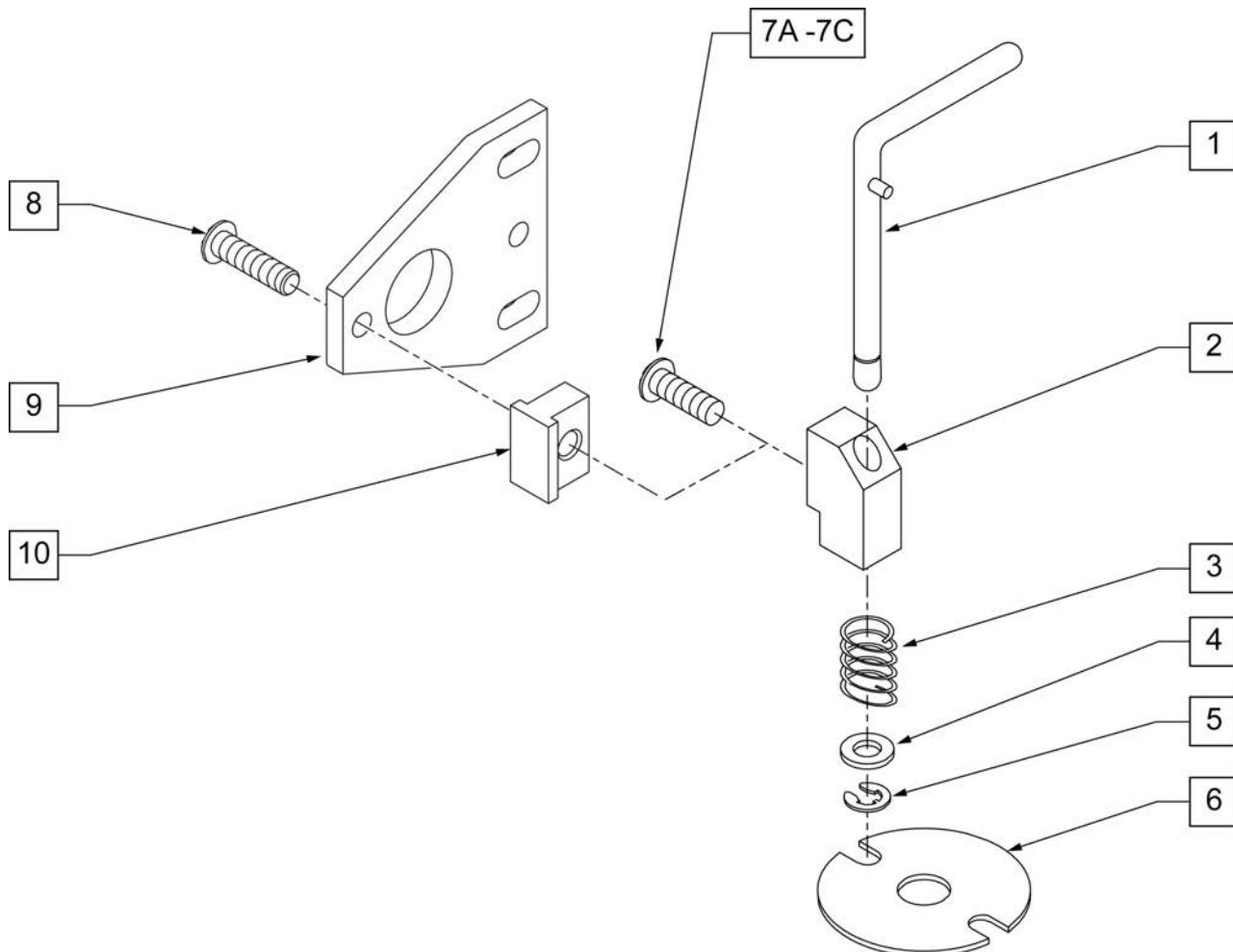

Pos.	Item Number	Description	Remarks
1	880761	PIN LOCK ARM W/ PIN S/A	RIGHT
1	880762	PIN LOCK ARM W/ PIN S/A	LEFT
2	500262	CASTER PIN LOCK BLOCK S/A BLK	
3	321141	.650L x 6 COIL SPRING	
4	106943	WASHER FLAT AN 1/4 ROHS	
5	342000	1/4" E-CLIP	
6	505012	CASTER PIN LOCK PLATE	
7A	106959	BHCS 1/4 -28 x 5/8 ROHS	USED W/ S/A HANGER FRT FRAME ONLY
7B	107015	HHCS 1/4 -28 x 1 3/4 ROHS	USED W/ TRANSIT OPTION
7C	106973	HHCS 1/4 -28 x 2 ROHS	USED FOR GP FIXED FRT FRAME ONLY
8	106959	1/4 -28 X 5/8 BH C/S	
9	505667	PIN LOCK PLATE HI-ANGLE CASTER	
10	505666	PIN LOCK BLOCK HI-ANGLE CASTER	
1-5	880750	CASTER PIN LOCK S/A	RIGHT
1-5	880751	CASTER PIN LOCK S/A	LEFT
1-6,7A	900806	CASTER PIN LOCK ASSM S/A OFF CHAIR	USED W/ S/A HANGER FRT FRAME ONLY
1-6,7B	900804	CASTER PIN LOCK KIT TRIANGULAR CASTER NON-S/A OFF CHAIR	USED W/ TRANSIT OPTION
1-6,7C	900807	CASTER PIN LOCK ASSM GP/GPV	USED FOR GP FIXED FRT FRAME ONLY


# CASTER PLATE

(07/2010)

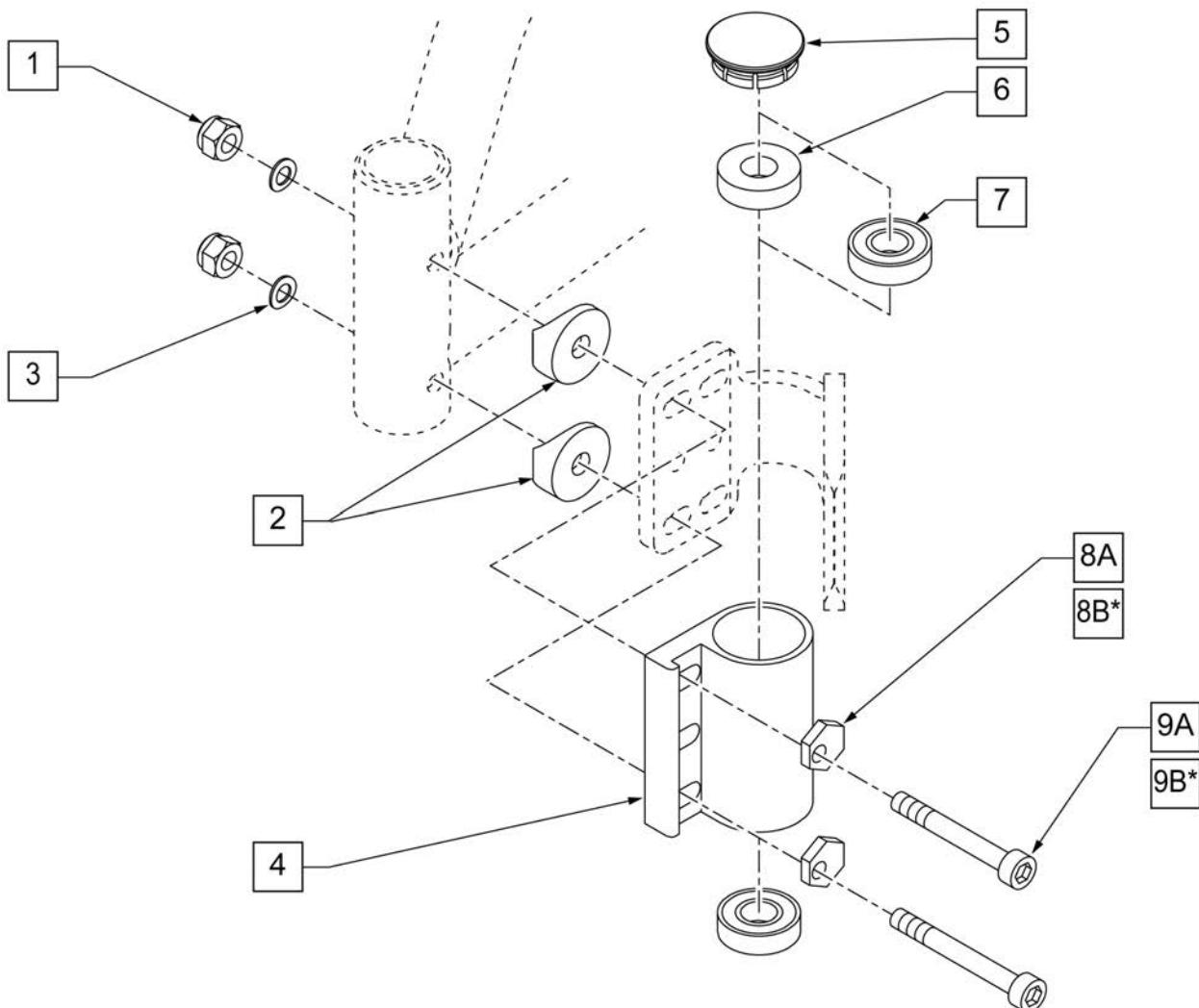

Pos.	Item Number	Description	Remarks
1	106941	NUT AUTOMATION 1/4-28 ROHS	
2	561086	1" HALF SADDLE 1/4" HOLE	
3	104414	WAVE WASHER 6 MM	
5	561070	CASTER CAP PLASTIC	
5	920028	CASTER CAP PLASTIC	PAIR
6	561050	ANTI-FLUTTER BUSHING	
7	349500	1 1/8 x 1/2 BEARING	
8A	262128	WASHER ECCENTRIC HD 1/4" HOLE	
*8B	481430	ECCENTRIC WASHER TRANSPORT	
9A	109809	SHCS 1/4 -28 x 2 ROHS	
*9B	108791	SHCS 1/4 -28 x 2 1/4 SHLDR ROHS	
4,6,7	780200	CASTER PLATE SNGLE BRNG BLK	RIGHT
4,6,7	780201	CASTER PLATE SNGLE BRNG BLK	LEFT
4,7	780210	CASTER PLATE DBL BRNG BLK	RIGHT
4,7	780212	CASTER PLATE DBL BRNG BLK	LEFT
1-3,5,8-10	920474	CASTER PLATE HI ANGLE HDWR KIT	



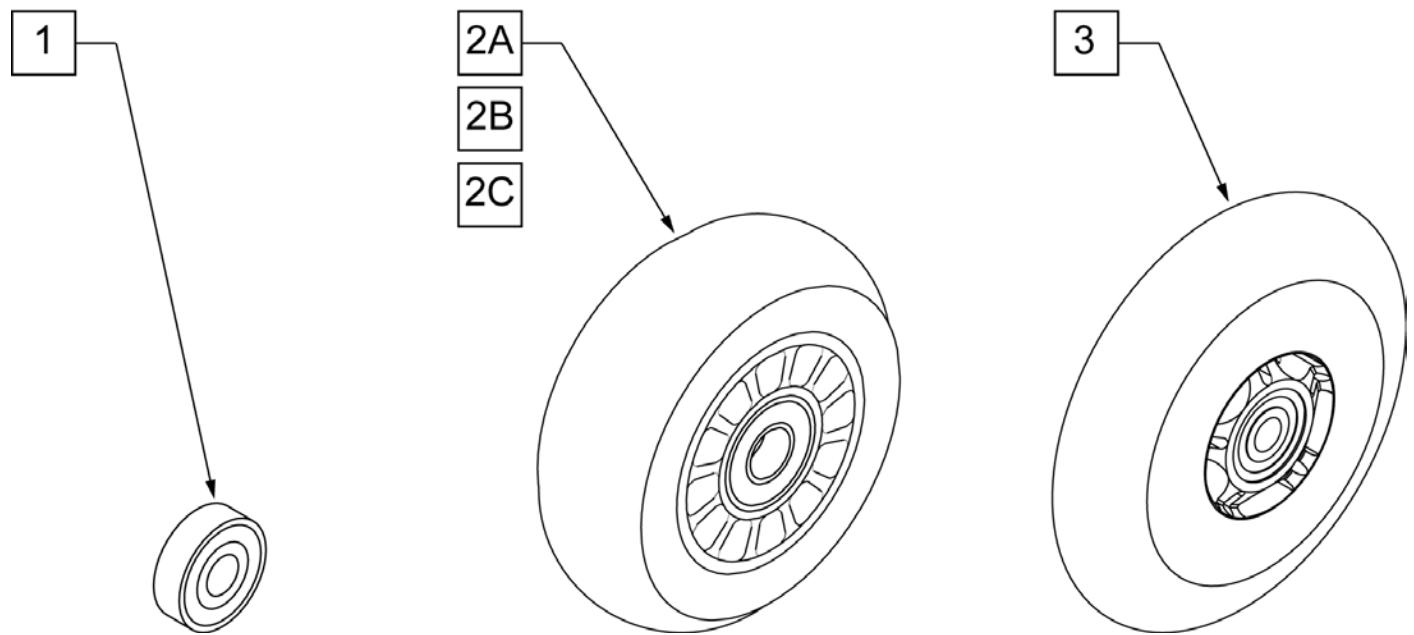


Pos.	Item Number	Description	HCPC	Remarks
1	349502	7/8"x 5/16" BEARING		
2A	400108	4" MICRO CASTER LIGHTED	K0077	4" MICRO LIGHTED
2B	920025	3" MICRO CASTER W/ BEARING -PAIR	K0077	PAIR 3" MICRO
2C	400107	3" MICRO CASTER LIGHTED	K0077	3" MICRO LIGHTED
3	052129	5" MICRO CASTER LIGHTED	K0077	5" MICRO LIGHTED


NOTE: 7" and 8" Casters are not available on the Q2 or the GT.

Note: If using 6" casters on Q7, Anti Flutter should be installed. See Standard Multi Position Fork page.

Note: Casters wider than 1" will not fit on Quickie Ti Titanium forks.

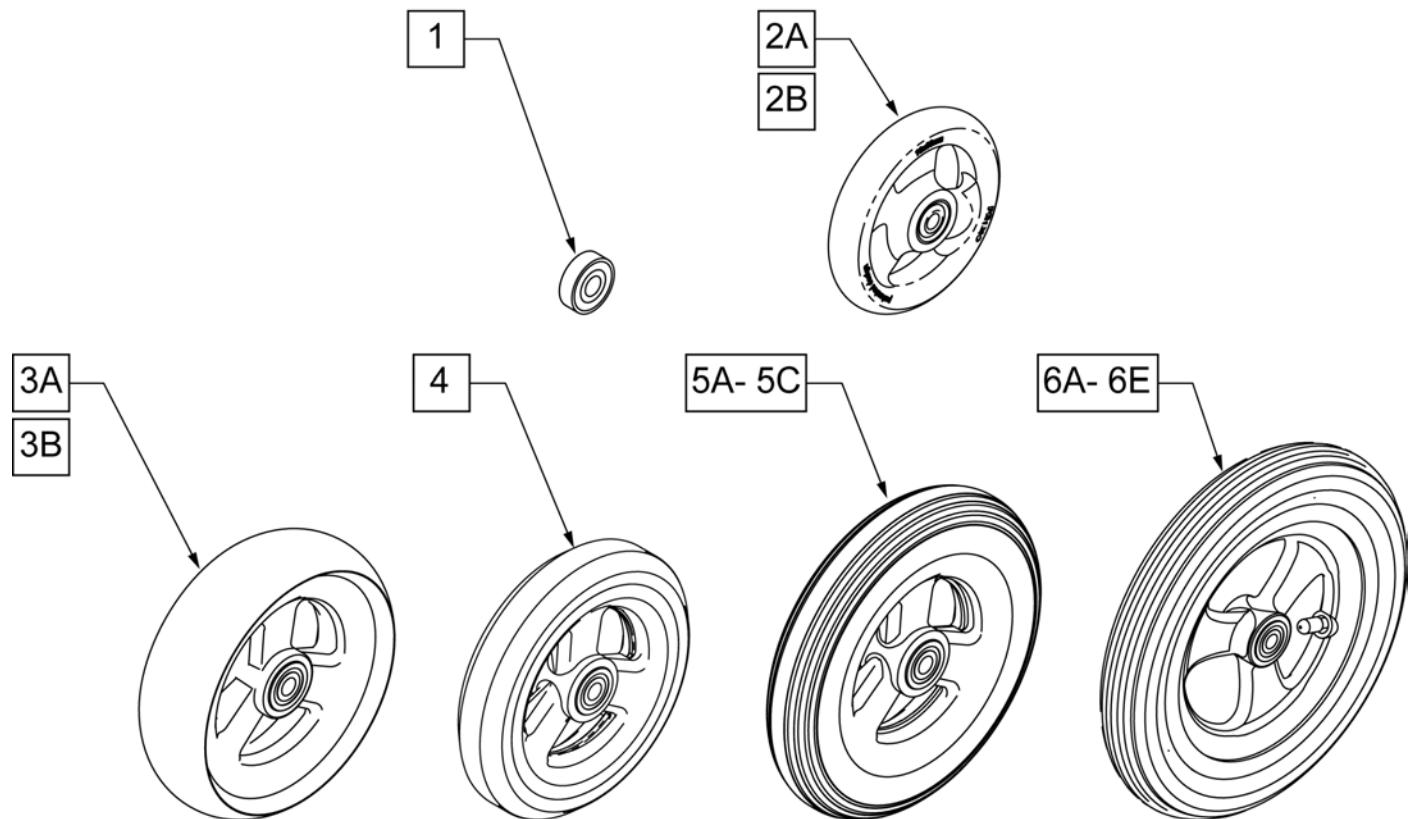


<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
1	349502	7/8"x 5/16" BEARING	
2A	101957	4"X1-1/4 POLY 3 SPOKE CSTR	4" X 1.25 SEMI-PNEUMATIC
2B	101098	CASTER 4" PLASTIC/POLY W/BRG	4" LOW PROFILE POLY
3A	101259	5"X1-1/2" POLY 3 SPOKE CSTR	5" x 1.5" SEMI-PNEUMATIC
3B	101255	6"X1-1/2" POLY 3 SPOKE CSTR	6" x 1.5" SEMI-PNEUMATIC
4	101254	5" LOW POLY 3 SPOKE CASTER	5" LOW PROFILE POLY
5A	101256	6" SEMI PNEU 3 SPOKE CASTER	6" POLY
5B	101257	6" PNEUMATIC 3 SPOKE CASTER	6" PNEUMATIC
5C	383245	6" PNEUMATIC TIRE & TUBE	
6A	400212	CASTER 8" PNEU 3 SPK ASSM	8" PNEUMATIC
6B	400213	CASTER 8" POLY 3 SPK ASSM	8" POLY
6C	400214	CASTER 8" LOW POLY 3 SPK ASSM	7" x LOW PROFILE POLY
6D	383255	8" PNEUMATIC TIRE & TUBE	
6E	400215	CASTER 8" WIDE POLY 3 SPK ASSM	8" x 1.5" LOW POLY

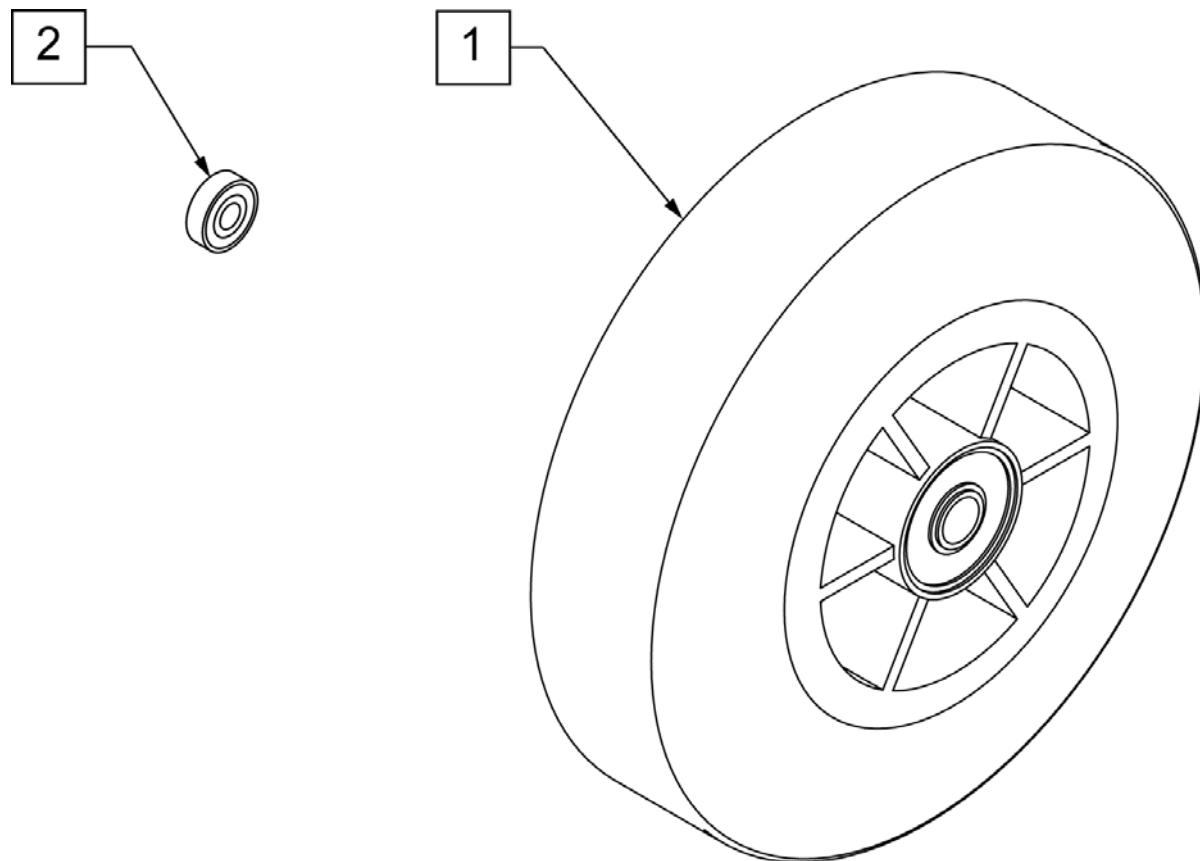


Pos.	Item Number	Description	HCPC	Remarks
1	400103	CASTER 5" SOFT ROLL W/BRNGS	K0077	5" SOFT ROLL SEMI PNEU (GREY)
2	349502	7/8"x 5/16" BEARING		

**8" x 2" CASTERS**


(05/2011)

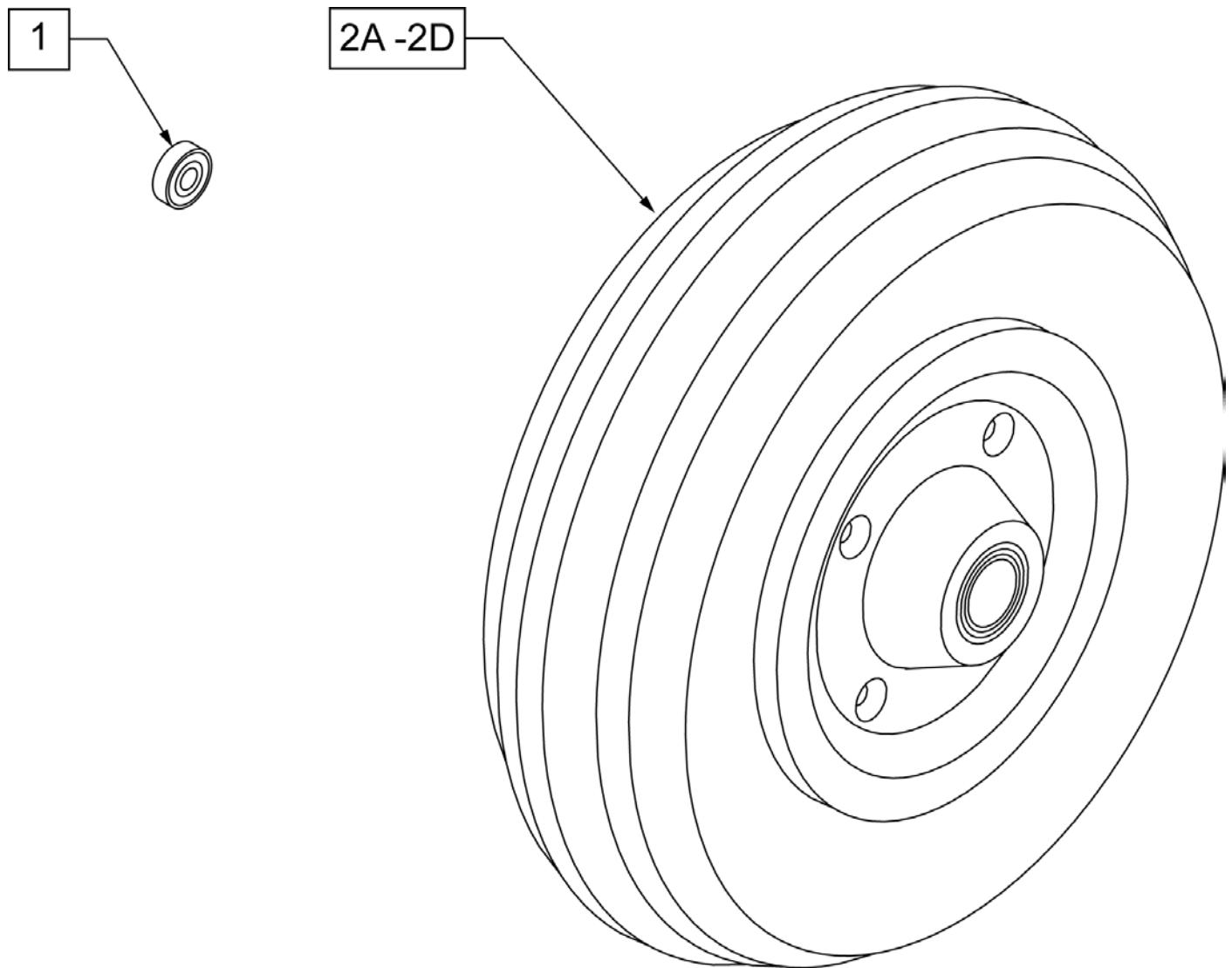

Pos.	Item Number	Description	Remarks
1	349502	7/8"x 5/16" BEARING	
2A	400700	8" X 2" CMPST W/PNEU & BEARING	8" x 2" PNEUMATIC
2B	400702	8" X 2" CMPST W/PNEU INSRT&BRG	8" x 2" PNEUMATIC W/ INSERT
2C	384700	TIRE PNEUMATIC 8" x 2"	
2D	384730	INNER TUBE CASTER 8" x 2"	

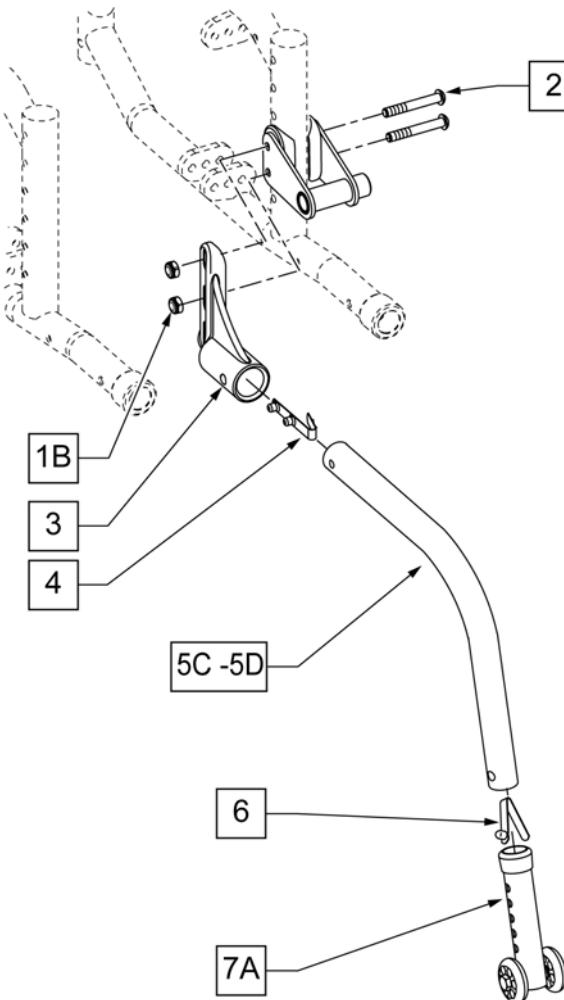


For use w/ 18-24" Wheels

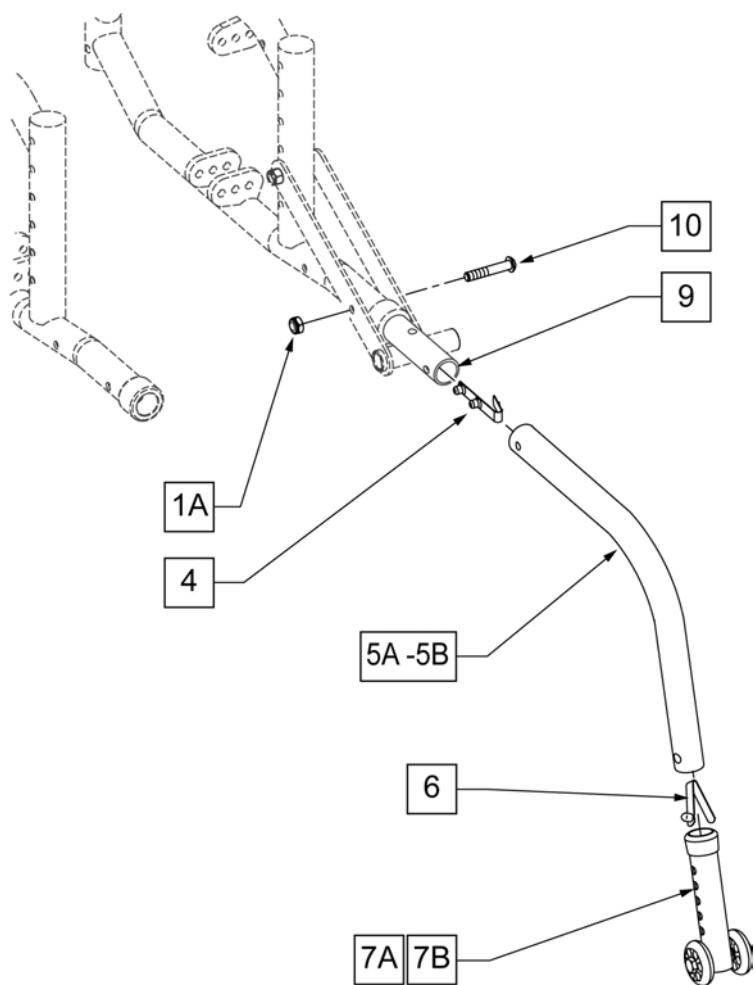

For use w/ 12" &amp; 16" Wheels

Pos.	Item Number	Description	Remarks
1A	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
1B	106934	NUT NYLOCK 1/4 -28 ROHS	
2	106966	BHCS 1/4 -28 x 2 1/4 ROHS	
3	920407	ANTI-TIP RECEIVERS	PAIR
4	301109	SNAP BUTTON DUAL ONE SIDE	
4,5A,6	880985	ANTI TIP TUBE TS W/ SNAP BUTTON	FOR USE W/ 12" REAR WHEELS
4,5B,6	882898	ANTI TIP TUBE KDZ R BL W/ SNAP BUTTON	RIGHT -FOR USE W/ 16" REAR WHEELS
4,5B,6	882899	ANTI TIP TUBE KDZ L BL W/ SNAP BUTTON	LEFT -FOR USE W/ 16" REAR WHEELS
4,5C,6	882966	ANTI TIP TUBE X/S R W/ SNAP BUTTON	RIGHT -FOR USE W/ 18", 20", 22" REAR WHEELS
4,5C,6	882967	ANTI TIP TUBE X/S L W/ SNAP BUTTON	LEFT -FOR USE W/ 18", 20", 22" REAR WHEELS
4,5D,6	882974	ANTI TIP TUBE EXTND REG R W/ SNAP	RIGHT -FOR USE W/ 24" REAR WHEELS
4,5D,6	882975	ANTI TIP TUBE EXTND REG L W/ SNAP	LEFT -FOR USE W/ 24" REAR WHEELS
6	301100	SNAP BUTTON 1/4" SINGLE	
7A	881512	ANTI-TIP SLEEVE W/WHEELS BLK (4H)	USED FOR 16", 18", 20", 22" & 24" REAR WHEELS
7B	880486	ANTI TIP SLEEVE W/ WHLS BZ/ZP	USED FOR 12" WHEELS ONLY
8	561004	1" SADDLE 1/4" HOLE	
9	109734	TUBE EXTENSION FRAME	
10	112390	BHCS 1/4 -28 x 1 3/4 FULL ROHS	
4,5A,6,7B	880510	ANTI TIP TUBE TS 12" WHEEL	For 12" WHEELS


<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
4,5A,6,7B	900705	ANTI TIP ASSM TS 12" WHEEL	PAIR
4,5B,6,7A	882904	ANTI TIP ASSM KIDZ R BLACK	For 16" WHEELS -RIGHT
4,5B,6,7A	882905	ANTI TIP ASSM KIDZ L BLACK	For 16" WHEELS -LEFT
4,5B,6,7A	900733	ANTI TIP ASSM KIDZ BLK	PAIR
4,5C,6,7A	882908	ANTI TIP ASSM SMALL EXT R BLK	For 18", 20",22" WHEELS - RIGHT
4,5C,6,7A	882909	ANTI TIP ASSM SMALL EXT L BLK	For 18", 20",22" WHEELS - LEFT
4,5C,6,7A	900696	ANTI TIP ASSM SMALL EXTEND	PAIR
4,5D,6,7A	882910	ANTI TIP ASSM REG EXTEND R BLK	For 24" WHEELS -RIGHT
4,5D,6,7A	882911	ANTI TIP ASSM REG EXTEND L BLK	For 24" WHEELS -LEFT
4,5D,6,7A	900697	ANTI-TIP ASSM REGULAR EXTEND	PAIR
1B,2,3	115730	A/T REC W/ HDW KIT R	RIGHT
1B,2,3	115731	A/T REC W/ HDW KIT L	LEFT

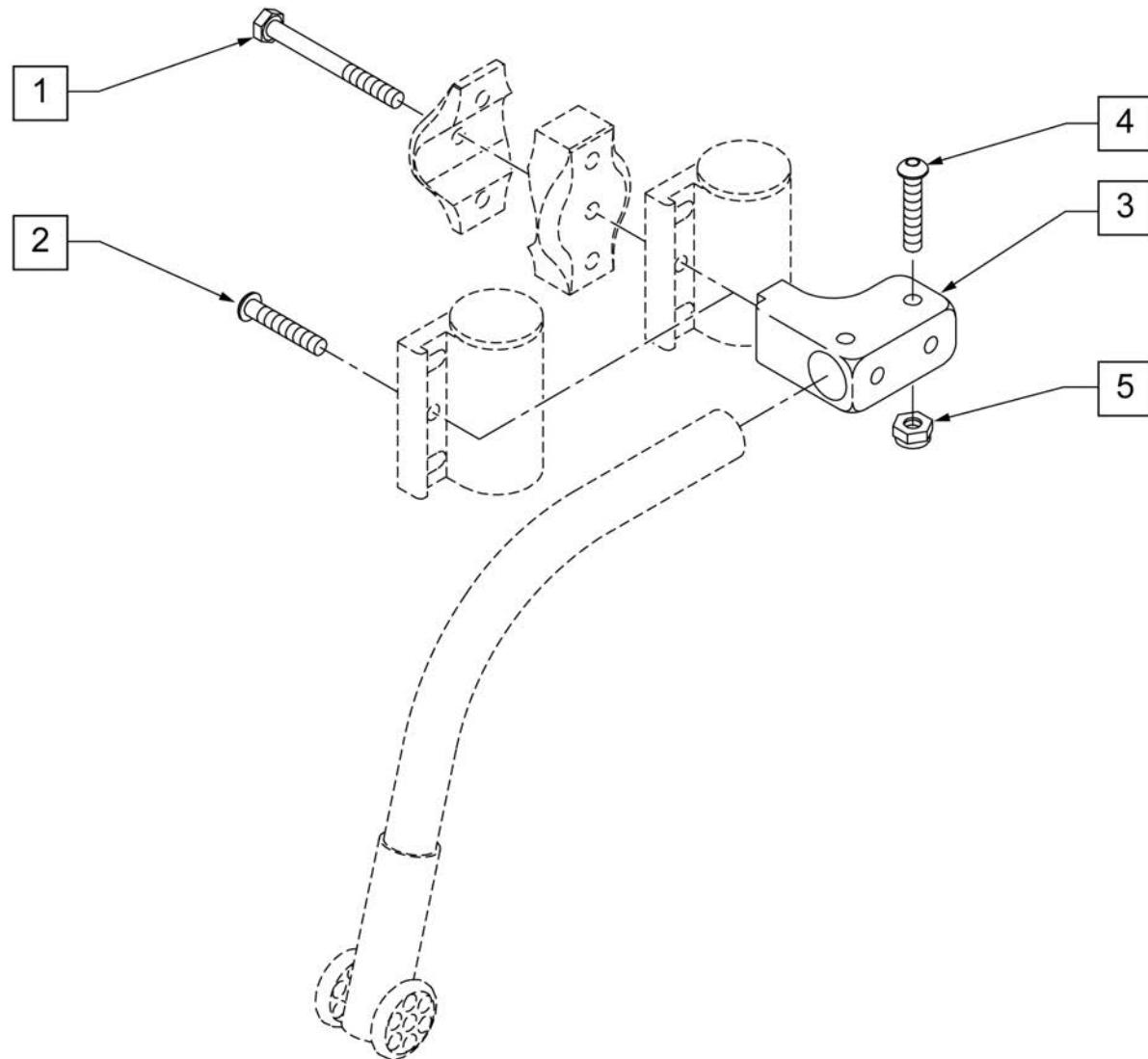

Pos.	Item Number	Description	Remarks
1	106973	HHCS 1/4 -28 x 2 ROHS	
2	106961	BHCS 1/4 -28 x 1 1/4 ROHS	
3	505340	ANTI-TIP RECEIVER BLOCK FRONT	
4	113283	BHCS 10 -32 x 1 1/8 ROHS	
5	106930	NUT NYLOCK 10 -32 ROHS	
1-5	920408	FRONT ANTI-TIP REC CASTER MOUNT	PAIR

## HUB LOCK for use w/ 12" AXLE BRACKETS

(12/2011)

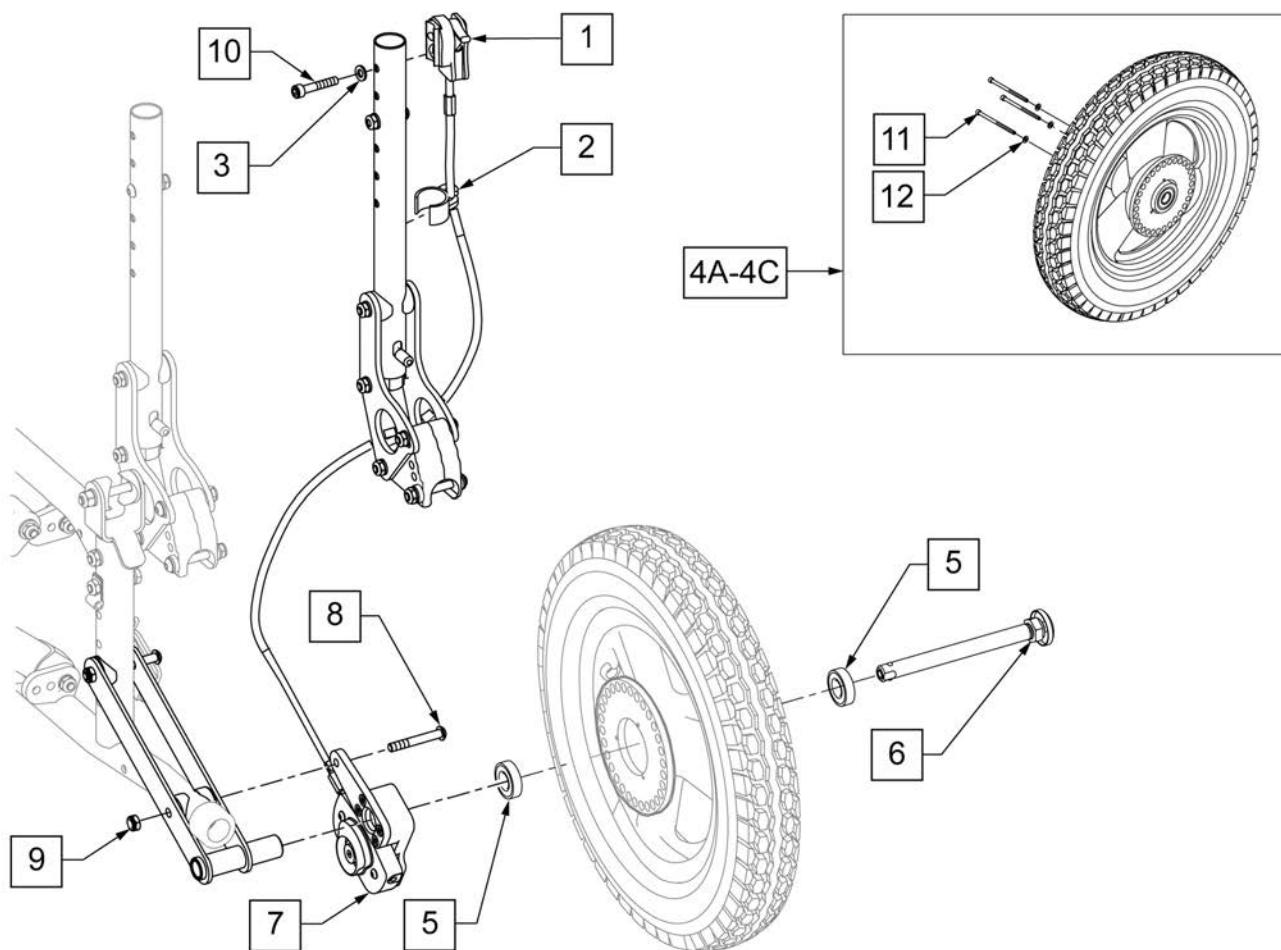


Pos.	Item Number	Description	Remarks
1	112426	HUB LOCK LEVER ASSM R	DOES NOT INCLUDE CABLE
1	112427	HUB LOCK LEVER ASSM L	DOES NOT INCLUDE CABLE
2	108134	CLIP CABLE 1" (QTY 10)	PACK OF 10
3	106943	WASHER FLAT AN 1/4 ROHS	
4A	112226-001	WHEEL HUBLOCK 12" PNEU	
4B	112226-002	WHEEL HUBLOCK 12" AIRLESS	
4C	112226-003	WHEEL HUBLOCK 12" LOW POLY	
4D	112227-001	WHEEL HUBLOCK 16" PNEU	
4E	112227-002	WHEEL HUBLOCK 16" AIRLESS	
5	349500	1 1/8" x 1/2" BEARING	
6	102065	QR AXLE W/ M12 NUT	
7	112816	LOCK MECH HUB LOCK ZTS 12" R	RIGHT-INCLUDES CABLE
7	112817	LOCK MECH HUB LOCK ZTS 12" L	LEFT -INCLUDES CABLE
8	112785	BHCS 1/4-28 X 2 FULL ROHS	
9	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
10	106976	SHCS 1/4 -28 x 1 1/2 ROHS	
11	112231	SHCS 6-32 X 2-1/4 ROHS	
12	112230	WASHER FLAT #6	

Pos.	Item Number	Description	Remarks
1-3,7-10	115732	ZTS HBLCK 12"AXL BRKT (PR)	PAIR - MUST ORDER WHEEL & AXLE SEPERATELY


# HUB LOCK for use w/ LONG AXLE BRACKET

(12/2011)

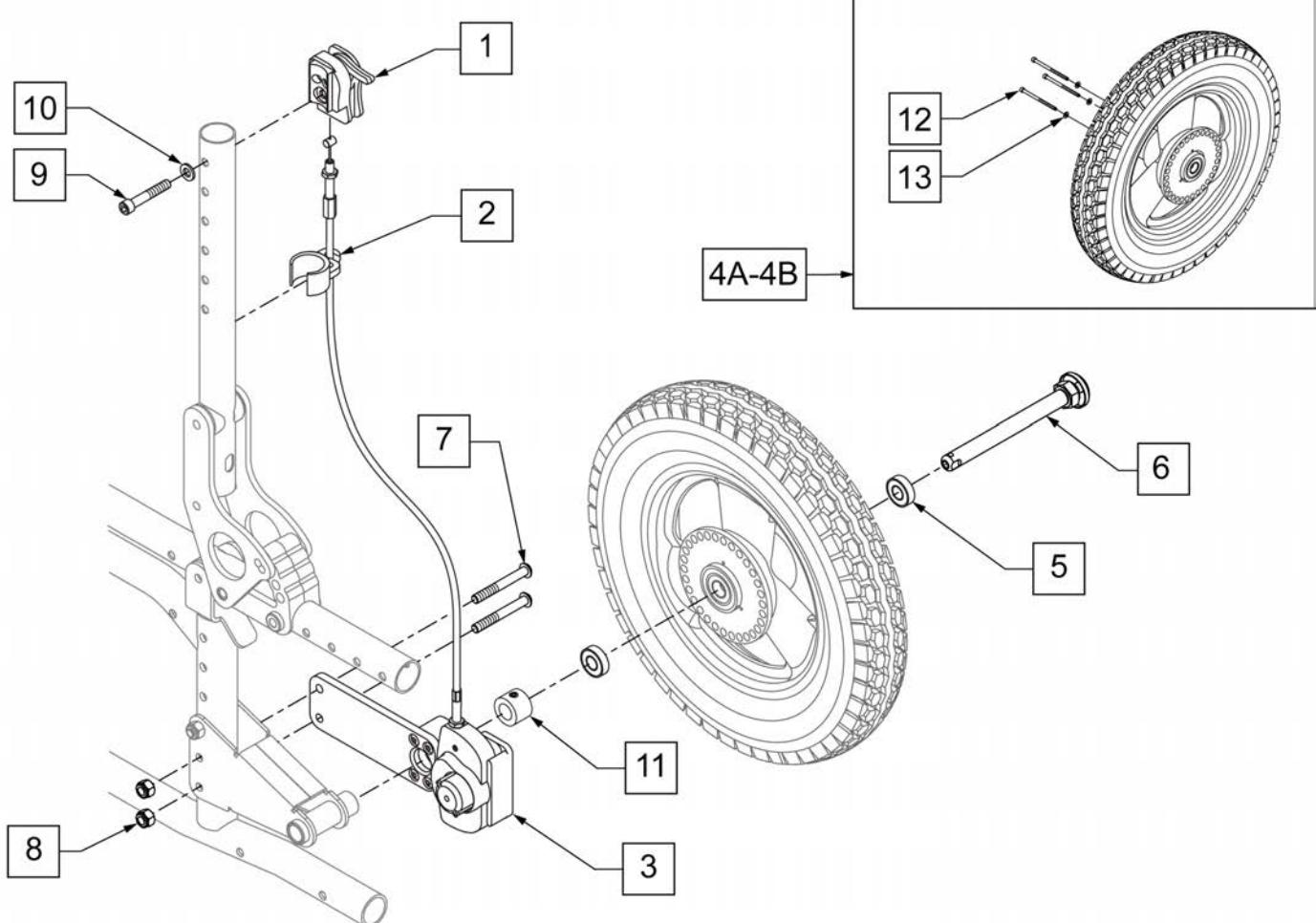


Pos.	Item Number	Description	Remarks
1	112426	HUB LOCK LEVER ASSM R	
1	112427	HUB LOCK LEVER ASSM L	
2	108134	CLIP CABLE 1" (QTY 10)	PACK OF 10
3	112819	LOCK ASSY HUB LOCK ZTS LONG R	RIGHT -INCLUDES CABLE
3	112820	LOCK ASSY HUB LOCK ZTS LONG L	LEFT -INCLUDES CABLE
4A	112227-001	WHEEL HUB LOCK 16IN PNEU	
4B	112227-002	WHEEL HUB LOCK 16IN AIRLESS	
5	349500	1 1/8" x 1/2" BEARING	
6	102065	QR AXLE W/ M12 NUT	
7	106964	BHCS 1/4-28 X 2 ROHS	
8	106934	NUT NYLOCK 1/4-28 ROHS	
9	106976	SHCS 1/4 -28 x 1 1/2 ROHS	
10	106943	WASHER FLAT AN 1/4 ROHS	
11	920117	MAG SPACER 6S W/ SET SCREW	
12	112231	SHCS 6-32 X 2-1/4 ROHS	
13	112230	WASHER FLAT #6	
1-3,7-11	115733	ZTS HBLCK LNG BRKT (PR)	PAIR - MUST ORDER WHEEL & AXLE SEPERATELY

## HIGH MOUNT PUSH WHEEL LOCK

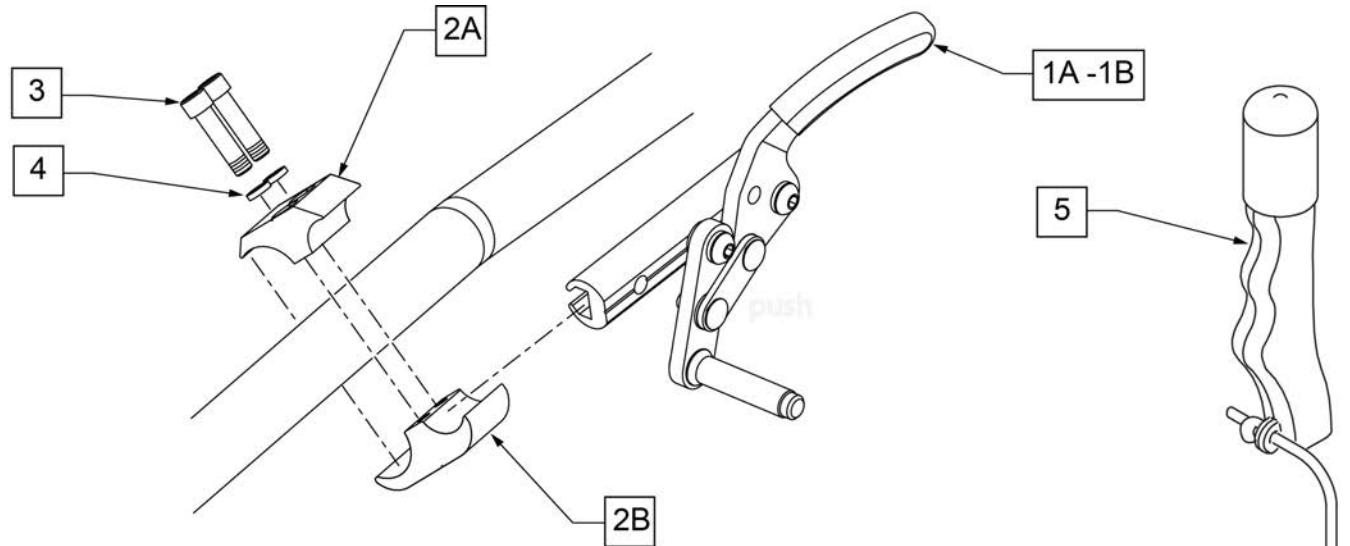


MEASURING GUIDE

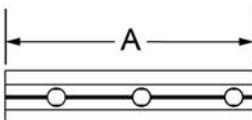

SLIDE WHEEL LOCK	DIM "A"
SHORT	3.50"
LONG	5.91"


WHEEL SIZE	AXLE PLATE	WHEEL LOCK CONFIG.	FRAME MOUNT TUBE
12"	12"	FORWARD	LOWER
12" LOW POLY	12"	REVERSE	LOWER
16"	SHORT	FORWARD	LOWER
	LONG	REVERSE	LOWER
18"	SHORT	FORWARD	LOWER
	LONG	REVERSE	LOWER
20"	SHORT	FORWARD	UPPER
	LONG	FORWARD	UPPER
22"	SHORT	FORWARD	UPPER
	LONG	FORWARD	UPPER
24"	SHORT	FORWARD	UPPER
	LONG	FORWARD	UPPER

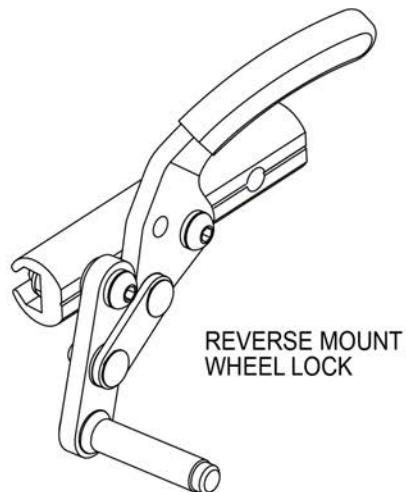


Pos.	Item Number	Description	Remarks
1A	104216-005	WHEEL LOCK PUSH R FWD LONG	
1A	104216-006	WHEEL LOCK PUSH L FWD LONG	
1B	104216-007	WHEEL LOCK PUSH R REV LONG	
1B	104216-008	WHEEL LOCK PUSH L REV LONG	
2A,2B,3,4	107639	WHL LCK CLMP ASM	
3	107032	SHCS 1/4 -28 x 1 ROHS	
4	107367	WASHER LOCK HI COLLAR 1/4 ROHS	
5	118068	WHEEL LOCK EXTENSION ASSY	each
1A,5	115734	WHLLCK PUSH R FWD LNG W/EXT	
1A,5	115735	WHLLCK PUSH L FWD LNG W/EXT	
1B,5	115736	WHLLCK PUSH R REV LNG W/EXT	
1B,5	115737	WHLLCK PUSH L REV LNG W/EXT	

# HIGH MOUNT PULL WHEEL LOCK

(06/2011)

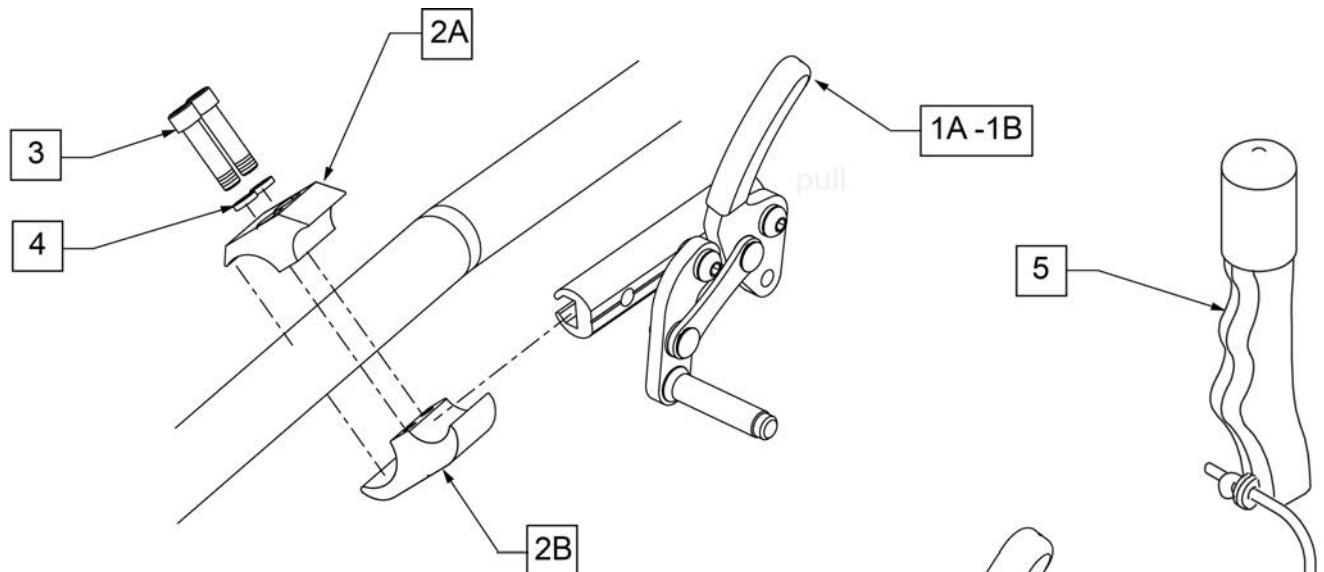


MEASURING GUIDE

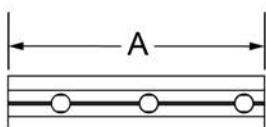

SLIDE WHEEL LOCK	DIM "A"
SHORT	3.50"
LONG	5.91"


WHEEL SIZE	AXLE PLATE	WHEEL LOCK CONFIG.	FRAME MOUNT TUBE
12"	12"	FORWARD	LOWER
12" LOW POLY	12"	REVERSE	LOWER
16"	12"	REVERSE	LOWER
	SHORT	FORWARD	LOWER
18"	LONG	REVERSE	LOWER
	SHORT	FORWARD	LOWER
20"	LONG	REVERSE	LOWER
	SHORT	FORWARD	UPPER
22"	LONG	FORWARD	UPPER
	SHORT	FORWARD	UPPER
24"	LONG	FORWARD	UPPER
	SHORT	FORWARD	UPPER

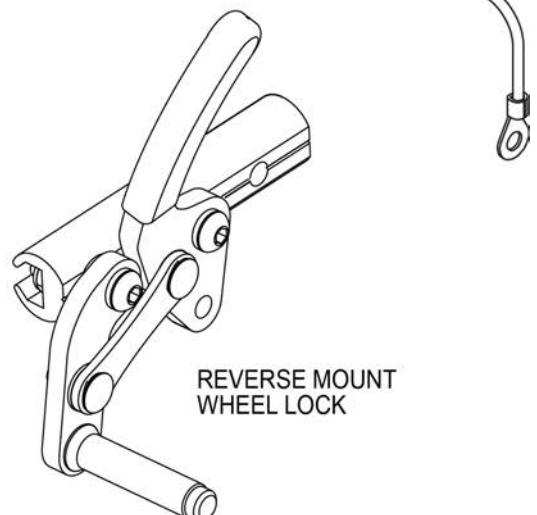

REVERSE MOUNT  
WHEEL LOCK

Pos.	Item Number	Description	Remarks
1A	104227-005	WHEEL LOCK PULL R FWD LONG	
1A	104227-006	WHEEL LOCK PULL L FWD LONG	
1B	104227-007	WHEEL LOCK PULL R REV LONG	
1B	104227-008	WHEEL LOCK PULL L REV LONG	
2A,2B,3,4	107639	WHL LCK CLMP ASM	
3	107032	SHCS 1/4 -28 x 1 ROHS	
4	107367	WASHER LOCK HI COLLAR 1/4 ROHS	
5	118068	WHEEL LOCK EXTENSION ASSY	each
1A,5	115738	WHLLCK PULL R FWD LNG W/EXT	
1A,5	115739	WHLLCK PULL L FWD LNG W/EXT	
1B,5	115740	WHLLCK PULL R REV LNG W/EXT	
1B,5	115741	WHLLCK PULL L REV LNG W/EXT	

NOTE: FOLDING FRAME SHOWN - IF MOUNTING TO A RIGID FRAME ITEMS 19-21 ARE NOT USED AND FOOT LOCK IS MOUNTING TO REAR STRUT.

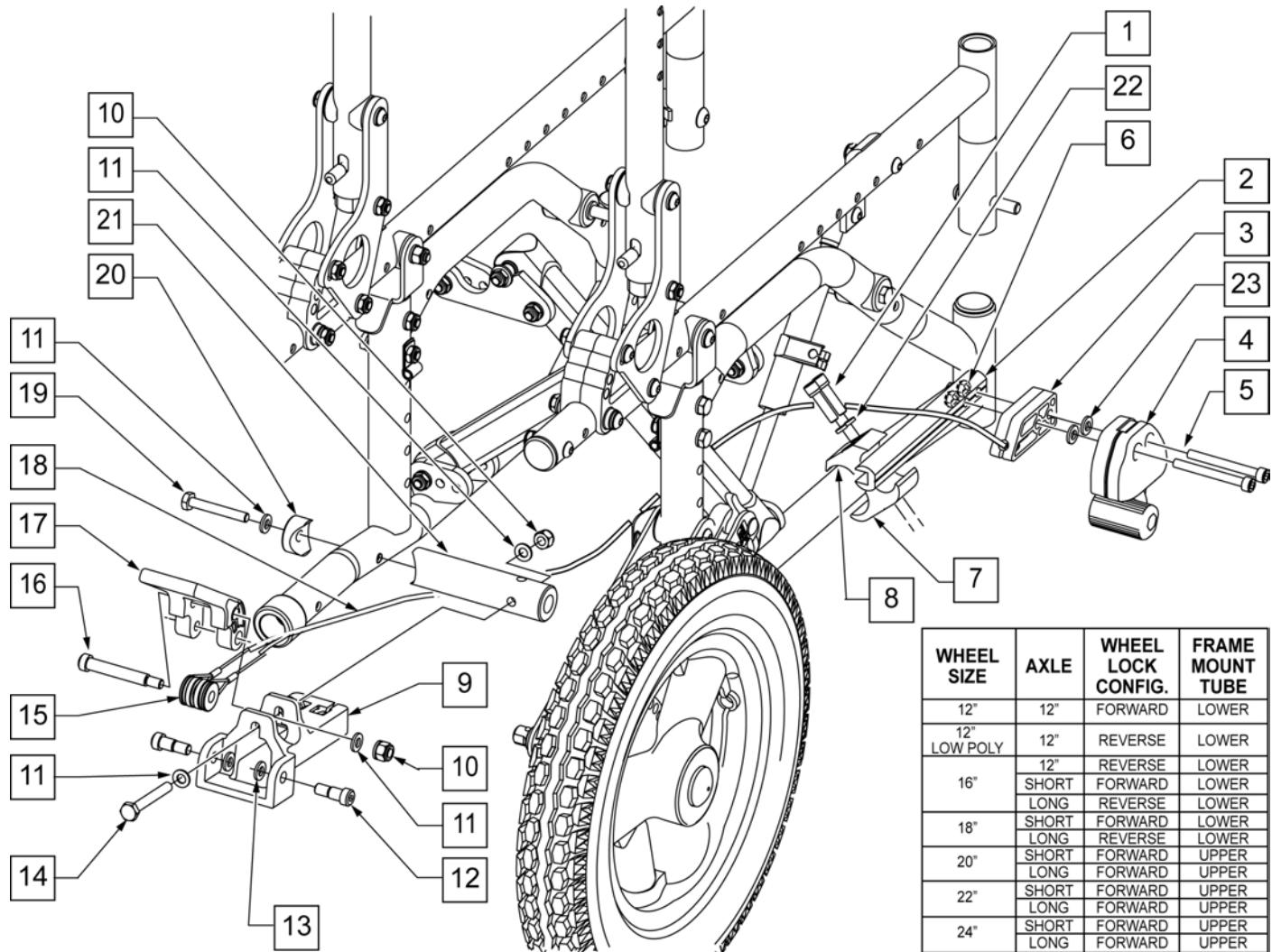


Pos.	Item Number	Description	Pos.	Item Number	Description
1	107032	SHCS 12/4 -28 x 1 ROHS	12	281048	SHLDR SCREW .312 DIA x .375 LG
2	112408	SLIDE ADJUSTABLE FOOT LOCK	13	481353	BEARING SPACER
3	506184	SPACER ATTENDANT WHEEL LOCK	14	106992	HHCS 1/4 -20 x 2 ROHS
4	119952	REMOTE LOCK RIGHT SIDE US	16	281041	SHLDR SCREW .312 DIA x 1.75 LG
4	119953	REMOTE LOCK LEFT SIDE US	19	107015	HHCS 1/4 -28 x 1 3/4 ROHS
5	103190-013	SHCS M6 x 1 x 60	20	483713	SADDLE ALUM 1" THICK P1/P5
6	104381	NUT HEX NYLOCK M6 x 1.0	21	112263	MOUNT PEDAL FOOT LOCK
1,7,8,22	107639	WHL LCK CLMP ASM	22	107367	WASHER LOCK HI COLLAR 1/4 ROHS
9	123517	ATTENDT WHL LCK MT BRKT	23	107383	WASHER FLAT M6 ROHS
10	106932	NUT NYLOCK 1/4 -20 ROHS	9-13,15-18	112419	ATTND W.L CABLE SUB ASSM SHT
11	106943	WASHER FLAT AN 1/4 ROHS	1-23	115742	FTLCK ZTS KIT

## WHEEL LOCK PUSH TO LOCK (QUICKIE STYLE)

[10/2011]

NOTE: QUICKIE STYLE WHEEL LOCKS ARE NOT OFFERED ON THE BREEZY 600 BUT WILL MOUNT DIRECTLY USING THE WHEEL LOCK CLAMP.

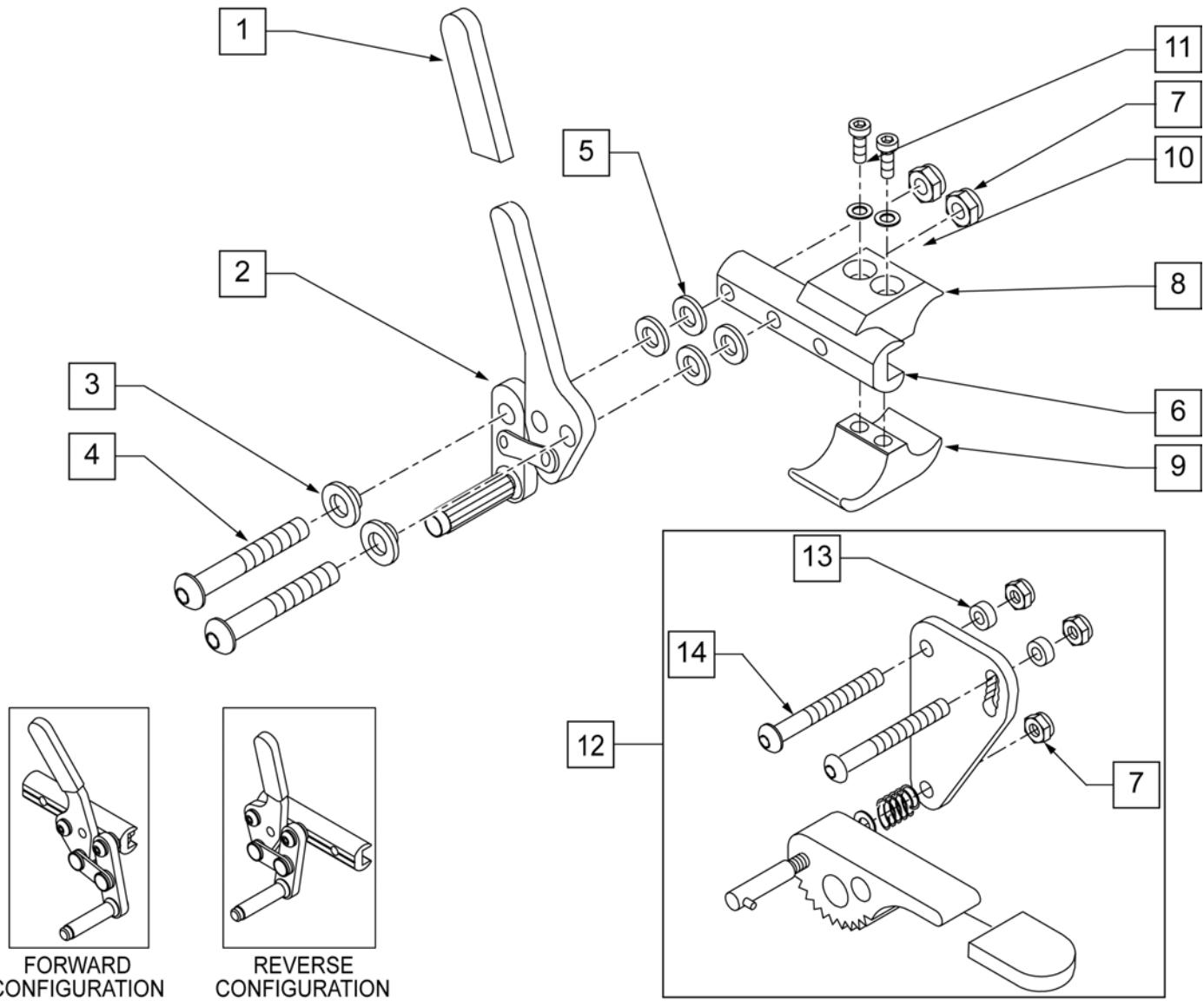


Pos.	Item Number	Description	Remarks
1	560500-QU	HB PLASTIC COVER	
1,2	881748	WHEEL LOCK ASSM R PUSH	
1,2	881749	WHEEL LOCK ASSM L PUSH	
3	560126	.250 ID .312 OD BUSHING PLASTIC	
4	226125	1/4-28 X 1 1/8 BH C/S	
5	561028	1/4" NYLON WASHER	
6	550374	WHEEL LOCK SLIDE BLOCK 3 1/2" BLK	
7	106941	1/4-28 AUTOMATION NUT	
8	550659	CLAMP 1 1/4" RECESS HALF BLK	
9	550658	CLAMP 1 1/4" THREADED BLK	

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
10	107367	WASHER LOCK HI COLLAR 1/4 ROHS	
11	107032	SHCS 1/4-28 X 1 ROHS	
12	880916	GRADE AID PLATE W/WHL & HDWR R	
12	880917	GRADE AID PLATE W/WHL & HDWR L	
6,12	880912	GRADE AID ASSM BLK R N/CLAMP	
6,12	880913	GRADE AID ASSM BLK L N/CLAMP	
13	483027	1/4 x 1/4 ALUMINUM SPACER BLK	
14	226130	1/4-28 x 1 1/2 BH C/S	
8-11	107639	WHL LCK CLMP ASM	
8-12A	880914	GRADE AID ASSM BLK R W/CLAMP	
8-12A	880915	GRADE AID ASSM BLK L W/CLAMP	
1-7	883010	WHL LOCK PUSH R W/O CLMP BLK	FORWARD CONFIG.
1-7	883011	WHL LOCK PUSH L W/O CLMP BLK	FORWARD CONFIG.
1-7	881762	WHL LOCK PUSH R REV N/ CLMP BLK	REVERSE CONFIG.
1-7	881763	WHL LOCK L PUSH REV N/ CLMP BLK	REVERSE CONFIG.
1-11	881965	WHL LOCK PUSH R W/ CLMP BLK	FORWARD CONFIG.
1-11	881966	WHL LOCK PUSH L W/ CLMP BLK	FORWARD CONFIG.
1-11	881973	WHL LOCK PUSH R REV W/CLMP BLK	REVERSE CONFIG.
1-11	881974	WHL LOCK PUSH L REV W/CLMP BLK	REVERSE CONFIG.
1-11	901551	WHEEL LOCK PUSH W/CLAMP BLK	PAIR, FORWARD CONFIG.
1-11	901542	WHEEL LOCK PUSH REV W/CLAMP BLK	PAIR, REVERSE CONFIG.
1-11,12	901557	WHEEL LOCK PUSH W/GRADE AID W/CLAMP BLK	PAIR
1-11,12	881969	WHL LOCK R PUSH W/ G.A. W/CLMP BLK	
1-11,12	881970	WHL LOCK L PUSH W/ G.A. W/CLMP BLK	
1-7,12	881754	WHL LOCK R PUSH W/G.A. N/ CLMP BLK	
1-7,12	881755	WHL LOCK L PUSH W/G.A. N/ CLMP BLK	

## WHEEL LOCK PULL TO LOCK (QUICKIE STYLE)


[10/2011]

NOTE: QUICKIE STYLE WHEEL LOCKS ARE NOT OFFERED ON THE BREEZY 600 BUT WILL MOUNT DIRECTLY USING THE WHEEL LOCK CLAMP.

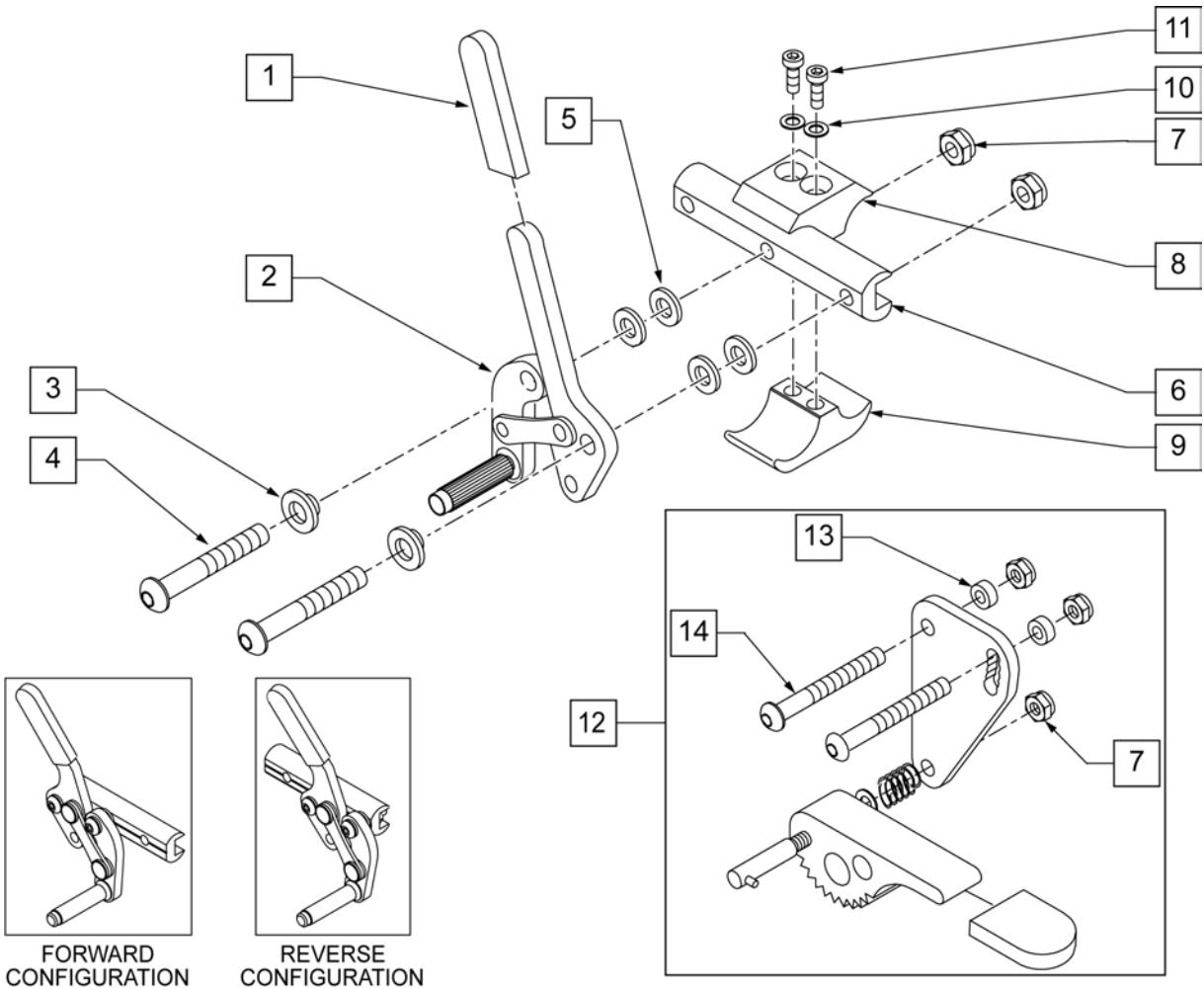

Pos.	Item Number	Description	Remarks
1	560500-QU	HB PLASTIC COVER	
3	560126	.250 ID .312 OD BUSHING PLASTIC	
4	226125	1/4-28 X 1 1/8 BH C/S	
5	561028	1/4" NYLON WASHER	
6	550374	WHEEL LOCK SLIDE BLOCK 3 1/2" BLK	
7	106941	1/4-28 AUTOMATION NUT	
8	550659	CLAMP 1 1/4" RECESS HALF BLK	
9	550658	CLAMP 1 1/4" THREADED BLK	
10	107367	WASHER LOCK HI COLLAR 1/4 ROHS	
11	107032	SHCS 1/4-28 X 1 ROHS	
12	880916	GRADE AID PLATE W/WHL & HDWR R	
12	880917	GRADE AID PLATE W/WHL & HDWR L	
6,12	880912	GRADE AID ASSM BLK R N/CLAMP	
6,12	880913	GRADE AID ASSM BLK L N/CLAMP	
13	483027	1/4 x 1/4 ALUMINUM SPACER BLK	
14	226130	1/4-28 x 1 1/2 BH C/S	

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
8-11	107639	WHL LCK CLMP ASM	
8-12	880914	GRADE AID ASSM BLK R W/CLAMP	
8-12	880915	GRADE AID ASSM BLK L W/CLAMP	
1-7	883012	WHL LOCK PULL R W/O CLMP BLK	FORWARD CONFIG.
1-7	883013	WHL LOCK PULL L W/O CLMP BLK	FORWARD CONFIG.
1-7	881764	WHL LOCK PULL R REV N/ CLMP BLK	REVERSE CONFIG.
1-7	881765	WHL LOCK L PULL REV N/ CLMP BLK	REVERSE CONFIG.
1-11	881967	WHL LOCK PULL R W/ CLMP BLK	FORWARD CONFIG.
1-11	881968	WHL LOCK PULL L W/ CLMP BLK	FORWARD CONFIG.
1-11	881975	WHL LOCK PULL R REV W/CLMP BLK	REVERSE CONFIG.
1-11	881976	WHL LOCK PULL L REV W/CLMP BLK	REVERSE CONFIG.
1-11	901555	WHEEL LOCK PULL W/CLAMP BLK	PAIR, FORWARD CONFIG.
1-11	901543	WHEEL LOCK PULL REV W/CLAMP BLK	PAIR, REVERSE CONFIG.
1-7,12	881756	WHL LOCK R PULL W/G.A. N/ CLMP BLK	
1-7,12	881757	WHL LOCK L PULL W/G.A. N/ CLMP BLK	
1-11,12	881971	WHL LOCK R PULL W/ G.A. W/CLMP BLK	
1-11,12	881972	WHL LOCK L PULL W/ G.A. W/CLMP BLK	
1-11,12	901558	WHEEL LOCK PULL W/GRADE AID W/CLAMP BLK	PAIR

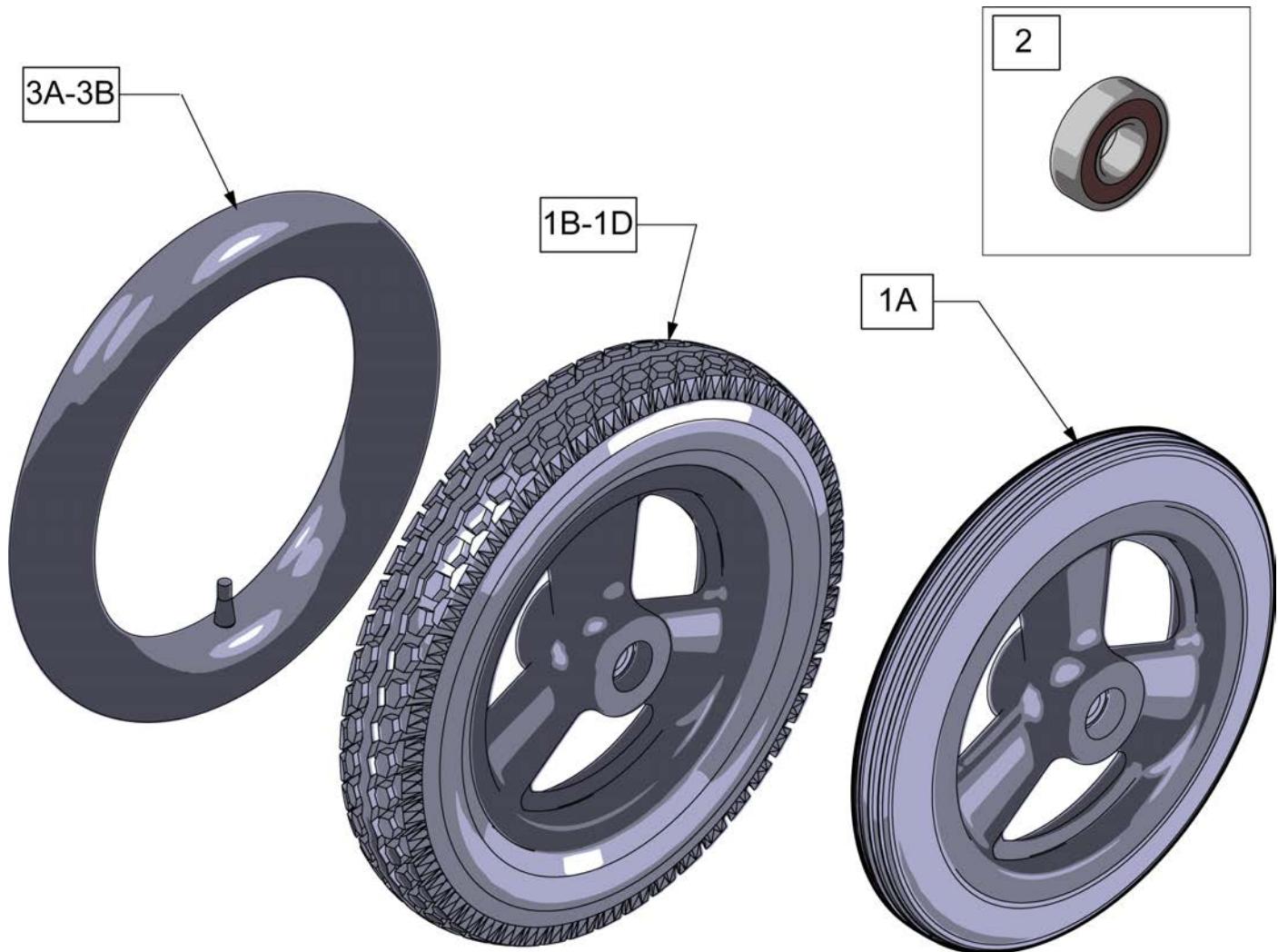

Pos.	Item Number	Description	Remarks
1A	101466	12 IN MAG POLY TIRE 3 SPOKE	Low Poly
1B	101661	12" MAG PNEU 3 SPOKE	
1C	101662	12" MAG PNEU INSRT 3 SPK	
1D	384706	TIRE PNEU 12-1/2" X 2-1/4"	Tire only
2	349500	1 1/8 X 1/2 BEARING	
3A	384736	INNER TUBE 12-1/2" X 2-1/4"	
3B	102972-001	12 1/2 IN AIRLESS INSERT	



3A-3B



1A-1B

2

Pos.	Item Number	Description	Remarks
1A	101667	16 IN MAG PNEU 3 SPOKE	
1B	101668	16 IN MAG PNEU W/INSERT	
1C	385237	16" X 1.75" PNEU TIRE	Tire Only
2	349500	1 1/8 X 1/2 BEARING	
3A	384738	16" X 1.75" INNER TUBE	
3B	102973-001	16IN AIRLESS INSERT	



Pos.	Item Number	Description	Remarks
1-3	401094	20" MAG W/BRNG 6 SPOKE	20"
1-3	401093	22" MAG W/BRNG 6 SPOKE	22"(501mm)
1-3	401092	24" MAG W/BRNG 6 SPOKE	24"(540mm)
2	349500	1 1/8 X 1/2 BEARING	
3	521564	MAG BEARING SPACER	
4	101932	SPCR .194ID X .375 OD X .750	Used when previously ordered with Omit handrims or Natural Fit Handrims

(10/2010)

**NOTE:** THIS WHEEL UTILIZES SCREW MOUNT HANDRIMS - THE HANDRIM HAS A THREADED HOLE THAT MOUNTS THE HANDRIM TO THE WHEEL THROUGH A SPACER AND FASTNER.

**NOTE:** IF MAG WHEEL ASSEMBLY WAS ORDER WITH OUT HANDRIM AND A HANDRIM IS ADDED TO THE WHEEL QTY 6 OF 561037 AND 281175 MUST BE ORDERED TO PROPERLY MOUNT THE HANDRIM.

**NOTE:** THIS WHEEL UTILIZES SCREW MOUNT HANDRIMS - THE HANDRIM HAS A THREADED HOLE THAT MOUNTS THE HANDRIM TO THE WHEEL THROUGH A SPACER AND FASTNER.



Pos.	Item Number	Description	Remarks
1A	116178	WHEEL ASSY MBL 20"	20" (451)
1B	116179	WHEEL ASSY MBL 22"	WHL RIM/SPOKE MBL 22 (501)
1C	116180	WHEEL ASSY MBL 24"	WHL RIM/SPOKE MBL 24 (540)
1D	116181	WHEEL ASSY MBL 25"	WHL RIM/SPOKE MBL 25 (559)
1E	116182	WHEEL ASSY MBL 26"	WHL RIM/SPOKE MBL 26 (590)
2	114097	BEARING, REPLACEMENT, MBL	12.7mm i.d. - 28mm o.d. - 8mm wide
3A	118100	MBL LT 20IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3B	118101	MBL LT 22IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3C	118102	MBL LT 24IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3D	118103	MBL LT 25IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3E	118104	MBL LT 26IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes

**NOTE:** THIS WHEEL UTILIZES TAB MOUNT HANDRIMS - THE HANDRIM HAS A STRAIGHT TAB THAT MOUNTS TO THE WHEEL THROUGH THE RIM.



Pos.	Item Number	Description	Remarks
1	762438	SPINERGY STD WHEEL 540 BLK SPK	24"(540mm)
1	762441	SPINERGY STD WHL 559 BLK SPK	25"(559mm)
1	762437	SPINERGY STD WHEEL 590 BLK SPK	26"(590mm)
1	762442	SPINERGY STD WHEEL 540 BLUE	24"(540mm)
1	762444	SPINERGY STD WHEEL 559 BLU	25"(559mm)
1	762446	SPINERGY STD WHEEL 590 BLUE	26"(590mm)
1	762443	SPINERGY STD WHEEL 540 RED	24"(540mm)
1	762445	SPINERGY STD WHEEL 559 RED	25"(559mm)
1	762447	SPINERGY STD WHEEL 590 RED	26"(590mm)
1	762456	SPINERGY STD WHEEL 540 WHT	24"(540mm)
1	762463	SPINERGY STD WHL 559 WHT SPK	25"(559mm)
1	762467	SPINERGY STD WHEEL 590 WHT	26"(590mm)
1	762433	SPINERGY STD WHEEL 540 YELLOW	24"(540mm)
1	762462	SPINERGY STD WHL 559 YLW SPK	25"(559mm)
1	762431	SPINERGY STD WHEEL 590 YELLOW	26"(590mm)
2	349500	1 1/8 X 1/2 BEARING	




Pos.	Item Number	Description	Remarks
1	384706	TIRE PNEU 12-1/2" X 2-1/4"	12-1/2" X 2-1/4"
1	385237	16" X 1.75" PNEU TIRE	
1	385225	TIRE PNEUMATIC 18"	18" x 1 3/8
1	385250	20" PNEUMATIC TIRE	20" x 1 3/8
1	385275	22" PNEUMATIC TIRE	22" x 1 3/8 (501mm)
1	385300	24" PNEUMATIC TIRE	24" x 1 3/8 (540mm)
1	385325	26" PNEUMATIC TIRE	26" x 1 3/8 (590mm)
1	384614	24" x 2.125 PNEUMATIC TIRE	24"x2 1/8 (540mm) used w/ M6




Pos.	Item Number	Description	Remarks
1	386232	20" POLY TIRE STD	20"
1	386242	22" POLY TIRE STD	22" (501mm)
1	386252	24" POLY TIRE STD	24" (540mm)

**LOW POLY TIRE**

(01/2010)

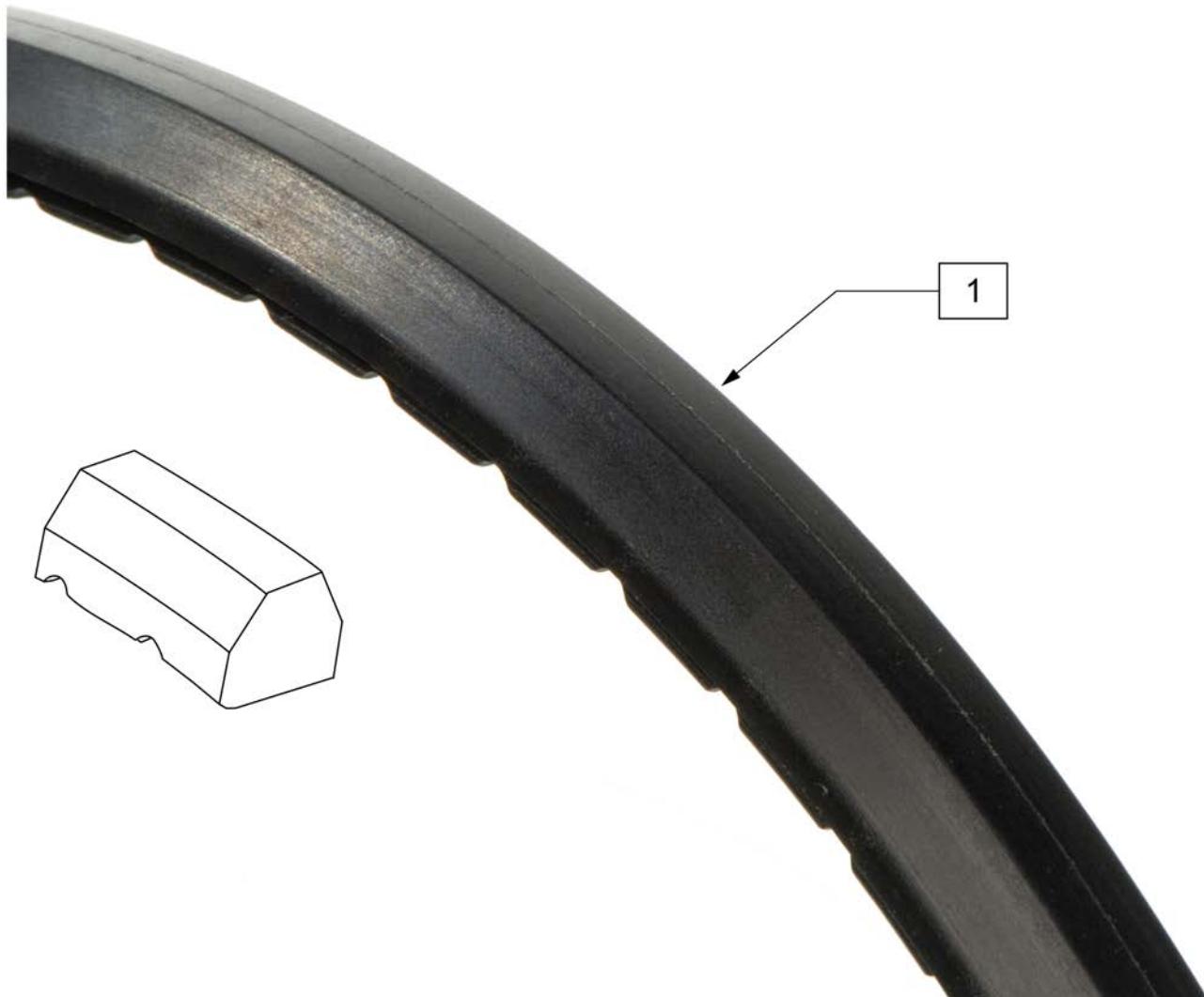


Pos.	Item Number	Description	Remarks
1	386262	20" LP POLY TIRE BR	20"
1	386263	22" LP POLY TIRE BR	22"(501mm)
1	386254	24" POLY LOW PROFILE	24"(540mm)

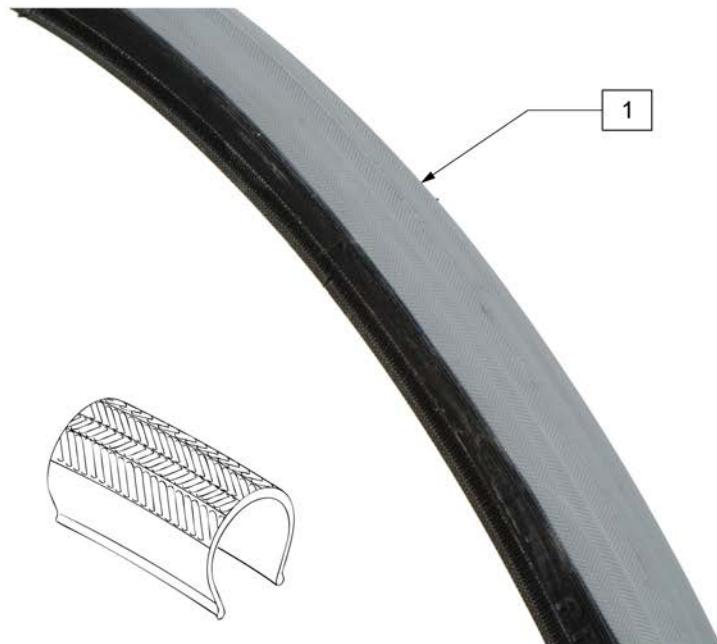

Pos.	Item Number	Description	Remarks
1	385280	22" x 1" HI PRES CLINCHER	22" (501mm)
1	389020	24" x 1" HI PRES CLNCHR TIRE	24" (540mm)
1	389012	QP 24" x 1" HT TURB BLK WR/BD 559	25" (559mm)
1	389030	26" x 1" HI PRES CLNCHR TIRE	26" (590mm)
1	105919	22" x 1" HI PRESS CLNCHR ULTRALIGHT	22" (489mm)




Pos.	Item Number	Description
1	385350	24" KEVLAR TIRE

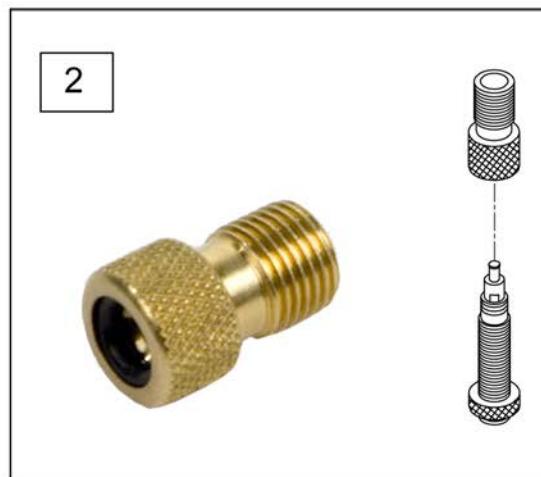

Pos.	Item Number	Description	Remarks
1A	382305	22" X 1" TUBE PRESTA VALVE	
1A	382330	24" TUBE PRESTA VALVE (540MM)	
1A	384545	QP 24" HT INNER TUBE (559MM)	
1A	382355	26" TUBE PRESTA VALVE (590MM)	
1A	384550	QP 26" HT INNER TUBE (700C)	
1B	384736	INNER TUBE 12-1/2" X 2-1/4"	
1B	384738	16" X 1.75" INNER TUBE	
1B	382255	TUBE 18" TIRE	18" X 1-3/4" TUBE
1B	382275	20" TUBE	20" 1-3/8" TUBE
1B	382300	22" TUBE	22" X 1-3/8" TUBE
1B	382336	TUBE PRIMO 24" 25-540 SV	24" X 1" TUBE
1B	382325	24 X 1.3/8IN TUBE REG	
1B	382334	TUBE INLINE 24 X 1.75	
1B	384551	26" X 1" TUBE SCHRADER	
1B	382350	26" TUBE	26" X 1-3/8" TUBE
1B	382361	TUBE 26 X 1.5/1.95/ S/V	
1B	382365	26 X 1.25/1.5 TUBE SHRADER	
1B	382375	24" x 2.125 TUBE	
1C	102972-001	12 1/2 IN AIRLESS INSERT	12-1/2" X 2-1/4"
1C	102973-001	16IN AIRLESS INSERT	16" X 1-3/4"
1C	102973-002	18IN AIRLESS INSERT	18" X 1-3/4"


<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
1C	102973-003	20IN AIRLESS INSERT	20" X 1-3/8"
1C	102972-002	AIRLESS INSERT 20 X 2-1/8 P3	
1C	102973-004	22IN AIRLESS INSERT	22" X 1-3/8"
1C	102973-005	24IN AIRLESS INSERT	24" X 1-3/8"
1C	102972-003	AIRLESS INSERT 24 X 2-1/8	
1C	108259	26" AIRLESS INSERT	26" X 1-3/8"
2	381230	VALVE ADAPTER PRESTA	

## GROWING SEAT PAN

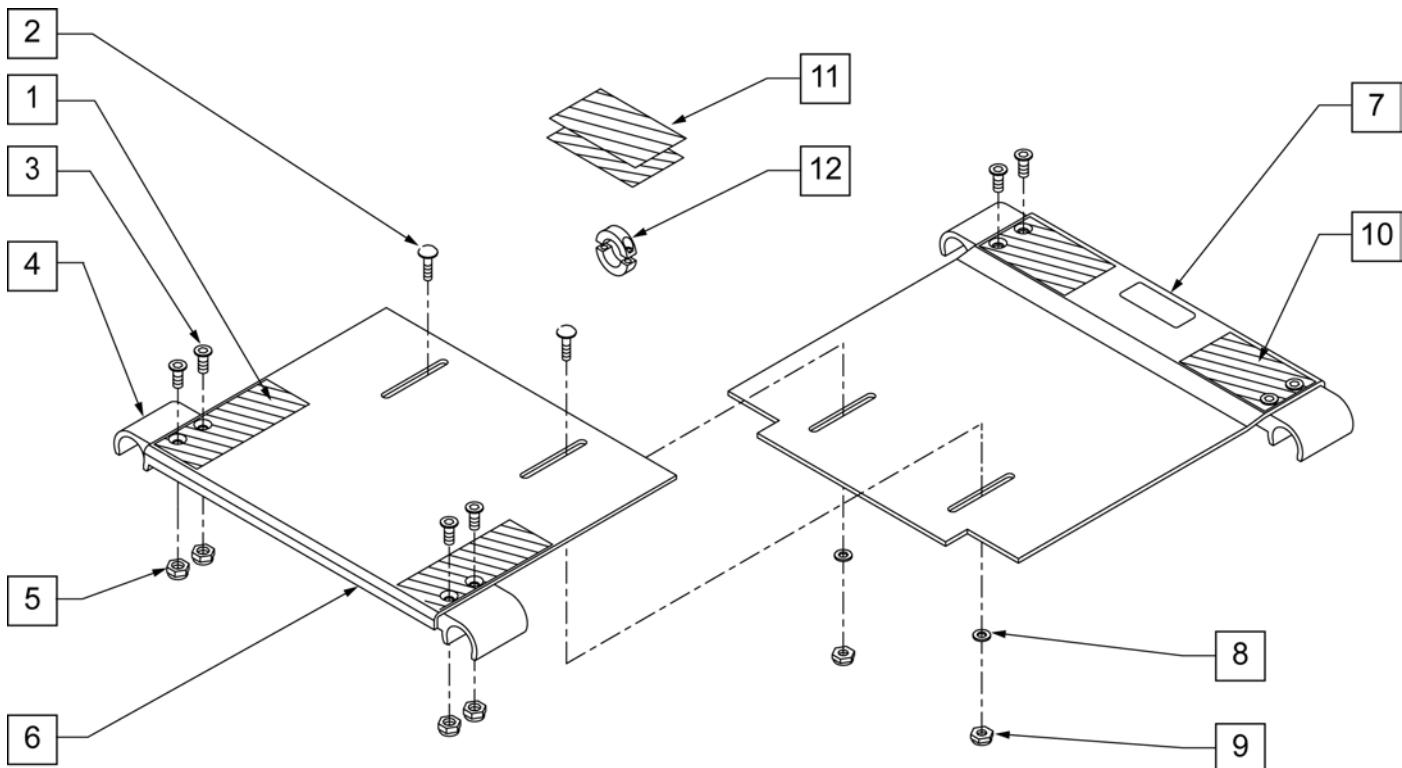

Pos.	Item Number	Description	Remarks
1	751047	HOOK FASTENER ZGS FRONT	
2	455036-QU	1/4-28 x 9/16 SPEC ROUND HD	
3	113278	FHCS 1/4 -28 x 3/4 ROHS	
4	560112	SEAT SNAP ZGS	
5	106934	NUT NYLOCK 1/4 -28 ROHS	
6	124340	SEAT PAN SMALL 10" x 10" ZGS	FITS ZGS ONLY FITS 10" -12" SEAT DEPTHS
6	124341	SEAT PAN SMALL 10" x 12" ZGS	FITS 10" -12" SEAT DEPTHS
6	124342	SEAT PAN SMALL 10" x 14" ZGS	FITS 10" -12" SEAT DEPTHS
6	124359	SEAT PAN SMALL 10 x 16" ZGS	FITS 10" -12" SEAT DEPTHS
6	124343	SEAT PAN MED 12" x 10" ZGS	FITS ZGS ONLY FITS 12" -16" SEAT DEPTHS
6	124344	SEAT PAN MED 12" x 12" ZGS	FITS 12" -16" SEAT DEPTHS
6	124345	SEAT PAN MED 12" x 14" ZGS	FITS 12" -16" SEAT DEPTHS
6	124360	SEAT PAN MED 12 x 16" ZGS	FITS 12" -16" SEAT DEPTHS
6	124346	SEAT PAN LARGE 14" x 10" ZGS	FITS ZGS ONLY FITS 14" -18" SEAT DEPTHS
6	124347	SEAT PAN LARGE 14" x 12" ZGS	FITS 14" -18" SEAT DEPTHS
6	124348	SEAT PAN LARGE 14" x 14" ZGS	FITS 14" -18" SEAT DEPTHS
6	124361	SEAT PAN LARGE 14 X 16" ZGS	FITS 14" -18" SEAT DEPTHS
6	124362	SEAT PAN X-LARGE 16 x 10" ZGS	FITS ZGS ONLY FITS 16" -20" DEPTHS
6	124363	SEAT PAN X-LARGE 16 x 12" ZGS	FITS 16" -20" SEAT DEPTHS
6	124364	SEAT PAN X-LARGE 16 x 14" ZGS	FITS 16" -20" SEAT DEPTHS
6	124365	SEAT PAN X-LARGE 16 x 16" ZGS	FITS 16" -20" SEAT DEPTHS
7	124350	SEAT PAN SMALL OFFSET 10" x 10" ZGS	FITS ZGS ONLY FITS 10" -12" SEAT DEPTHS
7	124351	SEAT PAN SMALL OFFSET 10" x 12" ZGS	FITS 10" -12" SEAT DEPTHS
7	124352	SEAT PAN SMALL OFFSET 10" x 14" ZGS	FITS 10" -12" SEAT DEPTHS
7	124370	SEAT PAN OFFSET SMALL 10 x 16" ZGS	FITS 10" -12" SEAT DEPTHS
7	124353	SEAT PAN MED OFFSET 12" x 10" ZGS	FITS ZGS ONLY FITS 14" -18" SEAT DEPTHS
7	124354	SEAT PAN MED OFFSET 12" x 12" ZGS	FITS 14" -18" SEAT DEPTHS
7	124355	SEAT PAN MED OFFSET 12" x 14" ZGS	FITS 14" -18" SEAT DEPTHS

Pos.	Item Number	Description	Remarks
7	124371	SEAT PAN OFFSET MED 12 x 16" ZGS	FITS 14" -18" SEAT DEPTHS
7	124356	SEAT PAN LARGE OFFSET 14" x 10" ZGS	FITS ZGS ONLY FITS 14" -18" SEAT DEPTHS
7	124357	SEAT PAN LARGE OFFSET 14" x 12" ZGS	FITS 14" -18" SEAT DEPTHS
7	124358	SEAT PAN LARGE OFFSET 14" x 14" ZGS	FITS 14" -18" SEAT DEPTHS
7	124372	SEAT PAN OFFSET LARGE 14 X 16" ZGS	FITS 14" -18" SEAT DEPTHS
7	124373	SEAT PAN OFFSET X-LARGE 16 x 10" ZGS	FITS ZGS ONLY FITS 16" -20" DEPTHS
7	124374	SEAT PAN OFFSET X-LARGE 16 x 12" ZGS	FITS 16" -20" SEAT DEPTHS
7	124375	SEAT PAN OFFSET X-LARGE 16 x 14" ZGS	FITS 16" -20" SEAT DEPTHS
7	124376	SEAT PAN OFFSET X-LARGE 16 x 16" ZGS	FITS 16" -20" SEAT DEPTHS
8	106943	WASHER FLAT AN 1/4 ROHS	
9	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
10	751048	HOOK FASTENER ZGS REAR	
11	751049	HOOK FASTENER ZGS FLOATING	
12	342048	ADJ SEAT STOP 1"	
1-12	703200	SEAT PAN ASSM SHORT 10-12"W ZGS	FITS ZGS ONLY FITS 10" -12" SEAT DEPTHS
1-12	703201	SEAT PAN ASSM SHORT 12-14"W ZGS	FITS 10" -12" SEAT DEPTHS
1-12	703202	SEAT PAN ASSM SHORT 14-16"W ZGS	FITS 10" -12" SEAT DEPTHS
1-12	703203	SEAT PAN ASSM SHORT 16-18"W ZGS	FITS 10" -12" SEAT DEPTHS
1-12	703210	SEAT PAN ASSM REG 10-12"W ZGS	FITS ZGS ONLY FITS 12" -16" SEAT DEPTHS
1-12	703211	SEAT PAN ASSM REG 12-14"W ZGS	FITS 12" -16" SEAT DEPTHS
1-12	703212	SEAT PAN ASSM REG 14-16"W ZGS	FITS 12" -16" SEAT DEPTHS
1-12	703213	SEAT PAN ASSM REG 16-18"W ZGS	FITS 12" -16" SEAT DEPTHS
1-12	703220	SEAT PAN ASSM LONG 10-12"W ZGS	FITS ZGS ONLY FITS 14" -18" SEAT DEPTHS
1-12	703221	SEAT PAN ASSM LONG 12-14"W ZGS	FITS 14" -18" SEAT DEPTHS
1-12	703222	SEAT PAN ASSM LONG 14-16"W ZGS	FITS 14" -18" SEAT DEPTHS
1-12	703223	SEAT PAN ASSM LONG 16-18"W ZGS	FITS 14" -18" SEAT DEPTHS
1-12	703230	SEAT PAN ASSM X-LONG 10-12"W ZGS	FITS ZGS ONLY FITS 16" -20" DEPTHS
1-12	703231	SEAT PAN ASSM X-LONG 12-14"W ZGS	FITS 16" -20" SEAT DEPTHS
1-12	703232	SEAT PAN ASSM X-LONG 14-16"W ZGS	FITS 16" -20" SEAT DEPTHS
1-12	703233	SEAT PAN ASSM X-LONG 16-18"W ZGS	FITS 16" -20" SEAT DEPTHS

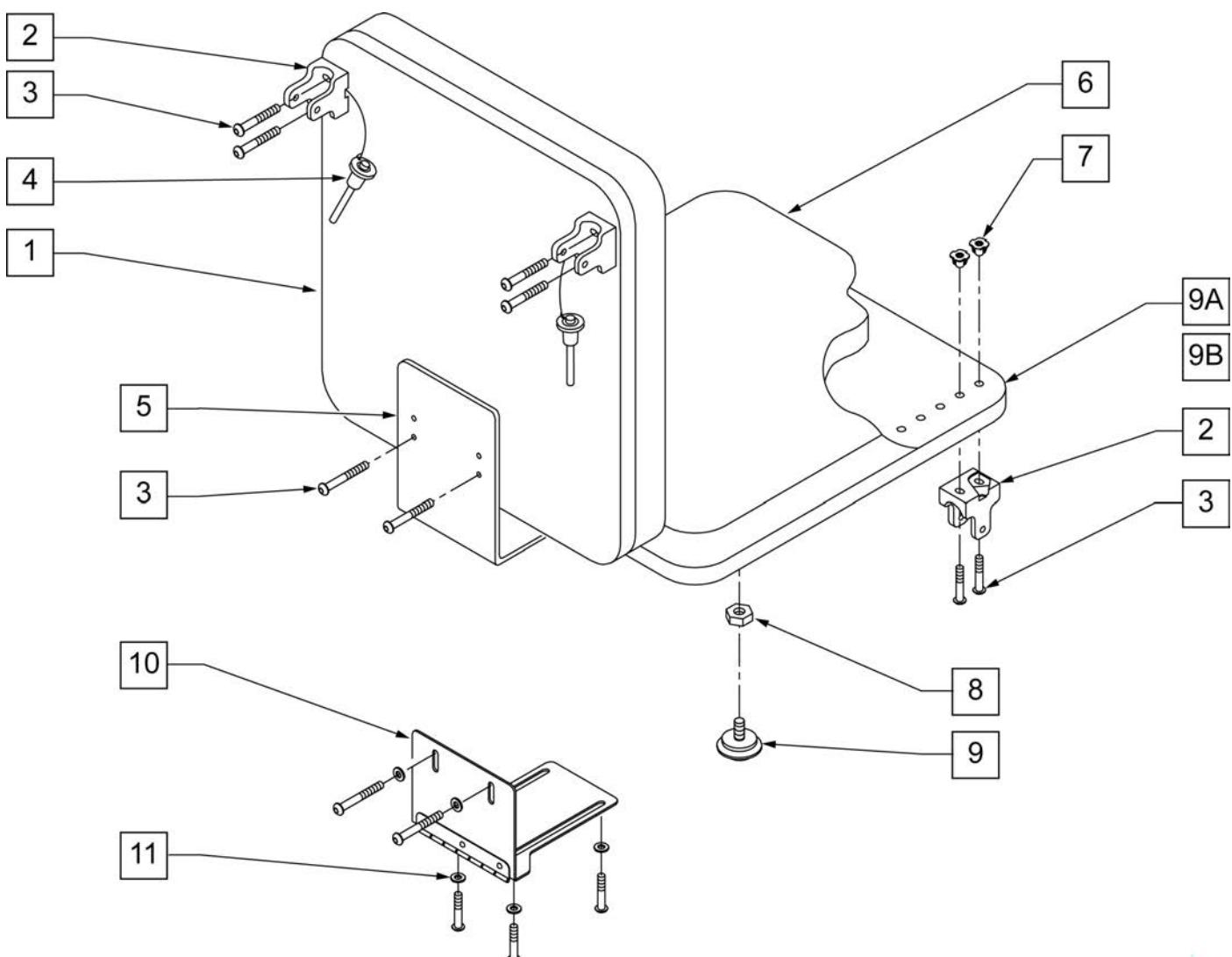**SOLID SEAT & SOLID BACK**

Pos.	Item Number	Description	Remarks
1,7	735050S	PLY BACK 10" LOW ZP's	
1,7	735051S	PLY BACK 10" MED ZP's	
1,7	735052S	PLY BACK 10" TALL ZP's	
1,7	735053S	PLY BACK 11" LOW ZP's	
1,7	735054S	PLY BACK 11" MED ZP's	
1,7	735055S	PLY BACK 11" TALL ZP's	
1,7	735056S	PLY BACK 12" LOW LOW ZPS	
1,7	735057S	PLY BACK 12" MED LOW ZPS	
1,7	735058S	PLY BACK 12" TALL LOW ZPS	
1,7	735059S	PLY BACK 13" LOW LOW ZPS	
1,7	735060S	PLY BACK 13" MED LOW ZPS	
1,7	735061S	PLY BACK 13" TALL LOW ZPS	
1,7	735062S	PLY BACK 14" LOW LOW ZPS	
1,7	735063S	PLY BACK 14" MED LOW ZPS	
1,7	735064S	PLY BACK 14" TALL LOW ZPS	
1,7	735065S	PLY BACK 15" LOW LOW ZPS	
1,7	735066S	PLY BACK 15" MED LOW ZPS	
1,7	735067S	PLY BACK 15" TALL LOW ZPS	

Pos.	Item Number	Description	Remarks
1,7	735068S	PLY BACK 16" LOW LOW ZPS	
1,7	735069S	PLY BACK 16" MED LOW ZPS	
1,7	735070S	PLY BACK 16" TALL LOW ZPS	
2	562609	BRACKET SEAT QTS/BACK Z2	
3	108107	BHCS 1/4-20 x 3/4 ROHS	
4	349020	STROLLER HANDLE RELEASE PIN	
5	505253	QT SEAT/BACK BRACKET NEW	
7	2400	1/4-20 X 5/16" T-NUTS 3 PRONG	
6,7	735000S	PLY SEAT 10 x 10/11 x 1/2 ZP	
6,7	735001S	PLY SEAT 10 x 11/12 x 1/2 ZP	
6,7	735002S	PLY SEAT 10 x 12/13 x 1/2 ZP	
6,7	735003S	PLY SEAT 10 x 13/14 x 1/2 ZP	
6,7	735004S	PLY SEAT 10 x 14/15 x 1/2 ZP	
6,7	735005S	PLY SEAT 10 x 15/16 x 1/2 ZP	
6,7	735006S	PLY SEAT 11 x 10/11 x 1/2 ZP	
6,7	735007S	PLY SEAT 11 x 11/12 x 1/2 ZP	
6,7	735008S	PLY SEAT 11 x 12/13 x 1/2 ZP	
6,7	735009S	PLY SEAT 11 x 13/14 x 1/2 ZP	
6,7	735010S	PLY SEAT 11 x 14/15 x 1/2 ZP	
6,7	735011S	PLY SEAT 11 x 15/16 x 1/2 ZP	
6,7	735012S	PLY SEAT 12 x 10/11 x 1/2 ZP	
6,7	735013S	PLY SEAT 12 x 11/12 x 1/2 ZP	
6,7	735014S	PLY SEAT 12 x 12/13 x 1/2 ZP	
6,7	735015S	PLY SEAT 12 x 13/14 x 1/2 ZP	
6,7	735016S	PLY SEAT 12 x 14/15 x 1/2 ZP	
6,7	735017S	PLY SEAT 12 x 15/16 x 1/2 ZP	
6,7	735018S	PLY SEAT 13 x 10/11 x 1/2 ZP	
6,7	735019S	PLY SEAT 13 x 11/12 x 1/2 ZP	
6,7	735020S	PLY SEAT 13 x 12/13 x 1/2 ZP	
6,7	735021S	PLY SEAT 13 x 13/14 x 1/2 ZP	
6,7	735022S	PLY SEAT 13 x 14/15 x 1/2 ZP	
6,7	735023S	PLY SEAT 13 x 15/16 x 1/2 ZP	
6,7	735024S	PLY SEAT 14 x 10/11 x 1/2 ZP	
6,7	735025S	PLY SEAT 14 x 11/12 x 1/2 ZP	
6,7	735026S	PLY SEAT 14 x 12/13 x 1/2 ZP	
6,7	735027S	PLY SEAT 14 x 13/14 x 1/2 ZP	
6,7	735028S	PLY SEAT 14 x 14/15 x 1/2 ZP	
6,7	735029S	PLY SEAT 14 x 15/16 x 1/2 ZP	
6,7	735030S	PLY SEAT 15 x 10/11 x 1/2 ZP	
6,7	735031S	PLY SEAT 15 x 11/12 x 1/2 ZP	
6,7	735032S	PLY SEAT 15 x 12/13 x 1/2 ZP	
6,7	735033S	PLY SEAT 15 x 13/14 x 1/2 ZP	
6,7	735034S	PLY SEAT 15 x 14/15 x 1/2 ZP	
6,7	735035S	PLY SEAT 15 x 15/16 x 1/2 ZP	
6,7	735036S	PLY SEAT 16 x 10/11 x 1/2 ZP	
6,7	735037S	PLY SEAT 16 x 11/12 x 1/2 ZP	
6,7	735038S	PLY SEAT 16 x 12/13 x 1/2 ZP	
6,7	735039S	PLY SEAT 16 x 13/14 x 1/2 ZP	
6,7	735040S	PLY SEAT 16 x 14/15 x 1/2 ZP	
6,7	735041S	PLY SEAT 16 x 15/16 x 1/2 ZP	
8	113281	NUT HEX THIN 1/4 -20 ROHS	
9	349560	ZP RUBBER STOP	
10	450990	SEAT TO BACK BRKT 90D CV	HINGED BRACKET
11	106943	WASHER FLAT AN 1/4 ROHS	
2-4,8,9	920451	ST/BCK HDWR KT STD BRKT ZP	
2-4,8,9,11	920452	ST/BCK HDWR KIT HNG BRKT ZP	




**TILT COMPONENTS DUAL TRIGGER CABLE W/ STROLLER HANDLE BACKPOST**

(03/2010)

**NOTE:** Cables can be routed in a CROSSED Configuration (Left trigger controls right mech lock) and a NON-CROSSED configuration (Left trigger controls left mech lock).

See Attached TRIGGER CABLE ORDERING GUIDE for identification & use.

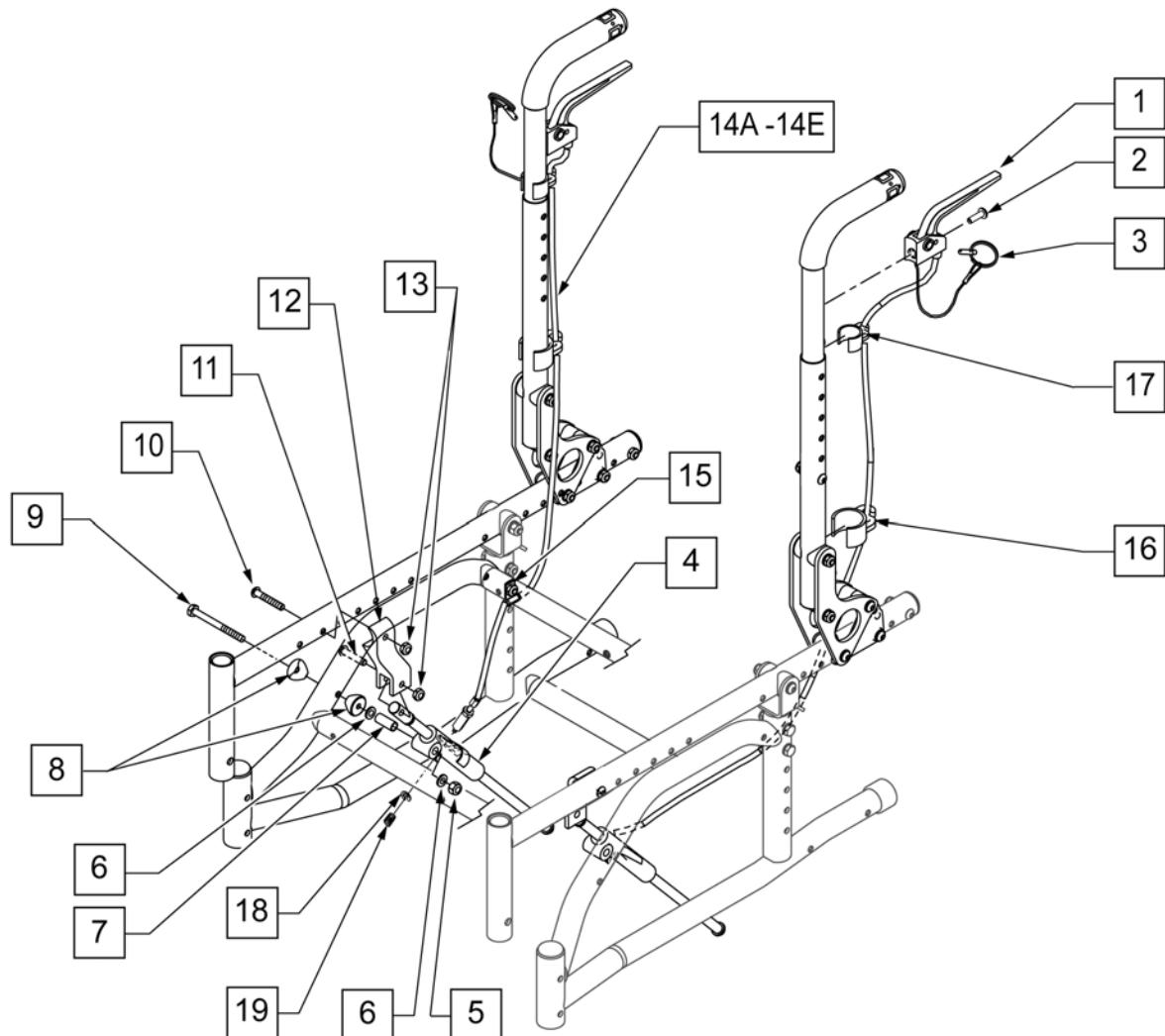


Pos.	Item Number	Description	Remarks
1,3	580110	PLUNGER PIN LEVER ASSM	
2	281120	10 -32 X 5/8 PAN HD PHIL	
3	349021	TRIGGER RELEASE PIN TS/QT/P5	
4	120625	MECH CYLINDER LOCK R BR RCLINR	RIGHT
4	120626	MECH CYLINDER LOCK L BR RCLINR	LEFT
5	106934	NUT NYLOCK 1/4 -28 ROHS	
6	106943	WASHER FLAT AN 1/4 ROHS	
7	520916	FOLDING ZTS MECH LOCK PVT SPACER FRM	
8	561004	1" SADDLE 1/4" HOLE	
9	107264	HHCS 5/16 -24 x 2 3/4 PL ROHS	
10	107028	BHCS 1/4 -28 x 1 1/2 ROHS	

Pos.	Item Number	Description	Remarks
11	107027	BHCS 1/4 -28 x 1 1/4 SHLDR ROHS	
12	500609	FOLDING ZTS MECH LOCK MNT SEAT FRM	
13	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
14A	112700-001	CABLE TILT ZTS SHORT	950 mm/ 37.40 Inches long. See TRIGGER CABLE ORDERING GUIDE.
14B	112700-002	CABLE TILT ZTS MEDIUM	1025 mm/ 40.35 Inches long. See TRIGGER CABLE ORDERING GUIDE.
14C	112700-003	CABLE TILT ZTS LONG	1100 mm/ 43.31 Inches long. See TRIGGER CABLE ORDERING GUIDE.
14D	112700-004	CABLE TILT ZTS X-LONG	1175 mm/ 46.26 Inches long. See TRIGGER CABLE ORDERING GUIDE.
14E	112700-005	CABLE TILT ZTS 2X-LONG	1250mm/ 49.21 inches long. See TRIGGER CABLE ORDERING GUIDE.
14F	112700-006	CABLE TILT ZTS 3X-LONG	1325mm/ 52.17 inches long. See TRIGGER CABLE ORDERING GUIDE.
15	112388	P-CLIP 3/16" VINYL COAT	
16	108134	CLIP CABLE 1" PACK (10)	PACK OF 10
17	108135	CLIP CABLE 7/8" PACK (10)	PACK OF 10
18	342000	1/4" E-CLIP	
19	321141	.650L X 6 COIL SPRING	
4,18	922055	ZTS MECH LOCK REPAIR KIT	PAIR

**TILT COMPONENTS DUAL TRIGGER CABLE W/ ANGLE ADJ PUSH HANDLE BACKPOST** (03/2010)

**NOTE:** Cables can be routed in a CROSSED Configuration (Left trigger controls right mech lock) and a NON-CROSSED configuration (Left trigger controls left mech lock).

See Attached TRIGGER CABLE ORDERING GUIDE for identification & use.

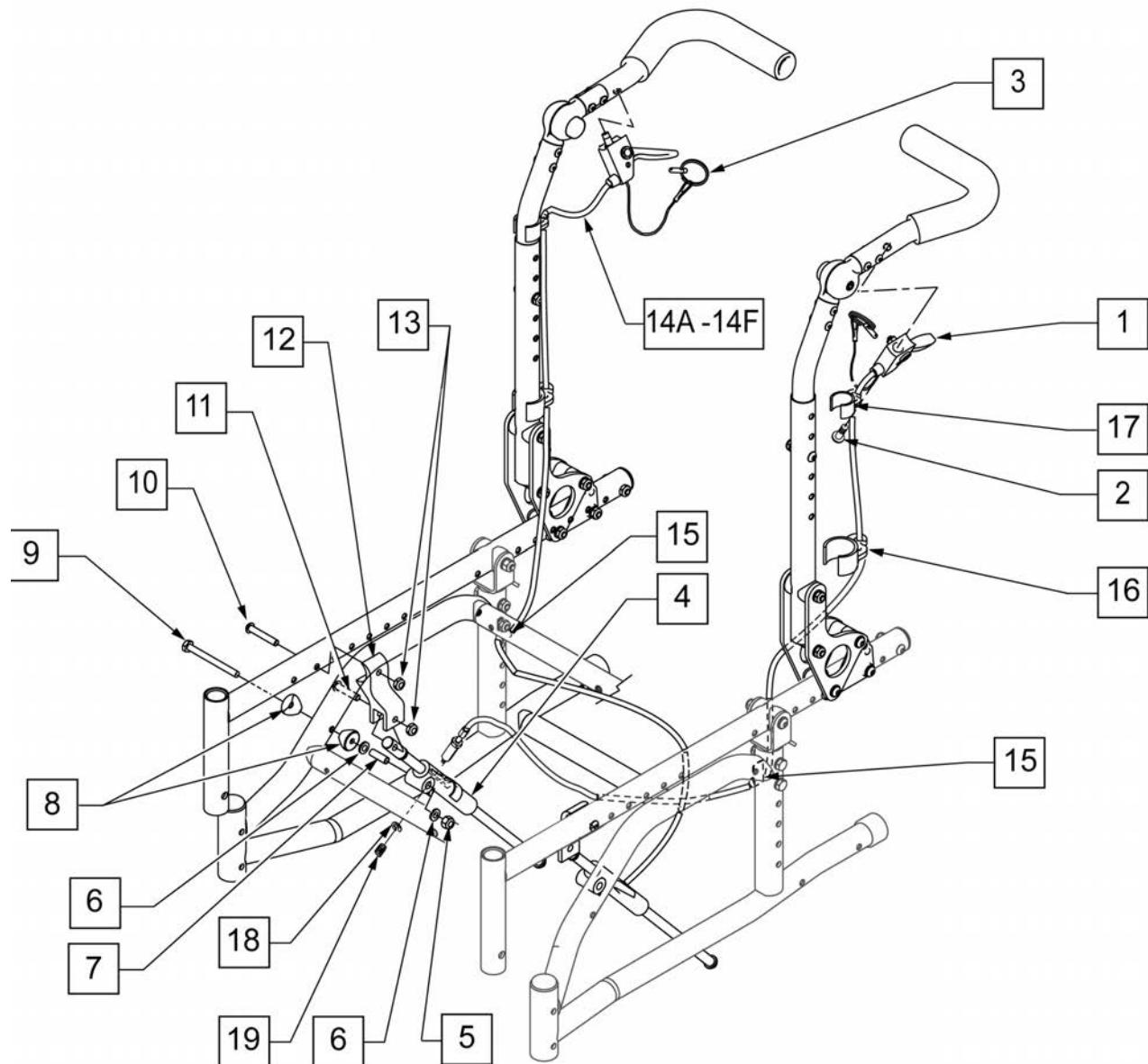


Pos.	Item Number	Description	Remarks
1,3	580109	PLUNGER PIN LEVER ASSM	
2	341497	10 -32 X 5/8 PAN HD PHIL	
3	349021	TRIGGER RELEASE PIN TS/QT/P5	
4	120625	MECH CYLINDER LOCK R BR RCLINR	RIGHT
4	120626	MECH CYLINDER LOCK L BR RCLINR	LEFT
5	106934	NUT NYLOCK 1/4 -28 ROHS	
6	106943	WASHER FLAT AN 1/4 ROHS	
7	520916	FOLDING ZTS MECH LOCK PVT SPACER FRM	

Pos.	Item Number	Description	Remarks
8	561004	1" SADDLE 1/4" HOLE	
9	107264	HHCS 5/16 -24 x 2 3/4 PL ROHS	
10	107028	BHCS 1/4 -28 x 1 1/2 ROHS	
11	107027	BHCS 1/4 -28 x 1 1/4 SHLDR ROHS	
12	500609	FOLDING ZTS MECH LOCK MNT SEAT FRM	
13	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
14A	112700-001	CABLE TILT ZTS SHORT	950 mm/ 37.40 Inches long. See TRIGGER CABLE ORDERING GUIDE.
14B	112700-002	CABLE TILT ZTS MEDIUM	1025 mm/ 40.35 Inches long. See TRIGGER CABLE ORDERING GUIDE.
14C	112700-003	CABLE TILT ZTS LONG	1100 mm/ 43.31 Inches long. See TRIGGER CABLE ORDERING GUIDE.
14D	112700-004	CABLE TILT ZTS X-LONG	1175 mm/ 46.26 Inches long. See TRIGGER CABLE ORDERING GUIDE.
14E	112700-005	CABLE TILT ZTS 2X-LONG	1250mm/ 49.21 inches long. See TRIGGER CABLE ORDERING GUIDE.
14F	112700-006	CABLE TILT ZTS 3X-LONG	1325mm/ 52.17 inches long. See TRIGGER CABLE ORDERING GUIDE.
15	112388	P-CLIP 3/16" VINYL COAT	
16	108134	CLIP CABLE 1" PACK (10)	PACK OF 10
17	108135	CLIP CABLE 7/8" PACK (10)	PACK OF 10
18	342000	1/4" E-CLIP	
19	321141	.650L X 6 COIL SPRING	
4,18	922055	ZTS MECH LOCK REPAIR KIT	PAIR

**SOLID SEAT & SOLID BACK**


[11/2009]

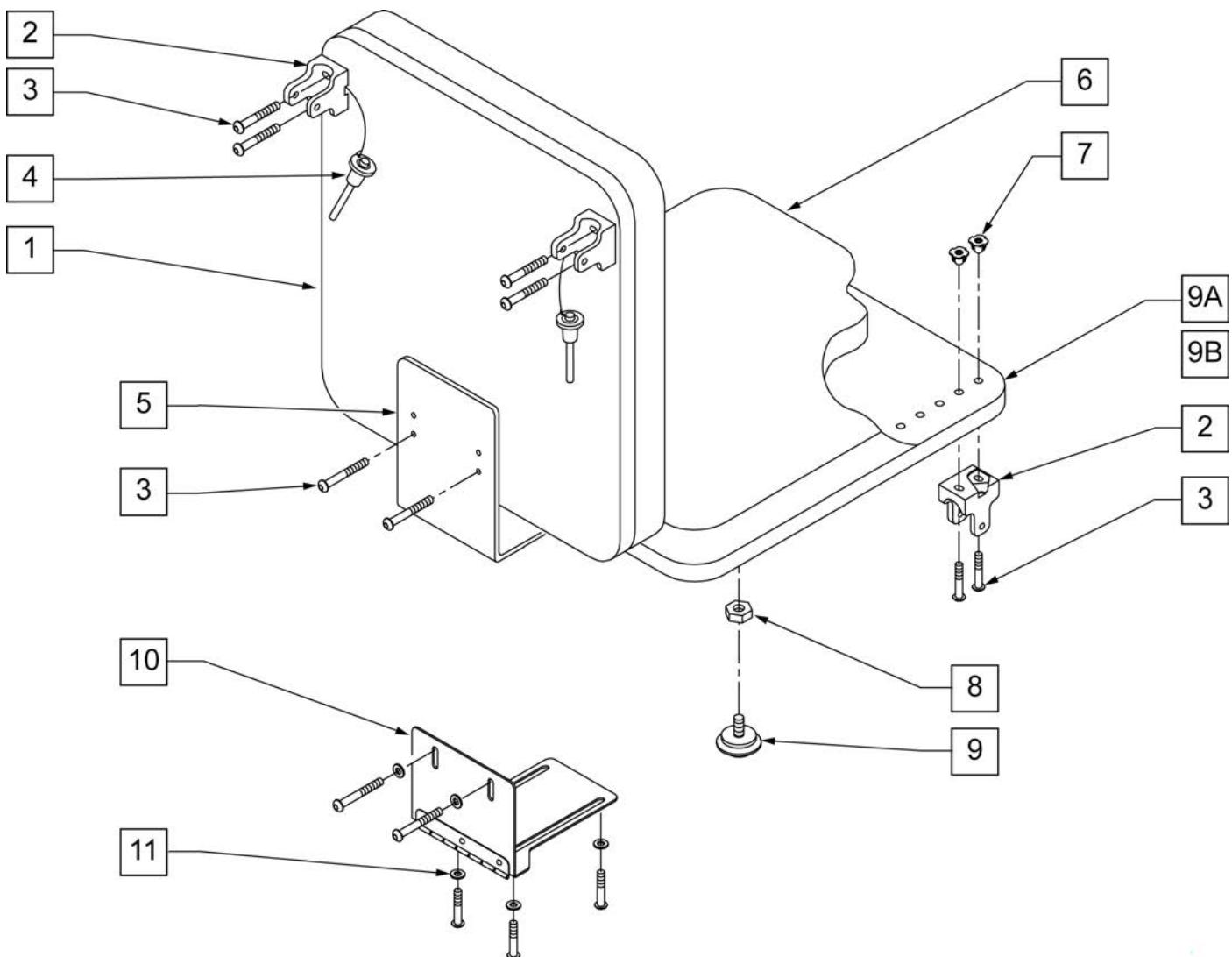


Pos.	Item Number	Description	Remarks
1,7	735050S	PLY BACK 10" LOW ZP's	
1,7	735051S	PLY BACK 10" MED ZP's	
1,7	735052S	PLY BACK 10" TALL ZP's	
1,7	735053S	PLY BACK 11" LOW ZP's	
1,7	735054S	PLY BACK 11" MED ZP's	
1,7	735055S	PLY BACK 11" TALL ZP's	
1,7	735056S	PLY BACK 12" LOW LOW ZPS	
1,7	735057S	PLY BACK 12" MED LOW ZPS	
1,7	735058S	PLY BACK 12" TALL LOW ZPS	
1,7	735059S	PLY BACK 13" LOW LOW ZPS	
1,7	735060S	PLY BACK 13" MED LOW ZPS	
1,7	735061S	PLY BACK 13" TALL LOW ZPS	
1,7	735062S	PLY BACK 14" LOW LOW ZPS	
1,7	735063S	PLY BACK 14" MED LOW ZPS	
1,7	735064S	PLY BACK 14" TALL LOW ZPS	
1,7	735065S	PLY BACK 15" LOW LOW ZPS	
1,7	735066S	PLY BACK 15" MED LOW ZPS	
1,7	735067S	PLY BACK 15" TALL LOW ZPS	

Pos.	Item Number	Description	Remarks
1,7	735068S	PLY BACK 16" LOW LOW ZPS	
1,7	735069S	PLY BACK 16" MED LOW ZPS	
1,7	735070S	PLY BACK 16" TALL LOW ZPS	
2	562609	BRACKET SEAT QTS/BACK Z2	
3	108107	BHCS 1/4-20 x 3/4 ROHS	
4	349020	STROLLER HANDLE RELEASE PIN	
5	505253	QT SEAT/BACK BRACKET NEW	
7	2400	1/4-20 X 5/16" T-NUTS 3 PRONG	
6,7	735000S	PLY SEAT 10 x 10/11 x 1/2 ZP	
6,7	735001S	PLY SEAT 10 x 11/12 x 1/2 ZP	
6,7	735002S	PLY SEAT 10 x 12/13 x 1/2 ZP	
6,7	735003S	PLY SEAT 10 x 13/14 x 1/2 ZP	
6,7	735004S	PLY SEAT 10 x 14/15 x 1/2 ZP	
6,7	735005S	PLY SEAT 10 x 15/16 x 1/2 ZP	
6,7	735006S	PLY SEAT 11 x 10/11 x 1/2 ZP	
6,7	735007S	PLY SEAT 11 x 11/12 x 1/2 ZP	
6,7	735008S	PLY SEAT 11 x 12/13 x 1/2 ZP	
6,7	735009S	PLY SEAT 11 x 13/14 x 1/2 ZP	
6,7	735010S	PLY SEAT 11 x 14/15 x 1/2 ZP	
6,7	735011S	PLY SEAT 11 x 15/16 x 1/2 ZP	
6,7	735012S	PLY SEAT 12 x 10/11 x 1/2 ZP	
6,7	735013S	PLY SEAT 12 x 11/12 x 1/2 ZP	
6,7	735014S	PLY SEAT 12 x 12/13 x 1/2 ZP	
6,7	735015S	PLY SEAT 12 x 13/14 x 1/2 ZP	
6,7	735016S	PLY SEAT 12 x 14/15 x 1/2 ZP	
6,7	735017S	PLY SEAT 12 x 15/16 x 1/2 ZP	
6,7	735018S	PLY SEAT 13 x 10/11 x 1/2 ZP	
6,7	735019S	PLY SEAT 13 x 11/12 x 1/2 ZP	
6,7	735020S	PLY SEAT 13 x 12/13 x 1/2 ZP	
6,7	735021S	PLY SEAT 13 x 13/14 x 1/2 ZP	
6,7	735022S	PLY SEAT 13 x 14/15 x 1/2 ZP	
6,7	735023S	PLY SEAT 13 x 15/16 x 1/2 ZP	
6,7	735024S	PLY SEAT 14 x 10/11 x 1/2 ZP	
6,7	735025S	PLY SEAT 14 x 11/12 x 1/2 ZP	
6,7	735026S	PLY SEAT 14 x 12/13 x 1/2 ZP	
6,7	735027S	PLY SEAT 14 x 13/14 x 1/2 ZP	
6,7	735028S	PLY SEAT 14 x 14/15 x 1/2 ZP	
6,7	735029S	PLY SEAT 14 x 15/16 x 1/2 ZP	
6,7	735030S	PLY SEAT 15 x 10/11 x 1/2 ZP	
6,7	735031S	PLY SEAT 15 x 11/12 x 1/2 ZP	
6,7	735032S	PLY SEAT 15 x 12/13 x 1/2 ZP	
6,7	735033S	PLY SEAT 15 x 13/14 x 1/2 ZP	
6,7	735034S	PLY SEAT 15 x 14/15 x 1/2 ZP	
6,7	735035S	PLY SEAT 15 x 15/16 x 1/2 ZP	
6,7	735036S	PLY SEAT 16 x 10/11 x 1/2 ZP	
6,7	735037S	PLY SEAT 16 x 11/12 x 1/2 ZP	
6,7	735038S	PLY SEAT 16 x 12/13 x 1/2 ZP	
6,7	735039S	PLY SEAT 16 x 13/14 x 1/2 ZP	
6,7	735040S	PLY SEAT 16 x 14/15 x 1/2 ZP	
6,7	735041S	PLY SEAT 16 x 15/16 x 1/2 ZP	
8	113281	NUT HEX THIN 1/4 -20 ROHS	
9	349560	ZP RUBBER STOP	
10	450990	SEAT TO BACK BRKT 90D CV	HINGED BRACKET
11	106943	WASHER FLAT AN 1/4 ROHS	
2-4,8,9	920451	ST/BCK HDWR KT STD BRKT ZP	
2-4,8,9,11	920452	ST/BCK HDWR KIT HNG BRKT ZP	

Part Changes, please see notes for important information

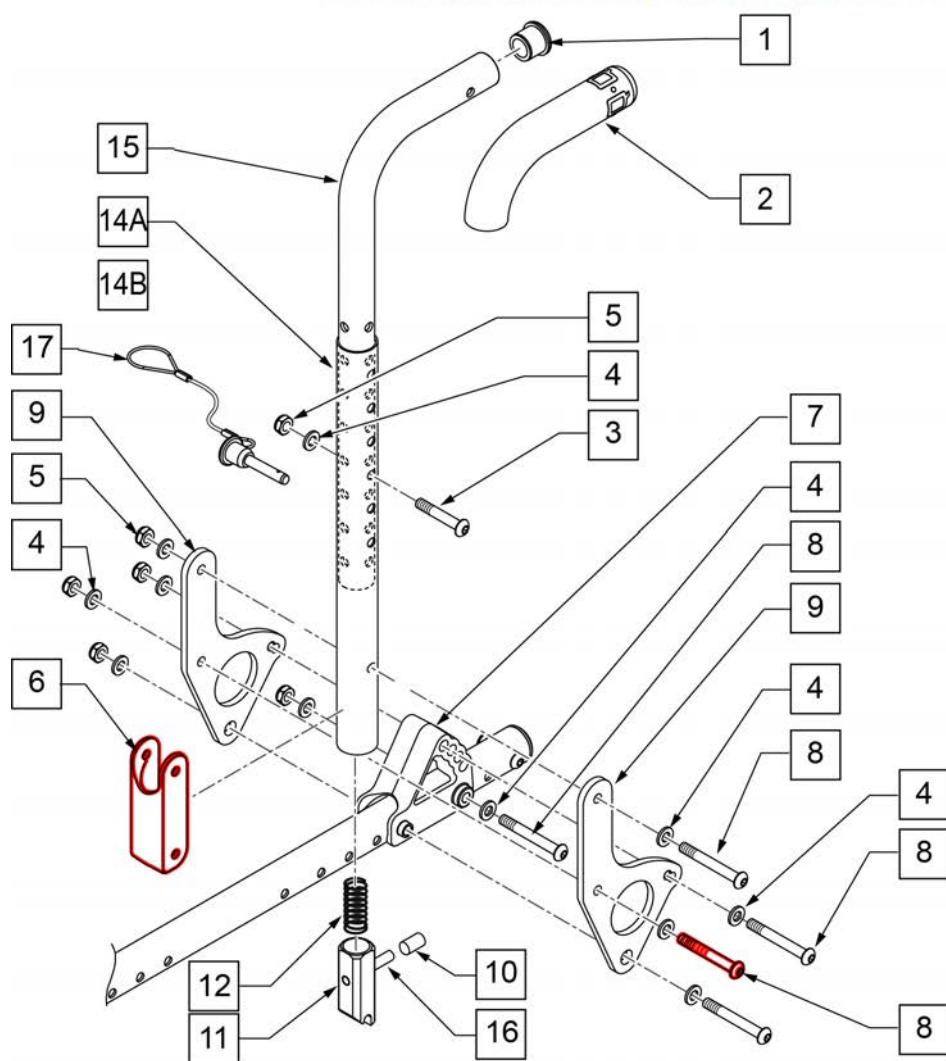


#### MEASURING GUIDE

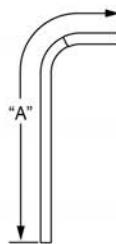

BACK TUBE	"A"
STD	17.4"

NOTE: ABOVE DIMENSIONS ARE FOR BACK TUBE ONLY AND DO NOT INCLUDE FOAM GRIP.

Discontinued Parts  
Replace w/ item 13 in pairs

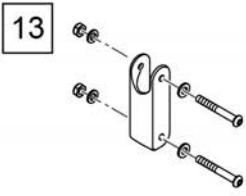


Pos.	Item Number	Description	Remarks
1	561007	7/8" INSERT PLUG .083	
2	920024	QUICKIE GRIP 7/8" W/INSTR SH.	
3	107027	BHCS 1/4 -28 x 1 1/4 SHLDR ROHS	
4	106943	WASHER FLAT AN 1/4 ROHS	
5	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
6	124612	BLOCK-LATCH STOP	
7	560517	BLOCK ANGLE BACKREST MOUNT	
8	106965	BHCS 1/4 -28 x 2 SHLDR ROHS	
9	109739	PLATE BACKREST	
10	126238	PUSH ON ROUND CAP	
11	560173	S/A LATCH .885 OD	
12	321110	S/A LATCH SPRING	
13	126099	BACK REST SADDLE W/ HDWR PAIR	Pair-Replacement Latch Stop Kit
13	126100	BACK REST SADDLE W/ HDWR EACH	Each-Replacement Latch Stop Kit
14A	520757	TUBE BACK FRAME LOWER	STD (16"-22" BACK HEIGHTS)
14B	107171	TUBE BACK FRAME LOWER -TALL	TALL (19"-25" BACK HEIGHTS)
1,2,15	111559	TUBE UPPER HEIGHT ADJ STROLLER	

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
16	481084	BACK REL PIN Z2/QT/EX	
17	111565	QR PIN 1/4" W/ LANYARD	
4-6,8-14A,16	110966-001	BACKREST LOWER HEIGHT ADJ ZP R STD	RIGHT - STD (16"-22" BACK HEIGHTS)
4-6,8-14B,16	110967-001	BACKREST LOWER HEIGHT ADJ ZP L STD	LEFT - STD (16"-22" BACK HEIGHTS)
4-6,8-14A,16	110966-002	BACKREST LOWER HEIGHT ADJ ZP R TALL	RIGHT - TALL (19"-25" BACK HEIGHTS)
4-6,8-14B,16	110967-002	BACKREST LOWER HEIGHT ADJ ZP L TALL	LEFT - TALL (19"-25" BACK HEIGHTS)
10-12,14A,16	681111	ADJ HEIGHT PUSH HANDLE LOWER TUBE W/ LATCH	STD (16"-22" BACK HEIGHTS)
10 -12, 14B,16	107170	ADJ HEIGHT PUSH HANDLE LOWER TUBE TL W/ LATCH	TALL (19"-25" BACK HEIGHTS)

**NOTE:** QUICK RELEASE PIN (ITEM 17) is used in place of ITEMS 3 -5 for SWING AWAY STROLLER HANDLES.

FIXED HEIGHT STROLLER HANDLE BACKPOST


[03/2010]

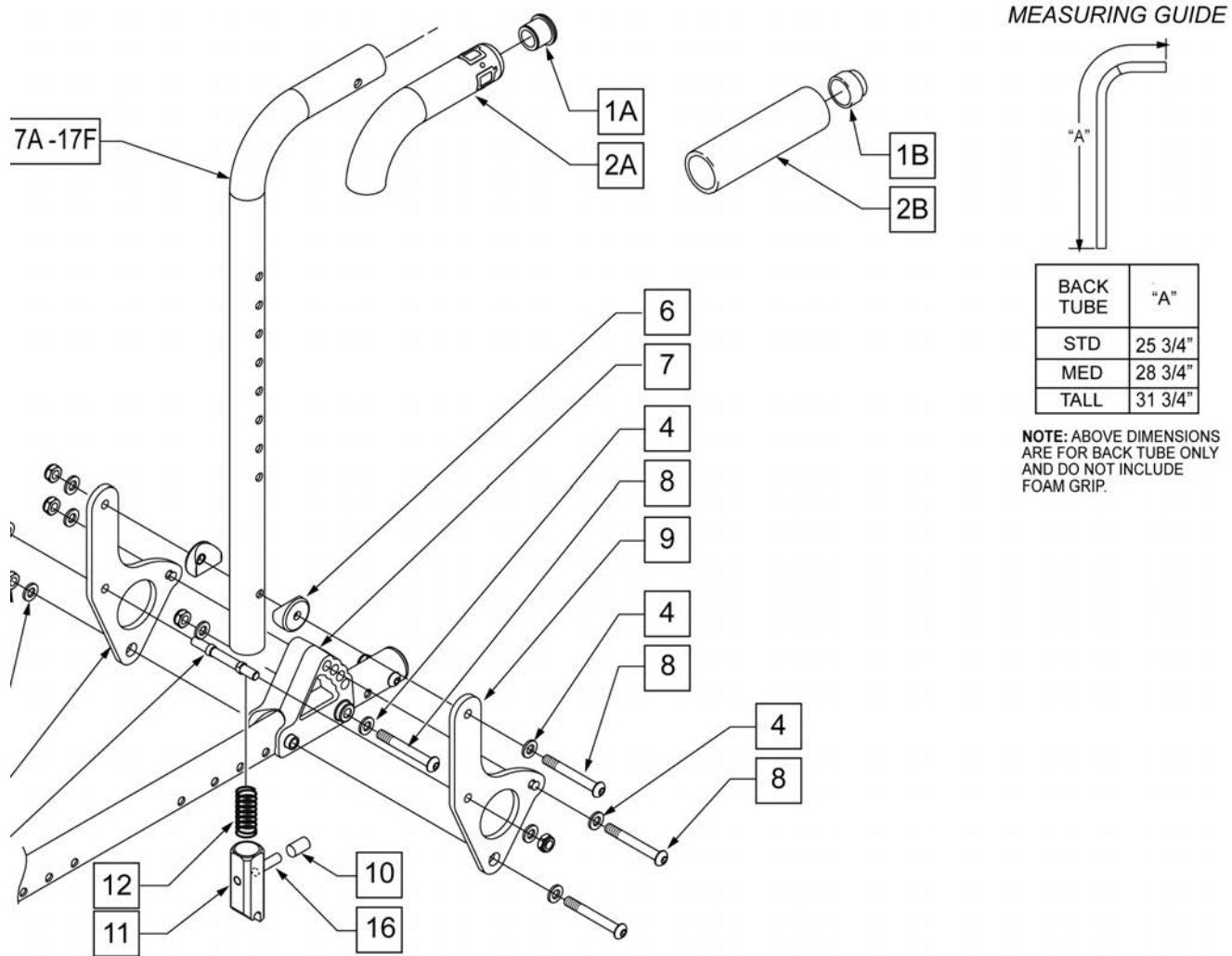

Pos.	Item Number	Description	Remarks
1A	561002	1" INSERT PLUG .083	
1B	561082	1" TUBE END BUSHING THIN	USED W/ REMOVABLE STROLLER HANDLE
2A	920027	QUICKIE GRIP 1" W/INSTR SHEET	
2B	920023	QUICKIE GRIP 1" W/HOLE W/INSTR	USED W/ REMOVABLE STROLLER HANDLE
4	106943	WASHER FLAT AN 1/4 ROHS	
5	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
6	561004	1" SADDLE 1/4" HOLE	
7	560517	BLOCK ANGLE BACKREST MOUNT	
8	106965	BHCS 1/4 -28 x 2 SHLDR ROHS	
9	109739	PLATE BACKREST	
10	126238	PUSH ON ROUND CAP	
11	483019	BACK RELEASE LATCH	
12	321110	S/A LATCH SPRING	
13	481042	BACK REL STOP PIN QTS	
16	481084	BACK REL PIN Z2/QT/EX	
1A,2A,4-6,8-17A	111401-001	BACKREST FIXED HT ZP R STD	RIGHT LOW- Includes Plates & Latch

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
1A,2A,4-6,8-17A	111402-001	BACKREST FIXED HT ZP L STD	LEFT LOW- Includes Plates & Latch
1A,2A,4-6,8-17B	111401-002	BACKREST FIXED HT ZP R MED	RIGHT MEDIUM- Includes Plates & Latch
1A,2A,4-6,8-17B	111402-002	BACKREST FIXED HT ZP L MED	LEFT MEDIUM- Includes Plates & Latch
1A,2A,4-6,8-17C	111401-003	BACKREST FIXED HT ZP R TA	RIGHT TALL- Includes Plates & Latch
1A,2A,4-6,8-17C	111402-003	BACKREST FIXED HT ZP L TALL	LEFT TALL- Includes Plates & Latch
1B,2B,4-6,8-17D	112365-001	BACKREST FXD HT STRLR ZP R STD	RIGHT LOW- Includes Plates & Latch - Used w/ removable stroller handle
1B,2B,4-6,8-17D	112366-001	BACKREST FXD HT STRLR ZP L STD	LEFT LOW- Includes Plates & Latch - Used w/ removable stroller handle
1B,2B,4-6,8-17E	112365-002	BACKREST FXD HT STRLR ZP R MED	RIGHT MEDIUM- Includes Plates & Latch - Used w/ removable stroller handle
1B,2B,4-6,8-17E	112366-002	BACKREST FXD HT STRLR ZP L MED	LEFT MEDIUM- Includes Plates & Latch - Used w/ removable stroller handle
1B,2B,4-6,8-17F	112365-003	BACKREST FXD HT STRLR ZP R LG	RIGHT TALL- Includes Plates & Latch - Used w/ removable stroller handle
1B,2B,4-6,8-17F	112366-003	BACKREST FXD HT STRLR ZP L TLL	LEFT TALL- Includes Plates & Latch - Used w/ removable stroller handle
1A,2A,17A	842080	STROLLER HANDLE W/ FOAM STD ZPS/P5/QT	LOW
1A,2A,17B	842079	STROLLER HANDLE W/ FOAM MED ZP'S/QT	MED
1A,2A,17C	842081	STROLLER HANDLE W/ FOAM TALL ZP'S/QT	TALL
1B,2B,17D	842067	EXT STROLLER HANDLE STD	LOW -Used w/ Removable Stroller Handle
1B,2B,17E	842066	EXT STROLLER HANDLE MED	MED -Used w/ Removable Stroller Handle
1B,2B,17F	842068	EXT STROLLER HANDLE TALL	TALL -Used w/ Removable Stroller Handle

Part Changes, please see notes for important information

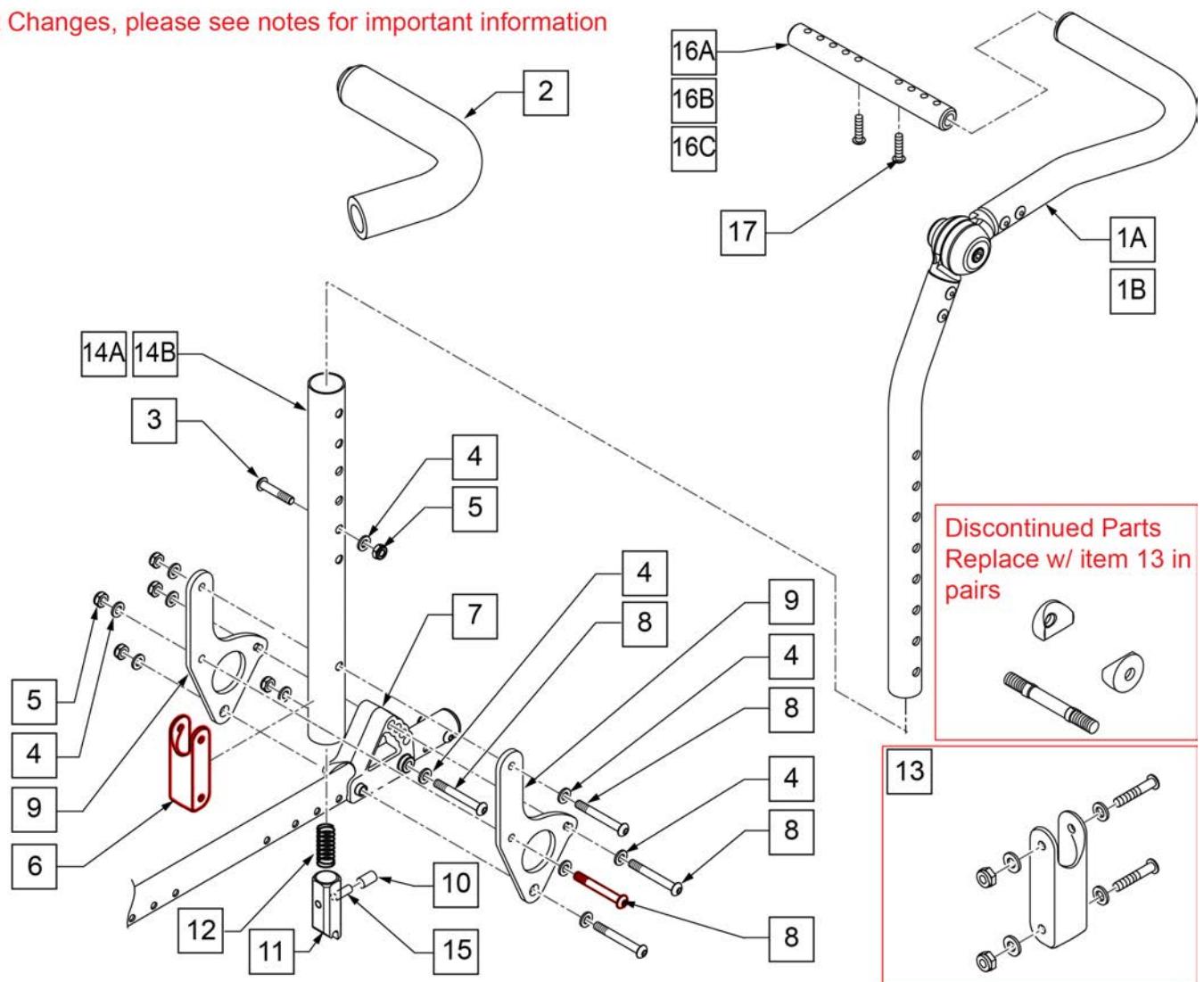


Pos.	Item Number	Description	Remarks
1A,2	110962-001	BACKREST UPPER HEIGHT ADJ PED R STD	RIGHT - STANDARD - INCLUDES FOAM
1A,2	110963-001	BACKREST UPPER HEIGHT ADJ PED L STD	LEFT - STANDARD - INCLUDES FOAM
1B,2	110962-002	BACKREST UPPER HEIGHT ADJ PED R LONG	RIGHT - LONG EXTENDED - INCLUDES FOAM
1B,2	110963-002	BACKREST UPPER HEIGHT ADJ PED L LONG	LEFT - LONG EXTENDED - INCLUDES FOAM
2	560638	FOAM GRIP .840 ID SHORT	
3	107027	BHCS 1/4 -28 x 1 1/4 SHLDR ROHS	
4	106943	WASHER FLAT AN 1/4 ROHS	
5	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
6	124612	BLOCK-LATCH STOP	
7	560517	BLOCK ANGLE BACKREST MOUNT	
8	106965	BHCS 1/4 -28 x 2 SHLDR ROHS	
9	109739	PLATE BACKREST	
10	126238	PUSH ON ROUND CAP	
11	560173	S/A LATCH .885 OD	
12	321110	S/A LATCH SPRING	
13	126099	BACK REST SADDLE W/ HDWR PAIR	Pair-Replacement Latch Stop Kit
13	126100	BACK REST SADDLE W/ HDWR EACH	Each-Replacement Latch Stop Kit

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
14A	520757	TUBE BACK FRAME LOWER	USED ON STD (17 -23" BACK HEIGHTS)
14B	107171	TUBE BACK FRAME LOWER TALL	USED ON TALL (20 -26" BACK HEIGHTS)
15	481084	BACK REL PIN Z2/QT/EX	
16A	050541-001	SHAFT GROWTH PED SHT	USED ON 10" - 14" WIDE CHAIRS
16B	050541-002	SHAFT GROWTH PED MED	USED ON 12" - 16" WIDE CHAIRS
16C	521392	SHAFT GROWTH HANDLE LONG	USED ON 14" - 18" WIDE CHAIRS
17	106960	BHCS 1/4 -28 x 7/8 ROHS	
4-6,8-14A,15	110966-001	BKRST LOWER HT ADJ ZP R STD	RIGHT - STANDARD HEIGHT (17 -23" BACK HEIGHTS)
4-6,8-14A,15	110967-001	BKRST LOWER HT ADJ ZP L STD	LEFT - STANDARD HEIGHT (17 -23" BACK HEIGHTS)
4-6,8-14B,15	110966-002	BKRST LOWER HT ADJ ZP R TALL	RIGHT - TALL HEIGHT (20 -26" BACK HEIGHTS)
4-6,8-14B,15	110967-002	BKRST LOWER HT ADJ ZP L TALL	LEFT - TALL HEIGHT (20 -26" BACK HEIGHTS)

# STROLLER HANDLE EXTENSION-REMOVABLE

[09/2013]

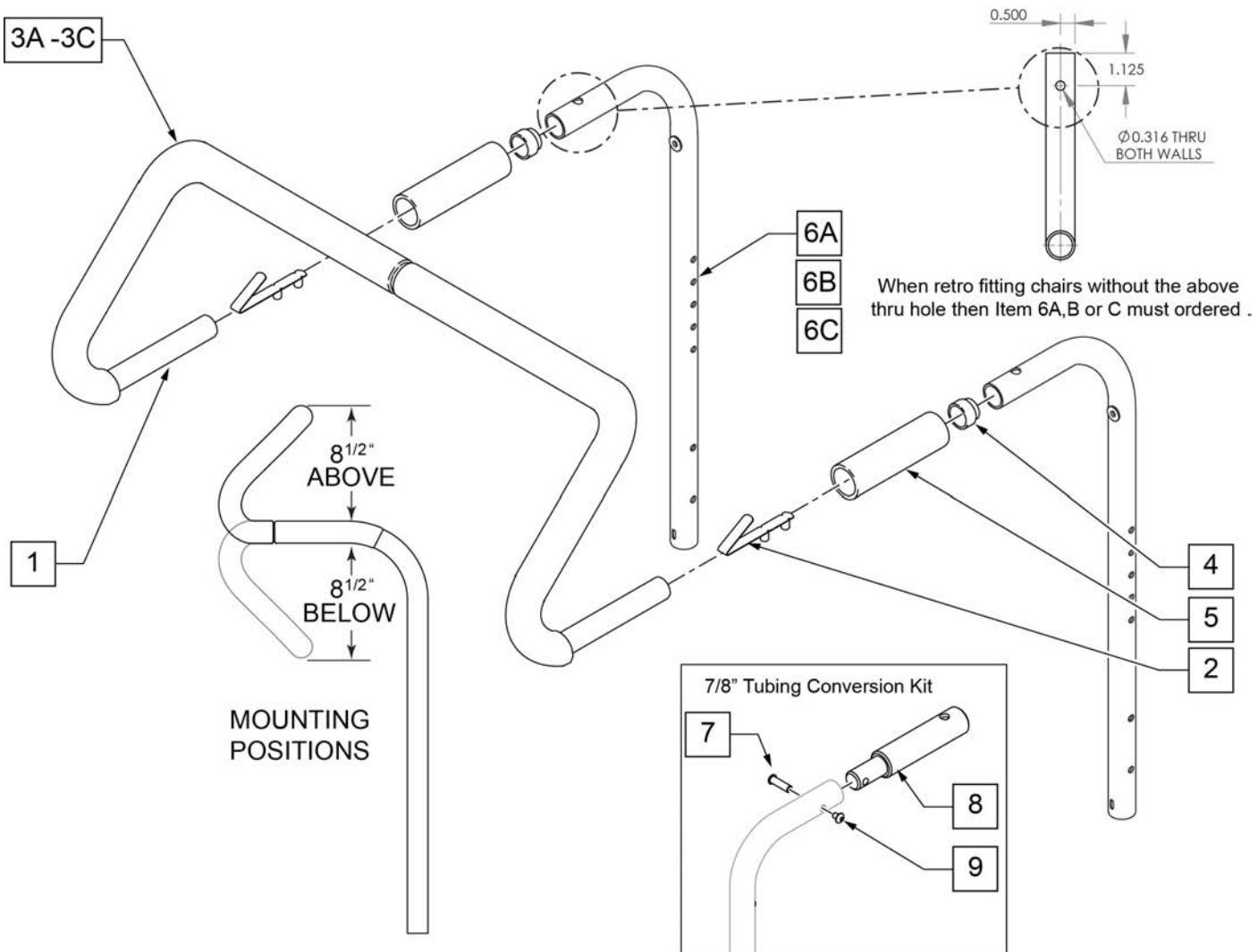


Pos.	Item Number	Description	Remarks
2	301108	5/16 SNAP BUTTON DUAL ONE SIDE	
3A	560615	STROLLER EXTENSION GRIP 10 -13"	10 -13" WIDE
3B	560616	STROLLER EXTENSION GRIP 14 -17"	14 -17" WIDE
3C	560617	STROLLER EXTENSION GRIP 18 -20"	18 -20" WIDE
4	561082	1" TUBE END BUSHING THIN	
5	920023	QUICKIE GRIP 1 W/ HOLE	
1, 2, 3A	842050	STROLLER EXTENSION ASSM 10"	
1, 2, 3A	842051	STROLLER EXTENSION ASSM 11"	
1, 2, 3A	842052	STROLLER EXTENSION ASSM 12"	
1, 2, 3A	842053	STROLLER EXTENSION ASSM 13"	
1, 2, 3B	842054	STROLLER EXTENSION ASSM 14"	
1, 2, 3B	842055	STROLLER EXTENSION ASSM 15"	
1, 2, 3B	842056	STROLLER EXTENSION ASSM 16"	
1, 2, 3B	842057	STROLLER EXTENSION ASSM 17"	
1, 2, 3C	842058	STROLLER EXTENSION ASSM 18"	
1, 2, 3C	842059	STROLLER EXTENSION ASSM 19	
1, 2, 3C	842063	STROLLER EXTENSION ASSM 20	
6A	842080	STROLLER HANDLE W/ FOAM STANDARD ZPS/P5/QT	STANDARD/LOW


<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
6B	842079	STROLLER HANDLE W/ FOAM MED ZPS/QT	MEDIUM
6C	842081	STROLLER HANDLE W/ FOAM TALL ZPS/QT	TALL
7-9	125324	STRLLR HNDL CNVERSN KIT 7/8 PH	To fit to 7/8" Push Handle

NOTE: Stroller handle extension works with 1" tubing backrest only. Example: DAB on Q2 or Standard backrest on positioning chairs.

NOTE: Not compatible with Swing-Away Adjustable Stroller Handle.

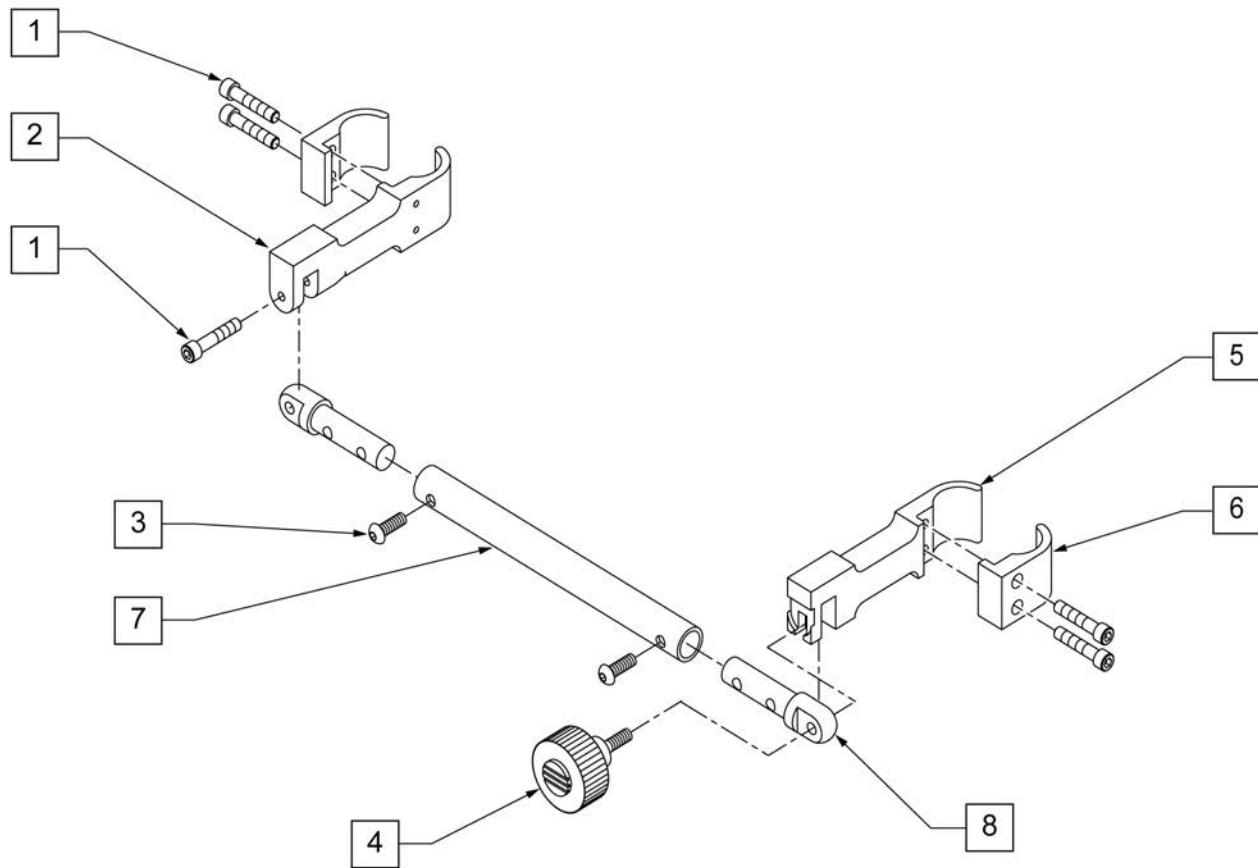


Pos.	Item Number	Description
1	107033	SHCS 1/4-20 X 1 ROHS
1-8	882056	RIGIDIZER ZTS 10 -12"
1-8	882057	RIGIDIZER ZTS 12 -14"
1-8	882058	RIGIDIZER ZTS 14 -16"
1-8	882059	RIGIDIZER ZTS 16 -18"

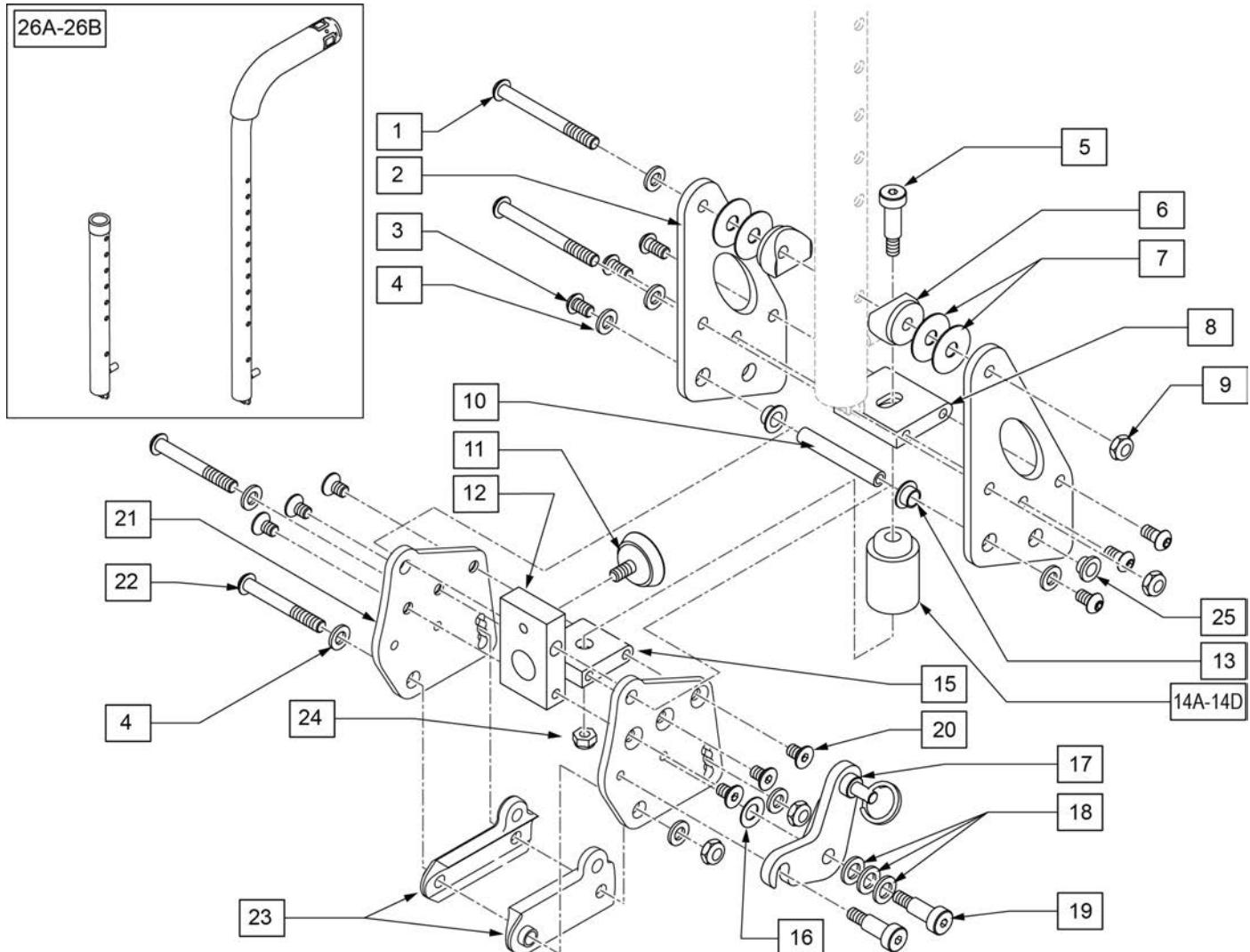


Pos.	Item Number	Description	Remarks
1	106967	BHCS 1/4-28 X 2-1/2 SHLDR ROHS	
3	103185-005	BHCS M6 X 1 X 12 FULL PL	
4	106943	WASHER FLAT AN 1/4 ROHS	
5	011649-003	SHLDR SCREW M6 X 8MM OD X 45MM	
6	561087	BACK REL SADDLE RIGID	
7	113268	WASHER NYLON 1IN OD X .312 ID	
9	106935	NUT NYLOCK THIN 1/4-28 ROHS	
11	349560	ZP RUBBER STOP	
14A	124044	SOFT RED SUSPENSION KIT	
14B	124060	MED BLUE SUSPENSION KIT	
14C	124064	FIRM GREEN SUSPENSION KIT	
14D	124082	X-FIRM BLACK SUSPENSION KIT	
16	262134	5/16 WASHER SPRING DISC	
18	106947	WASHER FLAT THIN AN 5/16 ROHS	
19	011649-001	SHLDR SCREW 8MM OD X 16MM LG	
20	013450	FHCS M6 X 1 X 10MM W/PCH	
22	106965	BHCS 1/4-28 X 2 SHLDR ROHS	
23	560518	SADDLE BACKREST Q2 DAB RIGHT	Right

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
23	560519	SADDLE BACKREST Q2 DAB LEFT	Left
24	104381	NUT HEX NYLOCK M6X1.0 INORG	
16-19,24,25	120943	DB LATCH REPLACEMENT KIT	Left & Right
1-25	126247	DYNAMIC BACKREST ASSM RH	
1-25	126248	DYNAMIC BACKREST ASSM LH	
1-25	121894	DAB DYNAMIC BACKREST ASSM RH	Right
1-25	121895	DAB DYNAMIC BACKREST ASSM LH	Left
1-26A	123944	ZP-Q2 DY BK PR ASM HGT ADJ STD	Pair
1-26A	123945	ZP-Q2 DY BK PR ASM HGT ADJ TLL	Pair
1-26B	123946	ZP-Q2 DY BK PR ASM STR EXT STD	Pair
1-26B	123948	ZP-Q2 DY BK PR ASM STR EXT MED	Pair
1-26B	123949	ZP-Q2 DY BK PR ASM STR EXT TLL	Pair
26A	123928	DYBCK ASSM ZPQ2 STD HGT ADJ RH	Right
26A	123929	DYBCK ASSM ZPQ2 STD HGT ADJ L	Left
26A	123930	DYBCK ASSM ZPQ2 TLL HGT ADJ RH	Right
26A	123931	DYBCK ASSM ZPQ2 TLL HGT ADJ LH	Left
26B	123932	DYBCK ZPQ2 STR EXT ASSM STD RH	Right
26B	123933	DYBCK ZPQ2 STR EXT ASSM STD LH	Left
26B	123935	DYBCK ZPQ2 STR EXT ASSM MED RH	Right
26B	123941	DYBCK ZPQ2 STR EXT ASSM MED LH	Left
26B	123942	DYBCK ZPQ2 STR EXT ASSM TLL RH	Right
26B	123943	DYBCK ZPQ2 STR EXT ASSM TLL LH	Left

## 60 DEG HANGER SWING IN-OUT

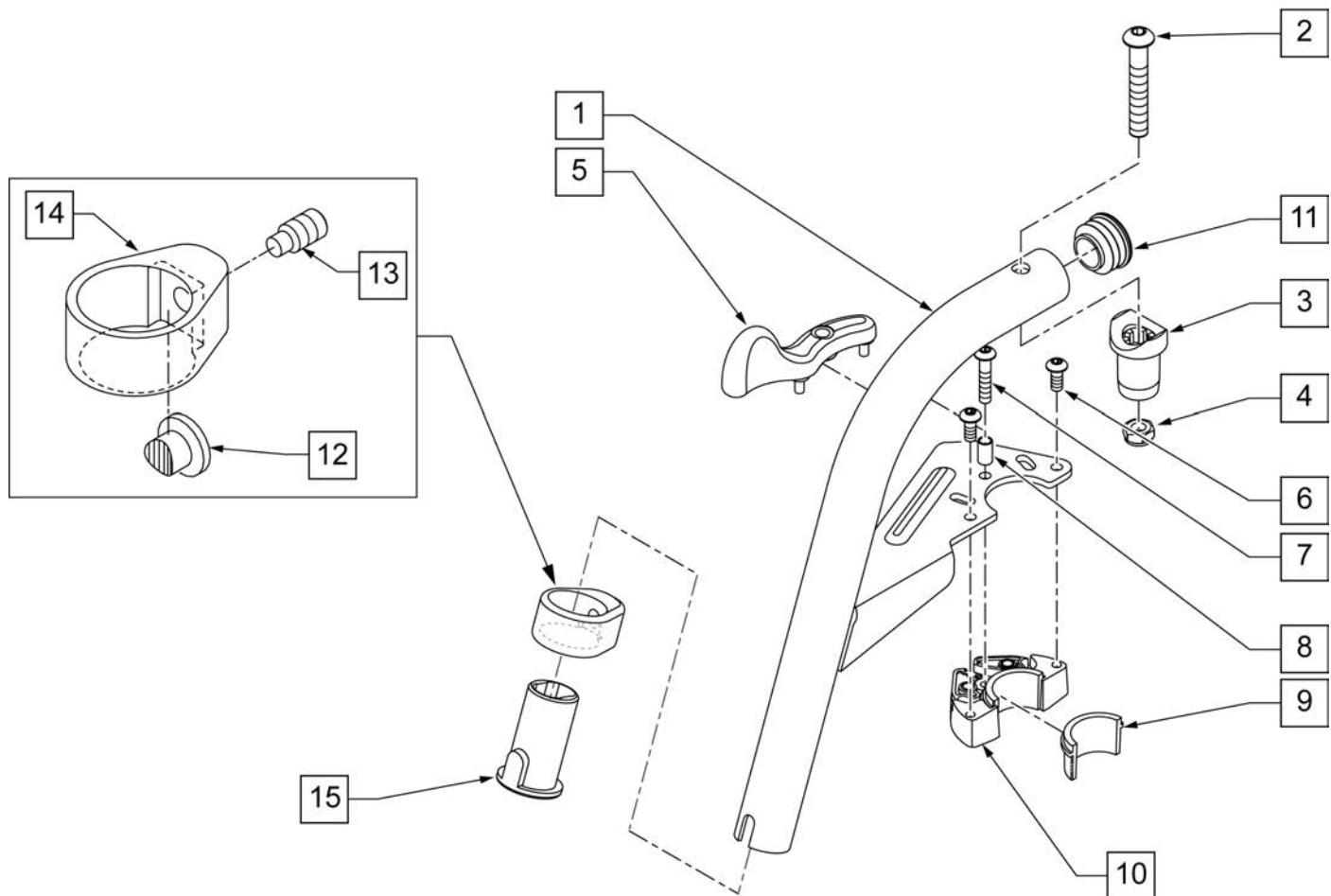


Pos.	Item Number	Description	Remarks
2	106983	5/16-18 X 2 BH C/S SHT THREAD	
3,4	561197	PIVOT SADDLE PLASTIC	
5	052296	SWING I/O LATCH LEVER RIGHT	Right
5	052297	SWING I/O LATCH LEVER LEFT	Left
6	102259-001	BHCS M5 X .8 X 12 W/PL INORG	
7	230494	M5-30 BTNN HD SOC C/S BLK	
8	052054	BUSHING PIVOT HANGER SWING I/O	
11	561062	1" INSERT PLUG .065 RIBS	
15	104160	BUSHING FOOTREST	
1-15	111243S	HANGER 60 Deg S/A ASSM R	SPECIFY COLOR,RIGHT
1-15	111242S	HANGER 60 Deg S/A ASSM L	SPECIFY COLOR,LEFT
12-15	107485	FOOTREST CLAMP ASSM	
6,9,10	120976	ASSY LATCH BLOCK ANTI LIFT	

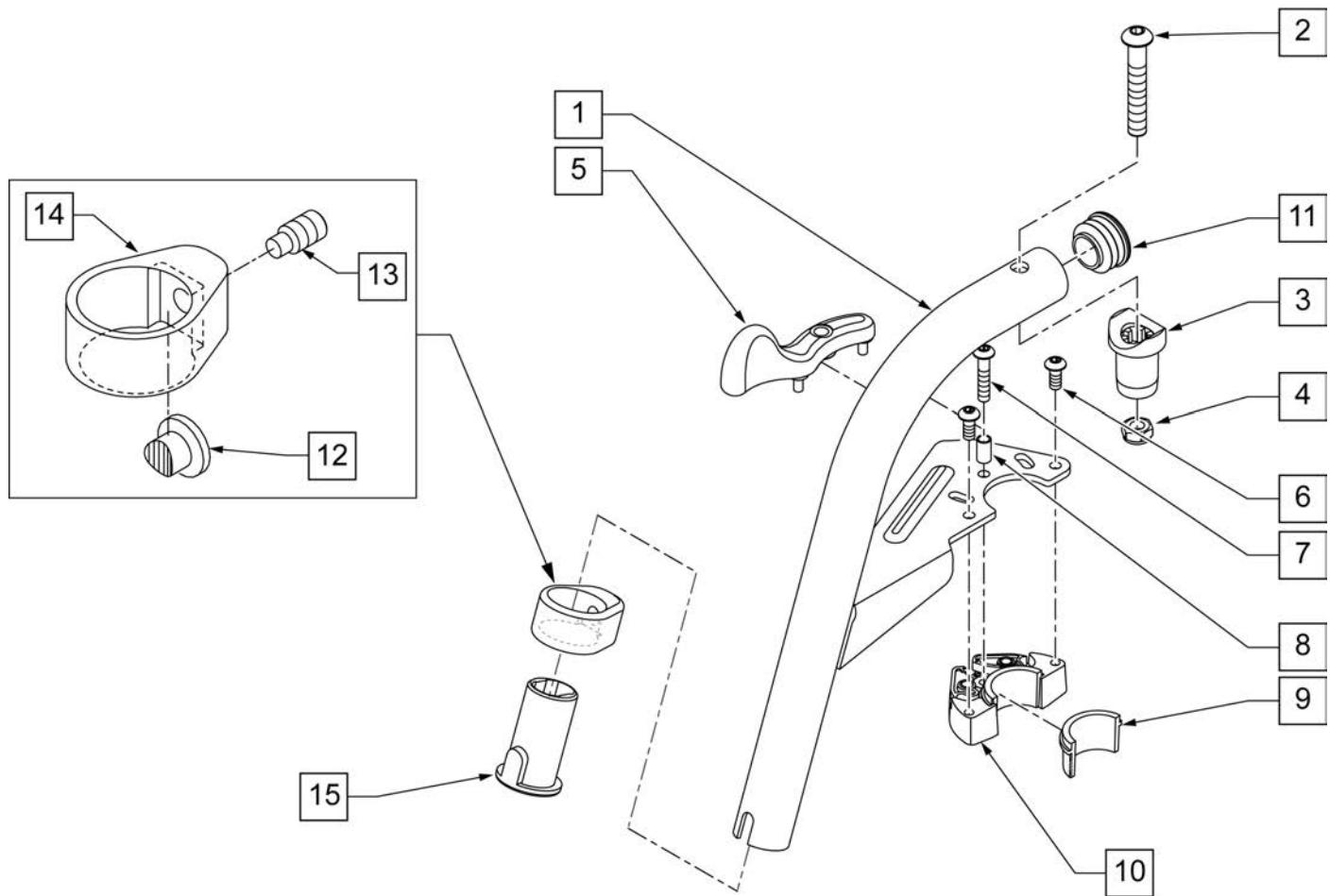

Pos.	Item Number	Description	Remarks
2	106983	5/16-18 X 2 BH C/S SHT THREAD	
3,4	561197	PIVOT SADDLE PLASTIC	
5	052296	SWING I/O LATCH LEVER RIGHT	Right
5	052297	SWING I/O LATCH LEVER LEFT	Left
6	102259-001	BHCS M5 X .8 X 12 W/PL INORG	
7	230494	M5-30 BTTN HD SOC C/S BLK	
8	052054	BUSHING PIVOT HANGER SWING I/O	
11	561062	1" INSERT PLUG .065 RIBS	
15	104160	BUSHING FOOTREST	
1-15	107671S	70 DEG SWING IN/OUT HANGER ASSEMBLY LEFT	SPECIFY COLOR,LEFT
1-15	107670S	70 DEG SWING IN/OUT HANGER ASSEMBLY RIGHT	SPECIFY COLOR,RIGHT
12-15	107485	FOOTREST CLAMP ASSM	
6,9,10	120976	ASSY LATCH BLOCK ANTI LIFT	

## 80 DEG HANGER SWING IN-OUT

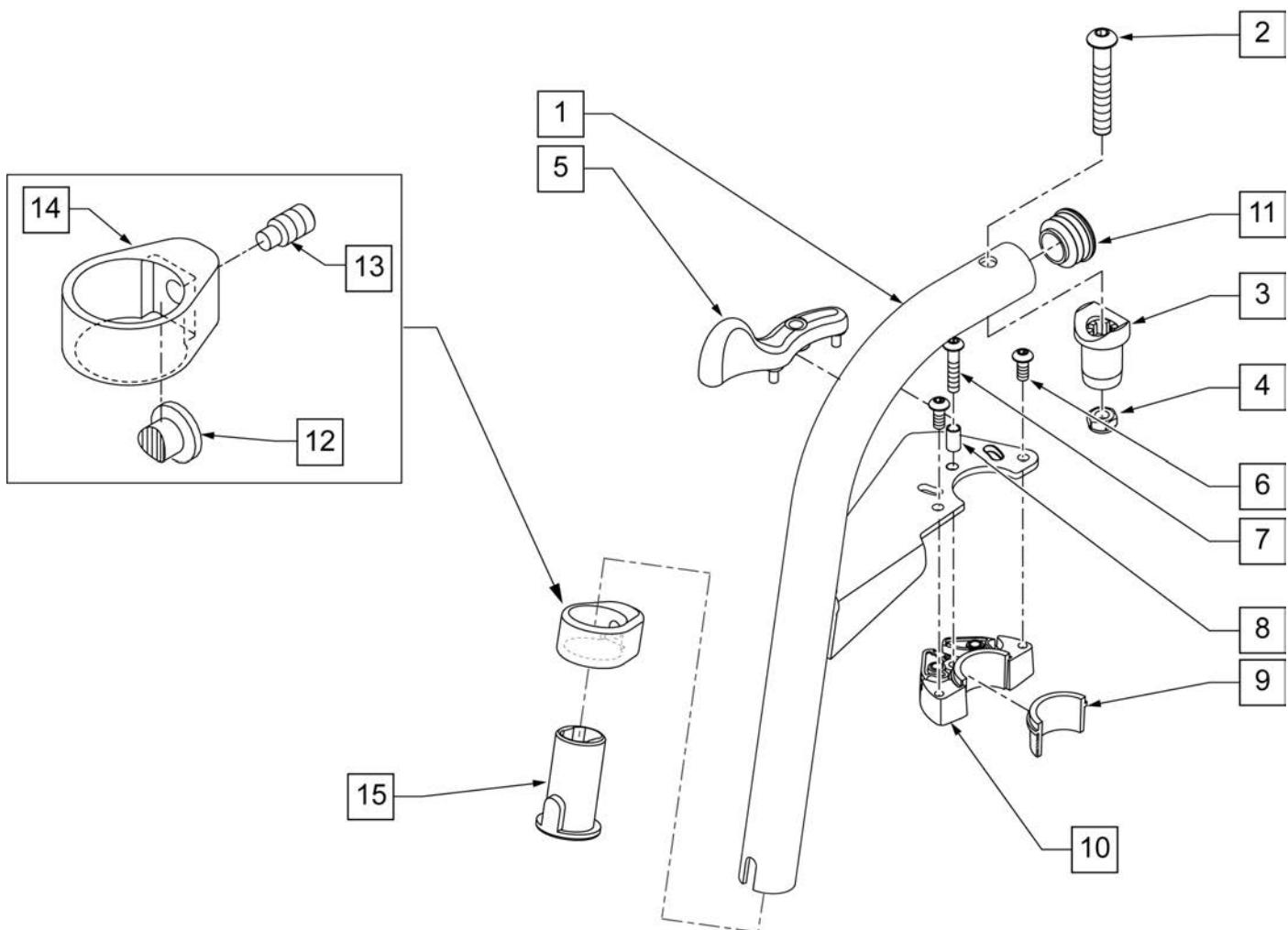


Pos.	Item Number	Description	Remarks
2	106983	5/16-18 X 2 BH C/S SHT THREAD	
3,4	561197	PIVOT SADDLE PLASTIC	
5	052296	SWING I/O LATCH LEVER RIGHT	Right
5	052297	SWING I/O LATCH LEVER LEFT	Left
6	102259-001	BHCS M5 X .8 X 12 W/PL INORG	
7	230494	M5-30 BTNN HD SOC C/S BLK	
8	052054	BUSHING PIVOT HANGER SWING I/O	
11	561062	1" INSERT PLUG .065 RIBS	
15	104160	BUSHING FOOTREST	
1-15	107673S	ASSEMBLY LEFT	SPECIFY COLOR,LEFT
1-15	107672S	ASSEMBLY RIGHT	SPECIFY COLOR,RIGHT
12-15	107485	FOOTREST CLAMP ASSM	
6,9,10	120976	ASSY LATCH BLOCK ANTI LIFT	

# COMPOSITE FOOTPLATE

[09/2011]

**NOTE:** FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.

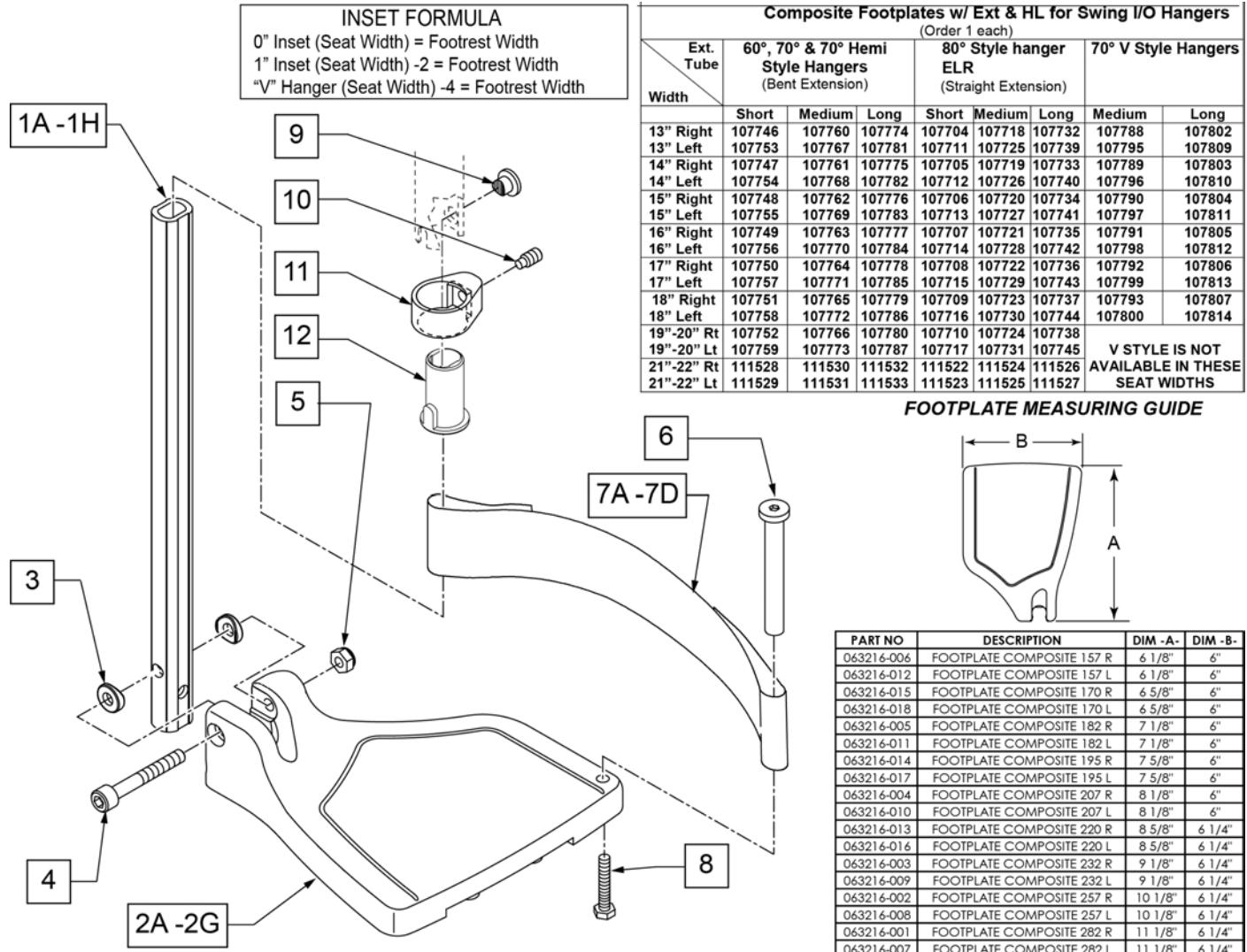

Pos.	Item Number	Description	Remarks
1A-5	120660	SHT STRT EXT COMP HDWR	80 deg hangers + ELR only
1B-5	120661	MED STRT EXT COMP HDWR	80 deg hangers + ELR only
1C-5	120662	LNG STRT EXT COMP HDWR	80 deg hangers + ELR only
1D-5	120663	SHT BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1E-5	120664	MED BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1F-5	120665	LNG BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1G-5	120666	SHT V EXT R COMP HDWR	70 deg V hangers only-LEFT
1G-5	120667	SHT V EXT L COMP HDWR	70 deg V hangers only-RIGHT
1H-5	120668	LNG V EXT R COMP HDWR	70 deg V hangers only-LEFT
1H-5	120669	LNG V EXT L COMP HDWR	70 deg V hangers only-RIGHT
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 13" Footrest Width
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 13" Footrest Width
2B	063216-015	FOOTPLATE COMPOSITE RIGHT	RIGHT 14" Footrest Width

Pos.	Item Number	Description	Remarks
2B	063216-018	FOOTPLATE COMPOSITE LEFT	LEFT 14" Footrest Width
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width
2D	063216-014	FOOTPLATE COMPOSITE RIGHT	RIGHT 16" Footrest Width
2D	063216-017	FOOTPLATE COMPOSITE LEFT	LEFT 16" Footrest Width
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width
2F	063216-013	FOOTPLATE COMPOSITE RIGHT	RIGHT 18" Footrest Width
2F	063216-016	FOOTPLATE COMPOSITE LEFT	LEFT 18" Footrest Width
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" & 20" Footrest Width
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 19" & 20" Footrest Width
2H	063216-002	FOOTPLATE COMPOSITE RIGHT	RIGHT 21" & 22" Footrest Width
2H	063216-008	FOOTPLATE COMPOSITE LEFT	LEFT 21" & 22" Footrest Width
3	104377	3/4" SADDLE THIN	
4	103175	SHCS M6 X 1 X 40 INORG	
5	104381	NUT HEX NYLOCK M6X1.0	Inserted in bottom of footplate.
6	104328	HEEL LOOP PEG	
7A	104380-001	ADJUSTABLE FOOT REST STRAP	14" wide
7B	104380-002	ADJUSTABLE FOOT REST STRAP	15-16" wide
7C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" wide
7D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" wide
8	102268-001	HHCS M6 X 1 X 25	
12	104160	BUSHING FOOTREST	
9-12	107485	FOOTREST CLAMP ASSM	
>>> 60°,70°,70° HEMI HANGERS - SHORT EXT - RIGHT			
1D,2A,3-6,7A,8	107746	COMP BNT SHT EXT ASM 13" W R	Right- 70D- 13" W- SHORT
1D,2B,3-6,7A,8	107747	COMP BNT SHT EXT ASM 14" W R	Right- 70D- 14" W- SHORT
1D,2C,3-6,7B,8	107748	COMP BNT SHT EXT ASM 15" W R	Right- 70D- 15" W- SHORT
1D,2D,3-6,7B,8	107749	COMP BNT SHT EXT ASM 16" W R	Right- 70D- 16" W- SHORT
1D,2E,3-6,7C,8	107750	COMP BNT SHT EXT ASM 17" W R	Right- 70D- 17" W- SHORT
1D,2F,3-6,7C,8	107751	COMP BNT SHT EXT ASM 18" W R	Right- 70D- 18" W- SHORT
1D,2G,3-6,7D,8	107752	COMP BNT SHT EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- SHORT
1D,2H,3-6,7D,8	111528	COMP BNT SHT EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- SHORT
>>> 60°,70°,70° HEMI HANGERS - SHORT EXT - LEFT			
1D,2A,3-6,7A,8	107753	COMP BNT SHT EXT ASM 13" W L	Left- 70D- 13 W- SHORT
1D,2B,3-6,7A,8	107754	COMP BNT SHT EXT ASM 14" W L	Left- 70D- 14 W- SHORT
1D,2C,3-6,7B,8	107755	COMP BNT SHT EXT ASM 15" W L	Left- 70D- 15 W- SHORT
1D,2D,3-6,7B,8	107756	COMP BNT SHT EXT ASM 16" W L	Left- 70D- 16 W- SHORT
1D,2E,3-6,7C,8	107757	COMP BNT SHT EXT ASM 17" W L	Left- 70D- 17 W- SHORT
1D,2F,3-6,7C,8	107758	COMP BNT SHT EXT ASM 18" W L	Left- 70D- 18 W- SHORT
1D,2G,3-6,7D,8	107759	COMP BNT SHT EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- SHORT
1D,2H,3-6,7D,8	111529	COMP BNT SHT EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- SHORT
>>> 60°,70°,70° HEMI HANGERS - MED EXT - RIGHT			
1E,2A,3-6,7A,8	107760	COMP BNT MED EXT ASM 13" W R	Right- 70D- 13" W- MED
1E,2B,3-6,7A,8	107761	COMP BNT MED EXT ASM 14" W R	Right- 70D- 14" W- MED
1E,2C,3-6,7B,8	107762	COMP BNT MED EXT ASM 15" W R	Right- 70D- 15" W- MED
1E,2D,3-6,7B,8	107763	COMP BNT MED EXT ASM 16" W R	Right- 70D- 16" W- MED
1E,2E,3-6,7C,8	107764	COMP BNT MED EXT ASM 17" W R	Right- 70D- 17" W- MED
1E,2F,3-6,7C,8	107765	COMP BNT MED EXT ASM 18" W R	Right- 70D- 18" W- MED
1E,2G,3-6,7D,8	107766	COMP BNT MED EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- MED
1E,2H,3-6,7D,8	111530	COMP BNT MED EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- MED
>>> 60°,70°,70° HEMI HANGERS - MED EXT - LEFT			
1E,2A,3-6,7A,8	107767	COMP BNT MED EXT ASM 13" W L	Left- 70D- 13 W- MED
1E,2B,3-6,7A,8	107768	COMP BNT MED EXT ASM 14" W L	Left- 70D- 14 W- MED
1E,2C,3-6,7B,8	107769	COMP BNT MED EXT ASM 15" W L	Left- 70D- 15 W- MED
1E,2D,3-6,7B,8	107770	COMP BNT MED EXT ASM 16" W L	Left- 70D- 16 W- MED

Pos.	Item Number	Description	Remarks
1E,2E,3-6,7C,8	107771	COMP BNT MED EXT ASM 17" W L	Left- 70D- 17 W- MED
1E,2F,3-6,7C,8	107772	COMP BNT MED EXT ASM 18" W L	Left- 70D- 18 W- MED
1E,2G,3-6,7D,8	107773	COMP BNT MED EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- MED
1E,2H,3-6,7D,8	111531	COMP BNT MED EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- MED
>>>		60°,70°,70° HEMI HANGERS - LONG EXT - RIGHT	
1F,2A,3-6,7A,8	107774	COMP BNT LONG EXT ASM 13" W R	Right- 70D- 13" W- LONG
1F,2B,3-6,7A,8	107775	COMP BNT LONG EXT ASM 14" W R	Right- 70D- 14" W- LONG
1F,2C,3-6,7B,8	107776	COMP BNT LONG EXT ASM 15" W R	Right- 70D- 15" W- LONG
1F,2D,3-6,7B,8	107777	COMP BNT LONG EXT ASM 16" W R	Right- 70D- 16" W- LONG
1F,2E,3-6,7C,8	107778	COMP BNT LONG EXT ASM 17" W R	Right- 70D- 17" W- LONG
1F,2F,3-6,7C,8	107779	COMP BNT LONG EXT ASM 18" W R	Right- 70D- 18" W- LONG
1F,2G,3-6,7D,8	107780	COMP BNT LONG EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- LONG
1F,2H,3-6,7D,8	111532	COMP BNT LONG EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- LONG
>>>		60°,70°,70° HEMI HANGERS - LONG EXT - LEFT	
1F,2A,3-6,7A,8	107781	COMP BNT LONG EXT ASM 13" W L	Left- 70D- 13 W- LONG
1F,2B,3-6,7A,8	107782	COMP BNT LONG EXT ASM 14" W L	Left- 70D- 14 W- LONG
1F,2C,3-6,7B,8	107783	COMP BNT LONG EXT ASM 15" W L	Left- 70D- 15 W- LONG
1F,2D,3-6,7B,8	107784	COMP BNT LONG EXT ASM 16" W L	Left- 70D- 16 W- LONG
1F,2E,3-6,7C,8	107785	COMP BNT LONG EXT ASM 17" W L	Left- 70D- 17 W- LONG
1F,2F,3-6,7C,8	107786	COMP BNT LONG EXT ASM 18" W L	Left- 70D- 18 W- LONG
1F,2G,3-6,7D,8	107787	COMP BNT LONG EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- LONG
1F,2H,3-6,7D,8	111533	COMP BNT LONG EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- LONG
>>>		70 DEGREE "V" HANGERS - MED EXT - RIGHT	
1G,2A,3-8	107788	COMP V MED EXT ASM 13W R	Right- 70D V hangers only 13 W- MED
1G,2B,3-8	107789	COMP V MED EXT ASM 14W R	Right- 70D V hangers only 14 W- MED
1G,2C,3-8	107790	COMP V MED EXT ASM 15W R	Right- 70D V hangers only 15 W- MED
1G,2D,3-8	107791	COMP V MED EXT ASM 16W R	Right- 70D V hangers only 16 W- MED
1G,2E,3-8	107792	COMP V MED EXT ASM 17W R	Right- 70D V hangers only 17 W- MED
1G,2F,3-8	107793	COMP V MED EXT ASM 18W R	Right- 70D V hangers only 18 W- MED
>>>		70 DEGREE "V" HANGERS - MED EXT - LEFT	
1G,2A,3-8	107795	COMP V MED EXT ASM 13W L	Left- 70D V hangers only 13 W- MED
1G,2B,3-8	107796	COMP V MED EXT ASM 14W L	Left- 70D V hangers only 14 W- MED
1G,2C,3-8	107797	COMP V MED EXT ASM 15W L	Left- 70D V hangers only 15 W- MED
1G,2D,3-8	107798	COMP V MED EXT ASM 16W L	Left- 70D V hangers only 16 W- MED
1G,2E,3-8	107799	COMP V MED EXT ASM 17W L	Left- 70D V hangers only 17 W- MED
1G,2F,3-8	107800	COMP V MED EXT ASM 18W L	Left- 70D V hangers only 18 W- MED
>>>		70 DEGREE "V" HANGERS - LONG EXT - RIGHT	
1H,2A,3-8	107802	COMP V LNG EXT ASM 13W R	Right- 70D V hangers only 13 W- LONG
1H,2B,3-8	107803	COMP V LNG EXT ASM 14W R	Right- 70D V hangers only 14 W- LONG
1H,2C,3-8	107804	COMP V LNG EXT ASM 15W R	Right- 70D V hangers only 15 W- LONG
1H,2D,3-8	107805	COMP V LNG EXT ASM 16W R	Right- 70D V hangers only 16 W- LONG
1H,2E,3-8	107806	COMP V LNG EXT ASM 17W R	Right- 70D V hangers only 17 W- LONG
1H,2F,3-8	107807	COMP V LNG EXT ASM 18W R	Right- 70D V hangers only 18 W- LONG
>>>		70 DEGREE "V" HANGERS - LONG EXT - LEFT	
1H,2A,3-8	107809	COMP V LNG EXT ASM 13W L	Left- 70D V hangers only 13 W- LONG
1H,2B,3-8	107810	COMP V LNG EXT ASM 14W L	Left- 70D V hangers only 14 W- LONG
1H,2C,3-8	107811	COMP V LNG EXT ASM 15W L	Left- 70D V hangers only 15 W- LONG
1H,2D,3-8	107812	COMP V LNG EXT ASM 16W L	Left- 70D V hangers only 16 W- LONG
1H,2E,3-8	107813	COMP V LNG EXT ASM 17W L	Left- 70D V hangers only 17 W- LONG
1H,2F,3-8	107814	COMP V LNG EXT ASM 18W L	Left- 70D V hangers only 18 W- LONG
>>>		80 DEGREE HANGERS - SHORT EXT - RIGHT	
1A,2A,3-6,7A,8	107704	COMP STRT SHT EXT ASM 13"W R	Right- 80D- 13" W- SHORT
1A,2B,3-6,7A,8	107705	COMP STRT SHT EXT ASM 14"W R	Right- 80D- 14" W- SHORT
1A,2C,3-6,7B,8	107706	COMP STRT SHT EXT ASM 15"W R	Right- 80D- 15" W- SHORT
1A,2D,3-6,7B,8	107707	COMP STRT SHT EXT ASM 16"W R	Right- 80D- 16" W- SHORT
1A,2E,3-6,7C,8	107708	COMP STRT SHT EXT ASM 17"W R	Right- 80D- 17" W- SHORT

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
1A,2F,3-6,7C,8	107709	COMP STRT SHT EXT ASM 18"W R	Right- 80D- 18" W- SHORT
1A,2G,3-6,7D,8	107710	COMP STRT SHT EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- SHORT
1A,2H,3-6,7D,8	111522	COMP STRT SHT EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- SHORT
>>>		80 DEGREE HANGERS - SHORT EXT - LEFT	
1A,2A,3-6,7A,8	107711	COMP STRT SHT EXT ASM 13"W L	LEFT- 80D- 13" W- SHORT
1A,2B,3-6,7A,8	107712	COMP STRT SHT EXT ASM 14"W L	LEFT- 80D- 14" W- SHORT
1A,2C,3-6,7B,8	107713	COMP STRT SHT EXT ASM 15"W L	LEFT- 80D- 15" W- SHORT
1A,2D,3-6,7B,8	107714	COMP STRT SHT EXT ASM 16"W L	LEFT- 80D- 16" W- SHORT
1A,2E,3-6,7C,8	107715	COMP STRT SHT EXT ASM 17"W L	LEFT- 80D- 17" W- SHORT
1A,2F,3-6,7C,8	107716	COMP STRT SHT EXT ASM 18"W L	LEFT- 80D- 18" W- SHORT
1A,2G,3-6,7D,8	107717	COMP STRT SHT EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- SHORT
1A,2H,3-6,7D,8	111523	COMP STRT SHT EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- SHORT
>>>		80 DEGREE HANGERS - MED EXT - RIGHT	
1B,2A,3-6,7A,8	107718	COMP STRT MED EXT ASM 13"W R	Right- 80D- 13" W- MED
1B,2B,3-6,7A,8	107719	COMP STRT MED EXT ASM 14"W R	Right- 80D- 14" W- MED
1B,2C,3-6,7B,8	107720	COMP STRT MED EXT ASM 15"W R	Right- 80D- 15" W- MED
1B,2D,3-6,7B,8	107721	COMP STRT MED EXT ASM 16"W R	Right- 80D- 16" W- MED
1B,2E,3-6,7C,8	107722	COMP STRT MED EXT ASM 17"W R	Right- 80D- 17" W- MED
1B,2F,3-6,7C,8	107723	COMP STRT MED EXT ASM 18"W R	Right- 80D- 18" W- MED
1B,2G,3-6,7D,8	107724	COMP STRT MED EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- MED
1B,2H,3-6,7D,8	111524	COMP STRT MED EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- MED
>>>		80 DEGREE HANGERS - MED EXT - LEFT	
1B,2A,3-6,7A,8	107725	COMP STRT MED EXT ASM 13"W L	LEFT- 80D- 13" W- MED
1B,2B,3-6,7A,8	107726	COMP STRT MED EXT ASM 14"W L	LEFT- 80D- 14" W- MED
1B,2C,3-6,7B,8	107727	COMP STRT MED EXT ASM 15"W L	LEFT- 80D- 15" W- MED
1B,2D,3-6,7B,8	107728	COMP STRT MED EXT ASM 16"W L	LEFT- 80D- 16" W- MED
1B,2E,3-6,7C,8	107729	COMP STRT MED EXT ASM 17"W L	LEFT- 80D- 17" W- MED
1B,2F,3-6,7C,8	107730	COMP STRT MED EXT ASM 18"W L	LEFT- 80D- 18" W- MED
1B,2G,3-6,7D,8	107731	COMP STRT MED EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- MED
1B,2H,3-6,7D,8	111525	COMP STRT MED EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- MED
>>>		80 DEGREE HANGERS - LONG EXT - RIGHT	
1C,2A,3-6,7A,8	107732	COMP STRT LONG EXT ASM 13"W R	Right- 80D- 13" W- LONG
1C,2B,3-6,7A,8	107733	COMP STRT LONG EXT ASM 14"W R	Right- 80D- 14" W- LONG
1C,2C,3-6,7B,8	107734	COMP STRT LONG EXT ASM 15"W R	Right- 80D- 15" W- LONG
1C,2D,3-6,7B,8	107735	COMP STRT LONG EXT ASM 16"W R	Right- 80D- 16" W- LONG
1C,2E,3-6,7C,8	107736	COMP STRT LONG EXT ASM 17"W R	Right- 80D- 17" W- LONG
1C,2F,3-6,7C,8	107737	COMP STRT LONG EXT ASM 18"W R	Right- 80D- 18" W- LONG
1C,2G,3-6,7D,8	107738	COMP STRT LONG EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- LONG
1C,2H,3-6,7D,8	111526	COMP STRT LONG EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- LONG
>>>		80 DEGREE HANGERS - LONG EXT - LEFT	
1C,2A,3-6,7A,8	107739	COMP STRT LONG EXT ASM 13"W L	LEFT- 80D- 13" W- LONG
1C,2B,3-6,7A,8	107740	COMP STRT LONG EXT ASM 14"W L	LEFT- 80D- 14" W- LONG
1C,2C,3-6,7B,8	107741	COMP STRT LONG EXT ASM 15"W L	LEFT- 80D- 15" W- LONG
1C,2D,3-6,7B,8	107742	COMP STRT LONG EXT ASM 16"W L	LEFT- 80D- 16" W- LONG
1C,2E,3-6,7C,8	107743	COMP STRT LONG EXT ASM 17"W L	LEFT- 80D- 17" W- LONG
1C,2F,3-6,7C,8	107744	COMP STRT LONG EXT ASM 18"W L	LEFT- 80D- 18" W- LONG
1C,2G,3-6,7D,8	107745	COMP STRT LONG EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- LONG
1C,2H,3-6,7D,8	111527	COMP STRT LONG EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- LONG

# COMPOSITE ANGLE ADJUSTABLE FOOT PLATE


[02/2010]

**NOTE:** FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.

INSET FORMULA			
0" Inset (Seat Width) = Footrest Width			
1" Inset (Seat Width) -2 = Footrest Width			
"V" Hanger (Seat Width) -4 = Footrest Width			

Composite Angle Adjustable "CAA" Footplate w/ Ext & HL for Swing I/O Hangers (Order 1 each)								
Ext. Tube Width	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger (Straight Extension)			70° V Style Hangers	
	Short	Medium	Long	Short	Medium	Long	Medium	Long
15" Right	107858	107872	107886	107816	107830	107844	107900	107914
15" Left	107865	107879	107893	107823	107837	107851	107907	107921
16" Right	107859	107873	107887	107817	107831	107845	107901	107915
16" Left	107866	107880	107894	107824	107838	107852	107908	107922
17" Right	107860	107874	107888	107818	107832	107846	107902	107916
17" Left	107867	107881	107895	107825	107839	107853	107909	107923
18" Right	107861	107875	107889	107819	107833	107847	107903	107917
18" Left	107868	107882	107896	107826	107840	107854	107910	107924
19" Right	107862	107876	107890	107820	107834	107848	V STYLE IS NOT	
19" Left	107869	107883	107897	107827	107841	107855		
20" Right	107863	107877	107891	107821	107835	107849		
20" Left	107870	107884	107898	107828	107842	107856		
21"-22" Rt	107864	107878	107892	107822	107836	107850		
21"-22" Lt	107871	107885	107899	107829	107843	107857		

## FOOTPLATE MEASURING GUIDE

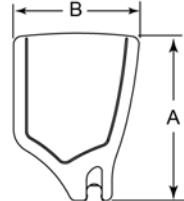

PART NO	DESCRIPTION	DIM -A-	DIM -B-
063216-006	FOOTPLATE COMPOSITE 157 R	6 1/8"	6"
063216-012	FOOTPLATE COMPOSITE 157 L	6 1/8"	6"
063216-015	FOOTPLATE COMPOSITE 170 R	6 5/8"	6"
063216-018	FOOTPLATE COMPOSITE 170 L	6 5/8"	6"
063216-005	FOOTPLATE COMPOSITE 182 R	7 1/8"	6"
063216-011	FOOTPLATE COMPOSITE 182 L	7 1/8"	6"
063216-014	FOOTPLATE COMPOSITE 195 R	7 5/8"	6"
063216-017	FOOTPLATE COMPOSITE 195 L	7 5/8"	6"
063216-004	FOOTPLATE COMPOSITE 207 R	8 1/8"	6"
063216-010	FOOTPLATE COMPOSITE 207 L	8 1/8"	6"
063216-013	FOOTPLATE COMPOSITE 220 R	8 5/8"	6 1/4"
063216-016	FOOTPLATE COMPOSITE 220 L	8 5/8"	6 1/4"
063216-003	FOOTPLATE COMPOSITE 232 R	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L	9 1/8"	6 1/4"
063216-002	FOOTPLATE COMPOSITE 257 R	10 1/8"	6 1/4"
063216-008	FOOTPLATE COMPOSITE 257 L	10 1/8"	6 1/4"
063216-001	FOOTPLATE COMPOSITE 282 R	11 1/8"	6 1/4"
063216-007	FOOTPLATE COMPOSITE 282 L	11 1/8"	6 1/4"

Pos.	Item Number	Description	Remarks
1A,6,7	120660	SHT STRT EXT COMP HDWR	80 deg hangers only
1B,6,7	120661	MED STRT EXT COMP HDWR	80 deg hangers only
1C,6,7	120662	LNG STRT EXT COMP HDWR	80 deg hangers only
1D,6,7	120663	SHT BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1E,6,7	120664	MED BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1F,6,7	120665	LNG BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1G,6,7	120666	SHT V EXT R COMP HDWR	70 deg V hangers only-LEFT
1G,6,7	120667	SHT V EXT L COMP HDWR	70 deg V hangers only-RIGHT
1H,6,7	120668	LNG V EXT R COMP HDWR	70 deg V hangers only-LEFT
1H,6,7	120669	LNG V EXT L COMP HDWR	70 deg V hangers only-RIGHT
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width
2B	063216-015	COMPOSITE FOOTPLATE RIGHT	RIGHT 16" Footrest Width

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
2B	063216-018	COMPOSITE FOOTPLATE LEFT	LEFT 16" Footrest Width
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width
2D	063216-014	COMPOSITE FOOTPLATE RIGHT	RIGHT 18" Footrest Width
2D	063216-017	COMPOSITE FOOTPLATE LEFT	LEFT 18" Footrest Width
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" Footrest Width
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 19" Footrest Width
2F	063216-013	COMPOSITE FOOTPLATE RIGHT	RIGHT 20" Footrest Width
2F	063216-016	COMPOSITE FOOTPLATE LEFT	LEFT 20" Footrest Width
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 21" & 22" Footrest Width
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 21" & 22" Footrest Width
3	108233	BHCS M6 X 1 X 40	
4	054242	ANGLE COVER	
5	054241	ANGLE SEGMENT	
6	103175	SHCS M6 X 1 X 40 INORG	
7	104381	NUT HEX NYLOCK M6X1.0	
8	102268-001	HHCS M6X1X25	Inserted in bottom of footplate.
9	104328	HEEL LOOP PEG	
10A	104380-001	ADJUSTABLE FOOT REST STRAP	15" Footrest Width
10B	104380-002	ADJUSTABLE FOOT REST STRAP	16" Footrest Width
10C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" Footrest Width
10D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" Footrest Width
11	104160	BUSHING FOOTREST	
11-14	107485	FOOTREST CLAMP ASSM	
>>> 60°,70°,70° HEMI HANGERS - SHORT EXT - RIGHT			
1D,2A,3-10A	107858	CAA BNT SHT EXT ASM 15W R	Right- 60 & 60° & 70° 15 W- SHORT
1D,2B,3-10B	107859	CAA BNT SHT EXT ASM 16W R	Right- 60° & 70° 16 W- SHORT
1D,2C,3-10B	107860	CAA BNT SHT EXT ASM 17W R	Right- 60° & 70° 17 W- SHORT
1D,2D,3-10C	107861	CAA BNT SHT EXT ASM 18W R	Right- 60° & 70° 18 W- SHORT
1D,2E,3-10C	107862	CAA BNT SHT EXT ASM 19W R	Right- 60° & 70° 19 W- SHORT
1D,2F,3-10D	107863	CAA BNT SHT EXT ASM 20W R	Right- 60° & 70° 20 W- SHORT
1D,2G,3-10D	107864	CAA BNT SHT EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- SHORT
>>> 60°,70°,70° HEMI HANGERS - SHORT EXT - LEFT			
1D,2A,3-10A	107865	CAA BNT SHT EXT ASM 15W L	Left- 60° & 70° 15 W- SHORT
1D,2B,3-10B	107866	CAA BNT SHT EXT ASM 16W L	Left- 60° & 70° 16 W- SHORT
1D,2C,3-10B	107867	CAA BNT SHT EXT ASM 17W L	Left- 60° & 70° 17 W- SHORT
1D,2D,3-10C	107868	CAA BNT SHT EXT ASM 18W L	Left- 60° & 70° 18 W- SHORT
1D,2E,3-10C	107869	CAA BNT SHT EXT ASM 19W L	Left- 60° & 70° 19 W- SHORT
1D,2F,3-10D	107870	CAA BNT SHT EXT ASM 20W L	Left- 60° & 70° 20 W- SHORT
1D,2G,3-10D	107871	CAA BNT SHT EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- SHORT
>>> 60°,70°,70° HEMI HANGERS - MED EXT - RIGHT			
1E,2A,3-10A	107872	CAA BNT MED EXT ASM 15W R	Right- 60° & 70° 15 W- MED
1E,2B,3-10B	107873	CAA BNT MED EXT ASM 16W R	Right- 60° & 70° 16 W- MED
1E,2C,3-10B	107874	CAA BNT MED EXT ASM 17W R	Right- 60° & 70° 17 W- MED
1E,2D,3-10C	107875	CAA BNT MED EXT ASM 18W R	Right- 60° & 70° 18 W- MED
1E,2E,3-10C	107876	CAA BNT MED EXT ASM 19W R	Right- 60° & 70° 19 W- MED
1E,2F,3-10D	107877	CAA BNT MED EXT ASM 20W R	Right- 60° & 70° 20 W- MED
1E,2G,3-10D	107878	CAA BNT MED EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- MED
>>> 60°,70°,70° HEMI HANGERS - MED EXT - LEFT			
1E,2A,3-10A	107879	CAA BNT MED EXT ASM 15W L	Left- 60° & 70° 15 W- MED
1E,2B,3-10B	107880	CAA BNT MED EXT ASM 16W L	Left- 60° & 70° 16 W- MED
1E,2C,3-10B	107881	CAA BNT MED EXT ASM 17W L	Left- 60° & 70° 17 W- MED
1E,2D,3-10C	107882	CAA BNT MED EXT ASM 18W L	Left- 60° & 70° 18 W- MED
1E,2E,3-10C	107883	CAA BNT MED EXT ASM 19W L	Left- 60° & 70° 19 W- MED
1E,2F,3-10D	107884	CAA BNT MED EXT ASM 20W L	Left- 60° & 70° 20 W- MED
1E,2G,3-10D	107885	CAA BNT MED EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- MED

Pos.	Item Number	Description	Remarks
>>>		60°,70°,70° HEMI HANGERS - LONG EXT - RIGHT	
1F,2A,3-10A	107886	CAA BNT LNG EXT ASM 15W R	Right- 60° & 70° 15 W- LONG
1F,2B,3-10B	107887	CAA BNT LNG EXT ASM 16W R	Right- 60° & 70° 16 W- LONG
1F,2C,3-10B	107888	CAA BNT LNG EXT ASM 17W R	Right- 60° & 70° 17 W- LONG
1F,2D,3-10C	107889	CAA BNT LNG EXT ASM 18W R	Right- 60° & 70° 18 W- LONG
1F,2E,3-10C	107890	CAA BNT LNG EXT ASM 19W R	Right- 60° & 70° 19 W- LONG
1F,2F,3-10D	107891	CAA BNT LNG EXT ASM 20W R	Right- 60° & 70° 20 W- LONG
1F,2G,3-10D	107892	CAA BNT LNG EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- LONG
>>>		60°,70°,70° HEMI HANGERS - LONG EXT - LEFT	
1F,2A,3-10A	107893	CAA BNT LNG EXT ASM 15W L	Left- 60° & 70° 15 W- LONG
1F,2B,3-10B	107894	CAA BNT LNG EXT ASM 16W L	Left- 60° & 70° 16 W- LONG
1F,2C,3-10B	107895	CAA BNT LNG EXT ASM 17W L	Left- 60° & 70° 17 W- LONG
1F,2D,3-10C	107896	CAA BNT LNG EXT ASM 18W L	Left- 60° & 70° 18 W- LONG
1F,2E,3-10C	107897	CAA BNT LNG EXT ASM 19W L	Left- 60° & 70° 19 W- LONG
1F,2F,3-10D	107898	CAA BNT LNG EXT ASM 20W L	Left- 60° & 70° 20 W- LONG
1F,2G,3-10D	107899	CAA BNT LNG EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- LONG
>>>		70° "V" HANGERS - MED EXT - RIGHT	
1G,2A,3-10A	107900	CAA V MED EXT ASM 15W R	Right- 70° V hangers only 15 W- MED
1G,2B,3-10B	107901	CAA V MED EXT ASM 16W R	Right- 70° V hangers only 16 W- MED
1G,2C,3-10B	107902	CAA V MED EXT ASM 17W R	Right- 70° V hangers only 17 W- MED
1G,2D,3-10C	107903	CAA V MED EXT ASM 18W R	Right- 70° V hangers only 18 W- MED
>>>		70° "V" HANGERS - MED EXT - LEFT	
1G,2A,3-10A	107907	CAA V MED EXT ASM 15W L	Left- 70° V hangers only 15 W- MED
1G,2B,3-10B	107908	CAA V MED EXT ASM 16W L	Left- 70° V hangers only 16 W- MED
1G,2C,3-10B	107909	CAA V MED EXT ASM 17W L	Left- 70° V hangers only 17 W- MED
1G,2D,3-10C	107910	CAA V MED EXT ASM 18W L	Left- 70° V hangers only 18 W- MED
>>>		70° "V" HANGERS - LONG EXT - RIGHT	
1H,2A,3-10A	107914	CAA V LNG EXT ASM 15W R	Right- 70° V hangers only 15 W- LONG
1H,2B,3-10B	107915	CAA V LNG EXT ASM 16W R	Right- 70° V hangers only 16 W- LONG
1H,2C,3-10B	107916	CAA V LNG EXT ASM 17W R	Right- 70° V hangers only 17 W- LONG
1H,2D,3-10C	107917	CAA V LNG EXT ASM 18W R	Right- 70° V hangers only 18 W- LONG
>>>		70° "V" HANGERS - LONG EXT - LEFT	
1H,2A,3-10A	107921	CAA V LNG EXT ASM 15W L	Left- 70° V hangers only 15 W- LONG
1H,2B,3-10B	107922	CAA V LNG EXT ASM 16W L	Left- 70° V hangers only 16 W- LONG
1H,2C,3-10B	107923	CAA V LNG EXT ASM 17W L	Left- 70° V hangers only 17 W- LONG
1H,2D,3-10C	107924	CAA V LNG EXT ASM 18W L	Left- 70° V hangers only 18 W- LONG
>>>		80° HANGERS - SHORT EXT - RIGHT	
1A,2A,3-10A	107816	CAA STRT SHT EXT ASM 15W R	Right- 80° 15 W- SHORT
1A,2B,3-10B	107817	CAA STRT SHT EXT ASM 16W R	Right- 80° 16 W- SHORT
1A,2C,3-10B	107818	CAA STRT SHT EXT ASM 17W R	Right- 80° 17 W- SHORT
1A,2D,3-10C	107819	CAA STRT SHT EXT ASM 18W R	Right- 80° 18 W- SHORT
1A,2E,3-10C	107820	CAA STRT SHT EXT ASM 19W R	Right- 80° 19 W- SHORT
1A,2F,3-10D	107821	CAA STRT SHT EXT ASM 20W R	Right- 80° 20 W- SHORT
1A,2G,3-10D	107822	CAA STRT SHT EXT ASM 21&22W R	Right- 80° 21&22 W- SHORT
>>>		80° HANGERS - SHORT EXT - LEFT	
1A,2A,3-10A	107823	CAA STRT SHT EXT ASM 15W L	Left- 80° 15 W- SHORT
1A,2B,3-10B	107824	CAA STRT SHT EXT ASM 16W L	Left- 80° 16 W- SHORT
1A,2C,3-10B	107825	CAA STRT SHT EXT ASM 17W L	Left- 80° 17 W- SHORT
1A,2D,3-10C	107826	CAA STRT SHT EXT ASM 18W L	Left- 80° 18 W- SHORT
1A,2E,3-10C	107827	CAA STRT SHT EXT ASM 19W L	Left- 80° 19 W- SHORT
1A,2F,3-10D	107828	CAA STRT SHT EXT ASM 20W L	Left- 80° 20 W- SHORT
1A,2G,3-10D	107829	CAA STRT SHT EXT ASM 21&22W L	Left- 80° 21&22 W- SHORT
>>>		80° HANGERS - MED EXT - RIGHT	
1B,2A,3-10A	107830	CAA STRT MED EXT ASM 15W R	Right- 80° 15 W- MED
1B,2B,3-10B	107831	CAA STRT MED EXT ASM 16W R	Right- 80° 16 W- MED
1B,2C,3-10B	107832	CAA STRT MED EXT ASM 17W R	Right- 80° 17 W- MED


<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
1B,2D,3-10C	107833	CAA STRT MED EXT ASM 18W R	Right- 80° 18 W- MED
1B,2E,3-10C	107834	CAA STRT MED EXT ASM 19W R	Right- 80° 19 W- MED
1B,2F,3-10D	107835	CAA STRT MED EXT ASM 20W R	Right- 80° 20 W- MED
1B,2G,3-10D	107836	CAA STRT MED EXT ASM 21&22W R	Right- 80° 21&22 W- MED
>>> 80° HANGERS - MED EXT - LEFT			
1B,2A,3-10A	107837	CAA STRT MED EXT ASM 15W L	Left- 80° 15 W- MED
1B,2B,3-10B	107838	CAA STRT MED EXT ASM 16W L	Left- 80° 16 W- MED
1B,2C,3-10B	107839	CAA STRT MED EXT ASM 17W L	Left- 80° 17 W- MED
1B,2D,3-10C	107840	CAA STRT MED EXT ASM 18W L	Left- 80° 18 W- MED
1B,2E,3-10C	107841	CAA STRT MED EXT ASM 19W L	Left- 80° 19 W- MED
1B,2F,3-10D	107842	CAA STRT MED EXT ASM 20W L	Left- 80° 20 W- MED
1B,2G,3-10D	107843	CAA STRT MED EXT ASM 21&22W L	Left- 80° 21&22 W- MED
>>> 80° HANGERS - LONG EXT - RIGHT			
1C,2A,3-10A	107844	CAA STRT LNG EXT ASM 15W R	Right- 80° 15 W- LONG
1C,2B,3-10B	107845	CAA STRT LNG EXT ASM 16W R	Right- 80° 16 W- LONG
1C,2C,3-10B	107846	CAA STRT LNG EXT ASM 17W R	Right- 80° 17 W- LONG
1C,2D,3-10C	107847	CAA STRT LNG EXT ASM 18W R	Right- 80° 18 W- LONG
1C,2E,3-10C	107848	CAA STRT LNG EXT ASM 19W R	Right- 80° 19 W- LONG
1C,2F,3-10D	107849	CAA STRT LNG EXT ASM 20W R	Right- 80° 20 W- LONG
1C,2G,3-10D	107850	CAA STRT LNG EXT ASM 21&22W R	Right- 80° 21&22 W- LONG
>>> 80° HANGERS - LONG EXT - LEFT			
1C,2A,3-10A	107851	CAA STRT LNG EXT ASM 15W L	Left- 80° 15 W- LONG
1C,2B,3-10B	107852	CAA STRT LNG EXT ASM 16W L	Left- 80° 16 W- LONG
1C,2C,3-10B	107853	CAA STRT LNG EXT ASM 17W L	Left- 80° 17 W- LONG
1C,2D,3-10C	107854	CAA STRT LNG EXT ASM 18W L	Left- 80° 18 W- LONG
1C,2E,3-10C	107855	CAA STRT LNG EXT ASM 19W L	Left- 80° 19 W- LONG
1C,2F,3-10D	107856	CAA STRT LNG EXT ASM 20W L	Left- 80° 20 W- LONG
1C,2G,3-10D	107857	CAA STRT LNG EXT ASM 21&22W L	Left- 80° 21&22 W- LONG

## KIDS ANGLE ADJ FOOTPLATE EXTENSION MOUNT

[11/2009]


**NOTE:** FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.

FOOTPLATE FORMULA	
0"	Inset (Seat Width) = Footrest Width
1"	Inset (Seat Width) -2 = Footrest Width
V Hanger (Seat Width)	-4 = Footrest Width

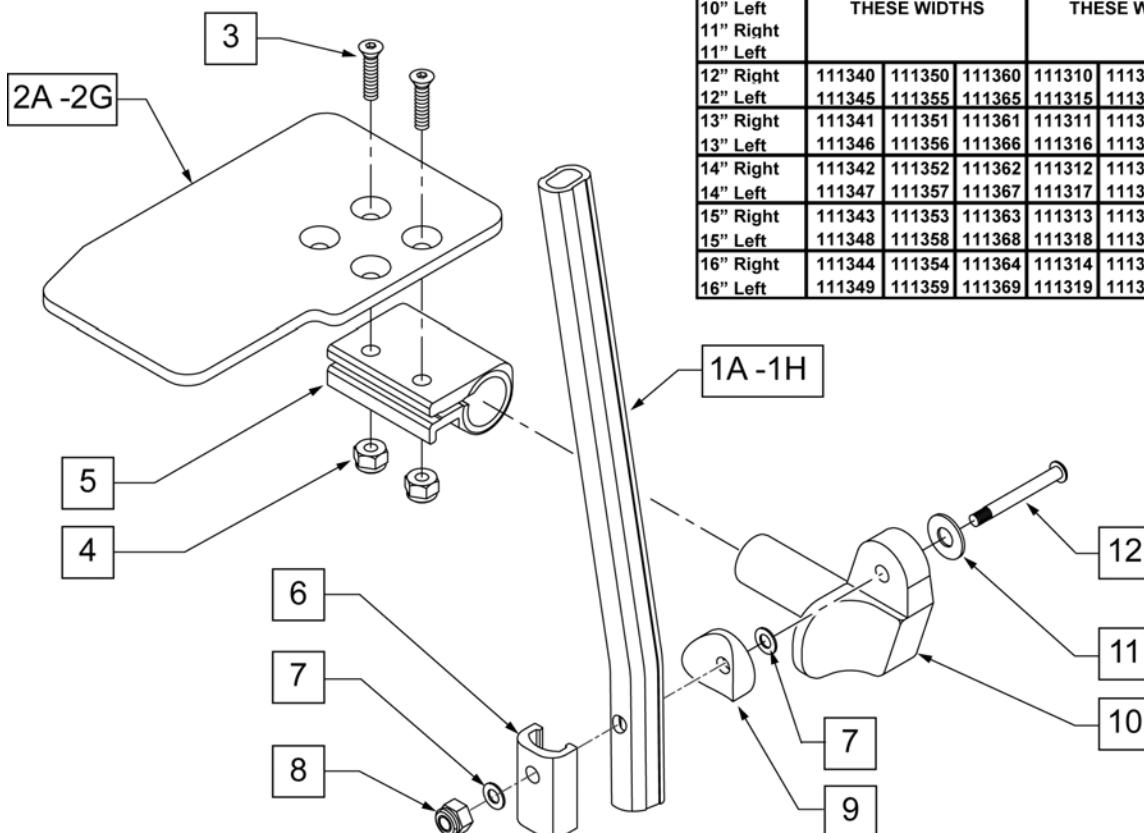

Kids Angle Adjustable "KAA" Footplates w/ Ext. for Swing I/O Hangers—leg strap not included (Order 1 each)									
Ext. Tube Width	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger (Straight Extension)			70° V Style Hangers		
	Short	Med	Long	Short	Med	Long	Med	Long	
10" Right	NOT AVAILABLE IN THESE WIDTHS			NOT AVAILABLE IN THESE WIDTHS			111370	111384	
10" Left							111377	111391	
11" Right							111371	111385	
11" Left							111378	111392	
12" Right	111340	111350	111360	111310	111320	111330	111372	111386	
12" Left	111345	111355	111365	111315	111325	111335	111379	111393	
13" Right	111341	111351	111361	111311	111321	111331	111373	111387	
13" Left	111346	111356	111366	111316	111326	111336	111380	111394	
14" Right	111342	111352	111362	111312	111322	111332	111374	111388	
14" Left	111347	111357	111367	111317	111327	111337	111381	111395	
15" Right	111343	111353	111363	111313	111323	111333	111375	111389	
15" Left	111348	111358	111368	111318	111328	111338	111382	111396	
16" Right	111344	111354	111364	111314	111324	111334	111376	111390	
16" Left	111349	111359	111369	111319	111329	111339	111383	111397	

### MEASURING GUIDE

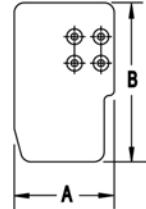

CHAIR WIDTH	"A"	"B"
10" W	3.55"	6.00"
11" W	4.05"	6.00"
12" W	4.55"	6.00"
13" W	5.05"	6.00"
14" W	5.55"	6.00"
15" W	6.05"	6.00"
16" W	6.55"	6.00"

Pos.	Item Number	Description	Remarks
1A	107119	TUBE FOOTREST EXT STRAIGHT SHORT	80 deg hangers only
1B	107120	TUBE FOOTREST EXT STRAIGHT MED	80 deg hangers only
1C	107121	TUBE FOOTREST EXT STRAIGHT LONG	80 deg hangers only
1D	107126	TUBE FOOTREST EXT SHORT	60,70,70 Hemi Hangers
1E	107127	TUBE FOOTREST EXT MED	60,70,70 Hemi Hangers
1F	107128	TUBE FOOTREST EXT LONG	60,70,70 Hemi Hangers
1G	106907	EXT TUBE V ONLY D2 MED LEFT	70 deg V hangers only-LEFT
1G	106908	EXT TUBE V ONLY D2 MED RIGHT	70 deg V hangers only-RIGHT
1H	106909	EXT TUBE V ONLY D2 X-LONG LEFT	70 deg V hangers only-LEFT
1H	106910	EXT TUBE V ONLY D2 X-LONG RIGHT	70 deg V hangers only-RIGHT
2A	500635	KIDS ANGL ADJ F/PLT 10" R	RIGHT -10" Footrest Width
2A	500636	KIDS ANGL ADJ F/PLT 10" L	LEFT -10" Footrest Width
2B	500637	KIDS ANGL ADJ F/PLT 11" R	RIGHT -11" Footrest Width

Pos.	Item Number	Description	Remarks
2B	500638	KIDS ANGL ADJ F/PLT 11" L	LEFT -11" Footrest Width
2C	500639	KIDS ANGL ADJ F/PLT 12" R	RIGHT -12" Footrest Width
2C	500640	KIDS ANGL ADJ F/PLT 12" L	LEFT -12" Footrest Width
2D	500641	KIDS ANGL ADJ F/PLT 13" R	RIGHT -13" Footrest Width
2D	500642	KIDS ANGL ADJ F/PLT 13" L	LEFT -13" Footrest Width
2E	500643	KIDS ANGL ADJ F/PLT 14" R	RIGHT -14" Footrest Width
2E	500644	KIDS ANGL ADJ F/PLT 14" L	LEFT -14" Footrest Width
2F	500645	KIDS ANGL ADJ F/PLT 15" R	RIGHT -15" Footrest Width
2F	500646	KIDS ANGL ADJ F/PLT 15" L	LEFT -15" Footrest Width
2G	500647	KIDS ANGL ADJ F/PLT 16" R	RIGHT -16" Footrest Width
2G	500648	KIDS ANGL ADJ F/PLT 16" L	LEFT -16" Footrest Width
3	106978	FHCS 1/4 -28 x 7/8 ROHS	
4	106935	NUT NYLOCK THIN 1/4-28 ROHS	
5	550350	3/4" CLAMP ADJ FT PLT	
6	107199	HANGER EXTENSION SADDLE	
7	106944	WASHER FLAT THIN AN 1/4 ROHS	
8	104381	NUT HEX NYLOCK M6 x 1.0	
9	104377	3/4" SADDLE THIN	
10	541002	FOOTREST MOUNTING BRKT 3/8 RAD R	RIGHT
10	541003	FOOTREST MOUNTING BRKT 3/8 RAD L	LEFT
11	106945	WASHER FLAT SAE 1/4 ROHS	
12	104399	SHCS M6 x 1 x 45 12mm PART THD	
1A,2C,3-12	111310	KAA STRT SHT EXT ASM 12W R	RIGHT- 80D- 12 W- SHORT
1A,2D,3-12	111311	KAA STRT SHT EXT ASM 13W R	RIGHT- 80D- 13 W- SHORT
1A,2E,3-12	111312	KAA STRT SHT EXT ASM 14W R	RIGHT- 80D- 14 W- SHORT
1A,2F,3-12	111313	KAA STRT SHT EXT ASM 15W R	RIGHT- 80D- 15 W- SHORT
1A,2G,3-12	111314	KAA STRT SHT EXT ASM 16W R	RIGHT- 80D- 16 W- SHORT
1A,2C,3-12	111315	KAA STRT SHT EXT ASM 12W L	LEFT- 80D- 12 W- SHORT
1A,2D,3-12	111316	KAA STRT SHT EXT ASM 13W L	LEFT- 80D- 13 W- SHORT
1A,2E,3-12	111317	KAA STRT SHT EXT ASM 14W L	LEFT- 80D- 14 W- SHORT
1A,2F,3-12	111318	KAA STRT SHT EXT ASM 15W L	LEFT- 80D- 15 W- SHORT
1A,2G,3-12	111319	KAA STRT SHT EXT ASM 16W L	LEFT- 80D- 16 W- SHORT
1B,2C,3-12	111320	KAA STRT MED EXT ASM 12W R	RIGHT- 80D- 12 W- MED
1B,2D,3-12	111321	KAA STRT MED EXT ASM 13W R	RIGHT- 80D- 13 W- MED
1B,2E,3-12	111322	KAA STRT MED EXT ASM 14W R	RIGHT- 80D- 14 W- MED
1B,2F,3-12	111323	KAA STRT MED EXT ASM 15W R	RIGHT- 80D- 15 W- MED
1B,2G,3-12	111324	KAA STRT MED EXT ASM 16W R	RIGHT- 80D- 16 W- MED
1B,2C,3-12	111325	KAA STRT MED EXT ASM 12W L	LEFT- 80D- 12 W- MED
1B,2D,3-12	111326	KAA STRT MED EXT ASM 13W L	LEFT- 80D- 13 W- MED
1B,2E,3-12	111327	KAA STRT MED EXT ASM 14W L	LEFT- 80D- 14 W- MED
1B,2F,3-12	111328	KAA STRT MED EXT ASM 15W L	LEFT- 80D- 15 W- MED
1B,2G,3-12	111329	KAA STRT MED EXT ASM 16W L	LEFT- 80D- 16 W- MED
1C,2C,3-12	111330	KAA STRT LNG EXT ASM 12W R	RIGHT- 80D- 12 W- LONG
1C,2D,3-12	111331	KAA STRT LNG EXT ASM 13W R	RIGHT- 80D- 13 W- LONG
1C,2E,3-12	111332	KAA STRT LNG EXT ASM 14W R	RIGHT- 80D- 14 W- LONG
1C,2F,3-12	111333	KAA STRT LNG EXT ASM 15W R	RIGHT- 80D- 15 W- LONG
1C,2G,3-12	111334	KAA STRT LNG EXT ASM 16W R	RIGHT- 80D- 16 W- LONG
1C,2C,3-12	111335	KAA STRT LNG EXT ASM 12W L	LEFT- 80D- 12 W- LONG
1C,2D,3-12	111336	KAA STRT LNG EXT ASM 13W L	LEFT- 80D- 13 W- LONG
1C,2E,3-12	111337	KAA STRT LNG EXT ASM 14W L	LEFT- 80D- 14 W- LONG
1C,2F,3-12	111338	KAA STRT LNG EXT ASM 15W L	LEFT- 80D- 15 W- LONG
1C,2G,3-12	111339	KAA STRT LNG EXT ASM 16W L	LEFT- 80D- 16 W- LONG
1D,2C,3-12	111340	KAA BNT SHT EXT ASM 12W R	RIGHT- 60 & 70D- 12 W- SHORT
1D,2D,3-12	111341	KAA BNT SHT EXT ASM 13W R	RIGHT- 60 & 70D- 13 W- SHORT
1D,2E,3-12	111342	KAA BNT SHT EXT ASM 14W R	RIGHT- 60 & 70D- 14 W- SHORT
1D,2F,3-12	111343	KAA BNT SHT EXT ASM 15W R	RIGHT- 60 & 70D- 15 W- SHORT

Pos.	Item Number	Description	Remarks
1D,2G,3-12	111344	KAA BNT SHT EXT ASM 16W R	RIGHT- 60 & 70D- 16 W- SHORT
1D,2C,3-12	111345	KAA BNT SHT EXT ASM 12W L	LEFT- 70D- 12 W- SHORT
1D,2D,3-12	111346	KAA BNT SHT EXT ASM 13W L	LEFT- 70D- 13 W- SHORT
1D,2E,3-12	111347	KAA BNT SHT EXT ASM 14W L	LEFT- 70D- 14 W- SHORT
1D,2F,3-12	111348	KAA BNT SHT EXT ASM 15W L	LEFT- 70D- 15 W- SHORT
1D,2G,3-12	111349	KAA BNT SHT EXT ASM 16W L	LEFT- 70D- 16 W- SHORT
1E,2C,3-12	111350	KAA BNT MED EXT ASM 12W R	RIGHT- 70D- 12 W- MED
1E,2D,3-12	111351	KAA BNT MED EXT ASM 13W R	RIGHT- 70D- 13 W- MED
1E,2E,3-12	111352	KAA BNT MED EXT ASM 14W R	RIGHT- 60 & 70D- 14 W- MED
1E,2F,3-12	111353	KAA BNT MED EXT ASM 15W R	RIGHT- 60 & 70D- 15 W- MED
1E,2G,3-12	111354	KAA BNT MED EXT ASM 16W R	RIGHT- 60 & 70D- 16 W- MED
1E,2C,3-12	111355	KAA BNT MED EXT ASM 12W L	LEFT- 60 & 70D- 12 W- MED
1E,2D,3-12	111356	KAA BNT MED EXT ASM 13W L	LEFT- 60 & 70D- 13 W- MED
1E,2E,3-12	111357	KAA BNT MED EXT ASM 14W L	LEFT- 60 & 70D- 14 W- MED
1E,2F,3-12	111358	KAA BNT MED EXT ASM 15W L	LEFT- 60 & 70D- 15 W- MED
1E,2G,3-12	111359	KAA BNT MED EXT ASM 16W L	LEFT- 60 & 70D- 16 W- MED
1F,2C,3-12	111360	KAA BNT LNG EXT ASM 12W R	RIGHT- 60 & 70D- 12 W- LONG
1F,2D,3-12	111361	KAA BNT LNG EXT ASM 13W R	RIGHT- 60 & 70D- 13 W- LONG
1F,2E,3-12	111362	KAA BNT LNG EXT ASM 14W R	RIGHT- 60 & 70D- 14 W- LONG
1F,2F,3-12	111363	KAA BNT LNG EXT ASM 15W R	RIGHT- 60 & 70D- 15 W- LONG
1F,2G,3-12	111364	KAA BNT LNG EXT ASM 16W R	RIGHT- 60 & 70D- 16 W- LONG
1F,2C,3-12	111365	KAA BNT LNG EXT ASM 12W L	LEFT- 60 & 70D- 12 W- LONG
1F,2D,3-12	111366	KAA BNT LNG EXT ASM 13W L	LEFT- 60 & 70D- 13 W- LONG
1F,2E,3-12	111367	KAA BNT LNG EXT ASM 14W L	LEFT- 60 & 70D- 14 W- LONG
1F,2F,3-12	111368	KAA BNT LNG EXT ASM 15W L	LEFT- 60 & 70D- 15 W- LONG
1F,2G,3-12	111369	KAA BNT LNG EXT ASM 16W L	LEFT- 60 & 70D- 16 W- LONG
1G,2A,3-12	111370	KAA V MED EXT ASM 10W R	RIGHT- 70D V hangers only 10 W- MED
1G,2B,3-12	111371	KAA V MED EXT ASM 11W R	RIGHT- 70D V hangers only 11 W- MED
1G,2C,3-12	111372	KAA V MED EXT ASM 12W R	RIGHT- 70D V hangers only 12 W- MED
1G,2D,3-12	111373	KAA V MED EXT ASM 13W R	RIGHT- 70D V hangers only 13 W- MED
1G,2E,3-12	111374	KAA V MED EXT ASM 14W R	RIGHT- 70D V hangers only 14 W- MED
1G,2F,3-12	111375	KAA V MED EXT ASM 15W R	RIGHT- 70D V hangers only 15 W- MED
1G,2G,3-12	111376	KAA V MED EXT ASM 16W R	RIGHT- 70D V hangers only 16 W- MED
1G,2A,3-12	111377	KAA V MED EXT ASM 10W L	LEFT- 70D V hangers only 10 W- MED
1G,2B,3-12	111378	KAA V MED EXT ASM 11W L	LEFT- 70D V hangers only 11 W- MED
1G,2C,3-12	111379	KAA V MED EXT ASM 12W L	LEFT- 70D V hangers only 12 W- MED
1G,2D,3-12	111380	KAA V MED EXT ASM 13W L	LEFT- 70D V hangers only 13 W- MED
1G,2E,3-12	111381	KAA V MED EXT ASM 14W L	LEFT- 70D V hangers only 14 W- MED
1G,2F,3-12	111382	KAA V MED EXT ASM 15W L	LEFT- 70D V hangers only 15 W- MED
1G,2G,3-12	111383	KAA V MED EXT ASM 16W L	LEFT- 70D V hangers only 16 W- MED
1H,2A,3-12	111384	KAA V LNG EXT ASM 10W R	RIGHT- 70D V hangers only 10 W- LONG
1H,2B,3-12	111385	KAA V LNG EXT ASM 11W R	RIGHT- 70D V hangers only 11 W- LONG
1H,2C,3-12	111386	KAA V LNG EXT ASM 12W R	RIGHT- 70D V hangers only 12 W- LONG
1H,2D,3-12	111387	KAA V LNG EXT ASM 13W R	RIGHT- 70D V hangers only 13 W- LONG
1H,2E,3-12	111388	KAA V LNG EXT ASM 14W R	RIGHT- 70D V hangers only 14 W- LONG
1H,2F,3-12	111389	KAA V LNG EXT ASM 15W R	RIGHT- 70D V hangers only 15 W- LONG
1H,2G,3-12	111390	KAA V LNG EXT ASM 16W R	RIGHT- 70D V hangers only 16 W- LONG
1H,2A,3-12	111391	KAA V LNG EXT ASM 10W L	LEFT- 70D V hangers only 10 W- LONG
1H,2B,3-12	111392	KAA V LNG EXT ASM 11W L	LEFT- 70D V hangers only 11 W- LONG
1H,2C,3-12	111393	KAA V LNG EXT ASM 12W L	LEFT- 70D V hangers only 12 W- LONG
1H,2D,3-12	111394	KAA V LNG EXT ASM 13W L	LEFT- 70D V hangers only 13 W- LONG
1H,2E,3-12	111395	KAA V LNG EXT ASM 14W L	LEFT- 70D V hangers only 14 W- LONG
1H,2F,3-12	111396	KAA V LNG EXT ASM 15W L	LEFT- 70D V hangers only 15 W- LONG
1H,2G,3-12	111397	KAA V LNG EXT ASM 16W L	LEFT- 70D V hangers only 16 W- LONG

[10/2011]


## ADULT ANGLE-ADJ FOOTPLATE EXT MOUNT

NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.


Adult Angle Adjustable "AAA" Footplates w/ Ext & HL for Swing I/O Hangers									
Ext. Tube	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger (Straight Extension)			70° V Style Hangers		
	(Order 1 each)			Short	Med	Long	Short	Med	Long
Width	Short	Med	Long	Short	Med	Long	Med	Long	
10"-11" Rt	NOT AVAILABLE IN THIS WIDTH			NOT AVAILABLE IN THIS WIDTH			111546	111550	
10"-11" Lt							111547	111551	
12"-13" Rt	111540	111542	111544	111534	111536	111538	111548	111552	
12"-13" Lt	111541	111543	111545	111535	111537	111539	111549	111553	
14"-15" Rt	107973	107987	108001	107928	107942	107956	108015	108029	
14"-15" Lt	107980	107994	108008	107935	107949	107963	108022	108036	
16"-17" Rt	107975	107989	108003	107930	107944	107958	108017	108031	
16"-17" Lt	107982	107996	108010	107937	107951	107965	108024	108038	
18"-22" Rt	107977	107991	108005	107932	107946	107960	108019	108033	
18"-22" Lt	107984	107998	108012	107939	107953	107968	108026	108040	

**INSET FORMULA**  
 0" Inset (Seat Width) = Footrest Width  
 1" Inset (Seat Width) -2 = Footrest Width  
 "V" Hanger (Seat Width) -4 = Footrest Width

## MEASURING GUIDE

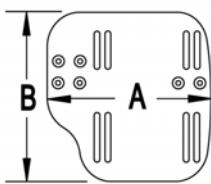

CHAIR WIDTH	"A"	"B"
10" - 11"	3 1/2"	7 1/4"
12" - 13"	4 1/2"	7 3/4"
14" - 15"	5 1/2"	7 3/4"
16" - 17"	6 1/2"	7 3/4"
18" - 20"	7 1/2"	7 3/4"

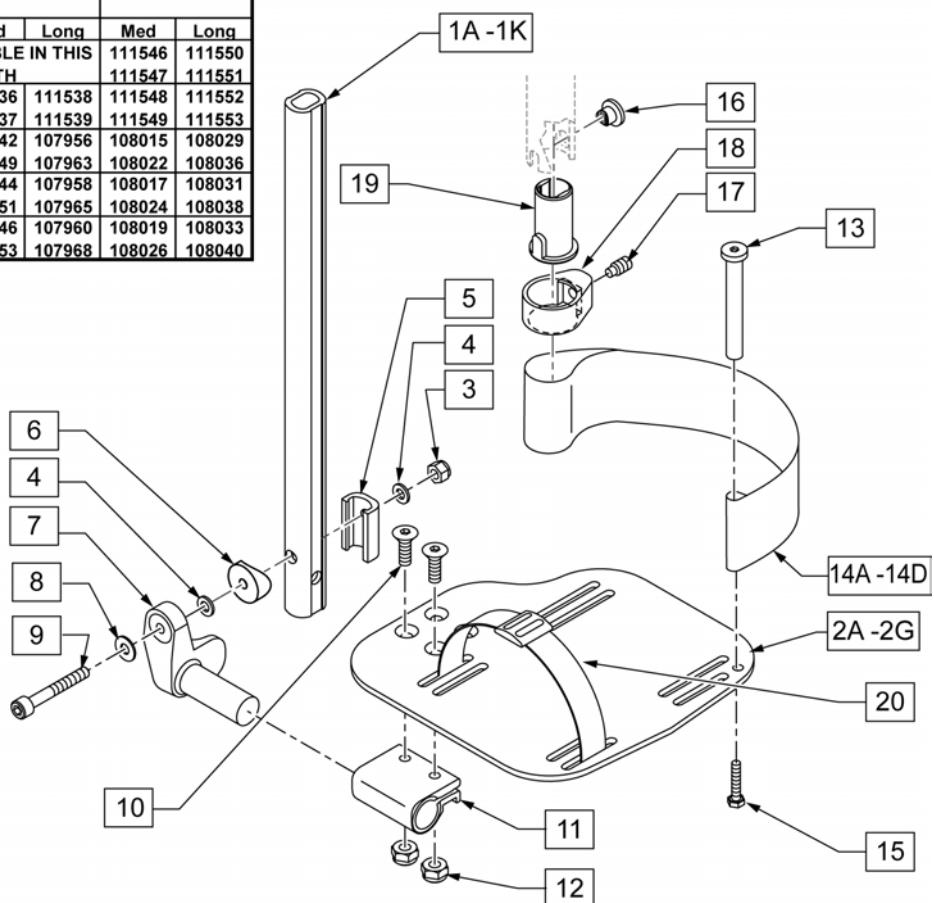


Pos.	Item Number	Description	Remarks
1A,3-5,8,9	120670	SHT STRT EXT AAA HDWR	80 deg Hangers Only
1B,3-5,8,9	120671	MED STRT EXT AAA HDWR	80 deg Hangers Only
1C,3-5,8,9	120672	LNG STRT EXT AAA HDWR	80 deg Hangers Only
1D,3-5,8,9	120673	SHT BNT EXT AAA HDWR	60,70,70 Hemi Hangers
1E,3-5,8,9	120674	MED BNT EXT AAA HDWR	60,70,70 Hemi Hangers
1F,3-5,8,9	120675	LNG BNT EXT AAA HDWR	60,70,70 Hemi Hangers
1G,3-5,8,9	120676	SHT V EXT R AAA HDWR	Left - 70 deg V Hangers Only
1H,3-5,8,9	120677	SHT V EXT L AAA HDWR	Right - 70 deg V Hangers Only
1I,3-5,8,9	120678	LNG V EXT R AAA HDWR	Left - 70 deg V Hangers Only
1J,3-5,8,9	120679	LNG V EXT L AAA HDWR	Right - 70 deg V Hangers Only
2A	500656	ADULT ANG ADJ F/PLT 10/11" R	RIGHT 10-11" Footrest Width
2A	500657	ADULT ANG ADJ F/PLT 10/11" L	LEFT 10-11" Footrest Width
2B	500658	ADULT ANG ADJ F/PLT 12/13" R	RIGHT 12-13" Footrest Width

Pos.	Item Number	Description	Remarks
2B	500659	ADULT ANG ADJ F/PLT 12/13" L	LEFT 12-13" Footrest Width
2C	500660	ADLT ANGL ADJ F/PLT 14/15	RIGHT 14-15" Footrest Width
2C	500661	ADLT ANGL ADJ F/PLT 14/15	LEFT 14-15" Footrest Width
2D	500662	ADLT ANGL ADJ F/PLT 16/17	RIGHT 16-17" Footrest Width
2D	500663	ADLT ANGL ADJ F/PLT 16/17	LEFT 16-17" Footrest Width
2E	500664	ADLT ANGL ADJ F/PLT 18-20	RIGHT 18-22" Footrest Width
2E	500665	ADLT ANGL ADJ F/PLT 18-20	LEFT 18-22" Footrest Width
3	104381	NUT HEX NYLOCK M6X1.0	
4	106944	WASHER FLAT THIN AN 1/4 ROHS	
5	107199	HANGER EXTENSION SADDLE	
6	104377	3/4" SADDLE THIN	
7	541002	FTRST MOUNTING BRKT-3/8 RAD R	RIGHT
7	541003	FTRST MOUNTING BRKT-3/8 RAD L	LEFT
8	106945	WASHER FLAT SAE 1/4 ROHS	
9	104399	SHCS M6X1X45 12 MM PART THD	
10	106978	FHCS 1/4-28 X 7/8 ROHS	
11	550350	3/4" CLMP ADJ FT PLT	
12	106935	NUT NYLOCK THIN 1/4-28 ROHS	
13	104328	POST HEEL LOOP	
14A	104380-001	ADJUSTABLE FOOT REST STRAP	12 - 14" WIDE
14B	104380-002	ADJUSTABLE FOOT REST STRAP	15 - 16" WIDE
14C	104380-003	ADJUSTABLE FOOT REST STRAP	17 - 18" WIDE
14D	104380-004	ADJUSTABLE FOOT REST STRAP	19 - 20" WIDE
15	102268-001	HHCS M6 X 1 X 25	
3-6,8,9	120659	EXT MNT HARDWARE 1 SIDE	
16-19	107485	FOOTREST CLAMP ASSM	
20	740260	TOE-ANKLE STRAP ADJ ADULT FOOTPLATE	
>>>		80° HANGERS - SHORT EXT - RIGHT	
1A,2B,3-14A,15	111534	AAA STRT SHT EXT ASM 12-13" W R	Right- 80D- 12-13" W- SHORT
1A,2C,3-14B,15	107928	AAA STRT SHT EXT ASM 14-15" W R	Right- 80D- 14-15" W- SHORT
1A,2D,3-14C,15	107930	AAA STRT SHT EXT ASM 16-17" W R	Right- 80D- 16-17" W- SHORT
1A,2E,3-14D,15	107932	AAA STRT SHT EXT ASM 18-22" W R	Right- 80D- 18-22" W- SHORT
>>>		80° HANGERS - SHORT EXT - LEFT	
1A,2B,3-14A,15	111535	AAA STRT SHT EXT ASM 12-13" W L	Left- 80D- 12-13" W- SHORT
1A,2C,3-14B,15	107935	AAA STRT SHT EXT ASM 14" W L	Left- 80D- 14-15" W- SHORT
1A,2D,3-14C,15	107937	AAA STRT SHT EXT ASM 16" W L	Left- 80D- 16-17" W- SHORT
1A,2E,3-14D,15	107939	AAA STRT SHT EXT ASM 18" W L	Left- 80D- 18-22" W- SHORT
>>>		80° HANGERS - MED EXT - RIGHT	
1B,2B,3-14A,15	111536	AAA STRT MED EXT ASM 12-13" W R	Right- 80D- 12-13" W- MED
1B,2C,3-14B,15	107942	AAA STRT MED EXT ASM 14-15" W R	Right- 80D- 14-15" W- MED
1B,2D,3-14C,15	107944	AAA STRT MED EXT ASM 16-17" W R	Right- 80D- 16-17" W- MED
1B,2E,3-14D,15	107946	AAA STRT MED EXT ASM 18-22" W R	Right- 80D- 18-22" W- MED
>>>		80° HANGERS - MED EXT - LEFT	
1B,2B,3-14A,15	111537	AAA STRT MED EXT ASM 12-13" W L	Left- 80D- 12-13" W- MED
1B,2C,3-14B,15	107949	AAA STRT MED EXT ASM 14" W L	Left- 80D- 14-15" W- MED
1B,2D,3-14C,15	107951	AAA STRT MED EXT ASM 16" W L	Left- 80D- 16-17" W- MED
1B,2E,3-14D,15	107953	AAA STRT MED EXT ASM 18" W L	Left- 80D- 18-22" W- MED
>>>		80° HANGERS - LONG EXT - RIGHT	
1C,2B,3-14A,15	111538	AAA STRT LNG EXT ASM 12-13" W R	Right- 80D- 12-13" W- LONG
1C,2C,3-14B,15	107956	AAA STRT LNG EXT ASM 14-15" W R	Right- 80D- 14-15" W- LONG
1C,2D,3-14C,15	107958	AAA STRT LNG EXT ASM 16-17" W R	Right- 80D- 16-17" W- LONG
1C,2E,3-14D,15	107960	AAA STRT LNG EXT ASM 18-22" W R	Right- 80D- 18-22" W- LONG
>>>		80° HANGERS - LONG EXT - LEFT	
1C,2B,3-14A,15	111539	AAA STRT LNG EXT ASM 12-13" W L	Left- 80D- 12-13" W- LONG
1C,2C,3-14B,15	107963	AAA STRT LNG EXT ASM 14W L	Left- 80D- 14-15" W- LONG
1C,2D,3-14C,15	107965	AAA STRT LNG EXT ASM 16W L	Left- 80D- 16-17" W- LONG

Pos.	Item Number	Description	Remarks
1C,2E,3-14D,15	107968	AAA STRT LNG EXT ASM 18W L	Left- 80D- 18-22" W- LONG
>>>		70° HANGERS - SHORT EXT - RIGHT	
1D,2B,3-14A,15	111540	AAA BNT SHT EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- SHORT
1D,2C,3-14B,15	107973	AAA BNT SHT EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- SHORT
1D,2D,3-14C,15	107975	AAA BNT SHT EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- SHORT
1D,2E,3-14D,15	107977	AAA BNT SHT EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- SHORT
>>>		70° HANGERS - SHORT EXT - LEFT	
1D,2B,3-14A,15	111541	AAA BNT SHT EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- SHORT
1D,2C,3-14B,15	107980	AAA BNT SHT EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- SHORT
1D,2D,3-14C,15	107982	AAA BNT SHT EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- SHORT
1D,2E,3-14D,15	107984	AAA BNT SHT EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- SHORT
>>>		70° HANGERS - MED EXT - RIGHT	
1E,2B,3-14A,15	111542	AAA BNT MED EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- MED
1E,2C,3-14B,15	107987	AAA BNT MED EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- MED
1E,2D,3-14C,15	107989	AAA BNT MED EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- MED
1E,2E,3-14D,15	107991	AAA BNT MED EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- MED
>>>		70° HANGERS - MED EXT - LEFT	
1E,2B,3-14A,15	111543	AAA BNT MED EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- MED
1E,2C,3-14B,15	107994	AAA BNT MED EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- MED
1E,2D,3-14C,15	107996	AAA BNT MED EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- MED
1E,2E,3-14D,15	107998	AAA BNT MED EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- MED
>>>		70° HANGERS - LONG EXT - RIGHT	
1F,2B,3-14A,15	111544	AAA BNT LONG EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- LONG
1F,2C,3-14B,15	108001	AAA BNT LONG EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- LONG
1F,2D,3-14C,15	108003	AAA BNT LONG EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- LONG
1F,2E,3-14D,15	108005	AAA BNT LONG EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- LONG
>>>		70° HANGERS - LONG EXT - LEFT	
1F,2B,3-14A,15	111545	AAA BNT LONG EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- LONG
1F,2C,3-14B,15	108008	AAA BNT LONG EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- LONG
1F,2D,3-14C,15	108010	AAA BNT LONG EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- LONG
1F,2E,3-14D,15	108012	AAA BNT LONG EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- LONG
>>>		70° "V" HANGERS - MED EXT - RIGHT	
1G,2A,3-15	111546	AAA V MED EXT ASM 10-11" W R	Right- 70D V hangers only 10-11" W- MED
1G,2B,3-15	111548	AAA V MED EXT ASM 12-13" W R	Right- 70D V hangers only 12-13" W- MED
1G,2C,3-15	108015	AAA V MED EXT ASM 14 -15" W R	Right- 70D V hangers only 14-15" W- MED
1G,2D,3-15	108017	AAA V MED EXT ASM 16 -17" W R	Right- 70D V hangers only 16-18" W- MED
1G,2E,3-15	108019	AAA V MED EXT ASM 18 -18" W R	Right- 70D V hangers only 18-22" W- MED
>>>		70° "V" HANGERS - MED EXT - LEFT	
1G,2A,3-15	111547	AAA V MED EXT ASM 10-11" W L	Left- 70D V hangers only 10-11" W- MED
1G,2B,3-15	111549	AAA V MED EXT ASM 12-13" W L	Left- 70D V hangers only 12-13" W- MED
1G,2C,3-15	108022	AAA V MED EXT ASM 14 -15" W L	Left- 70D V hangers only 14-15" W- MED
1G,2D,3-15	108024	AAA V MED EXT ASM 16 -17" W L	Left- 70D V hangers only 16-18" W- MED
1G,2E,3-15	108026	AAA V MED EXT ASM 18 -22: W L	Left- 70D V hangers only 18-22" W- MED
>>>		70° "V" HANGERS - LONG EXT - RIGHT	
1H,2A,3-15	111550	AAA V LONG EXT ASM 10-11" W R	Right- 70D V hangers only 10-11" W- LONG
1H,2B,3-15	111552	AAA V LONG EXT ASM 12-13" W R	Right- 70D V hangers only 12-13" W- LONG
1H,2C,3-15	108029	AAA V LONG EXT ASM 14 -15" W R	Right- 70D V hangers only 14-15" W- LONG
1H,2D,3-15	108031	AAA V LONG EXT ASM 16 -17" W R	Right- 70D V hangers only 16-18" W- LONG
1H,2E,3-15	108033	AAA V LONG EXT ASM 18 -22: W R	Right- 70D V hangers only 18-22" W- LONG
>>>		70° "V" HANGERS - LONG EXT - LEFT	
1H,2A,3-15	111551	AAA V LONG EXT ASM 10-11" W L	Left- 70D V hangers only 10-11" W- LONG
1H,2B,3-15	111553	AAA V LONG EXT ASM 12-13" W L	Left- 70D V hangers only 12-13" W- LONG
1H,2C,3-15	108036	AAA V LONG EXT ASM 14 -15" W L	Left- 70D V hangers only 14-15" W- LONG
1H,2D,3-15	108038	AAA V LONG EXT ASM 16 -17" W L	Left- 70D V hangers only 16-18" W- LONG
1H,2E,3-15	108040	AAA V LONG EXT ASM 18 -22" W L	Left- 70D V hangers only 18-22" W- LONG

# PLATFORM FLIP-UP FOOTPLATE

[07/2012]

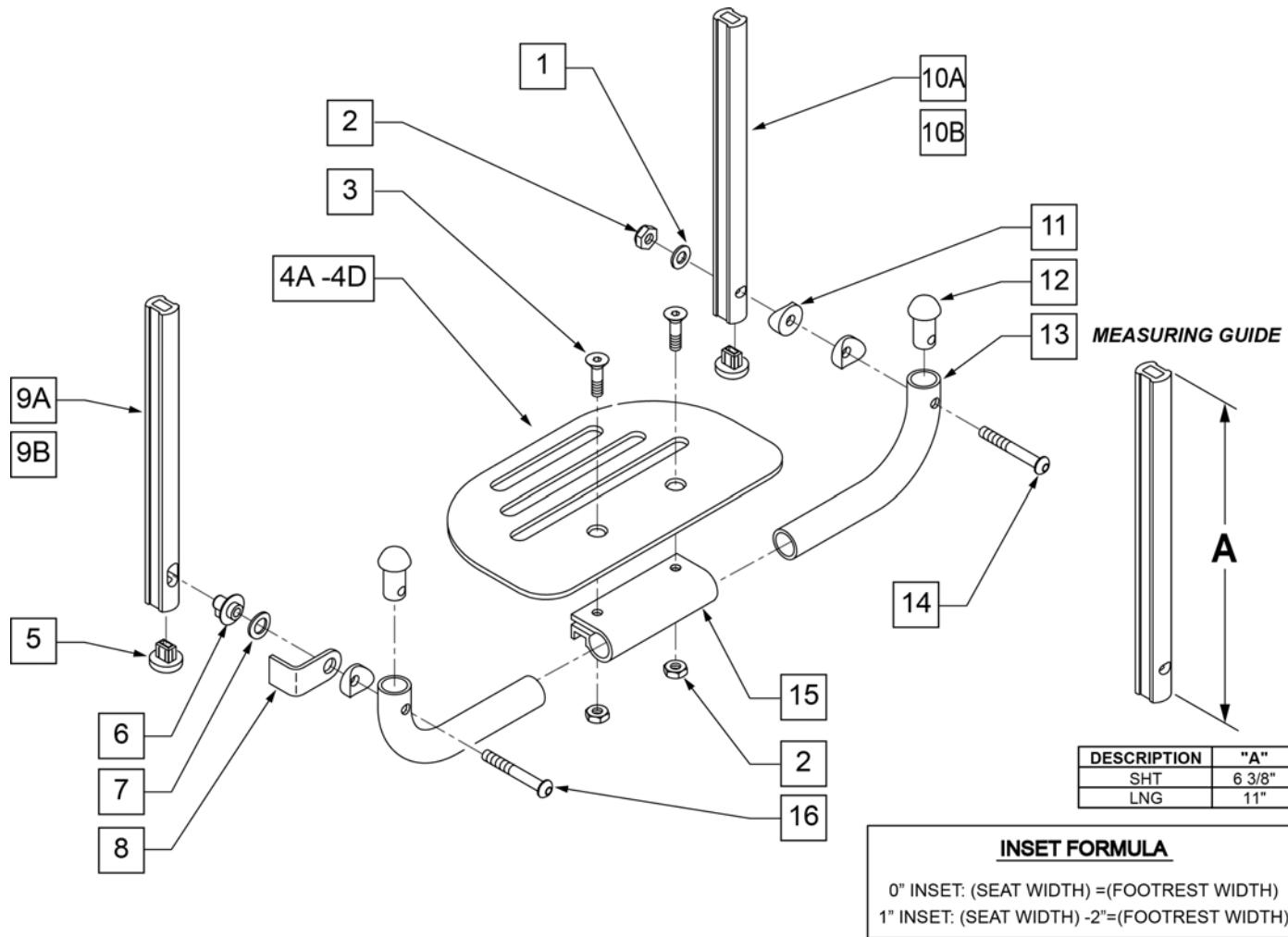


Pos.	Item Number	Description	Remarks
1	106944	WASHER FLAT THIN AN 1/4 ROHS	
2	106941	NUT AUTOMATION 1/4-28 ROHS	
3	106978	FHCS 1/4-28 X 7/8 ROHS	
4A	500651	PLATFORM F/PLT 8 x 5 1/2"	12 -14" Footrest Width
4B	500652	PLATFORM F/PLT 10" x 5 1/2"	15 -17" Footrest Width
4C	500653	PLATFORM F/PLT 12" x 5 1/2"	18 -20" Footrest Width
4D	500654	PLATFORM F/PLT 14" x 5 1/2	21 -22" Footrest Width
5	113221	PLUG DOUBLE D EXT TUBE	
6	540410	F/UP FOOTREST KEY STUD D2	
7	107385	WASHER CURVED 3/8 ROHS	
8	505078	FOOTREST RETAINING BRACKET	
9A	106875	EXT TBE F/UP KEY SHT	Short -Latch Side
9B	106876	EXT TBE F/UP KEY LNG	Long -Latch Side
10A	106873	EXT TBE F/UP SLT SHT	Short -Pivot Side
10B	106874	EXT TBE F/UP SLT LNG	Long -Pivot Side
11	561003	3/4" SADDLE THIN	
12	561034	SHORT DOME PLUG	
14	106964	BHCS 1/4-28 X 2 ROHS	
15	550351	PLATFORM FOOTPLATE CLAMP	


<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
16	107027	BHCS 1/4-28 X 1-1/4 SHLDR ROHS	
17A	740130	LEG STRAP 10"-14" BLK	12-14" Footrest Width
17B	740131	LEG STRAP 15"-17" BLK	15-17" Footrest Width
17C	740132	LEG STRAP 18"-20" BLK	18-20" Footrest Width
1-4A,6-8,11-16	900306	PLTFM F/P PLT BLK 12" D2 W/ INSTR	12" Footrest Width
1-4A,6-8,11-16	900307	PLTFM F/P PLT BLK 13" D2 W/ INSTR	13" Footrest Width
1-4A,6-8,11-16	900308	PLTFM F/P PLT BLK 14" D2 W /INSTR	14" Footrest Width
1-4B,6-8,11-16	900309	PLTFM F/P PLT BLK 15" D2 W /INSTR	15" Footrest Width
1-4B,6-8,11-16	900310	PLTFM F/P PLT BLK 16" D2 W /INSTR	16" Footrest Width
1-4B,6-8,11-16	900311	PLTFM F/P PLT BLK 17" D2 W /INSTR	17" Footrest Width
1-4C,6-8,11-16	900312	PLTFM F/P PLT BLK 18" D2 W /INSTR	18" Footrest Width
1-4C,6-8,11-16	900313	PLTFM F/P PLT BLK 19" D2 W /INSTR	19" Footrest Width
1-4C,6-8,11-16	900314	PLTFM F/P PLT BLK 20" D2 W/ INSTR	20" Footrest Width
1-4D,6-8,11-16	900315	PLTFM F/P PLT BLK 21" D2 W/ INSTR	21" Footrest Width
1-4D,6-8,11-16	900316	PLTFM F/P PLT BLK 22" D2 W/ INSTR	22" Footrest Width
		SHORT EXT ASSEMBLIES	
1-3,4A,5-8,9A,10B,11-16	108043	PLTFRM FLIP FTRST SHT 12W	12" wide (Short)
1-3,4A,5-8,9A,10B,11-16	108044	PLTFRM FLIP FTRST SHT 13W	13" wide (Short)
1-3,4A,5-8,9A,10B,11-16	108045	PLTFRM FLIP FTRST SHT 14W	14" wide (Short)
1-3,4B,5-8,9A,10B,11-16	108046	PLTFRM FLIP FTRST SHT 15W	15" wide (Short)
1-3,4B,5-8,9A,10B,11-16	108047	PLTFRM FLIP FTRST SHT 16W	16" wide (Short)
1-3,4B,5-8,9A,10B,11-16	108048	PLTFRM FLIP FTRST SHT 17W	17" wide (Short)
1-3,4C,5-8,9A,10B,11-16	108049	PLTFRM FLIP FTRST SHT 18W	18" wide (Short)
1-3,4C,5-8,9A,10B,11-16	108051	PLTFRM FLIP FTRST SHT 19W	19" wide (Short)
1-3,4C,5-8,9A,10B,11-16	108052	PLTFRM FLIP FTRST SHT 20W	20" wide (Short)
1-3,4D,5-8,9A,10B,11-16	111554	PLTFRM FLIP FTRST SHT 21W	21" wide (Short)
1-3,4D,5-8,9A,10B,11-16	111555	PLTFRM FLIP FTRST SHT 22W	22" wide (Short)
		LONG EXT ASSEMBLIES	
1-3,4A,5-8,9B,10A,11-16	108053	PLTFRM FLIP FTRST LONG 12W	12" wide (Long)
1-3,4A,5-8,9B,10A,11-16	108054	PLTFRM FLIP FTRST LONG 13W	13" wide (Long)
1-3,4A,5-8,9B,10A,11-16	108055	PLTFRM FLIP FTRST LONG 14W	14" wide (Long)
1-3,4B,5-8,9B,10A,11-16	108056	PLTFRM FLIP FTRST LONG 15W	15" wide (Long)
1-3,4B,5-8,9B,10A,11-16	108057	PLTFRM FLIP FTRST LONG 16W	16" wide (Long)
1-3,4B,5-8,9B,10A,11-16	108058	PLTFRM FLIP FTRST LONG 17W	17" wide (Long)
1-3,4C,5-8,9B,10A,11-16	108059	PLTFRM FLIP FTRST LONG 18W	18" wide (Long)
1-3,4C,5-8,9B,10A,11-16	108060	PLTFRM FLIP FTRST LONG 19W	19" wide (Long)
1-3,4C,5-8,9B,10A,11-16	108061	PLTFRM FLIP FTRST LONG 20W	20" wide (Long)
1-3,4D,5-8,9B,10A,11-16	111556	PLTFRM FLIP FTRST LONG 21W	21" wide (Long)
1-3,4D,5-8,9B,10A,11-16	111557	PLTFRM FLIP FTRST LONG 22W	22" wide (Long)

## HIGH MOUNT EXTENSION TUBE


[11/2009]

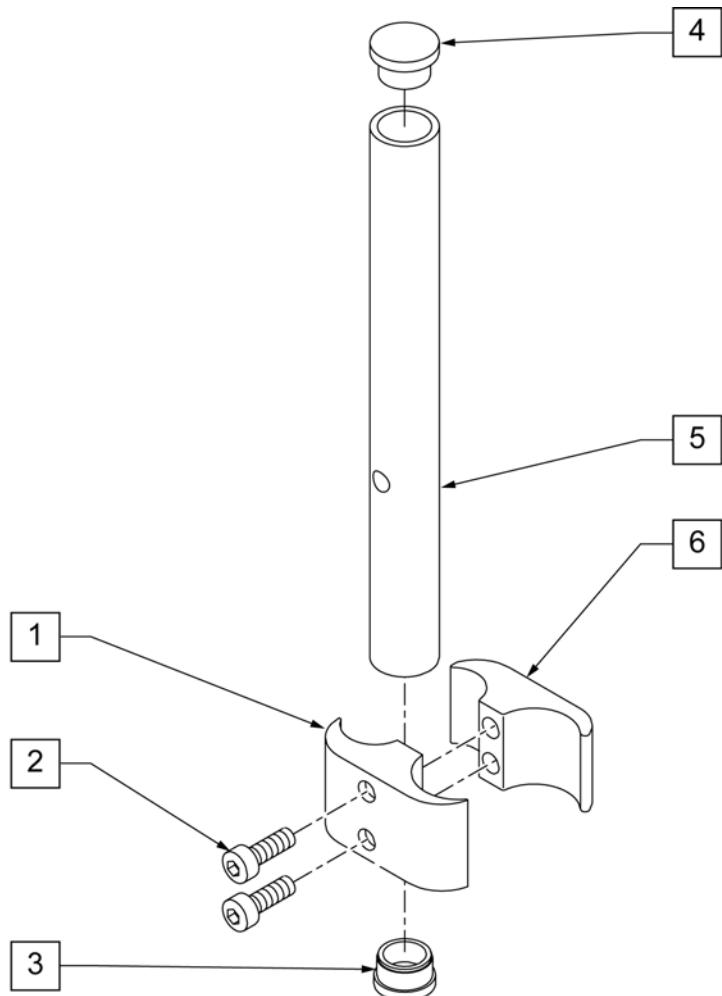

Pos.	Item Number	Description	Remarks
1	550657	CLAMP 1 1/4" RECESS FULL BLK	
2	107032	SHCS 1/4-28 X 1 ROHS	
3	561000-QU	3/4" INSERT PLUG .083	
4	482016	HIGH-MOUNT FL FT TUBE PLUG	
5	524100	H/M FL FT EXT TUBE BLK	
6	550658	CLAMP 1 1/4" THREADED BLK	
1,2,6	880538	HI-MOUNT FR CLAMP ASSEMBLY BLK	
1-6	921550S	H/M FL FT CLAMPS AND TUBES (PR)	Pair

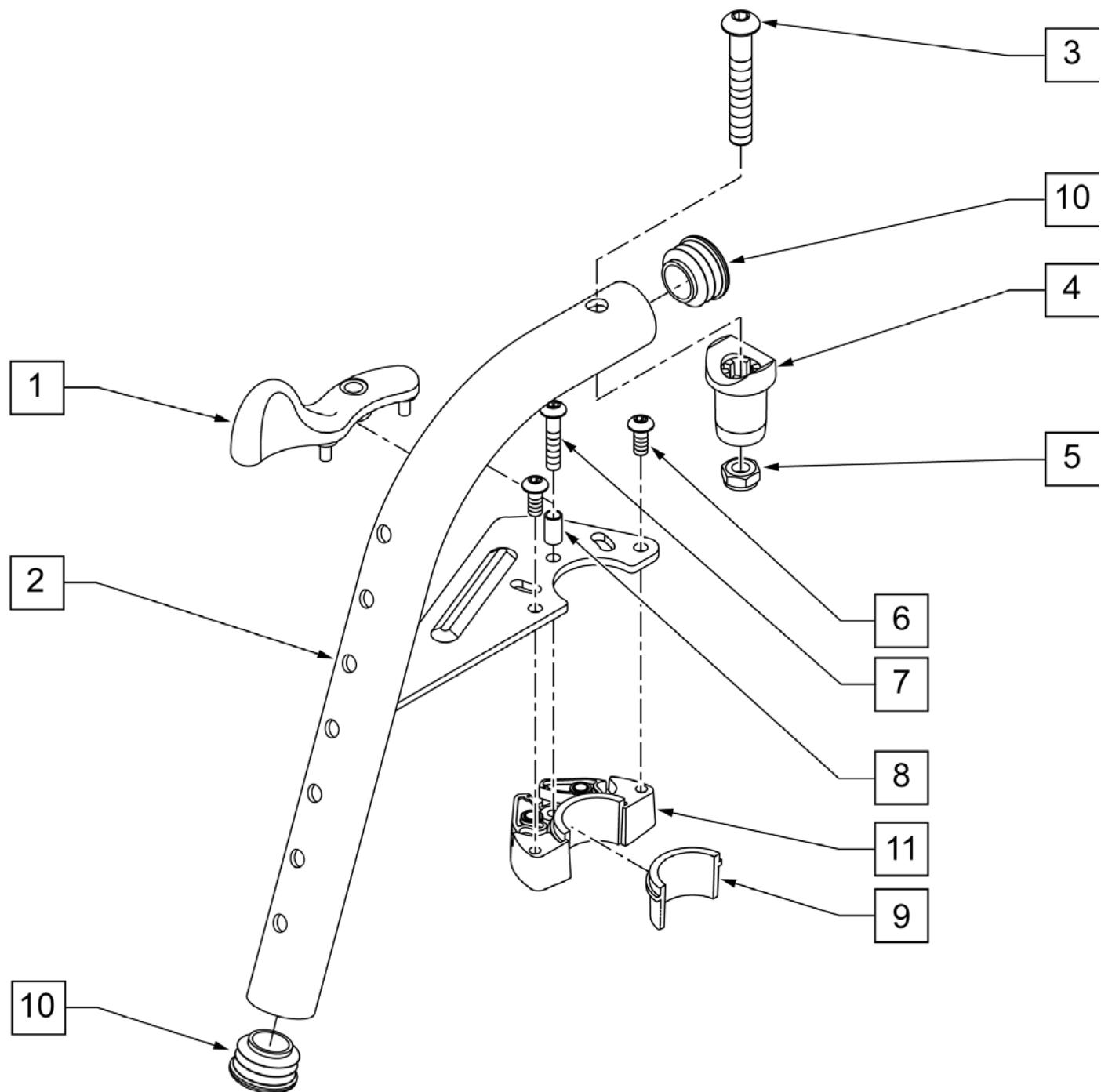


Pos.	Item Number	Description	Remarks
1	052296	SWING I/O LATCH LEVER R W/ PINS	RIGHT
1	052297	SWING I/O LATCH LEVER L W/ PINS	LEFT
3	106983	BHCS 5/16 -18 x 2 SHLDR ROHS	
4	561197	PLASTIC PIVOT SADDLE	Includes nylock nut
5	202117	5/16-18 NYLOCK NUT THICK BLK	
6	102259-001	BHCS M5 X .8 X 12 BLK W/ PL	
7	230494	M5-30 BTTN HD SOC C/S BLK	
8	052054	BUSHING PIVOT HANGER SWING I/O	
10	561062	1" INSERT PLUG .065 RIBS	
6,9,11	120976	ASSY LATCH BLOCK ANTI LIFT	
1-11	101153S	70D FM SWING I/O HNGR ASSY R	RIGHT
1-11	101154S	70D FM SWING I/O HNGR ASSY L	LEFT

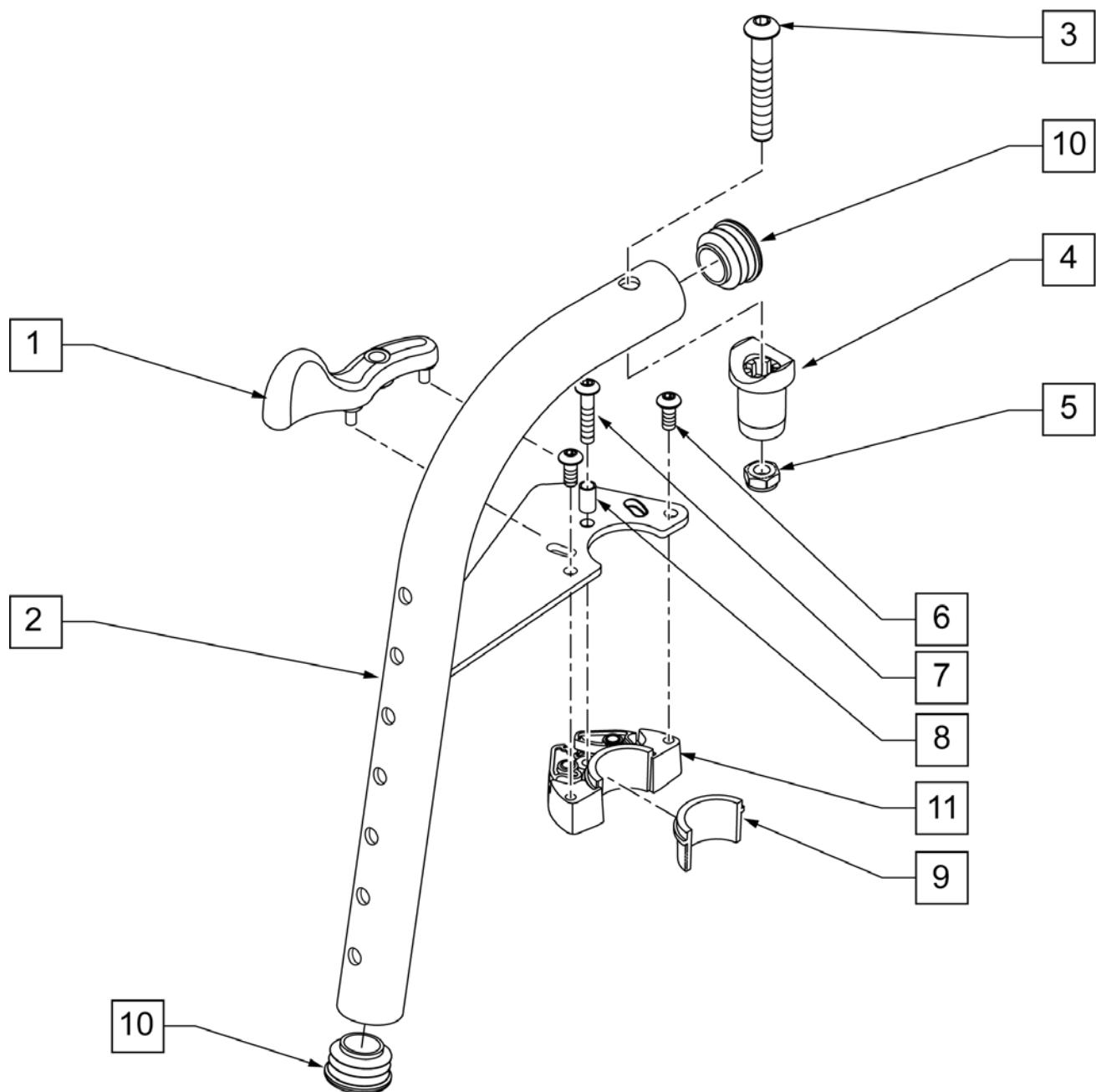


<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
1	052296	SWING I/O LATCH LEVER R w/ PINS	RIGHT
1	052297	SWING I/O LATCH LEVER L w/ PINS	LEFT
3	106983	BHCS 5/16-18 x 2 SHLDR ROHS	
4	561197	PLASTIC PIVOT SADDLE	Includes nylock nut
5	202117	5/16-18 NYLOCK NUT THICK BLK	
6	102259-001	BHCS M5 X .8 X 12 BLK W/PL	
7	230494	M5-30 BTTN HD SOC C/S BLK	
8	052054	BUSHING PIVOT HANGER SWING I/O	
10	561062	1" INSERT PLUG .065 RIBS	
6,9,11	120976	ASSY LATCH BLOCK ANTI LIFT	
1A,2A,3-11	101155S	80D FM SWNG I/O HNGR ASSY R	RIGHT
1B,2B,3-11	101156S	80D FM SWNG I/O HNGR ASSY L	LEFT

**NOTE:** Effective on LX/LXI as of 10/24/2008.

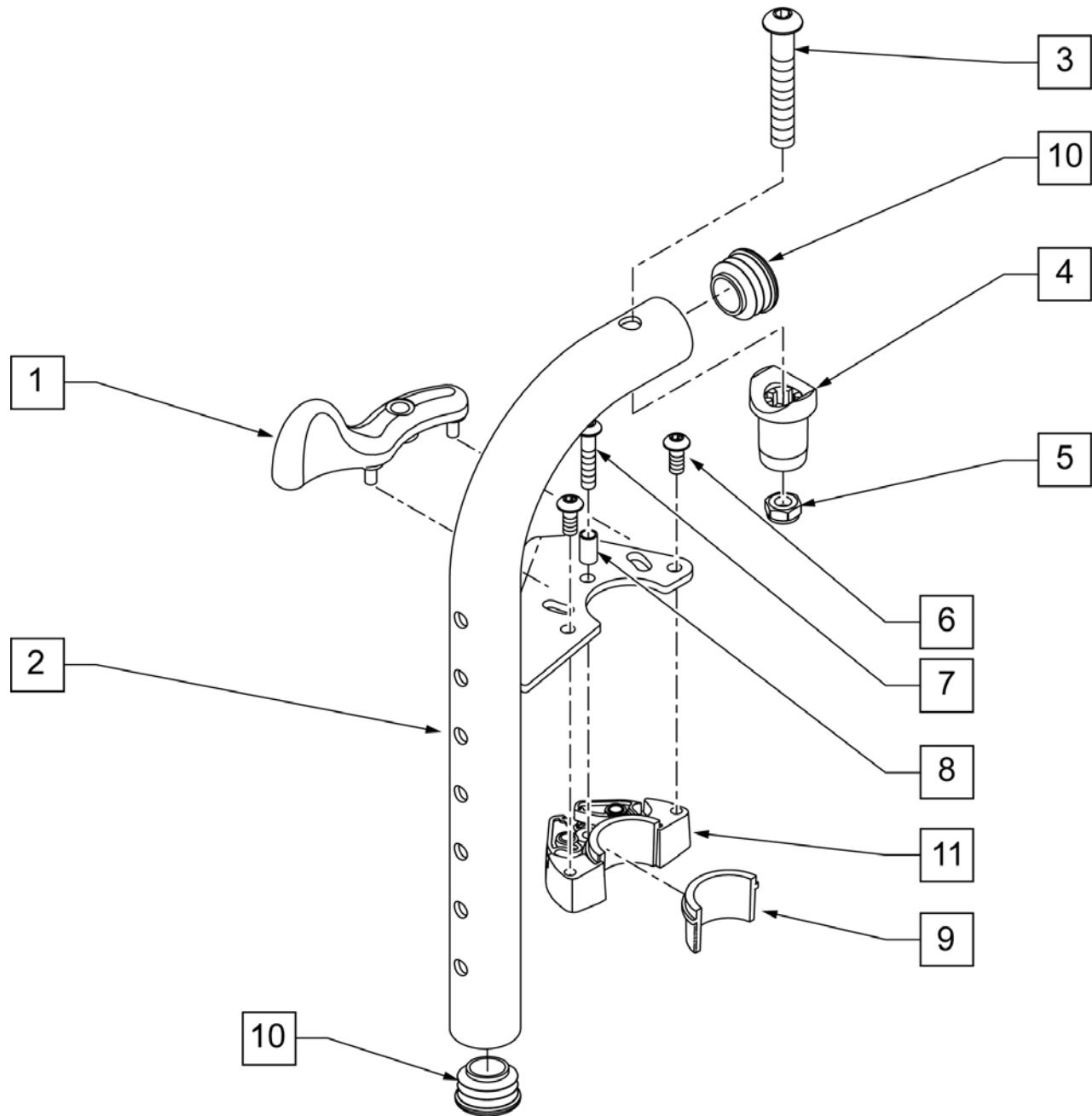

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
1	052296	SWING I/O LATCH LEVER R w/ PINS	RIGHT
1	052297	SWING I/O LATCH LEVER L w/ PINS	LEFT
3	106983	BHCS 5/16-18 x 2 SHLDR ROHS	
4	561197	PLASTIC PIVOT SADDLE	
5	202117	5/16-18 NYLOCK NUT THICK BLK	
6	102259-001	BHCS M5 x .8 x 12 W /PL INORG	
7	230494	M5-30 BTTN HD SOC C/S BLK	
8	052054	BUSHING PIVOT HANGER SWING I/O	
10	561062	1" INSERT PLUG .065 RIBS	
6,9,11	120976	ASSY LATCH BLOCK ANTI LIFT	
1A,2A,3-11	101157S	90D FM SWNG I/O Hngr ASSY R	RIGHT
1B,2B,3-11	101158S	90D FM SWNG I/O Hngr ASSY L	LEFT

**NOTE:** \* ORDER ITEM 16 WHEN NOT USING FOOTREST EXTENSION.

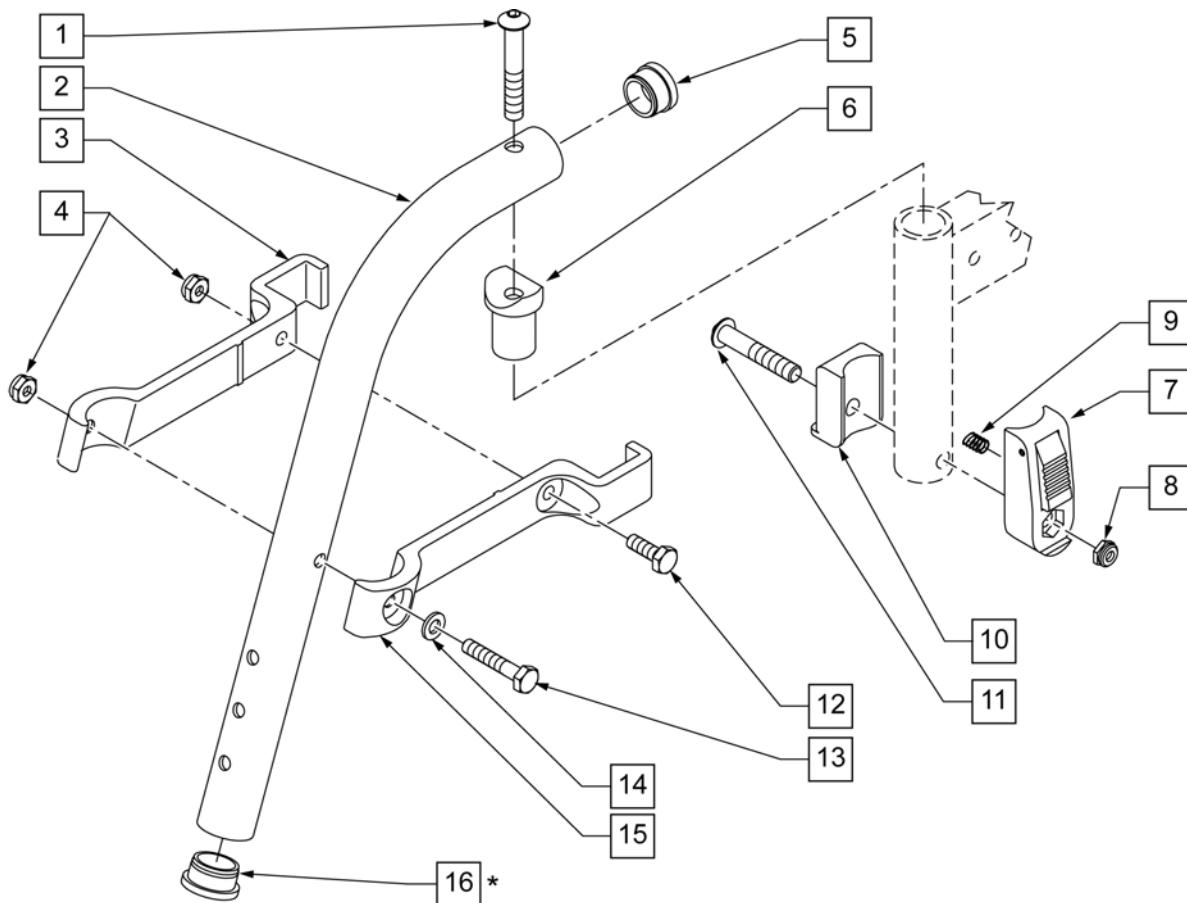


Pos.	Item Number	Description	Remarks
1	106983	BHCS 5/16 -18 x 2 SHLDR ROHS	
3	550152	T45 HANGER BRACKET 70° R	
4	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
5	561002	1" INSERT PLUG .083	
6	561197	PLASTIC PIVOT SADDLE	
7	880053	FOOTREST BLOCK T45	
8	202117	5/16-18 NYLOCK NUT THICK BLK	
9	321141	.650 x 6 COIL SPRING	
10	560684	T45 FOOTREST BLOCK RECESSED	
11	107029	BHCS 5/16 -18 x 1 5/8 SHLD ROHS	
12	107014	HHCS 1/4 -28 x 3/4 ROHS	
13	106971	HHCS 1/4 -28 x 1 3/8 ROHS	
14	106943	WASHER FLAT AN 1/4 ROHS	
15	550153	T45 HANGER BRACKET 70° L	
16	561002	1" INSERT PLUG .083	ORDER WHEN NOT USING FOOTREST EXTENSION
7-11	920720	FOOTREST BLOCK ASSEMBLY W/BOLT HD	
1-6,12-15	641210S	HANGER 70D PNTD R HD	
1-6,12-15	641211S	HANGER 70D PNTD L HD	

# 80 DEG LIFT-OFF HANGER


[09/2010]

**NOTE:** \* ORDER ITEM 16 WHEN NOT USING FOOTREST EXTENSION.

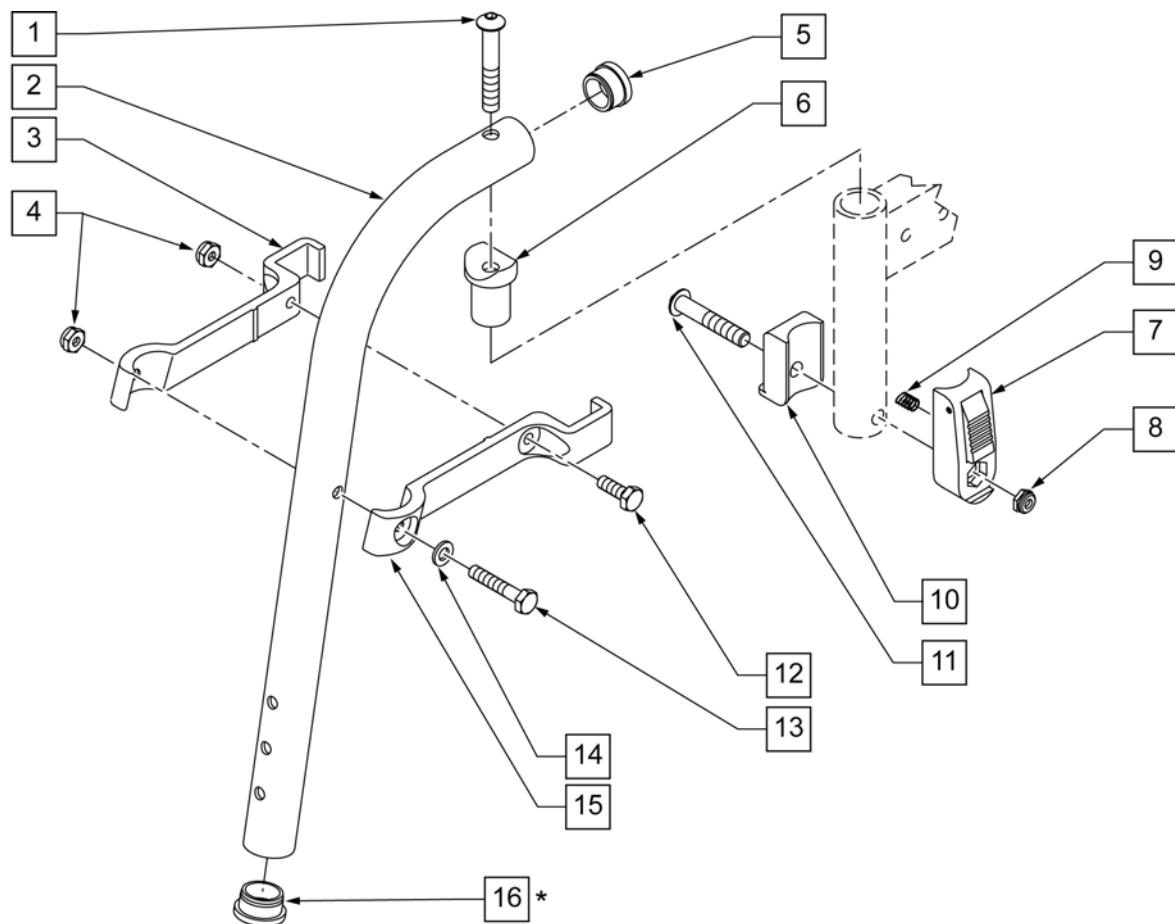

Pos.	Item Number	Description	Remarks
1	106983	BHCS 5/16 -18 x 2 SHLDR ROHS	
3	550154	T45 HANGER BRACKET 80ø R	
4	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
5	561002	1" INSERT PLUG .083	
6	561197	PLASTIC PIVOT SADDLE	
7	880053	FOOTREST BLOCK T45	
8	202117	5/16-18 NYLOCK NUT THICK BLK	
9	321141	.650L x 6 COIL SPRING	
10	560684	T45 FOOTREST BLOCK RECESSED	
11	107029	BHCS 5/16 -18 x 1 5/8 SHLD ROHS	
12	107014	HHCS 1/4 -28 x 3/4 ROHS	
13	106971	HHCS 1/4 -28 x 1 3/8 ROHS	
14	106943	WASHER FLAT AN 1/4 ROHS	
15	550155	T45 HANGER BRACKET 80ø L	
16	561002	1" INSERT PLUG .083	ORDER WHEN NOT USING FOOTREST EXTENSION
7-11	920720	FOOTREST BLOCK ASSEMBLY W/BOLT HD	
1-6,12-15	641200S	HANGER 80D PNTD R HD	
1-6,12-15	641201S	HANGER 80D PNTD L HD	


NOTE: THIS HANGER HAS BEEN TESTED AND APPROVED FOR OCCUPANTS THAT WEIGH LESS THAN 150 POUNDS.

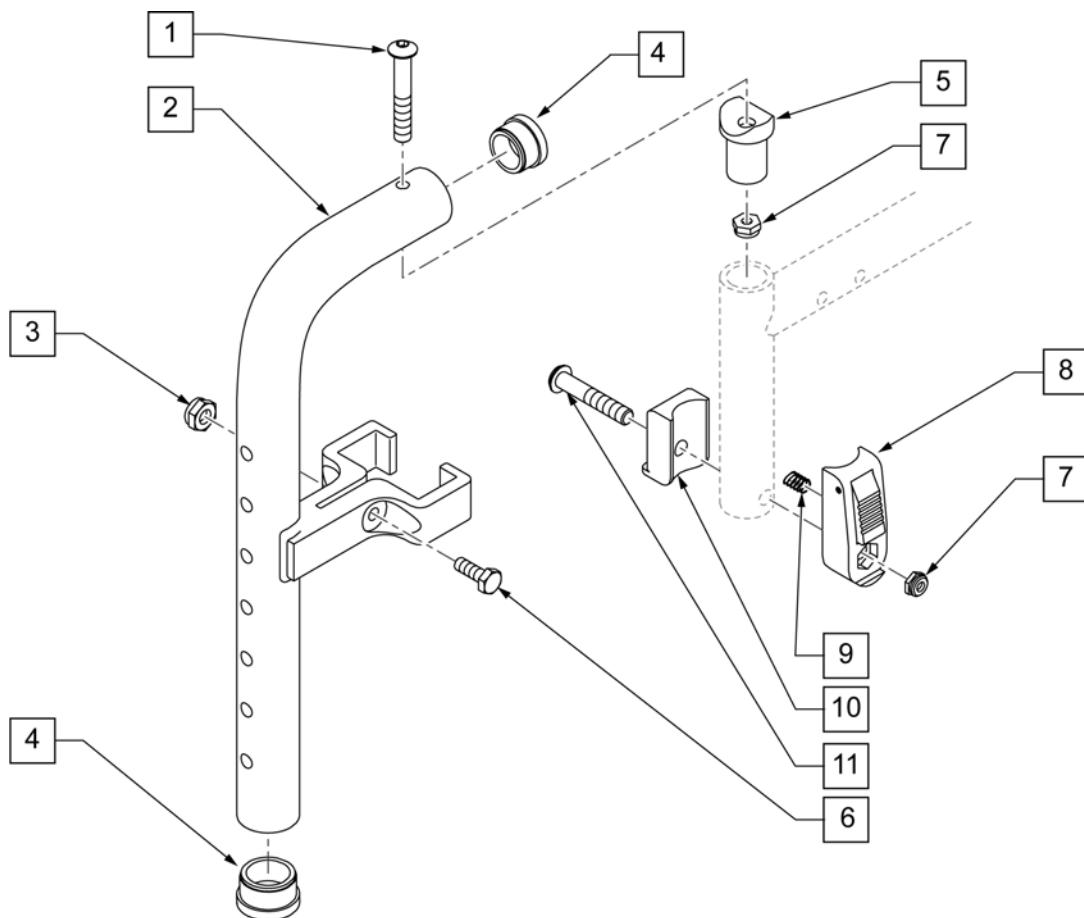

Pos.	Item Number	Description	Remarks
1	106983	BHCS 5/16 -18 x 2 SHLDR ROHS	
3	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
4	561002	1" INSERT PLUG .083	
5	561197	PLASTIC PIVOT SADDLE	Includes nylock nut
6	107013	HHC 1/4 -28 x 5/8 ROHS	
7	880053	FOOTREST BLOCK T45	
8	202117	5/16-18 NYLOCK NUT THICK BLK	
9	321141	.650L x 6 COIL SPRING	
10	560684	T45 FOOTREST BLOCK RECESSED	
11	107029	BHCS 5/16 -18 x 1 5/8 SHLD ROHS	
7-11	920720	FOOTREST BLOCK ASSM W/BOLT HD	
1-7	885005S	90 DEG F/D H/D L/O HNGR ASSM PNTD	


## FRONT MOUNT HANGER EXT

[02/2010]

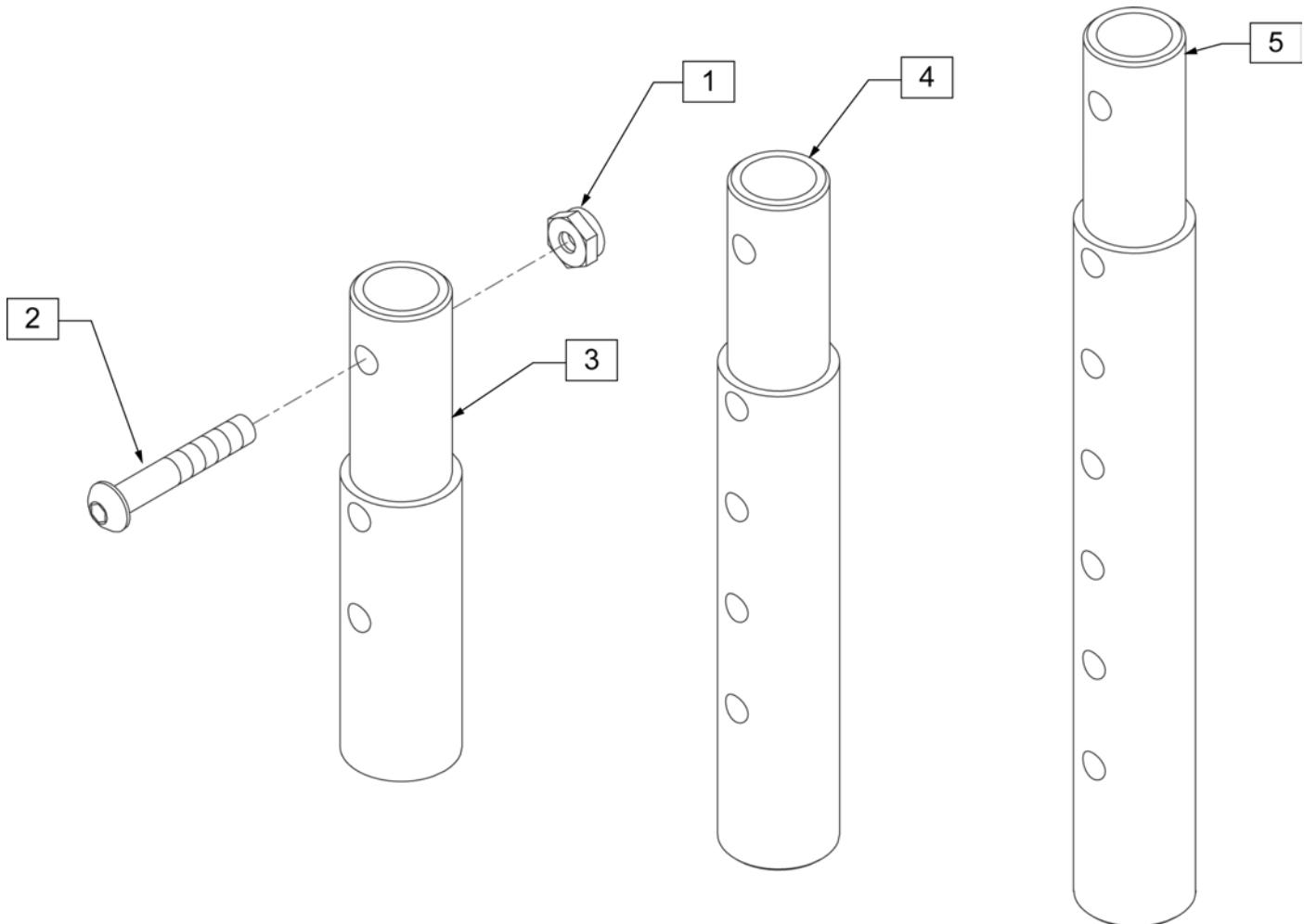

Pos.	Item Number	Description	Remarks
1	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
2	106961	BHCS 1/4 -28 x 1 1/4 ROHS	
1-3	111505	2" FOOTREST EXTENSION TUBE W/ HDWR ASSM	2" EXT (PAIR)
1, 2, 4	111506	4" FOOTREST EXTENSION TUBE W/ HDWR ASSM	4" EXT (PAIR)
1, 2, 5	111507	6" FOOTREST EXTENSION TUBE W/ HDWR ASSM	6" EXT (PAIR)


## KIDS ANGLE ADJ FOOTPLATE FRONT MOUNT

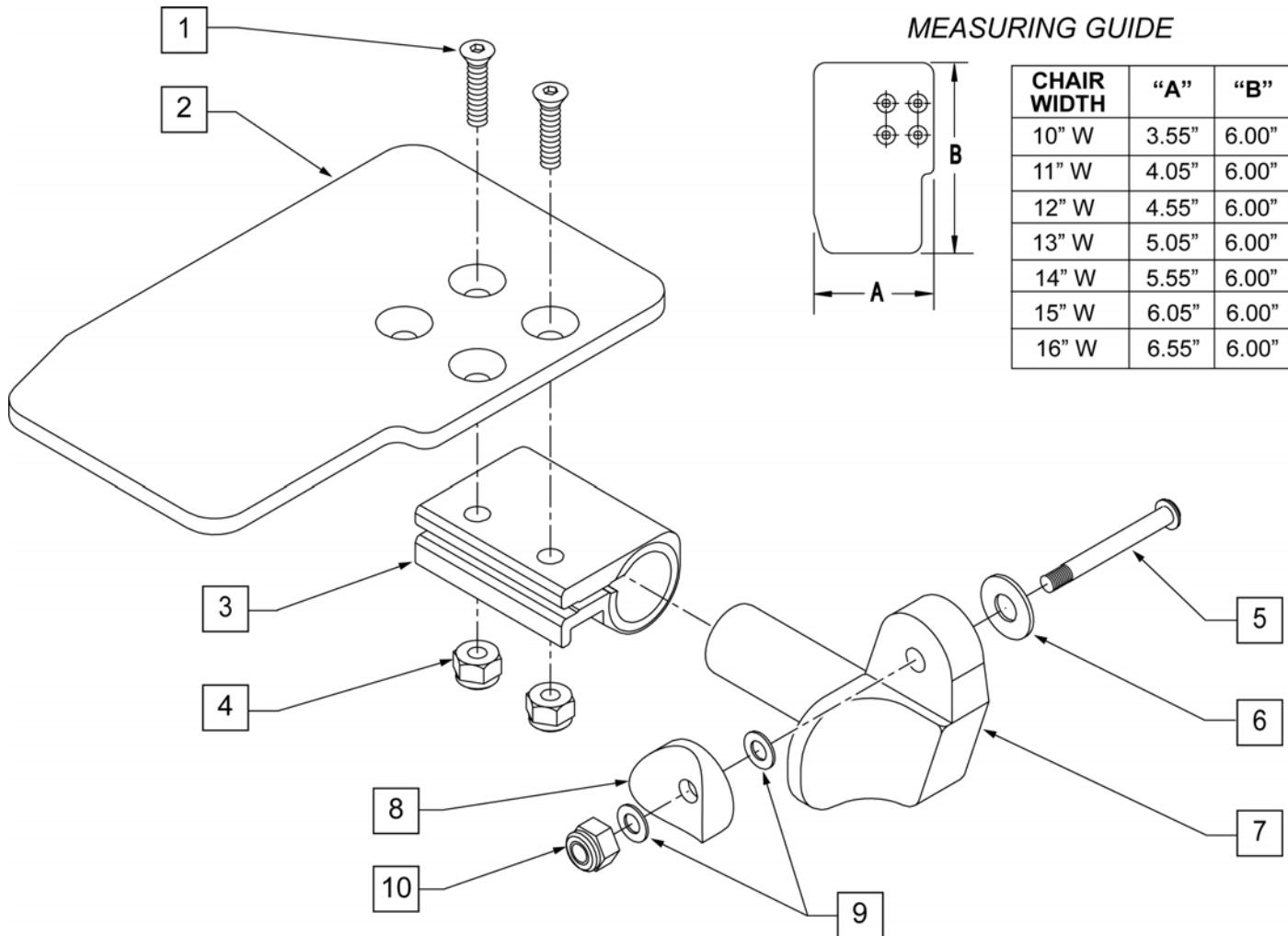

Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1	106978	FHCS 1/4-28 X 7/8 ROHS		7	541001	FTRST MOUNTING BRKT 1 RAD	LEFT
2	500635	KIDS ANGL ADJ F/PLT 10	RIGHT	8	561004	1 SADDLE 1/4 HOLE	
2	500636	KIDS ANGL ADJ F/PLT 10	LEFT	9	106943	WASHER FLAT THIN AN 1/4	
2	500637	KIDS ANGL ADJ F/PLT 11	RIGHT	10	106941	NUT AUTOMATION 1/4-28 ROHS	
2	500638	KIDS ANGL ADJ F/PLT 11	LEFT	1-10	828821	KIDS ANG ADJ F/PLT ASSM 10	RIGHT
2	500639	KIDS ANGL ADJ F/PLT 12	RIGHT	1-10	828822	KIDS ANG ADJ F/PLT ASSM 10	LEFT
2	500640	KIDS ANGL ADJ F/PLT 12	LEFT	1-10	828823	KIDS ANG ADJ F/PLT ASSM 11	RIGHT
2	500641	KIDS ANGL ADJ F/PLT 13	RIGHT	1-10	828824	KIDS ANG ADJ F/PLT ASSM 11	LEFT
2	500642	KIDS ANGL ADJ F/PLT 13	LEFT	1-10	828825	KIDS ANG ADJ F/PLT ASSM 12	RIGHT
2	500643	KIDS ANGL ADJ F/PLT 14	RIGHT	1-10	828826	KIDS ANG ADJ F/PLT ASSM 12	LEFT
2	500644	KIDS ANGL ADJ F/PLT 14	LEFT	1-10	828827	KIDS ANG ADJ F/PLT ASSM 13	RIGHT
2	500645	KIDS ANGL ADJ F/PLT 15	RIGHT	1-10	828828	KIDS ANG ADJ F/PLT ASSM 13	LEFT
2	500646	KIDS ANGL ADJ F/PLT 15	LEFT	1-10	828829	KIDS ANG ADJ F/PLT ASSM 14	RIGHT
2	500647	KIDS ANGL ADJ F/PLT 16	RIGHT	1-10	828830	KIDS ANG ADJ F/PLT ASSM 14	LEFT
2	500648	KIDS ANGL ADJ F/PLT 16	LEFT	1-10	828831	KIDS ANG ADJ F/PLT ASSM 15	RIGHT
3	550350	3/4 CLAMP ADJ FOOTPLATE		1-10	828832	KIDS ANG ADJ F/PLT ASSM 15	LEFT
4	106935	NUT NYLOCK THIN 1/4-28 ROHS		1-10	828833	KIDS ANG ADJ F/PLT ASSM 16	RIGHT
5	226147	1/4-28 X 1.85 BH C/S BLK ZN		1-10	828834	KIDS ANG ADJ F/PLT ASSM 16	LEFT
6	106945	WASHER FLAT SAE 1/4 ROHS		1-10	921226	KIDS ANG ADJ F/PLT ASSM 10	PAIR
7	541000	FTRST MOUNTING BRKT 1 RAD	RIGHT	1-10	921227	KIDS ANG ADJ F/PLT ASSM 11	PAIR


Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1-10	921228	KIDS ANG ADJ F/PLT ASSM 12	PAIR	1-10	921231	KIDS ANG ADJ F/PLT ASSM 15	PAIR
1-10	921229	KIDS ANG ADJ F/PLT ASSM 13	PAIR	1-10	921232	KIDS ANG ADJ F/PLT ASSM 16	PAIR
1-10	921230	KIDS ANG ADJ F/PLT ASSM 14	PAIR				

## ADULT ANGLE-ADJ FOOTPLATE FRONT MOUNT

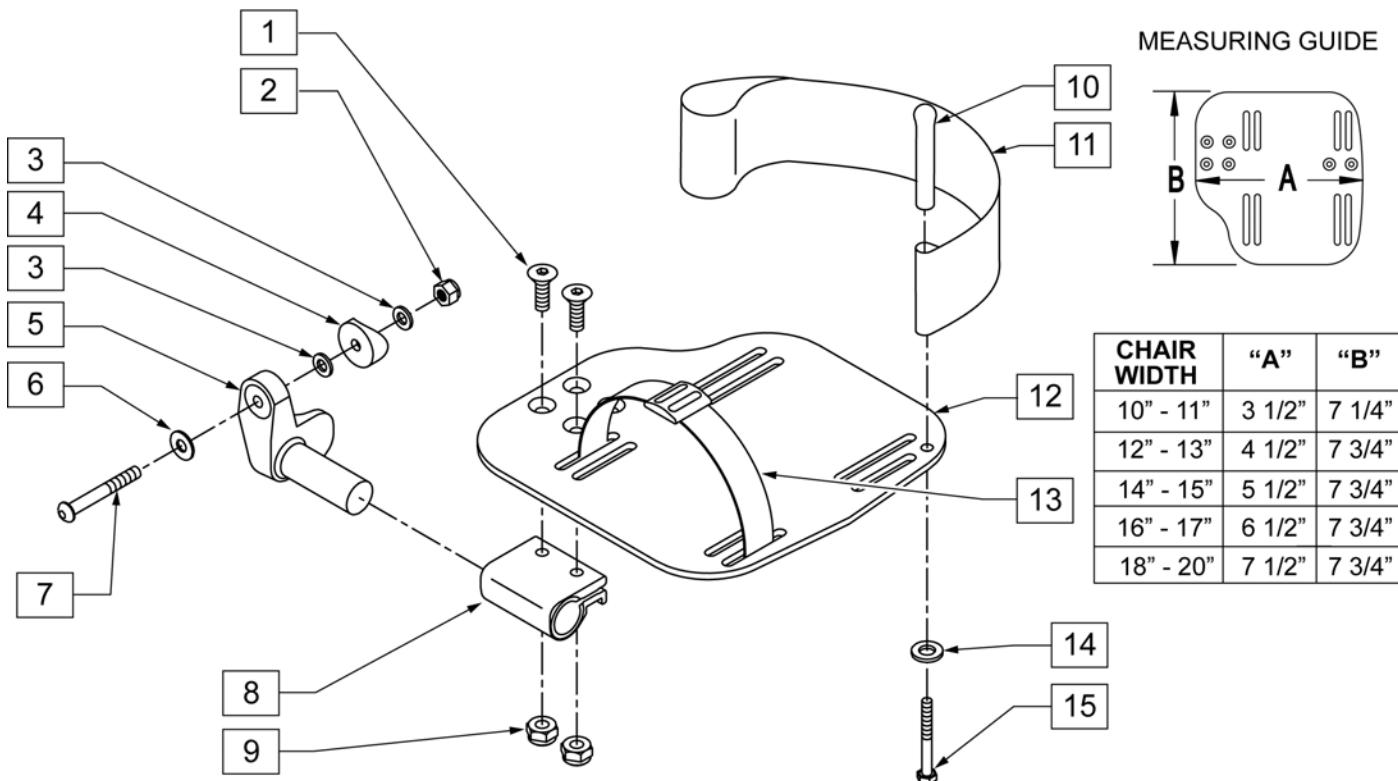

Pos.	Item Number	Description	Remarks
1	106978	FHCS 1/4-28 X 7/8 ROHS	
2	106941	NUT AUTOMATION 1/4-28 ROHS	
3	106943	WASHER FLAT AN 1/4 ROHS	
4	561004	1" SADDLE 1/4" HOLE	
5	541000	FOOTREST MOUNTING BRKT-1" RAD	RIGHT
5	541001	FOOTREST MOUNTING BRKT-1" RAD	LEFT
6	106945	WASHER FLAT SAE 1/4 ROHS	
7	106964	BHCS 1/4-28 X 2	
8	550350	3/4" CLAMP ADJ FOOTPLATE	
9	106935	NUT NYLOCK THIN 1/4-28 ROHS	
10	561040	HEEL LOOP PEG	
11	741153	HEEL LOOP 18-20 CMPST/14-15 ANG ADJ	USED W/ 14-15"/16-17" ADULT ANG ADJ FTPLTS
11	741253	HEEL LOOP ADLT ADJ F/PLT 18-20	USED W/ 18 -20" ADULT ANG ADJ FTPLTS
12	500656	ADULT ANGL ADJ F/PLT 10/11	RIGHT
12	500657	ADULT ANGL ADJ F/PLT 10/11	LEFT
12	500658	ADULT ANGL ADJ F/PLT 12/13	RIGHT
12	500659	ADULT ANGL ADJ F/PLT 12/13	LEFT
12	500660	ADULT ANGL ADJ F/PLT 14/15	RIGHT
12	500661	ADULT ANGL ADJ F/PLT 14/15	LEFT
12	500662	ADULT ANGL ADJ F/PLT 16/17	RIGHT
12	500663	ADULT ANGL ADJ F/PLT 16/17	LEFT
12	500664	ADULT ANGL ADJ F/PLT 18-20	RIGHT
12	500665	ADULT ANGL ADJ F/PLT 18-20	LEFT
13	740260	TOE-ANKLE STRAP ADJ ADULT FOOTPLATE	
14	106943	WASHER FLAT AN 1/4 ROHS	


<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
15	106958	HHCS 1/4-20 X 1 ROHS	
1-9,12	820544	FP ADULT ADJ 90D 10/11 R	RIGHT
1-9,12	820554	FP ADULT ADJ 90D 10/11 L	LEFT
1-9,12	820546	FP ADULT ADJ 90D 12/13 R	RIGHT
1-9,12	820556	FP ADULT ADJ 90D 12/13 L	LEFT
1-9,12	820472	FP ADULT ADJ 90D 14/15 R	RIGHT
1-9,12	820473	FP ADULT ADJ 90D 14/15 L	LEFT
1-9,12	820474	FP ADULT ADJ 90D 16/17 R	RIGHT
1-9,12	820475	FP ADULT ADJ 90D 16/17 L	LEFT
1-9,12	820476	FP ADULT ADJ 90D 18-20 R	RIGHT
1-9,12	820477	FP ADULT ADJ 90D 18-20 L	LEFT
1-9,12	921474	ADULT ANGL ADJ 90D 10/11 OMIT H/L	PAIR
1-9,12	921476	ADULT ANGL ADJ 90D 12/13 OMIT H/L	PAIR
1-9,12	921330	ADULT ANGL ADJ 90D 14/15 OMIT H/L	PAIR
1-9,12	921331	ADULT ANGL ADJ 90D 16/17 OMIT H/L	PAIR
1-9,12	921332	ADULT ANGL ADJ 90D 18-20 OMIT H/L	PAIR
1-12,14,15	820540	F/PLT ADULT ADJ 90D 10/11 R W/ HL	RIGHT
1-12,14,15	820550	F/PLT ADULT ADJ 90D 10/11 L W/ HL	LEFT
1-12,14,15	820542	F/PLT ADULT ADJ 90D 12/13 R W/ HL	RIGHT
1-12,14,15	820552	F/PLT ADULT ADJ 90D 12/13 L W/ HL	LEFT
1-12,14,15	820478	F/PLT ADULT ADJ 90D 14/15 R W/ HL	RIGHT
1-12,14,15	820479	F/PLT ADULT ADJ 90D 14/15 L W/ HL	LEFT
1-12,14,15	820480	F/PLT ADULT ADJ 90D 16/17 R W/ HL	RIGHT
1-12,14,15	820481	F/PLT ADULT ADJ 90D 16/17 L W/ HL	LEFT
1-12,14,15	820482	F/PLT ADULT ADJ 90D 18-20 R W/ HL	RIGHT
1-12,14,15	820483	F/PLT ADULT ADJ 90D 18-20 L W/ HL	LEFT
1-12,14,15	921470	F/PLT ADULT ADJ 90D 10/11 W/ HL	PAIR
1-12,14,15	921472	F/PLT ADULT ADJ 90D 12/13 W/ HL	PAIR
1-12,14,15	921333	F/PLT ADULT ADJ 90D 14/15 W/ HL	PAIR
1-12,14,15	921334	F/PLT ADULT ADJ 90D 16/17 W/ HL	PAIR
1-12,14,15	921335	F/PLT ADULT ADJ 90D 18-20 W/ HL	PAIR

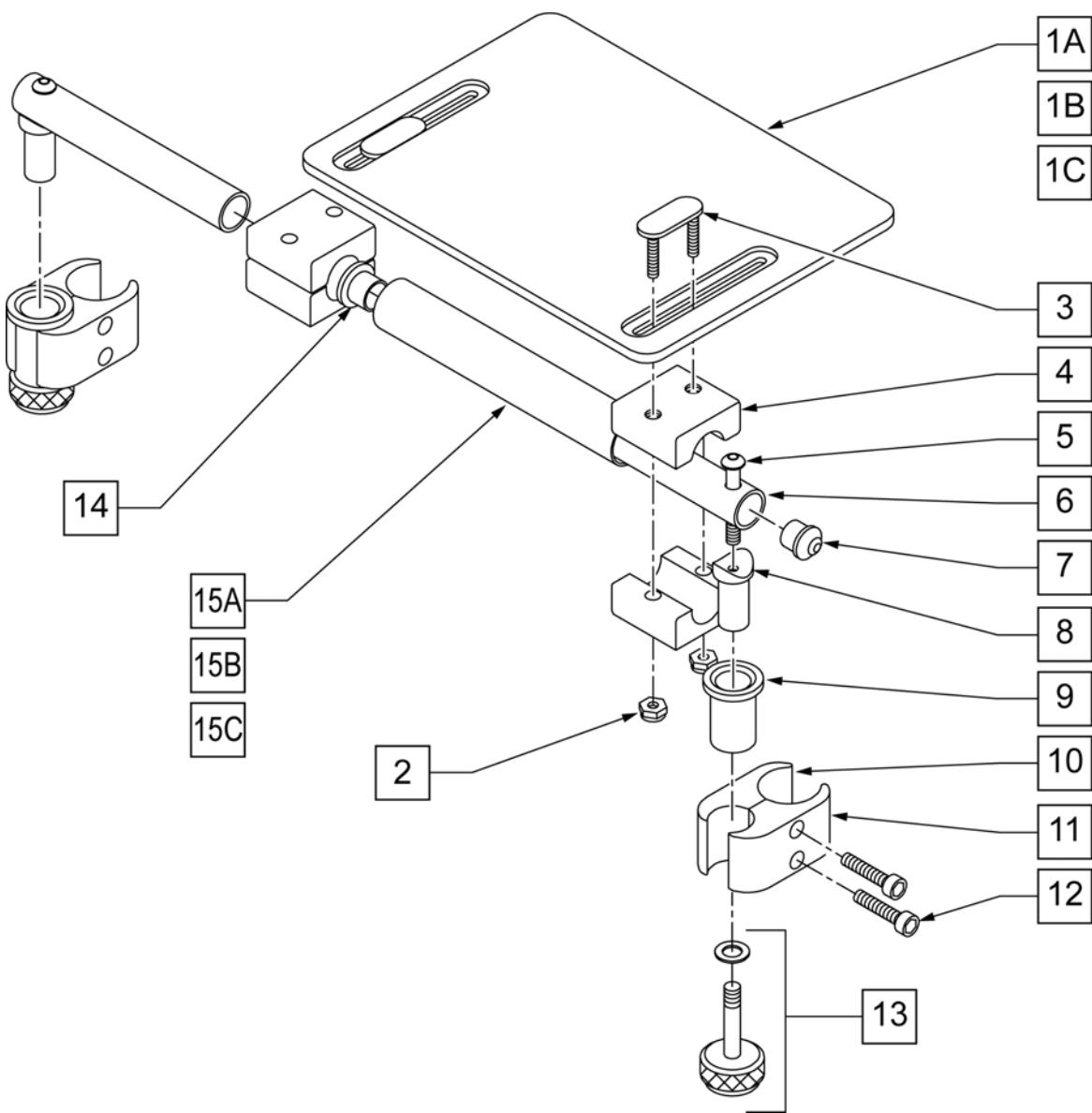

Pos.	Item Number	Description	Remarks
1A	562610	SLD FOOTPLATE ONLY 8" x 8" (10"-12" W)	10 -12" WIDE
1B	562612	SLD FOOTPLATE ONLY 10" x 8" (13"-14" W)	13 -14" WIDE
1C	500709	SLD FT PLT ALUM ONLY 12" x 8" (15"-16" W)	15 -16" WIDE
2	106933	NUT NYLOCK THIN 1/4-20 ROHS	
3	330120	SLD FT/ANGLE ADJ DOUBLE T BOLT	
4	550444	SOLID FOOT/ANG ADJ CLAMP BLK	
5	107028	BHCS 1/4-28 X 1-1/2 ROHS	
7	561034	SHORT DOME PLUG	
8	483236	SOLID FOOT INSERT SADDLE BLK	
10	550658	CLAMP 1 1/4" THREADED BLK	
11	550657	CLAMP 1 1/4" RECESS FULL BLK	
12	107032	SHCS 1/4-28 X 1 ROHS	
9,13	880559	SOLID FOOT LOCK KNOB ASSEMBLY	

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
14	560402	3/4" ID NY-LINER	
15A	521621	SLD FOOT SUPPORT TUBE BLK 10"-12"	10 -12" WIDE
15B	521622	SLD FOOT SUPPORT TUBE BLK 13"-14"	13 -14" WIDE
15C	521624	SLD FOOT SUPPORT TUBE BLK 15"-16"	15 -16" WIDE
5-8	880183	SLD FT PLT PIVOT TUBE SUB-ASSM	
1A,2-14, 15A	921801	SLD FT PLT ASSEMBLY 8" x 8" (10"-12" W)	10 -12" WIDE
1B,2-14, 15B	921803	SOLID FOOTPLATE ASSEMBLY 10" x 8" (13"-14" W)	13 -14" WIDE
1C,2-14, 15C	921805	SOLID FOOTPLATE ASSEMBLY 12" x 8" (15"-16" W)	15 -16" WIDE


NOTE: Order Item 27 (561002) when NOT using Footrest Extension.

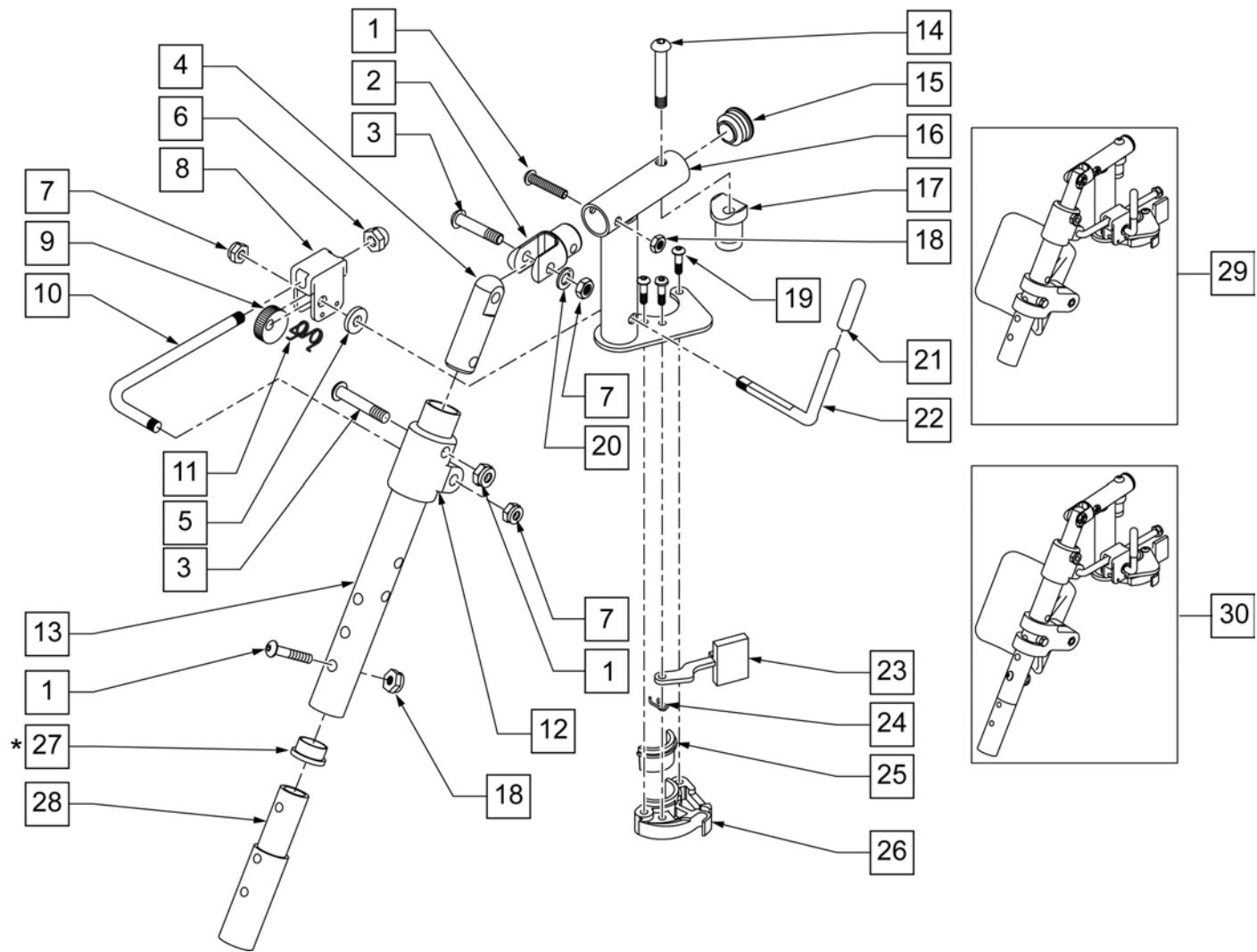

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
1	107027	BHCS 1/4-28 X 1-1/4 SHLDR ROHS	
2	505281	ELR CLEVIS	
3	107037	BHCS 5/16-24 X 1-1/2 SHLD ROHS	
4	540252	ELR PIVOT PRS	
5	561059	3/4 OD x .318 ID WASHER	
6	202109	5/16-24 ACORN NUT HEX	
7	106939	NUT NYLOCK THIN 5/16-24 ROHS	
8	880990	ELR LOCK HOUSING W/PIN	
9	481119	ELR CAM LOCKING POWDERED METAL	
10	481505	ELR LOCK ROD 90D	
11	321106	ELR TORSION SPRING	
12	560430	ELR ROD UPPER PIVOT	
13	505097S	ELR KIDS TUBE	SPECIFY COLOR
14	106983	BHCS 5/16-18 X 2 SHLDR ROHS	

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
15	561062	1" INSERT PLUG .065 RIBS	
17	561197	PLASTIC PIVOT SADDLE	
18	106935	NUT NYLOCK THIN 1/4-28 ROHS	
19	107035	BHCS 10-24 X .725 PL ROHS	
20	263100	5/16 AN WASHER FLAT	
21	561049	ELR RELEASE ROD COVER	
23	561079	S/A LATCH LEVER	
24	321136	S/A SPRING L RED	
24	321137	S/A SPRING R CLEAR	
27	561002	1" INSERT PLUG .083	
29	923067S	ELR HNGR ASSM 90D W/PAD	PAIR
30	923068S	ELR HNGR ASSM 90D W/PAD 2" EXT	PAIR
1,18,28	111505	2" FTREST EXT W/ HDWR	
25,26	881500	LATCH BLOCK ASSM R	RIGHT
25,26	881501	LATCH BLOCK ASSM L	LEFT
21,22	882411	ELR 90D ACTUATION ARM ASSM	
1-4,7,14-20,23-28	882413	ELR PIVOT BLOCK ASSM R	RIGHT
1-4,7,14-20,23-28	882414	ELR PIVOT BLOCK ASSM L	LEFT
1-26	641458S	ELR HNGR 90D W/O PAD PNTD R	RIGHT
1-26	641459S	ELR HNGR 90D W/O PAD PNTD L	LEFT
1-26,28	641468S	HANGER ELR 90 W/O PAD R 2" EXT	RIGHT
1-26,28	641469S	HANGER ELR 90 W/O PAD L 2" EXT	LEFT
19,23-26	881521	S/A LATCH KIT CASTING RIGHT	RIGHT
19,23-26	881522	S/A LATCH KIT CASTING LEFT	LEFT


## 90 DEG ELEVATING CALF PAD

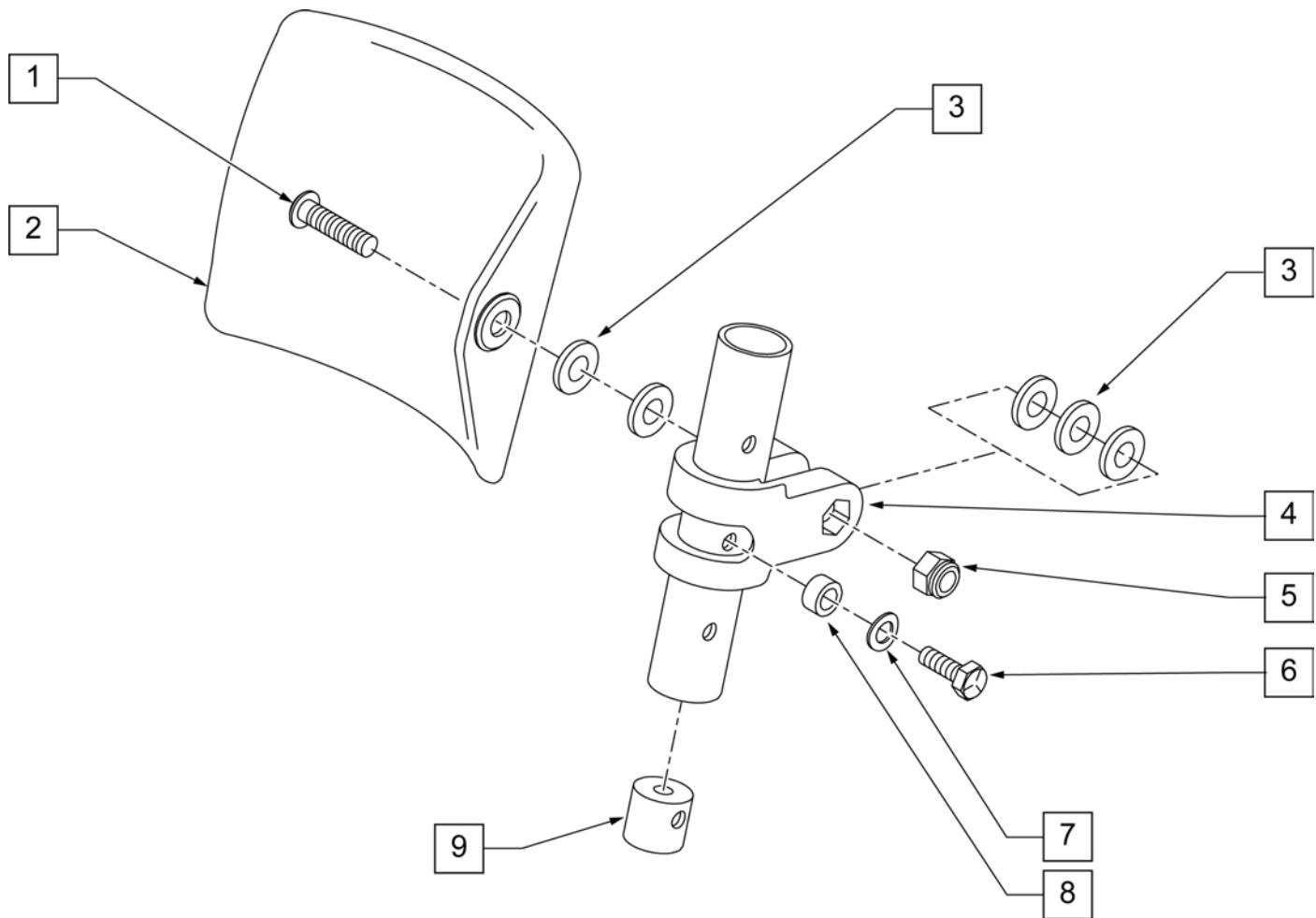

Pos.	Item Number	Description
1	226196	5/16-18 x 2 BH C/S ZN
2	755013	CALF PAD THROUGH STYLE KIDS
3	561016	1" WASHER 5/16" HOLE
4	561097	ELR ELEVATING TUBE CLAMP
5	106937	5/16 -18 NYLOCK NUT THIN
6	107017	1/4-20 x 1 1/4 G8 HEX C/S
7	262130	6mm WASHER WAVE
8	483027	1/4 x 1/4 ALUMINUM SPACER
9	500151	KIDS ALR RETAINING PLUG
1-9	880533	ELR TROUGH PAD ASSM KIDS


## FRONT MOUNT HANGER EXT

[02/2010]

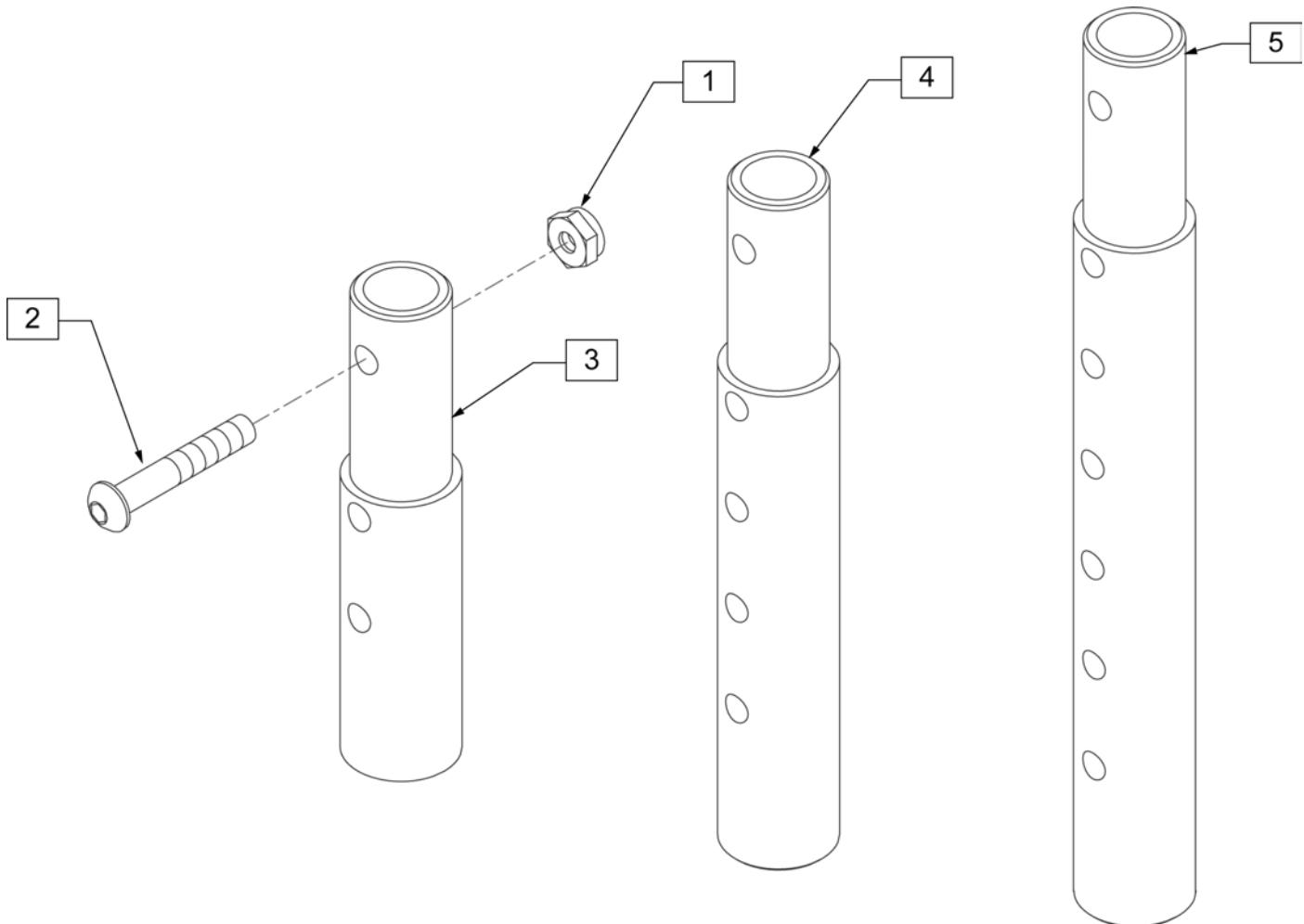

Pos.	Item Number	Description	Remarks
1	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
2	106961	BHCS 1/4 -28 x 1 1/4 ROHS	
1-3	111505	2" FOOTREST EXTENSION TUBE W/ HDWR ASSM	2" EXT (PAIR)
1, 2, 4	111506	4" FOOTREST EXTENSION TUBE W/ HDWR ASSM	4" EXT (PAIR)
1, 2, 5	111507	6" FOOTREST EXTENSION TUBE W/ HDWR ASSM	6" EXT (PAIR)


## KIDS ANGLE ADJ FOOTPLATE FRONT MOUNT

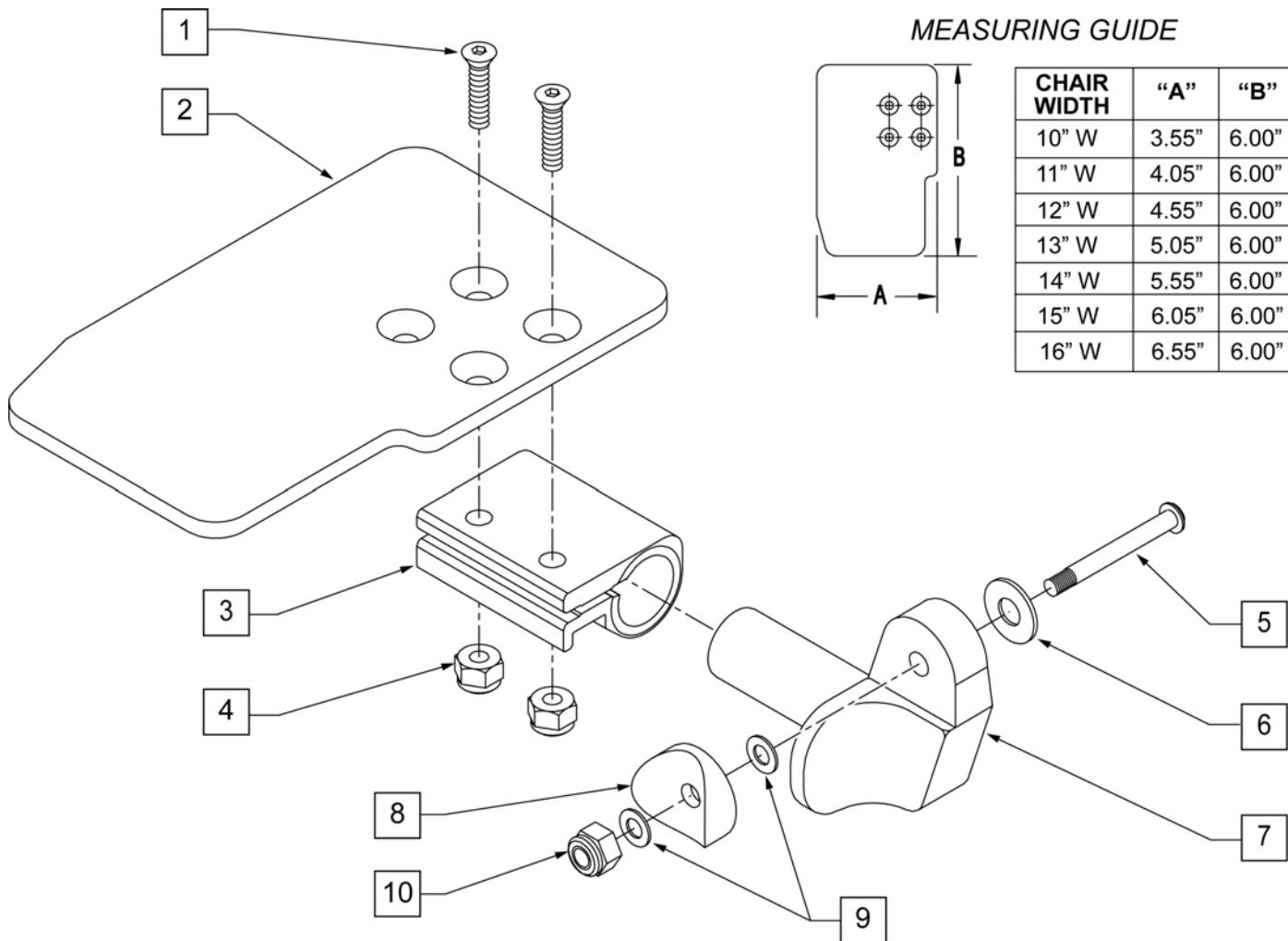

Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1	106978	FHCS 1/4-28 X 7/8 ROHS		7	541001	FTRST MOUNTING BRKT 1 RAD	LEFT
2	500635	KIDS ANGL ADJ F/PLT 10	RIGHT	8	561004	1 SADDLE 1/4 HOLE	
2	500636	KIDS ANGL ADJ F/PLT 10	LEFT	9	106943	WASHER FLAT THIN AN 1/4	
2	500637	KIDS ANGL ADJ F/PLT 11	RIGHT	10	106941	NUT AUTOMATION 1/4-28 ROHS	
2	500638	KIDS ANGL ADJ F/PLT 11	LEFT	1-10	828821	KIDS ANG ADJ F/PLT ASSM 10	RIGHT
2	500639	KIDS ANGL ADJ F/PLT 12	RIGHT	1-10	828822	KIDS ANG ADJ F/PLT ASSM 10	LEFT
2	500640	KIDS ANGL ADJ F/PLT 12	LEFT	1-10	828823	KIDS ANG ADJ F/PLT ASSM 11	RIGHT
2	500641	KIDS ANGL ADJ F/PLT 13	RIGHT	1-10	828824	KIDS ANG ADJ F/PLT ASSM 11	LEFT
2	500642	KIDS ANGL ADJ F/PLT 13	LEFT	1-10	828825	KIDS ANG ADJ F/PLT ASSM 12	RIGHT
2	500643	KIDS ANGL ADJ F/PLT 14	RIGHT	1-10	828826	KIDS ANG ADJ F/PLT ASSM 12	LEFT
2	500644	KIDS ANGL ADJ F/PLT 14	LEFT	1-10	828827	KIDS ANG ADJ F/PLT ASSM 13	RIGHT
2	500645	KIDS ANGL ADJ F/PLT 15	RIGHT	1-10	828828	KIDS ANG ADJ F/PLT ASSM 13	LEFT
2	500646	KIDS ANGL ADJ F/PLT 15	LEFT	1-10	828829	KIDS ANG ADJ F/PLT ASSM 14	RIGHT
2	500647	KIDS ANGL ADJ F/PLT 16	RIGHT	1-10	828830	KIDS ANG ADJ F/PLT ASSM 14	LEFT
2	500648	KIDS ANGL ADJ F/PLT 16	LEFT	1-10	828831	KIDS ANG ADJ F/PLT ASSM 15	RIGHT
3	550350	3/4 CLAMP ADJ FOOTPLATE		1-10	828832	KIDS ANG ADJ F/PLT ASSM 15	LEFT
4	106935	NUT NYLOCK THIN 1/4-28 ROHS		1-10	828833	KIDS ANG ADJ F/PLT ASSM 16	RIGHT
5	226147	1/4-28 X 1.85 BH C/S BLK ZN		1-10	828834	KIDS ANG ADJ F/PLT ASSM 16	LEFT
6	106945	WASHER FLAT SAE 1/4 ROHS		1-10	921226	KIDS ANG ADJ F/PLT ASSM 10	PAIR
7	541000	FTRST MOUNTING BRKT 1 RAD	RIGHT	1-10	921227	KIDS ANG ADJ F/PLT ASSM 11	PAIR


Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1-10	921228	KIDS ANG ADJ F/PLT ASSM 12	PAIR	1-10	921231	KIDS ANG ADJ F/PLT ASSM 15	PAIR
1-10	921229	KIDS ANG ADJ F/PLT ASSM 13	PAIR	1-10	921232	KIDS ANG ADJ F/PLT ASSM 16	PAIR
1-10	921230	KIDS ANG ADJ F/PLT ASSM 14	PAIR				

## ADULT ANGLE-ADJ FOOTPLATE FRONT MOUNT

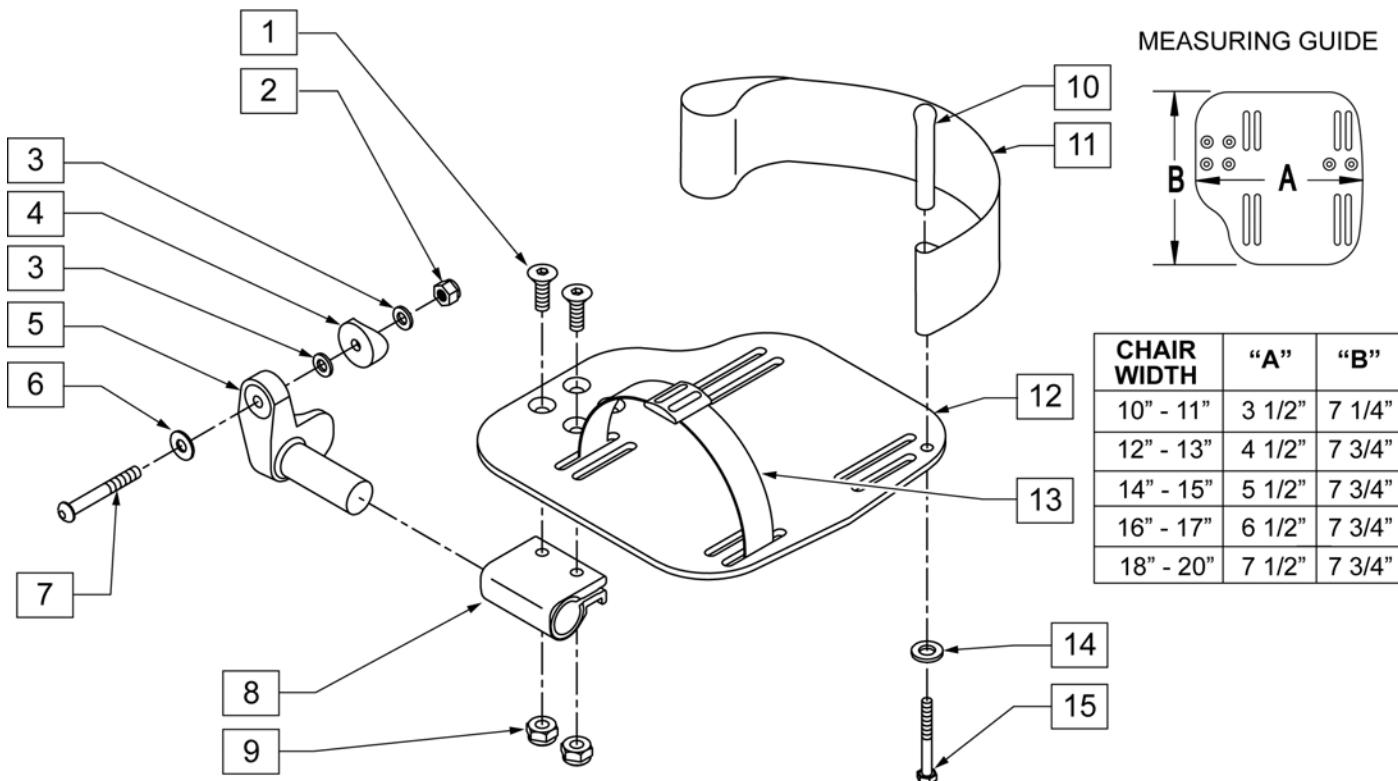


Pos.	Item Number	Description	Remarks
1	106978	FHCS 1/4-28 X 7/8 ROHS	
2	106941	NUT AUTOMATION 1/4-28 ROHS	
3	106943	WASHER FLAT AN 1/4 ROHS	
4	561004	1" SADDLE 1/4" HOLE	
5	541000	FOOTREST MOUNTING BRKT-1" RAD	RIGHT
5	541001	FOOTREST MOUNTING BRKT-1" RAD	LEFT
6	106945	WASHER FLAT SAE 1/4 ROHS	
7	106964	BHCS 1/4-28 X 2	
8	550350	3/4" CLAMP ADJ FOOTPLATE	
9	106935	NUT NYLOCK THIN 1/4-28 ROHS	
10	561040	HEEL LOOP PEG	
11	741153	HEEL LOOP 18-20 CMPST/14-15 ANG ADJ	USED W/ 14-15"/16-17" ADULT ANG ADJ FTPLTS
11	741253	HEEL LOOP ADLT ADJ F/PLT 18-20	USED W/ 18 -20" ADULT ANG ADJ FTPLTS
12	500656	ADULT ANGL ADJ F/PLT 10/11	RIGHT
12	500657	ADULT ANGL ADJ F/PLT 10/11	LEFT
12	500658	ADULT ANGL ADJ F/PLT 12/13	RIGHT
12	500659	ADULT ANGL ADJ F/PLT 12/13	LEFT
12	500660	ADULT ANGL ADJ F/PLT 14/15	RIGHT
12	500661	ADULT ANGL ADJ F/PLT 14/15	LEFT
12	500662	ADULT ANGL ADJ F/PLT 16/17	RIGHT
12	500663	ADULT ANGL ADJ F/PLT 16/17	LEFT
12	500664	ADULT ANGL ADJ F/PLT 18-20	RIGHT
12	500665	ADULT ANGL ADJ F/PLT 18-20	LEFT
13	740260	TOE-ANKLE STRAP ADJ ADULT FOOTPLATE	
14	106943	WASHER FLAT AN 1/4 ROHS	

Pos.	Item Number	Description	Remarks
15	106958	HHCS 1/4-20 X 1 ROHS	
1-9,12	820544	FP ADULT ADJ 90D 10/11 R	RIGHT
1-9,12	820554	FP ADULT ADJ 90D 10/11 L	LEFT
1-9,12	820546	FP ADULT ADJ 90D 12/13 R	RIGHT
1-9,12	820556	FP ADULT ADJ 90D 12/13 L	LEFT
1-9,12	820472	FP ADULT ADJ 90D 14/15 R	RIGHT
1-9,12	820473	FP ADULT ADJ 90D 14/15 L	LEFT
1-9,12	820474	FP ADULT ADJ 90D 16/17 R	RIGHT
1-9,12	820475	FP ADULT ADJ 90D 16/17 L	LEFT
1-9,12	820476	FP ADULT ADJ 90D 18-20 R	RIGHT
1-9,12	820477	FP ADULT ADJ 90D 18-20 L	LEFT
1-9,12	921474	ADULT ANGL ADJ 90D 10/11 OMIT H/L	PAIR
1-9,12	921476	ADULT ANGL ADJ 90D 12/13 OMIT H/L	PAIR
1-9,12	921330	ADULT ANGL ADJ 90D 14/15 OMIT H/L	PAIR
1-9,12	921331	ADULT ANGL ADJ 90D 16/17 OMIT H/L	PAIR
1-9,12	921332	ADULT ANGL ADJ 90D 18-20 OMIT H/L	PAIR
1-12,14,15	820540	F/PLT ADULT ADJ 90D 10/11 R W/ HL	RIGHT
1-12,14,15	820550	F/PLT ADULT ADJ 90D 10/11 L W/ HL	LEFT
1-12,14,15	820542	F/PLT ADULT ADJ 90D 12/13 R W/ HL	RIGHT
1-12,14,15	820552	F/PLT ADULT ADJ 90D 12/13 L W/ HL	LEFT
1-12,14,15	820478	F/PLT ADULT ADJ 90D 14/15 R W/ HL	RIGHT
1-12,14,15	820479	F/PLT ADULT ADJ 90D 14/15 L W/ HL	LEFT
1-12,14,15	820480	F/PLT ADULT ADJ 90D 16/17 R W/ HL	RIGHT
1-12,14,15	820481	F/PLT ADULT ADJ 90D 16/17 L W/ HL	LEFT
1-12,14,15	820482	F/PLT ADULT ADJ 90D 18-20 R W/ HL	RIGHT
1-12,14,15	820483	F/PLT ADULT ADJ 90D 18-20 L W/ HL	LEFT
1-12,14,15	921470	F/PLT ADULT ADJ 90D 10/11 W/ HL	PAIR
1-12,14,15	921472	F/PLT ADULT ADJ 90D 12/13 W/ HL	PAIR
1-12,14,15	921333	F/PLT ADULT ADJ 90D 14/15 W/ HL	PAIR
1-12,14,15	921334	F/PLT ADULT ADJ 90D 16/17 W/ HL	PAIR
1-12,14,15	921335	F/PLT ADULT ADJ 90D 18-20 W/ HL	PAIR

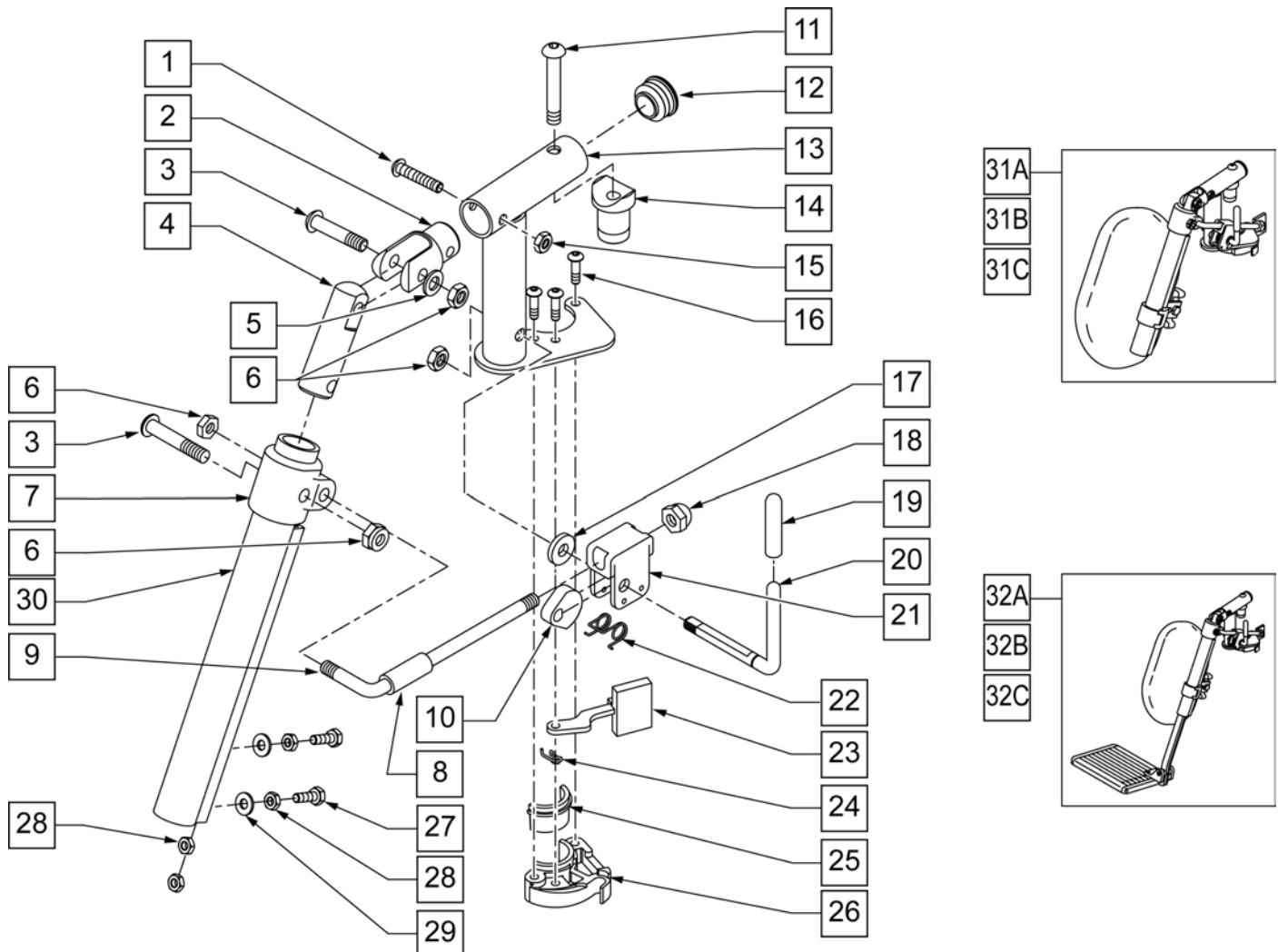

Pos.	Item Number	Description	Remarks
1	106961	1/4-28 X 1 1/4 BH C/S	
2	505281	ELR CLEVIS	
3	107037	BHCS 5/16-24 X 1-1/2 SHLD ROHS	
4	540252	ELR PIVOT PRS	
5	106946	WASHER FLAT AN 5/16 ROHS	
6	106939	NUT NYLOCK THIN 5/16-24 ROHS	
7	560420	ELR UPPER PIVOT PLST	
8	560784	ELR ADJUSTMENT STOP	
9	481117	ELR LOCK ROD	
10	481119	ELR CAM LOCKING POWDERED METAL	
11	106982	BHCS 5/16 -18 x 1 3/4 SHLD ROHS	
12	561062	1" INSERT PLUG .065 RIBS	
14	122729	ELR ALUM PIVOT SADDLE	
15	106935	NUT NYLOCK THIN 1/4-28 ROHS	
16	107035	BHCS 10-24 X .725 PL ROHS	
17	561059	3/4 OD x .318 ID WASHER	
18	202109	5/16-24 ACORN NUT HEX	
19	561049	ELR RELEASE ROD COVER	
19,20	882410S	ELR ACTUATION ARM ASSEMBLY	


<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
21	880990	ELR LOCK HOUSING W/PIN	
22	321106	ELR TORSION SPRING	
23	561079	S/A LATCH LEVER	
24	321136	S/A SPRING L RED	LEFT
24	321137	S/A SPRING R CLEAR	RIGHT
27	107014	1/4-28 x 3/4 HEX C/S	
28	107249	NUT HEX THIN 1/4-28 ROHS	
29	106944	WASHER FLAT THIN AN 1/4 ROHS	
30	550722S	ELR ADJUSTMENT TUBE	
6,8-10,17-22	881077S	ELR LOCK ROD/ARM ASSEMBLY	
25,26	881500	LATCH BLOCK ASSM	RIGHT
25,26	881501	LATCH BLOCK ASSM	LEFT
1-6,11-16,20-24	882413	ELR PIVOT BLOCK ASSM	RIGHT
1-6,11-16,20-24	882414	ELR PIVOT BLOCK ASSM	LEFT
1-26	881422	ELR PIVOT ASSM RIGHT	RIGHT
1-26	881423	ELR PIVOT ASSM LEFT	LEFT
1-30	641452S	ELR Hngr W/O PAD PNTD MNL	RIGHT
1-30	641453S	ELR Hngr W/O PAD PNTD MNL	LEFT
31A	923082S	ELR Hngr ASSM 14-15 W /PAD (PR)	PAIR
31B	923041S	ELR Hngr ASSM W/PAD 16-17"	PAIR
31C	923061S	ELR Hngr ASSM W/PAD 18-20"	PAIR
32A	923088S	ELR Hngr REG EXT 14 15" F/PLT	Pair-includes short extensions, 11.5" - 15.5"
32B	923084S	ELR Hngr REG EXT 16 17" F/PLT	Pair-includes short extensions, 11.5" - 15.5"
32C	923085S	ELR Hngr REG EXT 18-20" F/PLT	Pair-includes short extensions, 11.5" - 15.5"
16,23-26	881521	S/A LATCH KIT CASTING RIGHT	RIGHT
16,23-26	881522	S/A LATCH KIT CASTING LEFT	LEFT

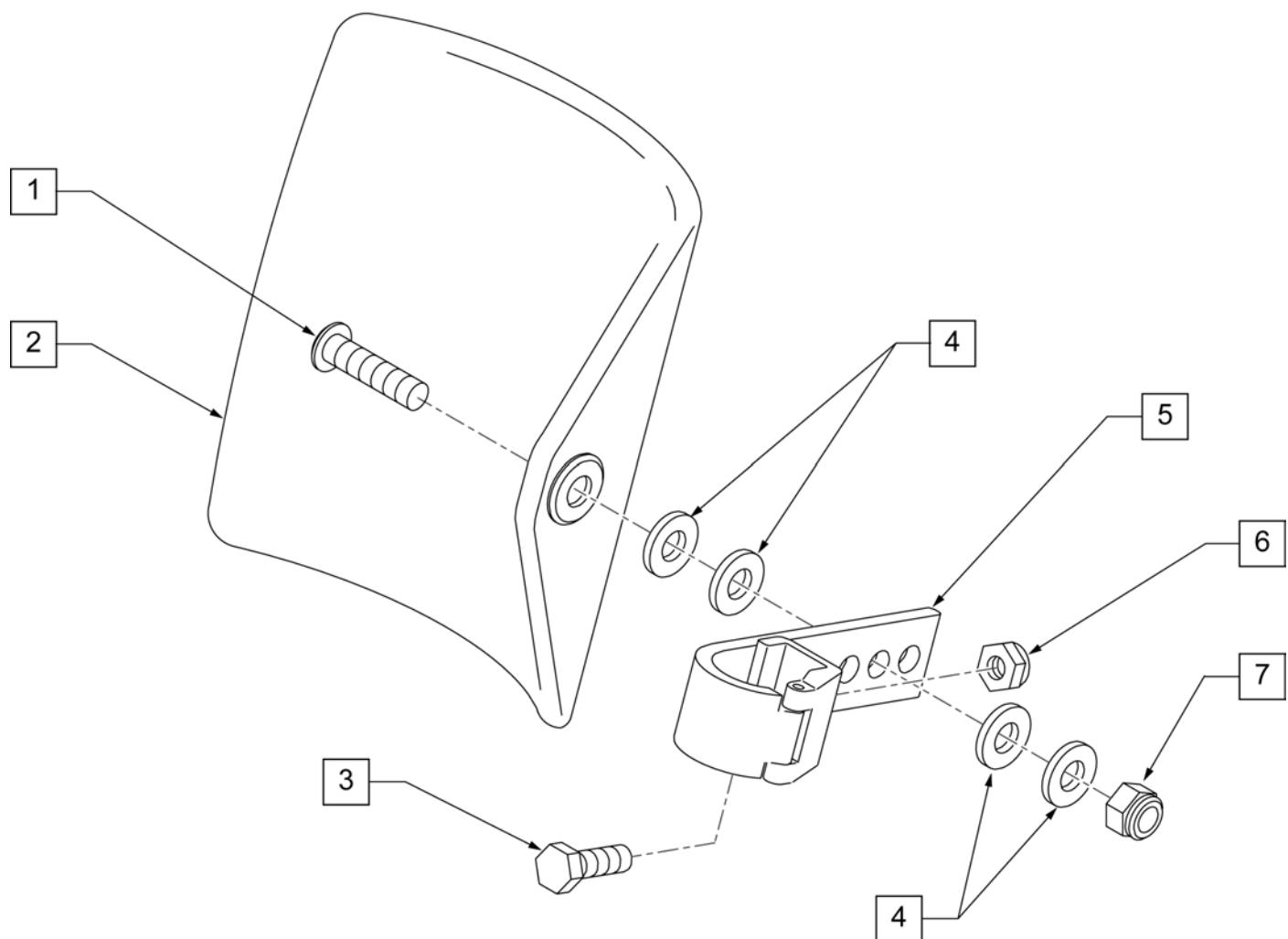

Pos.	Item Number	Description
1	106980	BHCS 5/16-18 X 1-1/4 ROHS
2	755014	CALF PAD TROUGH STYLE 14"/16"
3	107013	HHCS 1/4-28 X 5/8 ROHS
4	561016	1" WASHER 5/16" HOLE
5	881413	CALF PAD BRACKET ASSEMBLY
6	106934	NUT NYLOCK 1/4-28 ROHS
7	202117	5/16-18 NYLOCK NUT THICK BLK
1-7	881414	CALF PAD J-HOOK 14"-15" ASSEMBLY

**ELEVATING HANGER CALF PAD 16-20"**


[03/2011]

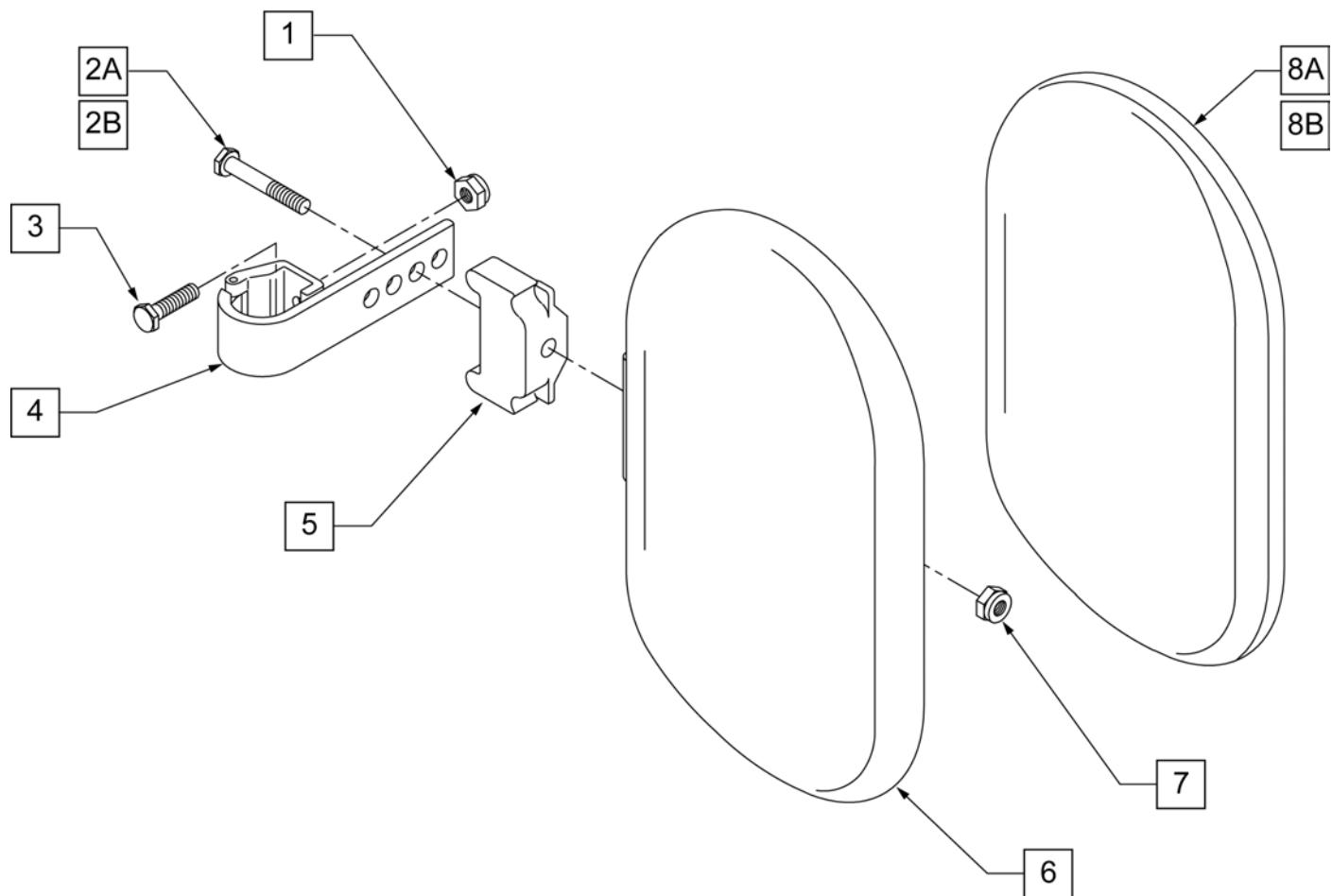

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
1	106934	NUT NYLOCK 1/4-28 ROHS	
2A	107018	HHCS 5/16-18 X 2-1/2 ROHS	16"-17" wide only
2B	107019	HHCS 5/16-18 X 3-1/2 ROHS	18"-20" wide only
3	107013	HHCS 1/4-28 X 5/8 ROHS	
4	881413	CALF PAD BRACKET ASSEMBLY	
5	560199	ELR CALF HINGE SPACER	18"-20" wide only
6	560108	CALF PAD MOLDED 16"-20"	
7	106936	NUT NYLOCK 5/16-18 ROHS	
8A	751045	CALF PAD COVER	
8B	751046	CALF PAD COVER (BREEZY)	
1,2A,3,4,6,7,8A	881416	CALF MOLDED 16"-17" ASSEMBLY	
1,2B,3-7,8A	881412	CALF PAD MOLDED 18"-20" ASSEMBLY	

**NOTE:** IF REPLACEING AN EXTENSION TUBE ON AN MANUAL ALR PLEASE REFER TO THE ROUND EXT TUBE PARTS MANUAL PAGE.


**NOTE:** WHEN ORDERING REPLACEMENT EXTENSION TUBES PLEASE VERIFY IF YOU HAVE THE OLD STYLE DOUBLE D EXTENSION TUBE OR THE NEW STYLE DOUBLE D EXTENSION TUBE. THIS CAN BE DONE BY REVIEWING THE HANGER THE EXTENSION TUBE IS USED ON. IF THE HANGER UTILIZES A SINGLE SET SCREW AT THE BOTTOM OF THE HANGER TUBE TO ATTACH THE EXTENSION TUBE TO THE HANGER, THEN THIS UTILIZES THE NEW STYLE DOUBLE D EXT TUBE ALL OTHER HANGERS REQUIRING A DOUBLE D EXTENSION TUBE UTILIZE THE OLD STYLE.

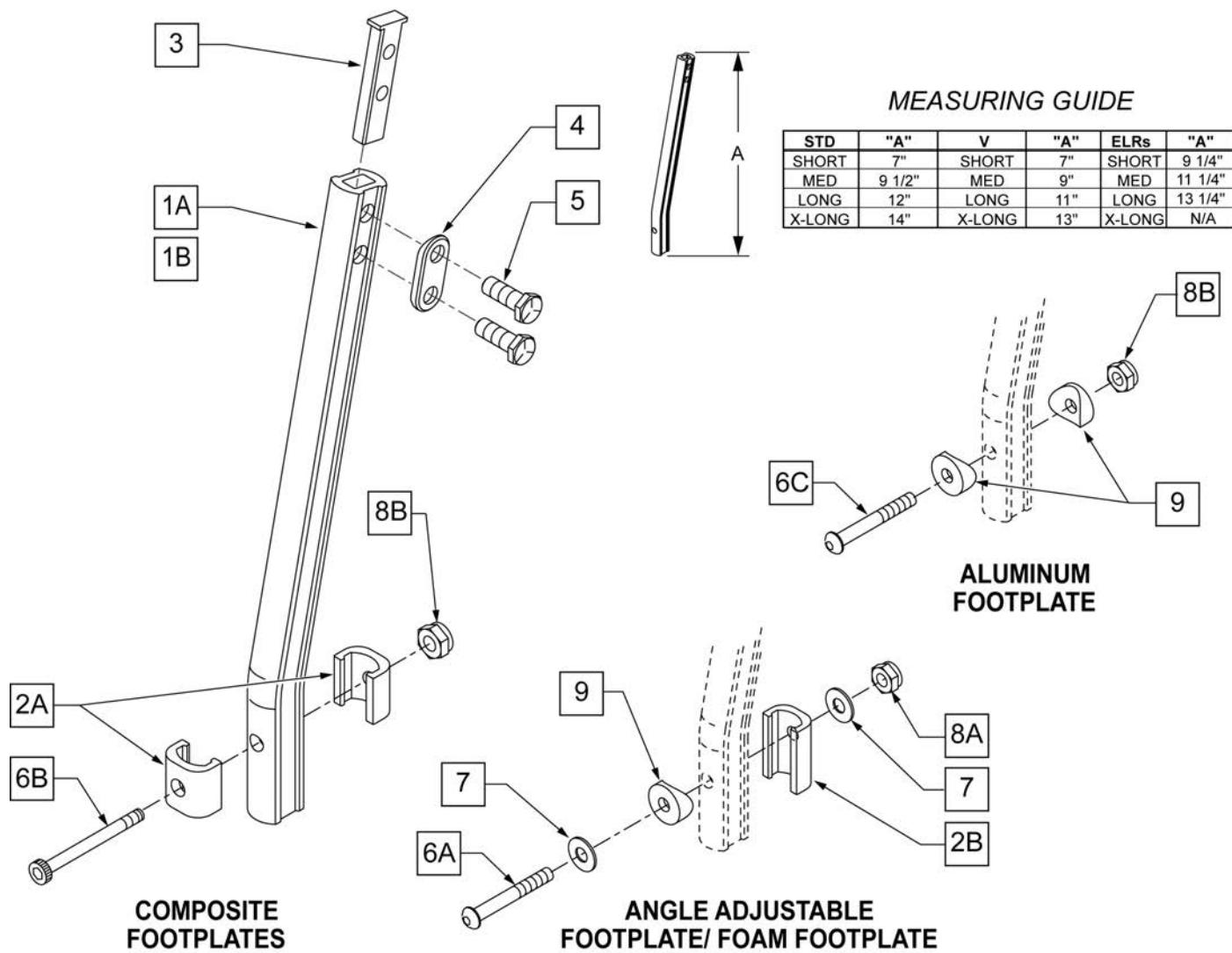

Pos.	Item Number	Description	Remarks
1A	550819	EXT TUBE S/A ONLY SHT	Short Ext-Swing Away Hanger
1A	550820	EXT TUBE S/A ONLY MED	Med Ext-Swing Away Hanger
1A	550821	EXT TUBE S/A ONLY LONG	Long Ext-Swing Away Hanger
1A	550822	EXT TUBE S/A ONLY X-LONG	X-Long Ext-Swing Away Hanger
1B	550834	EXT TUBE V IN ONLY D2 SHT R	Right Short Ext-Swing Away Hanger
1B	550835	EXT TUBE V IN ONLY D2 SHT L	Left Short Ext-Swing Away Hanger

Pos.	Item Number	Description	Remarks
1B	550836	EXT TUBE V IN ONLY D2 MED R	Right Med Ext-Swing Away Hanger
1B	550837	EXT TUBE V IN ONLY D2 MED L	Left Med Ext-Swing Away Hanger
1B	550838	EXT TUBE V IN ONLY D2 LONG R	Right Long Ext-Swing Away Hanger
1B	550839	EXT TUBE V IN ONLY D2 LONG L	Left Long Ext-Swing Away Hanger
1B	550840	EXT TUBE V IN ONLY D2 X-LONG R	Right X-Long Ext-Swing Away Hanger
1B	550841	EXT TUBE V IN ONLY D2 X-LONG L	Left X-Long Ext-Swing Away Hanger
2A	483718	HANGER EXTENSION SADDLE	USE W/ COMPOSITE FTPLT
2B	483720	HANGER EXTENSION SADDLE LONG	USE W/ ANGLE ADJ FTPLT
1C	550764	S/A HANGER EXT D2 SHORT BLACK	Short Ext ELR Hangers
1C	550765	S/A HANGER EXT D2 MED BLACK	Medium Ext ELR Hangers
1C	550766	S/A HANGER EXT D2 LONG BLACK	Long Ext ELR Hangers
3	500078	INSERT D2 EXT TUBE	
4	483717	HANGER SADDLE	
5	221102	1/4-28 x 3/4 HEX HEAD C/S BLK W/PTCH	
6A	106962	1/4-28 x 1 3/4 BH C/S BLK ZN	USE W/ ANGLE ADJ FTPLT
6B	226165	1/4 -28 x 2 1/2 SOC C/S	USE W/ COMPOSITE FTPLT
6C	226147	1/4 -28 X 1.85 BH C/S	
7	106945	1/4" SAE WASHER	
8A	106941	1/4 -28 AUTOMATION NUT	USE W/ ANGLE ADJ FTPLT
8B	106935	1/4 -28 NYLOCK NUT THIN	USE W/ COMPOSITE FTPLT
9	561055	3/4 SADDLE 1" OD	USE W/ ANGLE ADJ FTPLT
1A,2A-5	921759	EXT TUBE S/A SHT ASSM PCKT	USE W/ COMPOSITE FTPLT
1A,2A-5	921760	EXT TUBE S/A MED ASSM PCKT	USE W/ COMPOSITE FTPLT
1A,2A-5	921761	EXT TUBE S/A LONG ASSM PCKT	USE W/ COMPOSITE FTPLT
1A,2A-5	921762	EXT TUBE S/A X-LONG ASSM PCKT	USE W/ COMPOSITE FTPLT
1B,2A-5	921751	EXT TUBE "V" D2 SHT R ASSM PCKT	RIGHT - USE W/ COMPOSITE FTPLT V HANGER
1B,2A-5	921752	EXT TUBE "V" D2 SHT L ASSM PCKT	LEFT - USE W/ COMPOSITE FTPLT V HANGER
1B,2A-5	921753	EXT TUBE "V" D2 MED R ASSM PCKT	RIGHT - USE W/ COMPOSITE FTPLT V HANGER
1B,2A-5	921754	EXT TUBE "V" D2 MED L ASSM PCKT	LEFT - USE W/ COMPOSITE FTPLT V HANGER
1B,2A-5	921755	EXT TUBE "V" D2 LONG R ASSM PCKT	RIGHT - USE W/ COMPOSITE FTPLT V HANGER
1B,2A-5	921756	EXT TUBE "V" D2 LONG L ASSM PCKT	LEFT - USE W/ COMPOSITE FTPLT V HANGER
1B,2A-5	921757	EXT TUBE "V" D2 X-LONG R ASSM PCKT	RIGHT - USE W/ COMPOSITE FTPLT V HANGER
1B,2A-5	921758	EXT TUBE "V" D2 X-LONG L ASSM PCKT	LEFT - USE W/ COMPOSITE FTPLT V HANGER
1A,2A-7,9	921772	EXT TUBE S/A SHT ANG ADJ PCKT	USE W/ ANGLE ADJ FTPLT
1A,2A-7,9	921773	EXT TUBE S/A MED ANG ADJ PCKT	USE W/ ANGLE ADJ FTPLT
1A,2A-7,9	921774	EXT TUBE S/A LONG ANG ADJ PCKT	USE W/ ANGLE ADJ FTPLT
1A,2A-7,9	921775	EXT TUBE S/A X-LONG ANG ADJ PCKT	USE W/ ANGLE ADJ FTPLT
1B,2A-7,9	921764	EXT TUBE "V" D2 SHT R ANG ADJ PCKT	RIGHT - USE W/ ANGLE ADJ FTPLT V HANGER
1B,2A-7,9	921765	EXT TUBE "V" D2 SHT L ANG ADJ PCKT	LEFT - USE W/ ANGLE ADJ FTPLT V HANGER
1B,2A-7,9	921766	EXT TUBE "V" D2 MED R ANG ADJ PCKT	RIGHT - USE W/ ANGLE ADJ FTPLT V HANGER
1B,2A-7,9	921767	EXT TUBE "V" D2 MED L ANG ADJ PCKT	LEFT - USE W/ ANGLE ADJ FTPLT V HANGER
1B,2A-7,9	921768	EXT TUBE "V" D2 LONG R ANG ADJ PCKT	RIGHT - USE W/ ANGLE ADJ FTPLT V HANGER
1B,2A-7,9	921769	EXT TUBE "V" D2 LONG L ANG ADJ PCKT	LEFT - USE W/ ANGLE ADJ FTPLT V HANGER
1B,2A-7,9	921770	EXT TUBE "V" D2 X-LONG R ANG AD PCKT	RIGHT - USE W/ ANGLE ADJ FTPLT V HANGER
1B,2A-7,9	921771	EXT TUBE "V" D2 X-LONG L ANG AD PCKT	LEFT - USE W/ ANGLE ADJ FTPLT V HANGER

## COMPOSITE FOOTPLATE

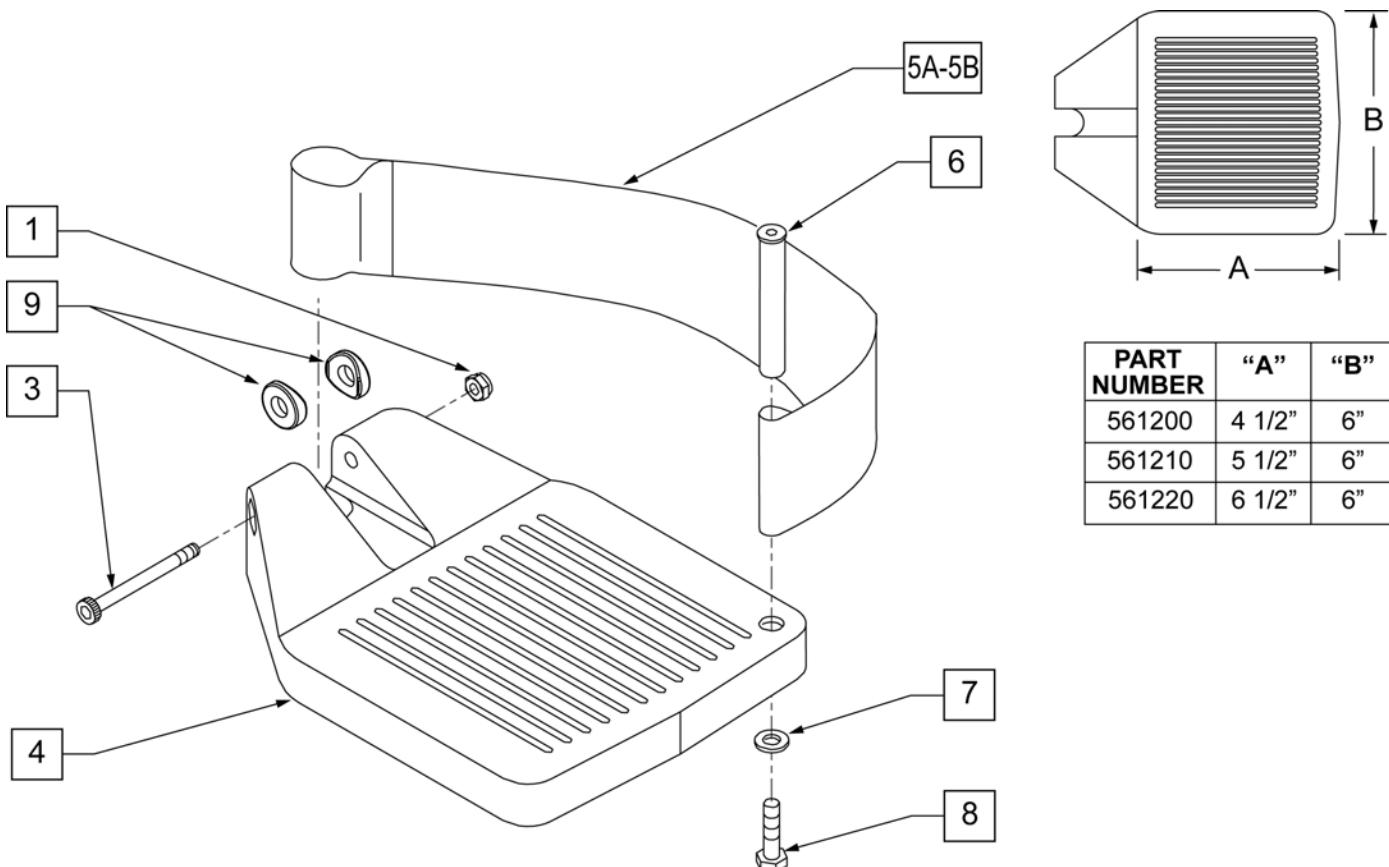


PART NUMBER	"A"	"B"
561200	4 1/2"	6"
561210	5 1/2"	6"
561220	6 1/2"	6"

Pos.	Item Number	Description	Remarks
1	106935	1/4 - 28 NYLOCK NUT	
3	226165	1/4 - 28 x 2 1/2 SOC C/S	
4	561200	COMPOSITE FOOTPLATE 14 -15 (16 -18 V )	14 -15 OR 16 -18V
4	561210	COMPOSITE FOOTPLATE 16 -17 (19 -20 V )	16 -17 OR 19 -20V
4	561220	COMPOSITE FOOTPLATE 18 -20	18 -20
5A	741151	HEEL LOOP 14-15 COMPOSITE (14 -15 or 16 -18V)	14 -15 OR 16 -18V
5A	741156	HEEL LOOP 16 -17 or 19 -20V	16 -17 OR 19 -20V
5A	741153	HEEL LOOP 18 -20 COMPOSITE/14-15 ANG ADJ	18 -20
5A	902270	HEEL LOOP COMPOSITE FT 14-15	PAIR 14 -15 OR 16 -18V
5A	902271	HEEL LOOP COMPOSITE FT 16-17	PAIR 16 -17 OR 19 -20V
5A	902272	HEEL LOOP COMPOSITE FT 18-20	PAIR 18 -20
5B	741154	HEEL LOOP 14-15 COMPOSITE HEMI	14 -15 HEMI
5B	741155	HEEL LOOP 16-17 COMPOSITE HEMI	16 -17 HEMI
5B	741153	HEEL LOOP 18-20 COMPOSITE HEMI	18 -10 HEMI
5B	902260	HEEL LOOPS HEMI COMPOSITE 14-15	PAIR 14 -15 HEMI
5B	902261	HEEL LOOPS HEMI COMPOSITE 16-17	PAIR 16 -17 HEMI
5B	902262	HEEL LOOPS HEMI COMPOSITE 18-20	PAIR 18 -20 HEMI
6	561040	HEEL LOOP PEG	
7	106944	1/4 AN WASHER FLAT THIN	
8	106958	HHCS 1/4-20 X 1 ROHS	
9	561003	3/4" SADDLE THIN	USED W/ ROUND EXT TUBES ONLY
1,3,4	921180	COMPOSITE FOOTPLATE ASSM 14-15	RIGHT (14 -15 OR 16 -18V) W/O HEEL LOOP
1,3,4	921181	COMPOSITE FOOTPLATE ASSM 14-15	LEFT (14 -15 OR 16 -18V) W/O HEEL LOOP
1,3,4	921182	COMPOSITE FOOTPLATE ASSM 16-17	RIGHT (16 -17 OR 19 -20V) W/O HEEL LOOP
1,3,4	921183	COMPOSITE FOOTPLATE ASSM 16-17	LEFT (16 -17 OR 19 -20V) W/O HEEL LOOP

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
1,3,4	921184	COMPOSITE FOOTPLATE ASSM 18-20	RIGHT (18 -20) W/O HEEL LOOP
1,3,4	921185	COMPOSITE FOOTPLATE ASSM 18-20	LEFT (18 -20) W/O HEEL LOOP
1,3,4	921151	COMPOSITE FOOTPLATE ASSM 14-15/16-18V	PAIR (14 -15 OR 16 -18V) W/O HEEL LOOP
1,3,4	921152	COMPOSITE FOOTPLATE ASSM 16-17/19-20V	PAIR (16 -17 OR 19 -20V) W/O HEEL LOOP
1,3,4	921153	COMPOSITE FOOTPLATE ASSM 18-20	PAIR (18 -20) W/O HEEL LOOP
1,3,4A-8	921190	COMPOSITE FOOTPLATE ASSM HL LP 14-15	RIGHT (14 -15 OR 16 -18V) W/ HEEL LOOP
1,3,4A-8	921191	COMPOSITE FOOTPLATE ASSM HL LP 14-15	LEFT (14 -15 OR 16 -18V) W/ HEEL LOOP
1,3,4A-8	921192	COMPOSITE FOOTPLATE ASSM HL LP 16-17	RIGHT (16 -17 OR 19 -20V) W/ HEEL LOOP
1,3,4A-8	921193	COMPOSITE FOOTPLATE ASSM HL LP 16-17	LEFT (16 -17 OR 19 -20V) W/ HEEL LOOP
1,3,4A-8	921194	COMPOSITE FOOTPLATE ASSM HL LP 18-20	RIGHT (18 -20) W/ HEEL LOOP
1,3,4A-8	921195	COMPOSITE FOOTPLATE ASSM HL LP 18-20	LEFT (18 -20) W/ HEEL LOOP
1,3,4A-8	921161	COMPOSITE FOOTPLATE ASSM HL 14-15/16-18V	PAIR (14 -15 OR 16 -18V) W/ HEEL LOOP
1,3,4A-8	921162	COMPOSITE FOOTPLATE ASSM HL 16-17/19-20V	PAIR (16 -17 OR 19 -20V) W/ HEEL LOOP
1,3,4A-8	921163	COMPOSITE FOOTPLATE ASSM HL 18-20	PAIR (18 -20) W/ HEEL LOOP
1,3,4B-8	921140	COMPOSITE FOOTPLATE ASSM HEMI HL LP 14-15	RIGHT (14 -15) W/ HEEL LOOP
1,3,4B-8	921141	COMPOSITE FOOTPLATE ASSM HEMI HL LP 14-15	LEFT (14 -15) W/ HEEL LOOP
1,3,4B-8	921142	COMPOSITE FOOTPLATE ASSM HEMI HL LP 16-17	RIGHT (16 -17) W/ HEEL LOOP
1,3,4B-8	921143	COMPOSITE FOOTPLATE ASSM HEMI HL LP 16-17	LEFT (16 -17) W/ HEEL LOOP
1,3,4B-8	921144	COMPOSITE FOOTPLATE ASSM HEMI HL LP 18-20	RIGHT (18 -20) W/ HEEL LOOP
1,3,4B-8	921145	COMPOSITE FOOTPLATE ASSM HEMI HL LP 18-20	LEFT (18 -20) W/ HEEL LOOP
1,3,4B-8	921167	COMPOSITE FOOTPLATE ASSM HEMI HL LPS 14-15	PAIR (14 -15) W/ HEEL LOOP
1,3,4B-8	921168	COMPOSITE FOOTPLATE ASSM HEMI HL LPS 16-17	PAIR (16 -17) W/ HEEL LOOP
1,3,4B-8	921169	COMPOSITE FOOTPLATE ASSM HEMI HL LPS 18-20	PAIR (18 -20) W/ HEEL LOOP

## KIDS ANGLE ADJUSTABLE FOOTPLATE EXT MOUNT

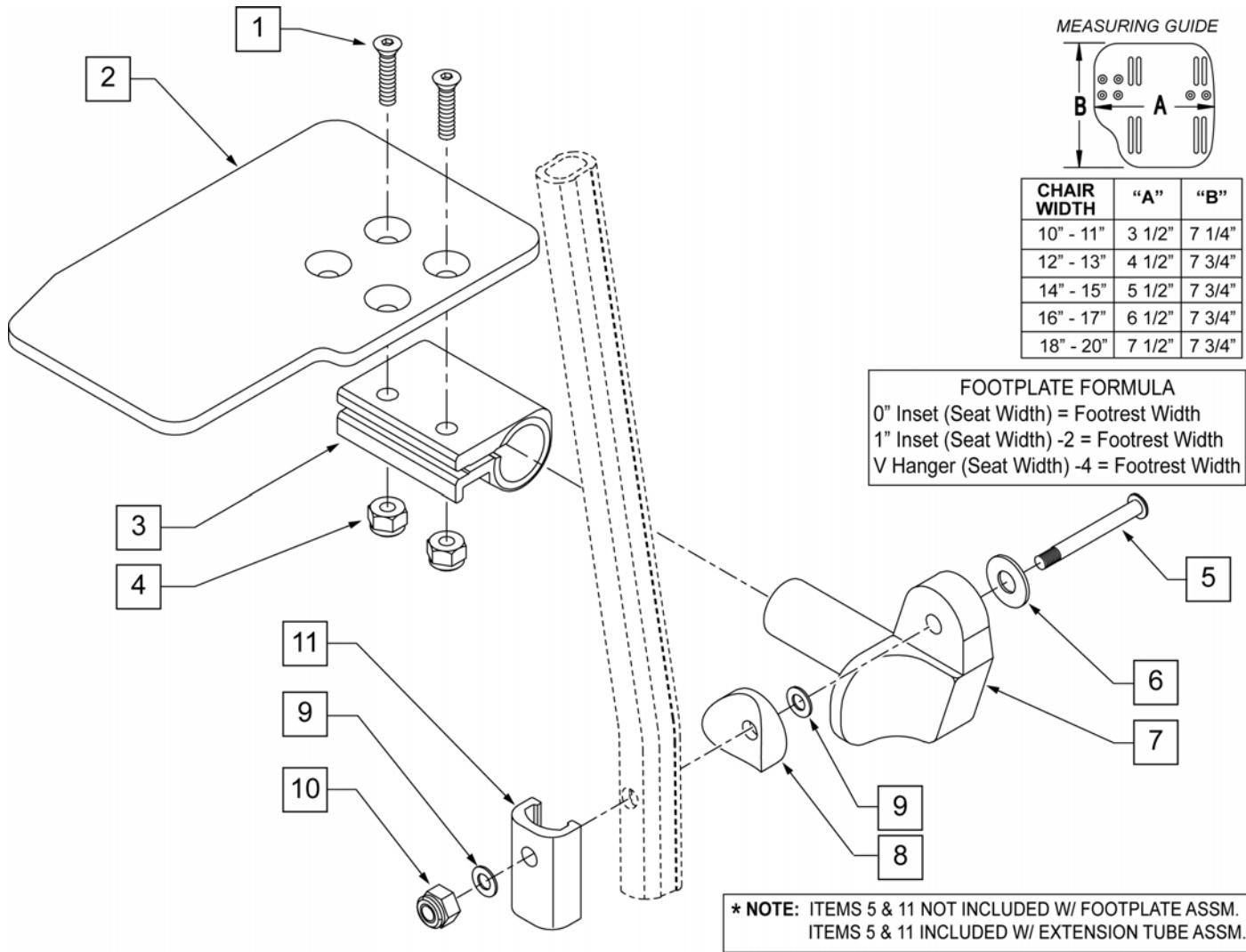

Pos.	Item Number	Description	Remarks
1	106978	FHCS 1/4 -28 x 7/8 ROHS	
2	500635	KIDS ANGL ADJ F/PLT 10" R	Right
2	500636	KIDS ANGL ADJ F/PLT 10" L	Left
2	500637	KIDS ANGL ADJ F/PLT 11" R	Right
2	500638	KIDS ANGL ADJ F/PLT 11" L	Left
2	500639	KIDS ANGL ADJ F/PLT 12" R	Right
2	500640	KIDS ANGL ADJ F/PLT 12" L	Left
2	500641	KIDS ANGL ADJ F/PLT 13" R	Right
2	500642	KIDS ANGL ADJ F/PLT 13" L	Left
2	500643	KIDS ANGL ADJ F/PLT 14" R	Right
2	500644	KIDS ANGL ADJ F/PLT 14" L	Left
2	500645	KIDS ANGL ADJ F/PLT 15" R	Right
2	500646	KIDS ANGL ADJ F/PLT 15" L	Left
2	500647	KIDS ANGL ADJ F/PLT 16" R	Right
2	500648	KIDS ANGL ADJ F/PLT 16" L	Left
3	550350	3/4" CLAMP ADJ FOOTPLATE	
4	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
5	106962	1/4-28 x 1 3/4 BH C/S BLK ZN (D2 EXTENSION TUBES)	

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
5	226167	1/4-28 x 1.625 BH C/S BLK (ROUND EXTENSION TUBES)	
6	106945	1/4 SAE WASHER	
7	541002	FTRST MOUNTING BRKT-3/8 RAD R	
7	541003	FTRST MOUNTING BRKT-3/8 RAD L	
8	561055	3/4" SADDLE 1" OD	
9	106944	WASHER FLAT THIN AN 1/4 ROHS	
10	106941	NUT AUTOMATION 1/4 -28 ROHS	
1-7,9,10	828850	KIDS ANG ADJ F/PLT EXT-MNT 10"R	Right
1-7,9,10	828851	KIDS ANG ADJ F/PLT EXT-MNT 10"L	Left
1-7,9,10	828852	KIDS ANG ADJ F/PLT EXT-MNT 11"R	Right
1-7,9,10	828853	KIDS ANG ADJ F/PLT EXT-MNT 11"L	Left
1-7,9,10	828854	KIDS ANG ADJ F/PLT EXT-MNT 12"R	Right
1-7,9,10	828855	KIDS ANG ADJ F/PLT EXT-MNT 12"L	Left
1-7,9,10	828856	KIDS ANG ADJ F/PLT EXT-MNT 13"R	Right
1-7,9,10	828857	KIDS ANG ADJ F/PLT EXT-MNT 13"L	Left
1-7,9,10	828858	KIDS ANG ADJ F/PLT EXT-MNT 14"R	Right
1-7,9,10	828859	KIDS ANG ADJ F/PLT EXT-MNT 14"L	Left
1-7,9,10	828860	KIDS ANG ADJ F/PLT EXT-MNT 15"R	Right
1-7,9,10	828861	KIDS ANG ADJ F/PLT EXT-MNT 15"L	Left
1-7,9,10	828862	KIDS ANG ADJ F/PLT EXT-MNT 16"R	Right
1-7,9,10	828863	KIDS ANG ADJ F/PLT EXT-MNT 16"L	Left
1-7,9,10	921280	KIDS ANG ADJ F/PLT EXT-MNT 10"	Pair
1-7,9,10	921281	KIDS ANG ADJ F/PLT EXT-MNT 11"	Pair
1-7,9,10	921282	KIDS ANG ADJ F/PLT EXT-MNT 12"	Pair
1-7,9,10	921283	KIDS ANG ADJ F/PLT EXT-MNT 13"	Pair
1-7,9,10	921284	KIDS ANG ADJ F/PLT EXT-MNT 14"	Pair
1-7,9,10	921285	KIDS ANG ADJ F/PLT EXT-MNT 15"	Pair
1-7,9,10	921286	KIDS ANG ADJ F/PLT EXT-MNT 16"	Pair


**NOTE:** FOR HIGH-MOUNT ADULT ANGLE-ADJUSTABLE FOOTPLATE, SEE "HIGH-MOUNT EXTENSION TUBE" PAGE.

**NOTE:** ORDER ITEM 4B FOR CHAIRS PRIOR TO 8/13/01.

**NOTE:** ONLY 1 WASHER PRIOR TO 8/13/01.

**NOTE:** ORDER PART 226167 PRIOR TO 8/13/01.


**NOTE:** See attached REFERENCE GUIDE.

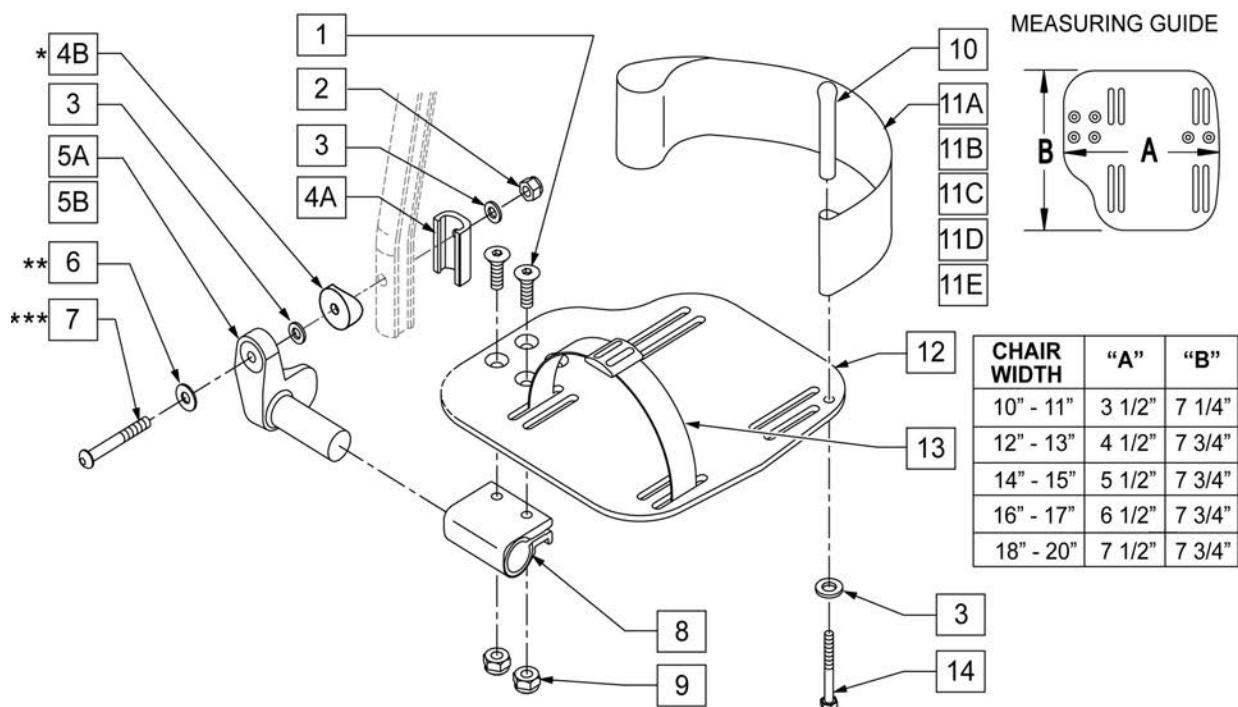

Pos.	Item Number	Description	Remarks
1	106978	1/4-28 x 7/8 FH C/S	
2	106941	1/4-28 AUTOMATION NUT	

Pos.	Item Number	Description	Remarks
3	106943	WASHER FLAT AN 1/4 ROHS	
4A	483720	HANGER EXTENSION SADDLE LONG	USE W/ DOUBLE D EXT TUBE
4B	561055	3/4" SADDLE 1" OD	USE W/ ROUND EXT TUBE
5	541002	FTRST MOUNTING BRKT-3/8 RAD R	RIGHT
5	541003	FTRST MOUNTING BRKT-3/8 RAD L	LEFT
6	106945	WASHER FLAT SAE 1/4 ROHS	
7A	106962	BHCS 1/4-28 X 1-3/4 ROHS	USE W/ DOUBLE D EXT TUBE
7B	226167	1/4-28 x 1.625 BH C/S BLK	USE W/ ROUND EXT TUBE
8	550350	3/4" CLMP ADJ FT PLT	
9	106935	NUT NYLOCK THIN 1/4-28 ROHS	
10	561040	HEEL LOOP PEG	
11A	741154	HEEL LOOP 14-15 CMPST HEMI	USED W/ 10-11" ADULT ANG ADJ FTPLTS
11B	741155	HEEL LOOP 16-17 CMPST HEMI	USED W/ 12 -13" ADULT ANG ADJ FTPLTS
11C	741153	H/L 18-20 CMPST/14-15 ANG ADJ	USED W/ 14-15"/16-17" ADULT ANG ADJ FTPLTS
11D	741253	HEEL LOOP ADLT ADJ F/PLT 18-20	USED W/ 18 -20" ADULT ANG ADJ FTPLTS
12	500656	ADLT ANGL ADJ F/PLT 10/11" R	RIGHT
12	500657	ADLT ANGL ADJ F/PLT 10/11" L	LEFT
12	500658	ADLT ANGL ADJ F/PLT 12/13" R	RIGHT
12	500659	ADLT ANGL ADJ F/PLT 12/13" L	LEFT
12	500660	ADLT ANGL ADJ F/PLT 14/15	RIGHT
12	500661	ADLT ANGL ADJ F/PLT 14/15	LEFT
12	500662	ADLT ANGL ADJ F/PLT 16/17	RIGHT
12	500663	ADLT ANGL ADJ F/PLT 16/17	LEFT
12	500664	ADLT ANGL ADJ F/PLT 18-20	RIGHT
12	500665	ADLT ANGL ADJ F/PLT 18-20	LEFT
13	740260	TOE/ANKLE STRAP ADJ ADLT F/PLT	
14	106958	1/4-20 x 1 HEX C/S	
1-3,5,6,8,9,12	820526	AD ANG ADJ F/PLT R 12"/13" OMIT HL	RIGHT
1-3,5,6,8,9,12	820536	AD ANG ADJ F/PLT L 12"/13" OMT HL	LEFT
1-3,5,6,8,9,12	820460	FP ADLT ADJ 14/15 OMIT HL	RIGHT
1-3,5,6,8,9,12	820462	FP ADLT ADJ 14/15 OMIT HL	LEFT
1-3,5,6,8,9,12	820464	FP ADLT ADJ 16/17 OMIT HL	RIGHT
1-3,5,6,8,9,12	820466	FP ADLT ADJ 16/17 OMIT HL	LEFT
1-3,5,6,8,9,12	820468	FP ADLT ADJ 18-20 OMIT HL	RIGHT
1-3,5,6,8,9,12	820470	FP ADLT ADJ 18-20 OMIT HL	LEFT
1-3,5,6,8,9,12	921466	ADLT ANGL ADJ F/PLT 12"/13" OMIT HL	PAIR
1-3,5,6,8,9,12	921320	FP ADLT ADJ 14/15 OMIT HL	PAIR
1-3,5,6,8,9,12	921322	FP ADLT ADJ 16/17 OMIT HL	PAIR
1-3,5,6,8,9,12	921324	FP ADLT ADJ 18-20 OMIT HL	PAIR
1-3,5,6,8,9,12	820522	AD ANGL ADJ F/PLT R 12/13" H/L	RIGHT
1-3,5,6,8,9,12	820532	AD ANGL ADJ F/PLT L 12/13" H/L	LEFT
1-3,5,6,8-10,11C, 12,14	820461	FP ADLT ADJ 14/15 HL	RIGHT
1-3,5,6,8-10,11C, 12,14	820463	FP ADLT ADJ 14/15 HL	LEFT
1-3,5,6,8-10,11C, 12,14	820465	FP ADLT ADJ 16/17 HL	RIGHT
1-3,5,6,8-10,11C, 12,14	820467	FP ADLT ADJ 16/17 HL	LEFT
1-3,5,6,8-10,11D, 12,14	820469	FP ADLT ADJ 18-20 HL	RIGHT
1-3,5,6,8-10,11D, 12,14	820471	FP ADLT ADJ 18-20 HL	LEFT
1-3,5,6,8-10,11B, 12,14	921462	ADLT ANGL ADJ F/PLATE 12"/13" HL	PAIR
1-3,5,6,8-10,11C, 12,14	921321	FP ADLT ADJ 14/15 HL	PAIR
1-3,5,6,8-10,11C, 12,14	921323	FP ADLT ADJ 16/17 HL	PAIR
1-3,5,6,8-10,11D, 12,14	921325	FP ADLT ADJ 18-20 HL	PAIR
1-3,6,7A,9-10,14	101533	ADLT ANG ADJ FP D2 HDWR KIT	FOR DOUBLE D EXT TUBES
1-3B,4B,6,7B,9,10,14	101534	ADLT ANG ADJ FP RND HDWR KIT	FOR ROUND EXT TUBES (Q2 HP ONLY)

**NOTE:** Left Side Shown

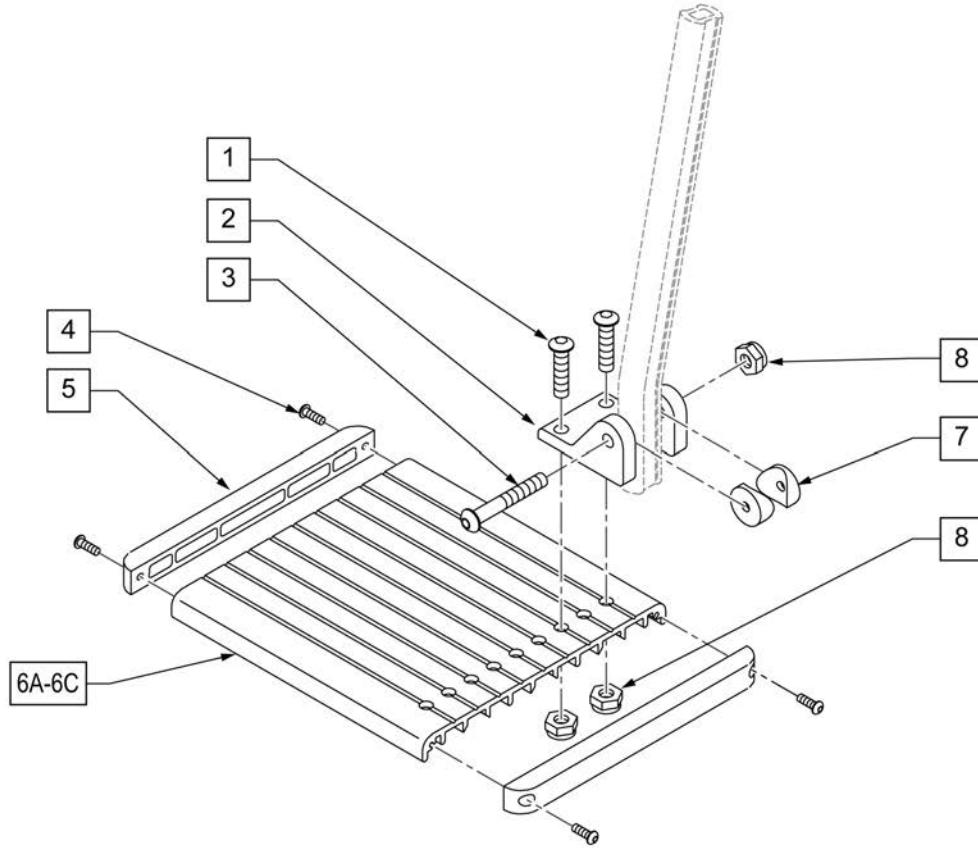


Pos.	Item Number	Description	Remarks
1	106959	1/4-28 x 5/8 BH C/S	
2	550739	FOOTPLATE MOUNTING BRACKET	
3	226147	1/4-28 x 1.85 BH C/S	
4	281043	#4 x 5/8 PAN PHIL S/TAP	
5	560101	END CAPS FOOTREST	
7	561003	3/4" SADDLE THIN	
8	106935	1/4-28 NYLOCK NUT THIN	
4,5,6A	882337	ELR FOOTPLATE ASSEMBLY 14 -15"	
4,5,6B	882338	ELR FOOTPLATE ASSEMBLY 16 -17"	
4,5,6C	882339	ELR FOOTPLATE ASSEMBLY 18 -20"	
1-5,6A-8	921400S	ELR FOOTPLATE KIT 14 -15"	Does not include Extension
1-5,6B-8	921401S	ELR FOOTPLATE KIT 16 -17"	Does not include Extension
1-5,6C-8	921402S	ELR FOOTPLATE KIT 18 -20"	Does not include Extension

# ARTICULATING HANGER

[10/2011]

**NOTE:** WHEN USING AN ARTICULATING LEGREST ASSEMBLY, THE PULL-TO-LOCK WHEEL LOCK MUST BE USED.

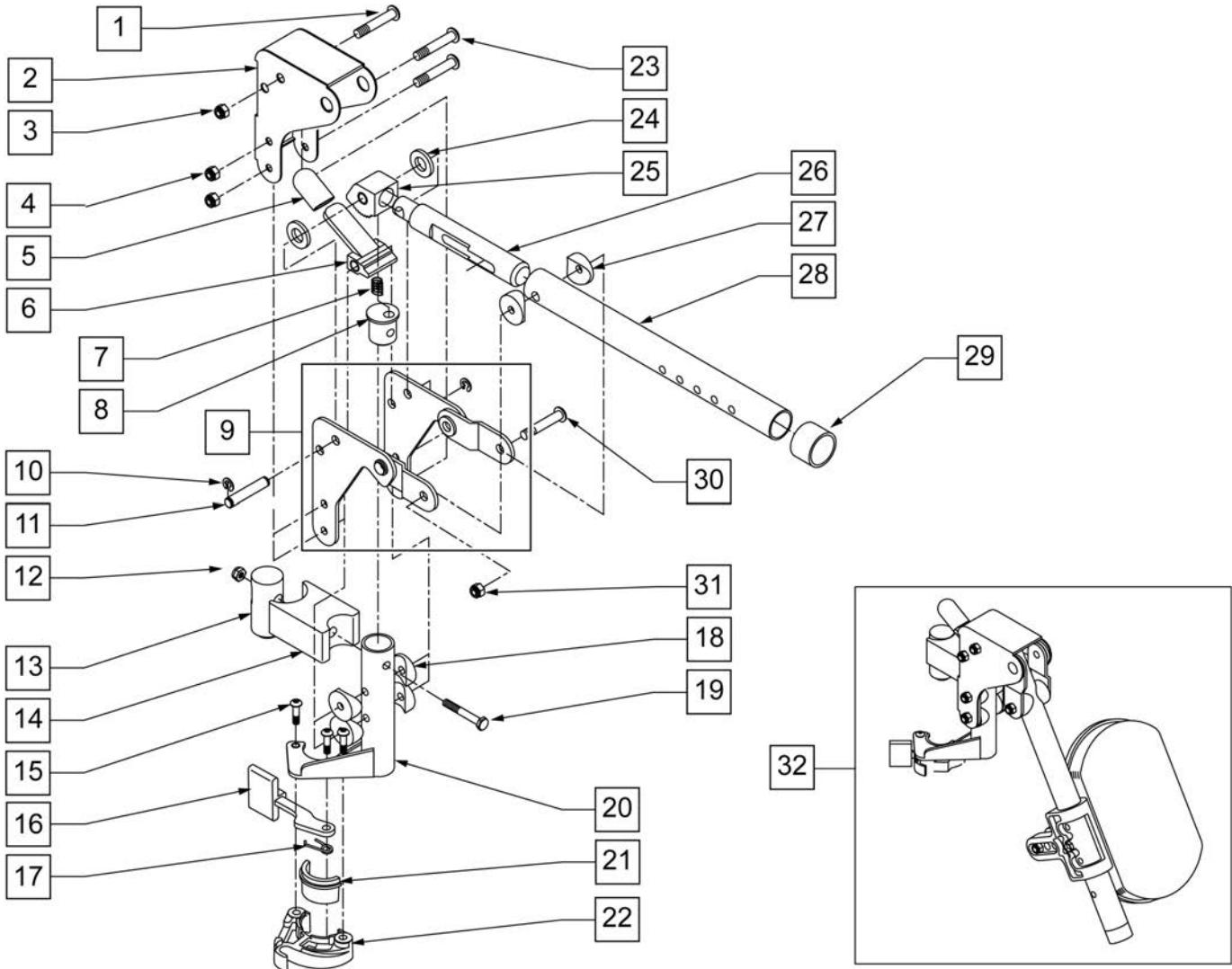


Pos.	Item Number	Description	Remarks
1	106983	5/16-18 x 2 BH C/S SHT THREAD	
2	562720	ALR COVER STD	
3	106937	5/16-18 NYLOCK NUT THIN	
4	106935	1/4-28 NYLOCK NUT THIN	
5	560501	ALR RELEASE LEVER COVER	
6	505902	ALR RELEASE LEVER	
7	321141	650L x 6 COIL SPRING	
8	483060	ALR SPRING PLUG	
9	880526	ALR BRACKET W/LINK R	RIGHT
9	880527	ALR BRACKET W/LINK L	LEFT
10	342000	1/4" E-CLIP	
11	481052	ALR RETAINING STUD	
12	106934	1/4-28 NYLOCK NUT	
13	122991	ALR PIVOT SADDLE WITH LEDGE	

Pos.	Item Number	Description	Remarks
14	122992	ALR PIVOT SADDLE BLOCK W/NOTCH	
15	226057	10/24 x .725 BTTN HD SOC BZ	
16	561079	S/A LATCH LEVER	
17	321136	S/A SPRING L RED	
17	321137	S/A SPRING R CLEAR	
18	561004	1" SADDLE 1/4" HOLE	
19	107016	1/4-28 x 3 G8 HEX C/S	
20	882466	ALR S/A TUBE ASSM L	LEFT
20	882467	ALR S/A TUBE ASSM R	RIGHT
23	106964	1/4-28 x 2 BH C/S	
24	561016	1" WASHER 5/16" HOLE	
25	505900	ALR RATCHET BLOCK	
26	483062	ALR RATCHET ROD	
27	561006	1" SADDLE 5/16" HOLE	
28	505026S	ALR TUBE ONLY	
29	561082	1" TUBE END BUSHING THIN	
30	226195	5/16-18 x 1.82 BH C/S	
31	106937	NUT NYLOCK THIN 5/16-18 ROHS	
21,22	881500	LATCH BLOCK ASSM R	RIGHT
21,22	881501	LATCH BLOCK ASSM L	LEFT
1-26	880092	HANGER ALR SUB ASSM R	RIGHT
1-26	880093	HANGER ALR SUB ASSM L	LEFT
1-31	641464S	HANGER ALR W/O PAD PNTD R	RIGHT
1-31	641465S	HANGER ALR W/O PAD PNTD L	LEFT
32	923051S	ALR HANGER ASSM W/PAD ALUM LB	PAIR
15-17,21,22	881521	S/A LATCH KIT CASTING RIGHT	
15-17,21,22	881522	S/A LATCH KIT CASTING LEFT	

## ARTICULATING HANGER CALF PAD


[09/2011]

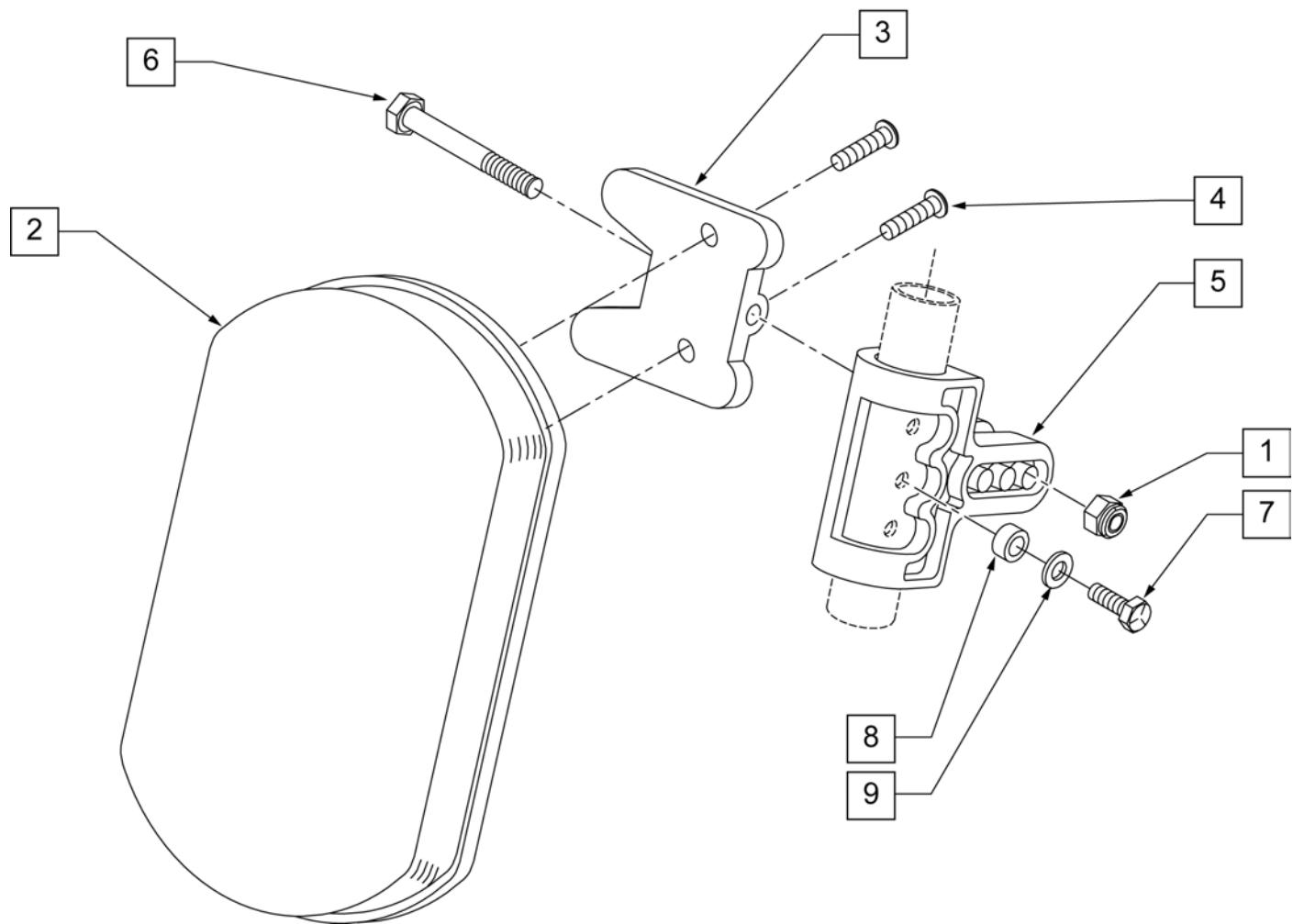


Pos.	Item Number	Description
1	106936	5/16-18 NYLOCK NUT THICK
2	755010	ALR/ELR PAD
3	561096	ALR/ELR CALF PAD BRACKET
4	281182	1/4-20 x 5/8 PAN PHIL
5	561099	CALF CLAMP ALR
6	107019	HHCS 5/16-18 X 3-1/2 ROHS
7	107017	1/4-20 x 1 1/4 G8 HEX C/S
8	483027	1/4 x 1/4 ALUMINUM SPACER BLK
9	104414	WAVE WASHER 6 MM
1-6	881399	ALR CALF PAD W/BRACKET
1,3-9	881398	ALR CALF CLAMP ASSM W/O PAD
1-9	881400	ALR CALF PAD ASSEMBLY (EA)

**NOTE:** (Discontinued on Quickie 2 for 60° and 70° Hangers 10/23/99- Parts only).

**NOTE:** Used on Articulating Hanger Assemblies.

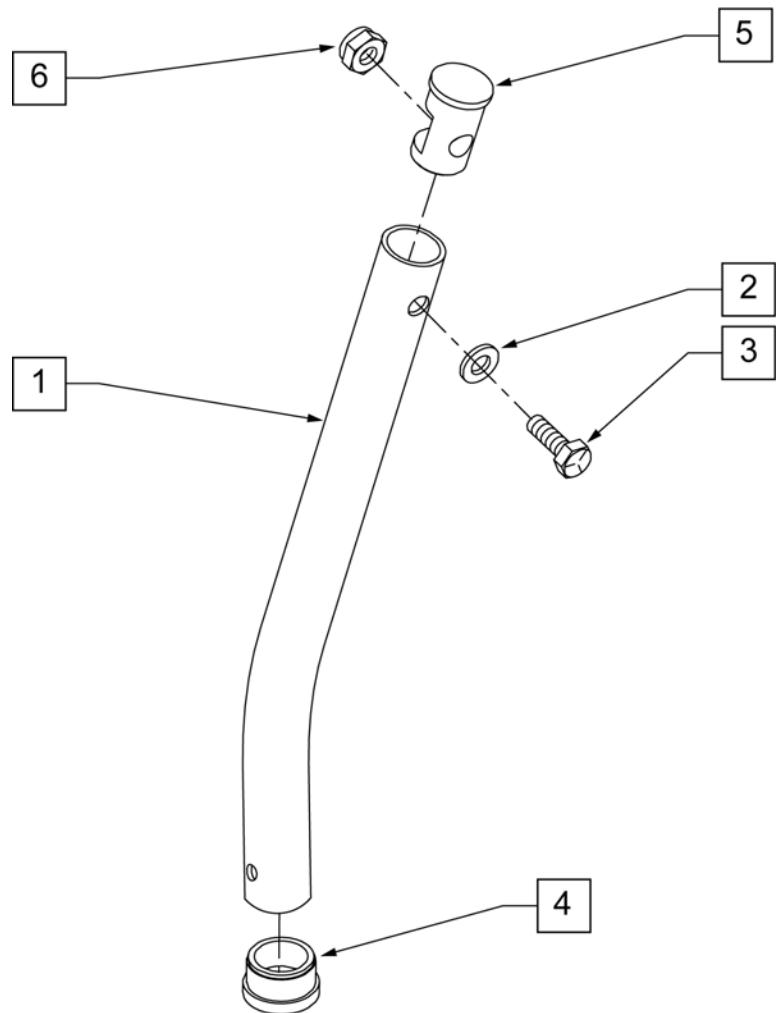


**MEASURING GUIDE**

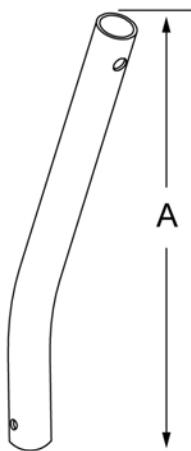

	"A"
SHORT	6 5/8"
MED	9"
LONG	11 5/8"
X-LONG	13 1/2"

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>
1	521010	FOOTREST EXT TUBE ONLY SHT BLK
1	521011	FOOTREST EXT TUBE ONLY MED BLK
1	521012	FOOTREST EXT TUBE ONLY LNG BLK
1	521013	FOOTREST EXT TUBE ONLY X-LNG BLK
2	104414	WAVE WASHER 6 MM
3	106958	HHCS 1/4-20 X 1 ROHS
4	561000-QU	3/4" INSERT PLUG .083
5	483553	FOOTREST INSERT PLUG Q2
6	106932	NUT NYLOCK 1/4-20 ROHS
1-6	921221	FOOTREST EXT TUBE SHT ASSM BLK (EA)
1-6	921222	FOOTREST EXT TUBE MED ASSM BLK (EA)
1-6	921223	FOOTREST EXT TUBE LNG ASSM BLK (EA)
1-6	921224	FOOTREST EXT TUBE X-LNG ASSM BLK (EA)

## COMPOSITE FOOTPLATE


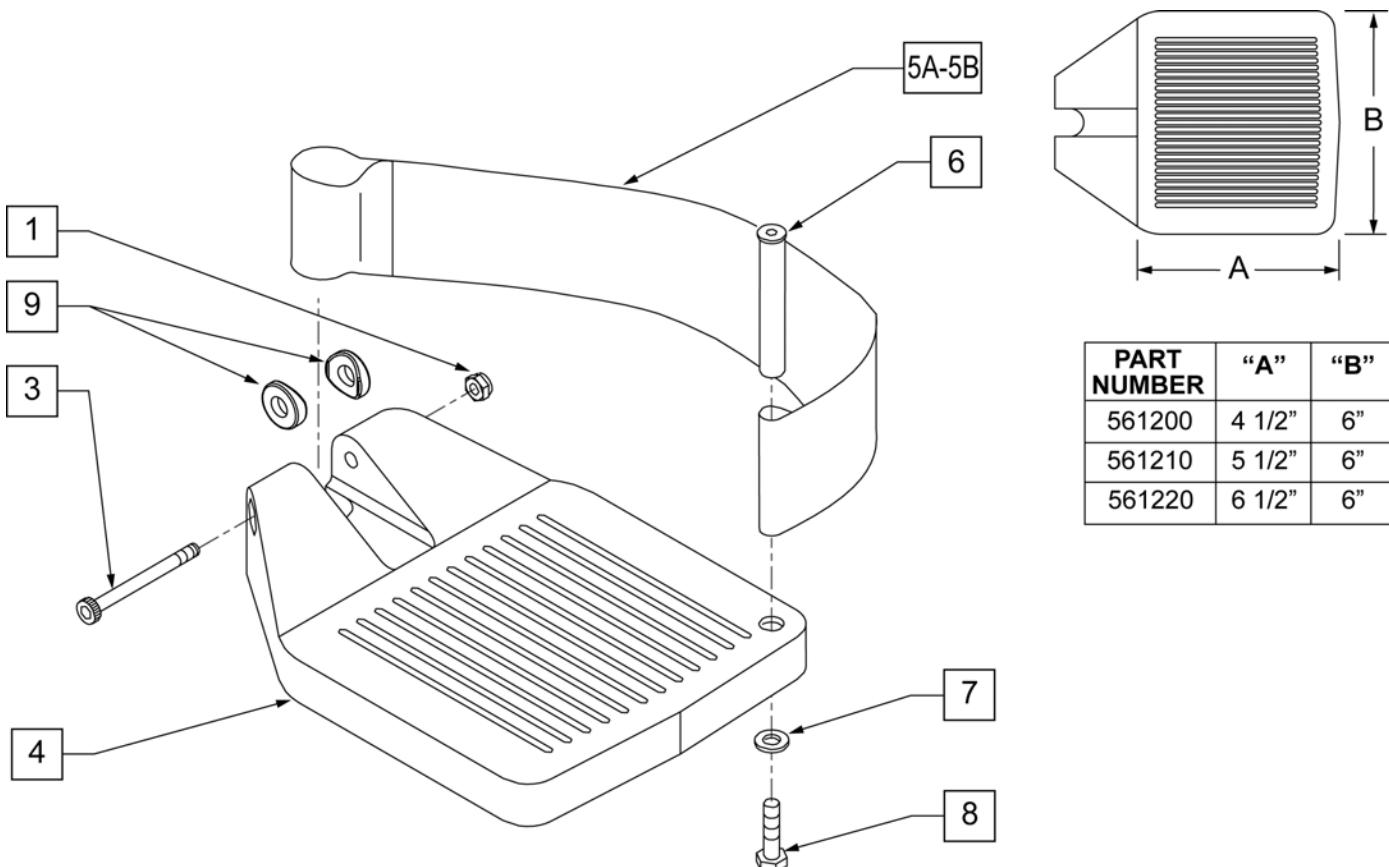


PART NUMBER	"A"	"B"
561200	4 1/2"	6"
561210	5 1/2"	6"
561220	6 1/2"	6"

Pos.	Item Number	Description	Remarks
1	106935	1/4 - 28 NYLOCK NUT	
3	226165	1/4 - 28 x 2 1/2 SOC C/S	
4	561200	COMPOSITE FOOTPLATE 14 -15 (16 -18 V )	14 -15 OR 16 -18V
4	561210	COMPOSITE FOOTPLATE 16 -17 (19 -20 V )	16 -17 OR 19 -20V
4	561220	COMPOSITE FOOTPLATE 18 -20	18 -20
5A	741151	HEEL LOOP 14-15 COMPOSITE (14 -15 or 16 -18V)	14 -15 OR 16 -18V
5A	741156	HEEL LOOP 16 -17 or 19 -20V	16 -17 OR 19 -20V
5A	741153	HEEL LOOP 18 -20 COMPOSITE/14-15 ANG ADJ	18 -20
5A	902270	HEEL LOOP COMPOSITE FT 14-15	PAIR 14 -15 OR 16 -18V
5A	902271	HEEL LOOP COMPOSITE FT 16-17	PAIR 16 -17 OR 19 -20V
5A	902272	HEEL LOOP COMPOSITE FT 18-20	PAIR 18 -20
5B	741154	HEEL LOOP 14-15 COMPOSITE HEMI	14 -15 HEMI
5B	741155	HEEL LOOP 16-17 COMPOSITE HEMI	16 -17 HEMI
5B	741153	HEEL LOOP 18-20 COMPOSITE HEMI	18 -10 HEMI
5B	902260	HEEL LOOPS HEMI COMPOSITE 14-15	PAIR 14 -15 HEMI
5B	902261	HEEL LOOPS HEMI COMPOSITE 16-17	PAIR 16 -17 HEMI
5B	902262	HEEL LOOPS HEMI COMPOSITE 18-20	PAIR 18 -20 HEMI
6	561040	HEEL LOOP PEG	
7	106944	1/4 AN WASHER FLAT THIN	
8	106958	HHCS 1/4-20 X 1 ROHS	
9	561003	3/4" SADDLE THIN	USED W/ ROUND EXT TUBES ONLY
1,3,4	921180	COMPOSITE FOOTPLATE ASSM 14-15	RIGHT (14 -15 OR 16 -18V) W/O HEEL LOOP
1,3,4	921181	COMPOSITE FOOTPLATE ASSM 14-15	LEFT (14 -15 OR 16 -18V) W/O HEEL LOOP
1,3,4	921182	COMPOSITE FOOTPLATE ASSM 16-17	RIGHT (16 -17 OR 19 -20V) W/O HEEL LOOP
1,3,4	921183	COMPOSITE FOOTPLATE ASSM 16-17	LEFT (16 -17 OR 19 -20V) W/O HEEL LOOP


<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
1,3,4	921184	COMPOSITE FOOTPLATE ASSM 18-20	RIGHT (18 -20) W/O HEEL LOOP
1,3,4	921185	COMPOSITE FOOTPLATE ASSM 18-20	LEFT (18 -20) W/O HEEL LOOP
1,3,4	921151	COMPOSITE FOOTPLATE ASSM 14-15/16-18V	PAIR (14 -15 OR 16 -18V) W/O HEEL LOOP
1,3,4	921152	COMPOSITE FOOTPLATE ASSM 16-17/19-20V	PAIR (16 -17 OR 19 -20V) W/O HEEL LOOP
1,3,4	921153	COMPOSITE FOOTPLATE ASSM 18-20	PAIR (18 -20) W/O HEEL LOOP
1,3,4A-8	921190	COMPOSITE FOOTPLATE ASSM HL LP 14-15	RIGHT (14 -15 OR 16 -18V) W/ HEEL LOOP
1,3,4A-8	921191	COMPOSITE FOOTPLATE ASSM HL LP 14-15	LEFT (14 -15 OR 16 -18V) W/ HEEL LOOP
1,3,4A-8	921192	COMPOSITE FOOTPLATE ASSM HL LP 16-17	RIGHT (16 -17 OR 19 -20V) W/ HEEL LOOP
1,3,4A-8	921193	COMPOSITE FOOTPLATE ASSM HL LP 16-17	LEFT (16 -17 OR 19 -20V) W/ HEEL LOOP
1,3,4A-8	921194	COMPOSITE FOOTPLATE ASSM HL LP 18-20	RIGHT (18 -20) W/ HEEL LOOP
1,3,4A-8	921195	COMPOSITE FOOTPLATE ASSM HL LP 18-20	LEFT (18 -20) W/ HEEL LOOP
1,3,4A-8	921161	COMPOSITE FOOTPLATE ASSM HL 14-15/16-18V	PAIR (14 -15 OR 16 -18V) W/ HEEL LOOP
1,3,4A-8	921162	COMPOSITE FOOTPLATE ASSM HL 16-17/19-20V	PAIR (16 -17 OR 19 -20V) W/ HEEL LOOP
1,3,4A-8	921163	COMPOSITE FOOTPLATE ASSM HL 18-20	PAIR (18 -20) W/ HEEL LOOP
1,3,4B-8	921140	COMPOSITE FOOTPLATE ASSM HEMI HL LP 14-15	RIGHT (14 -15) W/ HEEL LOOP
1,3,4B-8	921141	COMPOSITE FOOTPLATE ASSM HEMI HL LP 14-15	LEFT (14 -15) W/ HEEL LOOP
1,3,4B-8	921142	COMPOSITE FOOTPLATE ASSM HEMI HL LP 16-17	RIGHT (16 -17) W/ HEEL LOOP
1,3,4B-8	921143	COMPOSITE FOOTPLATE ASSM HEMI HL LP 16-17	LEFT (16 -17) W/ HEEL LOOP
1,3,4B-8	921144	COMPOSITE FOOTPLATE ASSM HEMI HL LP 18-20	RIGHT (18 -20) W/ HEEL LOOP
1,3,4B-8	921145	COMPOSITE FOOTPLATE ASSM HEMI HL LP 18-20	LEFT (18 -20) W/ HEEL LOOP
1,3,4B-8	921167	COMPOSITE FOOTPLATE ASSM HEMI HL LPS 14-15	PAIR (14 -15) W/ HEEL LOOP
1,3,4B-8	921168	COMPOSITE FOOTPLATE ASSM HEMI HL LPS 16-17	PAIR (16 -17) W/ HEEL LOOP
1,3,4B-8	921169	COMPOSITE FOOTPLATE ASSM HEMI HL LPS 18-20	PAIR (18 -20) W/ HEEL LOOP

## KIDS ANGLE ADJUSTABLE FOOTPLATE EXT MOUNT

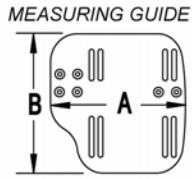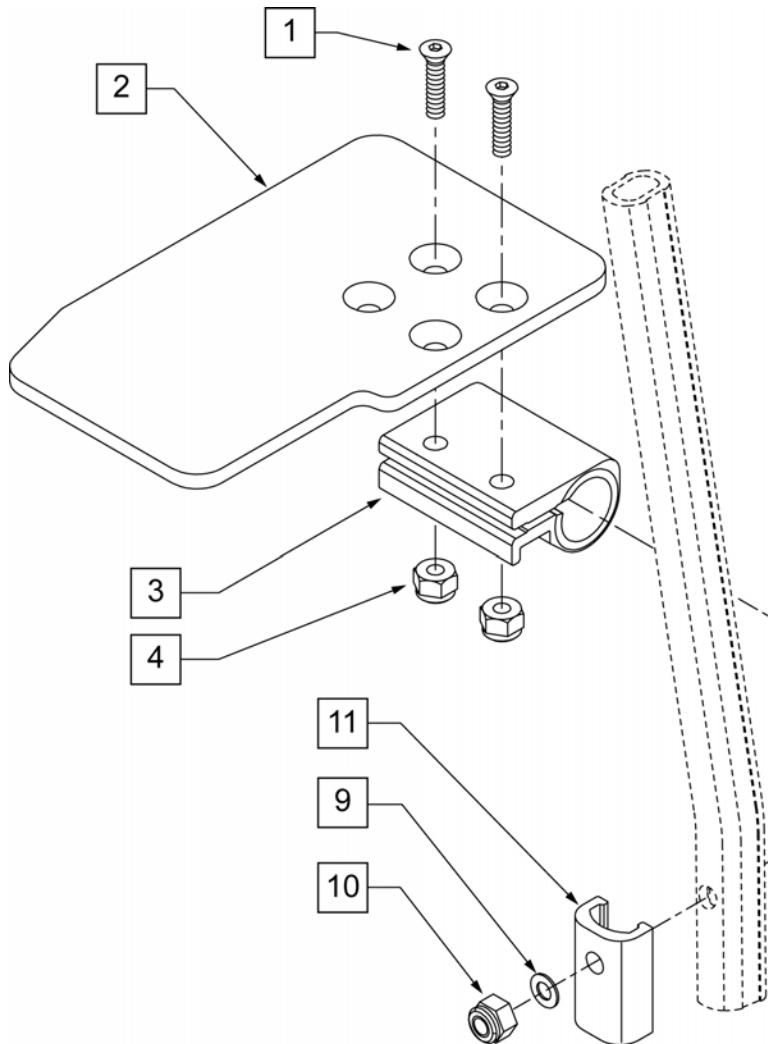

CHAIR WIDTH	"A"	"B"
10" - 11"	3 1/2"	7 1/4"
12" - 13"	4 1/2"	7 3/4"
14" - 15"	5 1/2"	7 3/4"
16" - 17"	6 1/2"	7 3/4"
18" - 20"	7 1/2"	7 3/4"

**FOOTPLATE FORMULA**  
 0" Inset (Seat Width) = Footrest Width  
 1" Inset (Seat Width) -2 = Footrest Width  
 V Hanger (Seat Width) -4 = Footrest Width

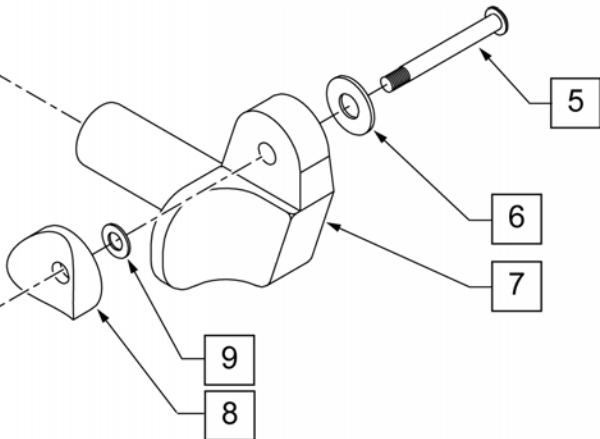

\* NOTE: ITEMS 5 & 11 NOT INCLUDED W/ FOOTPLATE ASSM.  
 ITEMS 5 & 11 INCLUDED W/ EXTENSION TUBE ASSM.

Pos.	Item Number	Description	Remarks
1	106978	FHCS 1/4 -28 x 7/8 ROHS	
2	500635	KIDS ANGL ADJ F/PLT 10" R	Right
2	500636	KIDS ANGL ADJ F/PLT 10" L	Left
2	500637	KIDS ANGL ADJ F/PLT 11" R	Right
2	500638	KIDS ANGL ADJ F/PLT 11" L	Left
2	500639	KIDS ANGL ADJ F/PLT 12" R	Right
2	500640	KIDS ANGL ADJ F/PLT 12" L	Left
2	500641	KIDS ANGL ADJ F/PLT 13" R	Right
2	500642	KIDS ANGL ADJ F/PLT 13" L	Left
2	500643	KIDS ANGL ADJ F/PLT 14" R	Right
2	500644	KIDS ANGL ADJ F/PLT 14" L	Left
2	500645	KIDS ANGL ADJ F/PLT 15" R	Right
2	500646	KIDS ANGL ADJ F/PLT 15" L	Left
2	500647	KIDS ANGL ADJ F/PLT 16" R	Right
2	500648	KIDS ANGL ADJ F/PLT 16" L	Left
3	550350	3/4" CLAMP ADJ FOOTPLATE	
4	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
5	106962	1/4-28 x 1 3/4 BH C/S BLK ZN (D2 EXTENSION TUBES)	

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
5	226167	1/4-28 x 1.625 BH C/S BLK (ROUND EXTENSION TUBES)	
6	106945	1/4 SAE WASHER	
7	541002	FTRST MOUNTING BRKT-3/8 RAD R	
7	541003	FTRST MOUNTING BRKT-3/8 RAD L	
8	561055	3/4" SADDLE 1" OD	
9	106944	WASHER FLAT THIN AN 1/4 ROHS	
10	106941	NUT AUTOMATION 1/4 -28 ROHS	
1-7,9,10	828850	KIDS ANG ADJ F/PLT EXT-MNT 10"R	Right
1-7,9,10	828851	KIDS ANG ADJ F/PLT EXT-MNT 10"L	Left
1-7,9,10	828852	KIDS ANG ADJ F/PLT EXT-MNT 11"R	Right
1-7,9,10	828853	KIDS ANG ADJ F/PLT EXT-MNT 11"L	Left
1-7,9,10	828854	KIDS ANG ADJ F/PLT EXT-MNT 12"R	Right
1-7,9,10	828855	KIDS ANG ADJ F/PLT EXT-MNT 12"L	Left
1-7,9,10	828856	KIDS ANG ADJ F/PLT EXT-MNT 13"R	Right
1-7,9,10	828857	KIDS ANG ADJ F/PLT EXT-MNT 13"L	Left
1-7,9,10	828858	KIDS ANG ADJ F/PLT EXT-MNT 14"R	Right
1-7,9,10	828859	KIDS ANG ADJ F/PLT EXT-MNT 14"L	Left
1-7,9,10	828860	KIDS ANG ADJ F/PLT EXT-MNT 15"R	Right
1-7,9,10	828861	KIDS ANG ADJ F/PLT EXT-MNT 15"L	Left
1-7,9,10	828862	KIDS ANG ADJ F/PLT EXT-MNT 16"R	Right
1-7,9,10	828863	KIDS ANG ADJ F/PLT EXT-MNT 16"L	Left
1-7,9,10	921280	KIDS ANG ADJ F/PLT EXT-MNT 10"	Pair
1-7,9,10	921281	KIDS ANG ADJ F/PLT EXT-MNT 11"	Pair
1-7,9,10	921282	KIDS ANG ADJ F/PLT EXT-MNT 12"	Pair
1-7,9,10	921283	KIDS ANG ADJ F/PLT EXT-MNT 13"	Pair
1-7,9,10	921284	KIDS ANG ADJ F/PLT EXT-MNT 14"	Pair
1-7,9,10	921285	KIDS ANG ADJ F/PLT EXT-MNT 15"	Pair
1-7,9,10	921286	KIDS ANG ADJ F/PLT EXT-MNT 16"	Pair


**NOTE:** FOR HIGH-MOUNT ADULT ANGLE-ADJUSTABLE FOOTPLATE, SEE "HIGH-MOUNT EXTENSION TUBE" PAGE.

**NOTE:** ORDER ITEM 4B FOR CHAIRS PRIOR TO 8/13/01.

**NOTE:** ONLY 1 WASHER PRIOR TO 8/13/01.



**NOTE:** ORDER PART 226167 PRIOR TO 8/13/01.

**NOTE:** See attached REFERENCE GUIDE.

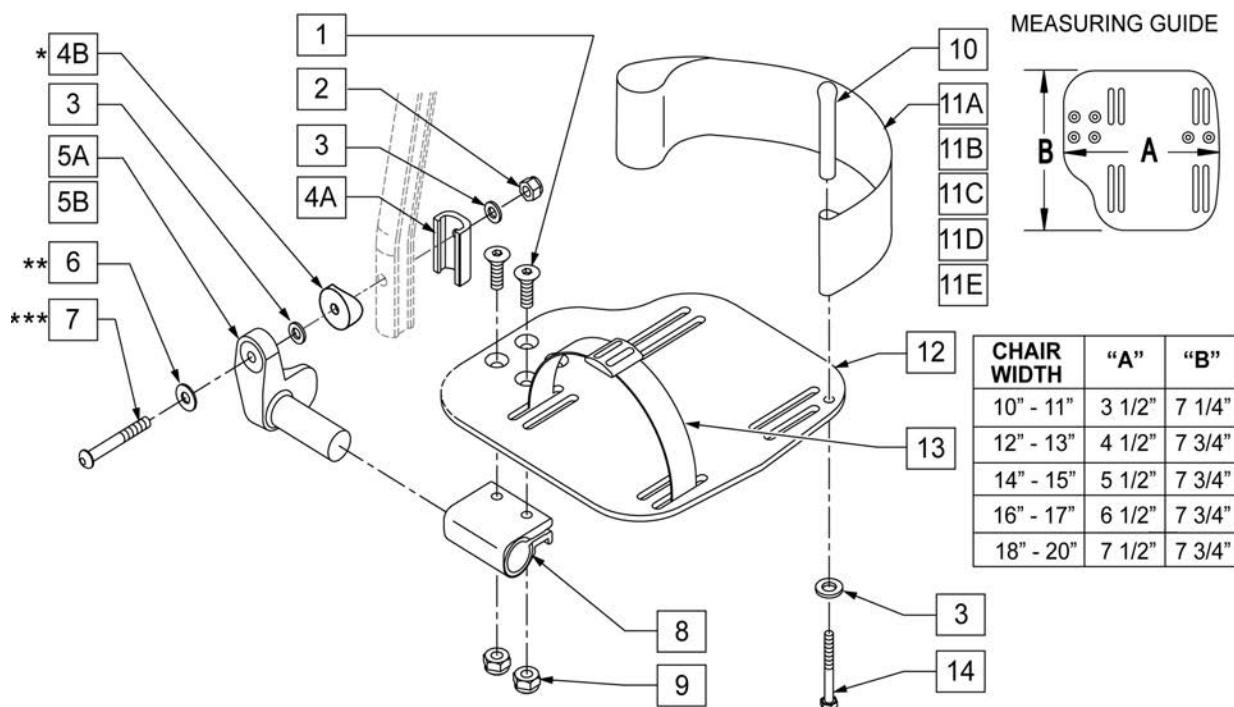

Pos.	Item Number	Description	Remarks
1	106978	1/4-28 x 7/8 FH C/S	
2	106941	1/4-28 AUTOMATION NUT	


Pos.	Item Number	Description	Remarks
3	106943	WASHER FLAT AN 1/4 ROHS	
4A	483720	HANGER EXTENSION SADDLE LONG	USE W/ DOUBLE D EXT TUBE
4B	561055	3/4" SADDLE 1" OD	USE W/ ROUND EXT TUBE
5	541002	FTRST MOUNTING BRKT-3/8 RAD R	RIGHT
5	541003	FTRST MOUNTING BRKT-3/8 RAD L	LEFT
6	106945	WASHER FLAT SAE 1/4 ROHS	
7A	106962	BHCS 1/4-28 X 1-3/4 ROHS	USE W/ DOUBLE D EXT TUBE
7B	226167	1/4-28 x 1.625 BH C/S BLK	USE W/ ROUND EXT TUBE
8	550350	3/4" CLMP ADJ FT PLT	
9	106935	NUT NYLOCK THIN 1/4-28 ROHS	
10	561040	HEEL LOOP PEG	
11A	741154	HEEL LOOP 14-15 CMPST HEMI	USED W/ 10-11" ADULT ANG ADJ FTPLTS
11B	741155	HEEL LOOP 16-17 CMPST HEMI	USED W/ 12 -13" ADULT ANG ADJ FTPLTS
11C	741153	H/L 18-20 CMPST/14-15 ANG ADJ	USED W/ 14-15"/16-17" ADULT ANG ADJ FTPLTS
11D	741253	HEEL LOOP ADLT ADJ F/PLT 18-20	USED W/ 18 -20" ADULT ANG ADJ FTPLTS
12	500656	ADLT ANGL ADJ F/PLT 10/11" R	RIGHT
12	500657	ADLT ANGL ADJ F/PLT 10/11" L	LEFT
12	500658	ADLT ANGL ADJ F/PLT 12/13" R	RIGHT
12	500659	ADLT ANGL ADJ F/PLT 12/13" L	LEFT
12	500660	ADLT ANGL ADJ F/PLT 14/15	RIGHT
12	500661	ADLT ANGL ADJ F/PLT 14/15	LEFT
12	500662	ADLT ANGL ADJ F/PLT 16/17	RIGHT
12	500663	ADLT ANGL ADJ F/PLT 16/17	LEFT
12	500664	ADLT ANGL ADJ F/PLT 18-20	RIGHT
12	500665	ADLT ANGL ADJ F/PLT 18-20	LEFT
13	740260	TOE/ANKLE STRAP ADJ ADLT F/PLT	
14	106958	1/4-20 x 1 HEX C/S	
1-3,5,6,8,9,12	820526	AD ANG ADJ F/PLT R 12"/13" OMIT HL	RIGHT
1-3,5,6,8,9,12	820536	AD ANG ADJ F/PLT L 12"/13" OMT HL	LEFT
1-3,5,6,8,9,12	820460	FP ADLT ADJ 14/15 OMIT HL	RIGHT
1-3,5,6,8,9,12	820462	FP ADLT ADJ 14/15 OMIT HL	LEFT
1-3,5,6,8,9,12	820464	FP ADLT ADJ 16/17 OMIT HL	RIGHT
1-3,5,6,8,9,12	820466	FP ADLT ADJ 16/17 OMIT HL	LEFT
1-3,5,6,8,9,12	820468	FP ADLT ADJ 18-20 OMIT HL	RIGHT
1-3,5,6,8,9,12	820470	FP ADLT ADJ 18-20 OMIT HL	LEFT
1-3,5,6,8,9,12	921466	ADLT ANGL ADJ F/PLT 12"/13" OMIT HL	PAIR
1-3,5,6,8,9,12	921320	FP ADLT ADJ 14/15 OMIT HL	PAIR
1-3,5,6,8,9,12	921322	FP ADLT ADJ 16/17 OMIT HL	PAIR
1-3,5,6,8,9,12	921324	FP ADLT ADJ 18-20 OMIT HL	PAIR
1-3,5,6,8,9,12	820522	AD ANGL ADJ F/PLT R 12/13" H/L	RIGHT
1-3,5,6,8,9,12	820532	AD ANGL ADJ F/PLT L 12/13" H/L	LEFT
1-3,5,6,8-10,11C, 12,14	820461	FP ADLT ADJ 14/15 HL	RIGHT
1-3,5,6,8-10,11C, 12,14	820463	FP ADLT ADJ 14/15 HL	LEFT
1-3,5,6,8-10,11C, 12,14	820465	FP ADLT ADJ 16/17 HL	RIGHT
1-3,5,6,8-10,11C, 12,14	820467	FP ADLT ADJ 16/17 HL	LEFT
1-3,5,6,8-10,11D, 12,14	820469	FP ADLT ADJ 18-20 HL	RIGHT
1-3,5,6,8-10,11D, 12,14	820471	FP ADLT ADJ 18-20 HL	LEFT
1-3,5,6,8-10,11B, 12,14	921462	ADLT ANGL ADJ F/PLATE 12"/13" HL	PAIR
1-3,5,6,8-10,11C, 12,14	921321	FP ADLT ADJ 14/15 HL	PAIR
1-3,5,6,8-10,11C, 12,14	921323	FP ADLT ADJ 16/17 HL	PAIR
1-3,5,6,8-10,11D, 12,14	921325	FP ADLT ADJ 18-20 HL	PAIR
1-3,6,7A,9-10,14	101533	ADLT ANG ADJ FP D2 HDWR KIT	FOR DOUBLE D EXT TUBES
1-3B,4B,6,7B,9,10,14	101534	ADLT ANG ADJ FP RND HDWR KIT	FOR ROUND EXT TUBES (Q2 HP ONLY)

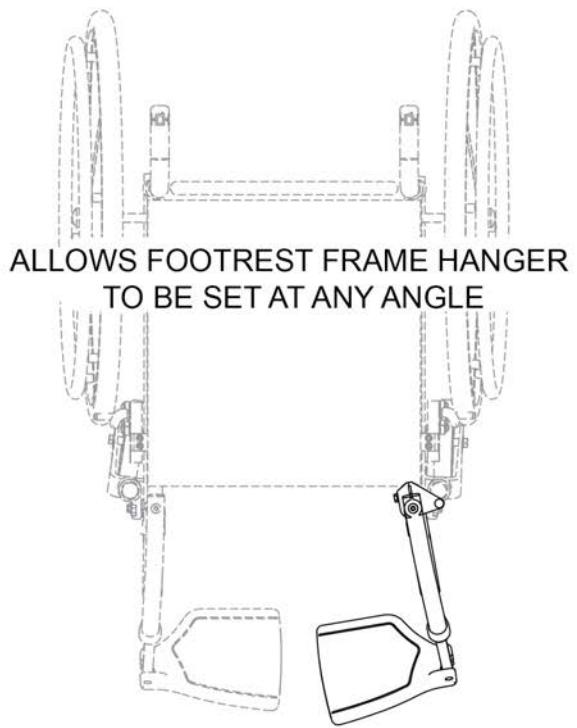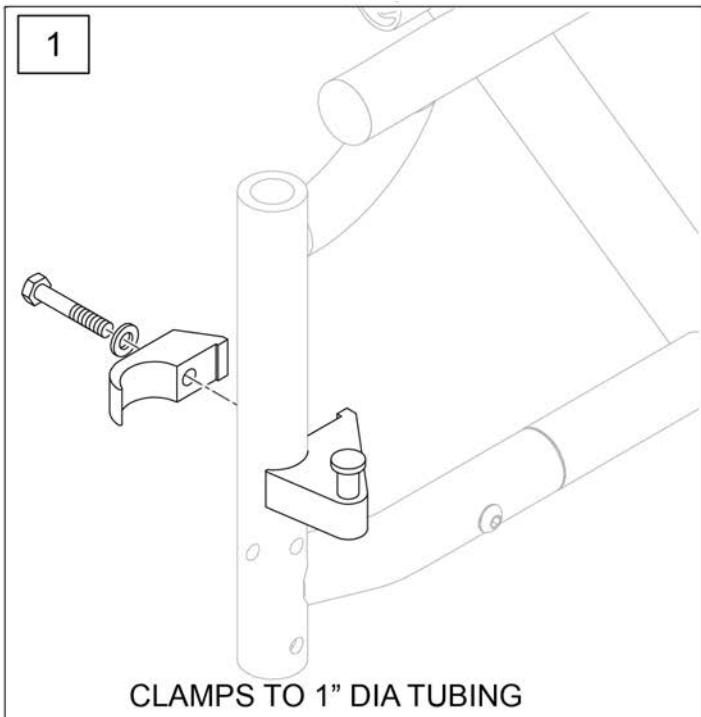

Pos.	Item Number	Description	Remarks
1	112807	LATCH STUD SUPPORT R MULTIPOS	Single
1	112808	LATCH STUD SUPPORT L MULTIPOS	Single
1	112809	LATCH STUD SUPPORT PR MULTIPOS	Pair

# SINGLE POST HEIGHT ADJUSTABLE ARMREST


[11/2011]

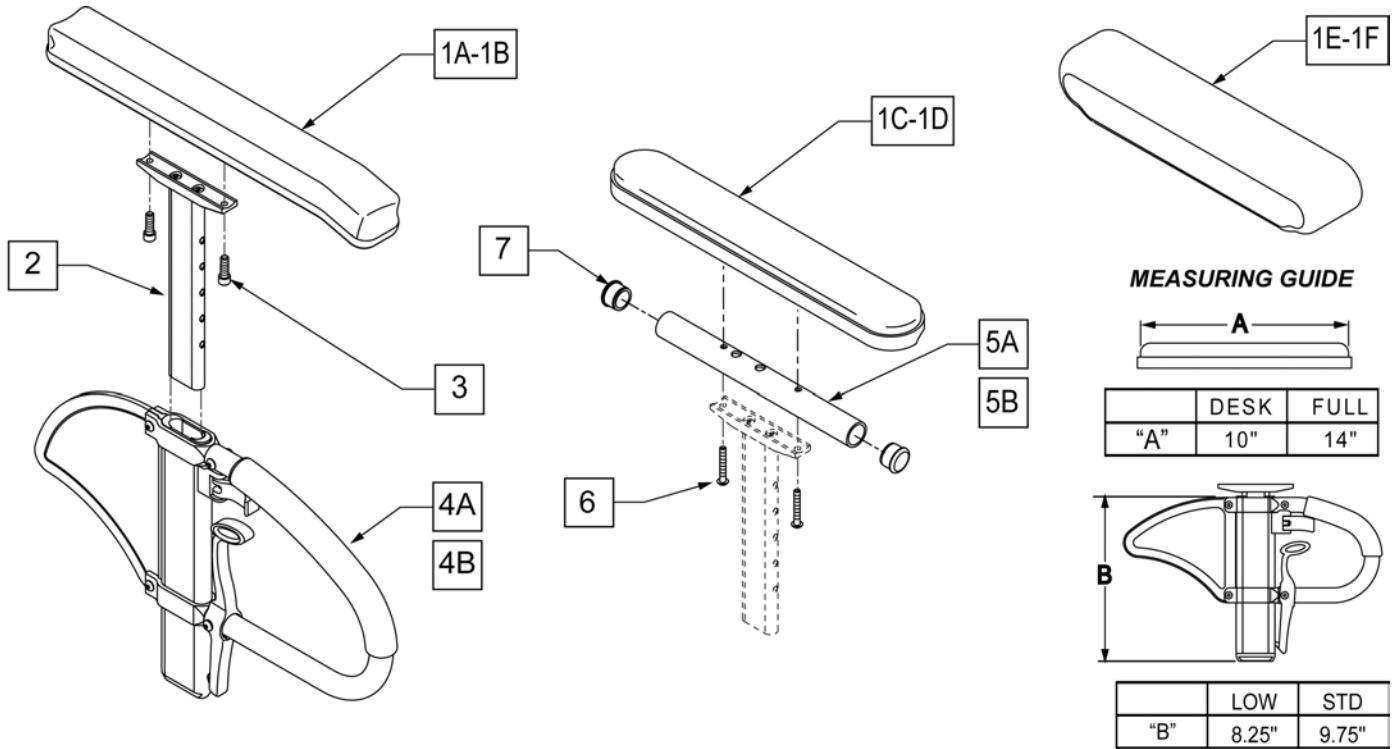

Pos.	Item Number	Description	Remarks
1A	104568-001	SPHA PAD DESK	CONTOUR DESK
1B	104568-002	SPHA PAD FULL	CONTOUR FULL
1C	561024	ARMREST PAD 10" DESK	CLASSIC DESK
1D	561031	ARMREST PAD 14" FULL	CLASSIC FULL
1E	561280	ARMREST PAD 10" LEFT W/F	Left-Waterfall Desk
1E	561281	ARMREST PAD 10" RIGHT W/F	Right-Waterfall Desk
1F	561284	ARMREST PAD 14" LEFT W/F	Left-Waterfall Full
1F	561285	ARMREST PAD 14" RIGHT W/F	Right-Waterfall Full
2	116312	SNGL POST UPPR ASSM LOW	LOW/SMALL
2	116313	SNGL POST UPPR ASSM STD	STANDARD
3	106956	SHCS 10-32 X 3/4 ROHS	
2,4A	104509-001	SPHA ARMREST ASSY STD	STANDARD
2,4B	104509-002	SPHA ARMREST ASSY LOW	LOW/SMALL
5A	521812	HT ADJ PD TUBE 10IN DESK POWE	Desk
5B	521813	HT ADJ PD TUBE 14IN FULL POWE	Full
6	107024	BHCS 10-32 X 1 1/2 ROHS	
7	560735	7/8" ROUND TUBE PLUG	
1C-4A-5A,6,7	108599	SPHA STD CLASIC DESK PAD R	Right
1C-4A-5A,6,7	108603	SPHA STD CLASIC DESK PAD L	Left
1D-4A-5B-7	108601	SPHA STD CLASIC FULL PAD R	Right
1D-4A-5B-7	108605	SPHA STD CLASIC FULL PAD L	Left
1A-4A	108595	SPHA STD CNTUR DESK PAD	
1B-4A	108597	SPHA STD CNTUR FULL PAD	
1C-4B-5A,6,7	108600	SPHA LOW CLASIC DESK PAD R	Right
1C-4B-5A,6,7	108604	SPHA LOW CLASIC DESK PAD L	Left
1D-4B-5B-7	108602	SPHA LOW CLASIC FULL PAD R	Right
1D-4B-5B-7	108606	SPHA LOW CLASIC FULL PAD L	Left
1A-4B	108596	SPHA LOW CNTUR DESK PAD	

Pos.	Item Number	Description	Remarks
1B-4B	108598	SPHA LOW CNTUR FULL PAD	

**NOTE:** If ordering for pediatric chairs with 20" or 22" wheels, you must order left armrest pad for right side, and right armrest pad for left side.

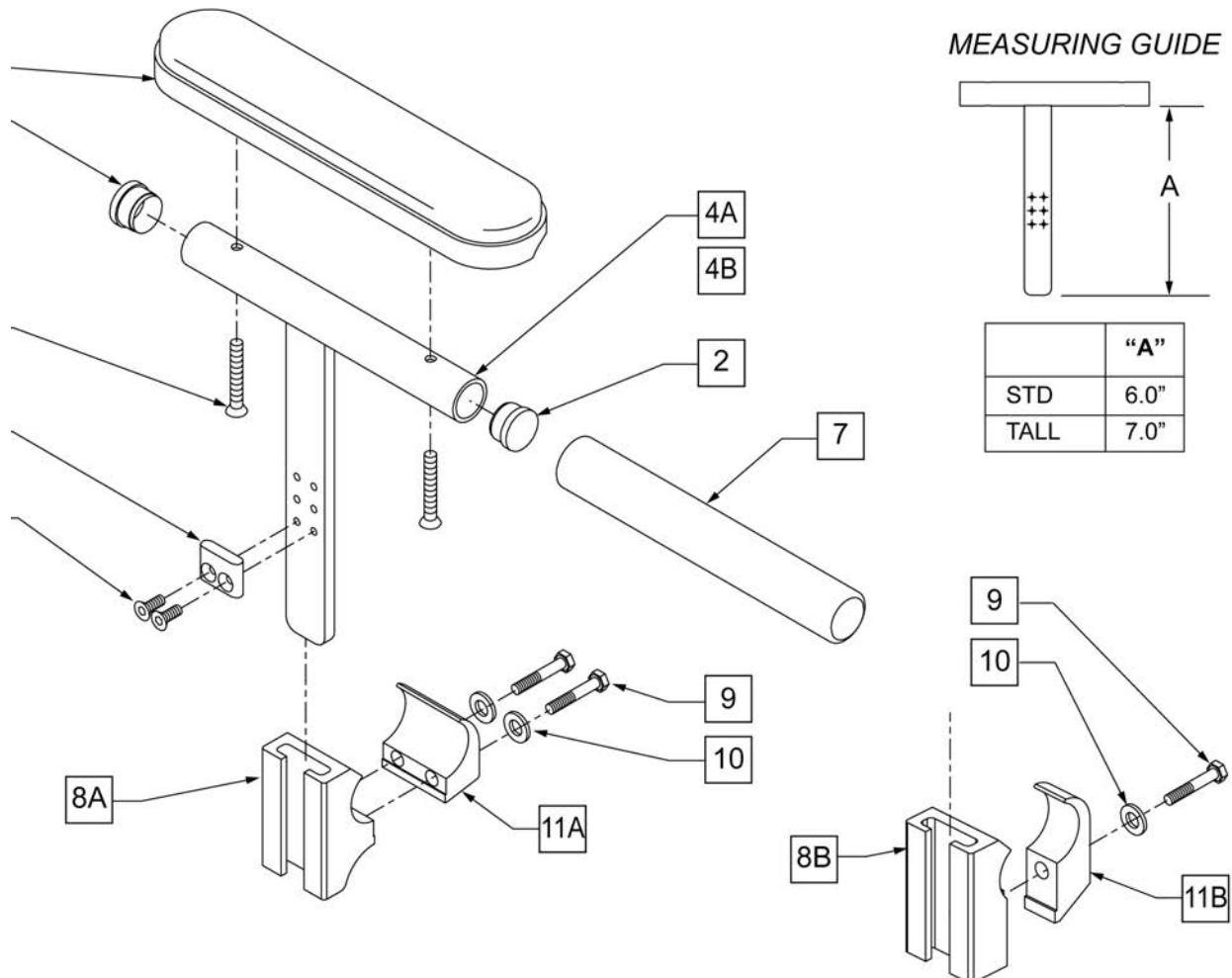


Pos.	Item Number	Description	Remarks
1	560780	ARMREST PAD SMALL 9"	
2	561007	7/8" INSERT PLUG .083	
3	107023	10-32 x 1 3/8 BH C/S BLK	
5	550105-QU	KIDZ ARM STOP	
6	113279	10-32 x 1/2 FH C/S BLK	
7	560309	ARMREST FOAM KIDZ	
8A	502037	TPHHA & SG CLAMP 1"	USED W/ ZIPPIE TS AND ZIPPIE GS
8B	550325	SIDE GUARD/KIDZ ARM REC	USED WITH ZIPPIE IRIS AND ZIPPIE 2
9	106971	1/4-28 x 1 3/8 HEX C/S	
10	106944	1/4 AN WASHER FLAT THIN	
11A	502038	TPHAR & SGR CLAMP 1"	USED W/ ZIPPIE TS AND ZIPPIE GS
11B	550251	CLAMP P300 CHRGNG/ADJ FL KDS	USED WITH ZIPPIE IRIS AND ZIPPIE 2
1-3,4A,5,6	880162	KIDS ARM STANDARD R	RIGHT
1-3,4A,5,6	880163	KIDS ARM STANDARD L	LEFT

Pos.	Item Number	Description	Remarks
1-3,4B,5,6	880164	KIDZ ARM TALL R	RIGHT
1-3,4B,5,6	880165	KIDZ ARM TALL L	LEFT
8A,9,10,11A	882681	T-PST/SDE GUARD REC W/CLMP ZTS	INCLUDES 1 RECEIVER -FITS ZIPPIE TS & GS
8B,9,10,11B	881200	SIDE GUARD/KIDZ ARM REC W/ CLAMP	INCLUDES 1 RECEIVER -FITS ZIPPIE IRIS & 2
2,4A,5-7	885010	KIDZ ARM STD W/FOAM R	RIGHT
2,4A,5-7	885011	KIDZ ARM STD W/FOAM L	LEFT
2,4B,5-7	885012	KIDZ ARM TALL W/FOAM R	RIGHT
2,4B,5-7	885013	KIDZ ARM TALL W/FOAM L	LEFT
1-3,4A,5,6,8A,9,10,11A	900795	KIDZ ARM ASSM STD W/REC ZTS	PAIR -FITS ZIPPIE TS & GS
1-3,4B,5,6,8A,9,10,11A	900796	KIDZ ARM ASSM TALL W/REC ZTS	PAIR -FITS ZIPPIE TS & GS
2,4A,5-7,8A,9,10,11A	900797	KDZ ARM ASSM STD W/FM W/RC ZTS	PAIR -FITS ZIPPIE TS & GS
2,4B,5-7,8A,9,10,11A	900798	KDZ ARM ASSM TLL W/FM W/RC ZTS	PAIR -FITS ZIPPIE TS & GS
1-3,4A,5,6,8B,9,10,11B	900693	KIDZ ARM ASSM STD W/REC	PAIR -FITS ZIPPIE IRIS & 2
1-3,4B,5,6,8B,9,10,11B	900694	KIDZ ARM ASSM TALL W/REC	PAIR -FITS ZIPPIE IRIS & 2
2,4A,5-7,8B,9,10,11B	900755	KIDZ ARM ASSM STD W/FOAM W/REC	PAIR -FITS ZIPPIE IRIS & 2
2,4B,5-7,8B,9,10,11B	900756	KIDZ ARM ASSM TALL W/FOAM W/REC	PAIR -FITS ZIPPIE IRIS & 2


# FLIP-UP ARMREST

[11/2009]

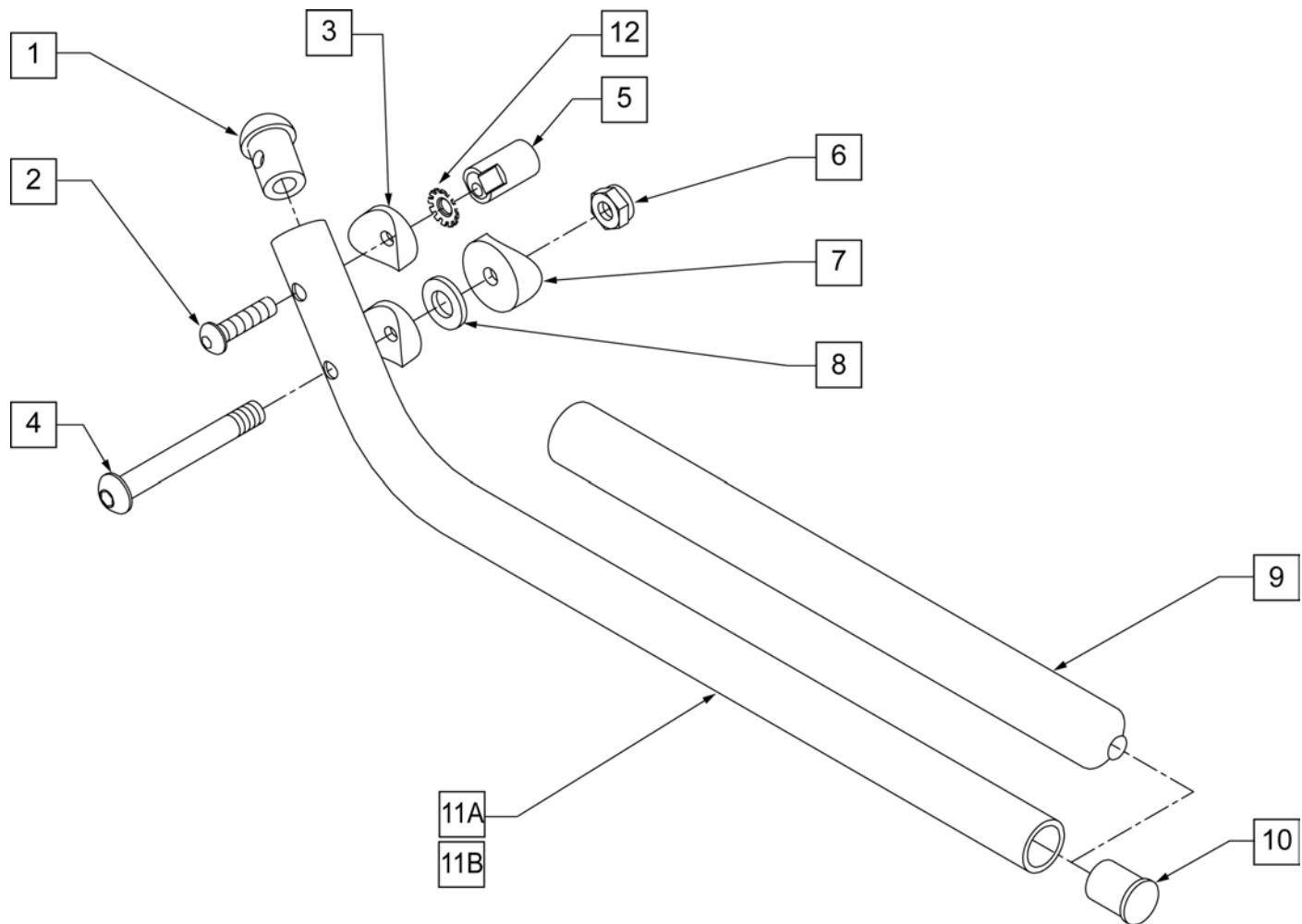

Pos.	Item Number	Description	Remarks
1	561034	SHORT DOME PLUG	
2	107028	1/4-28 x 1 1/2 BH C/S	
3	561003	3/4" SADDLE THIN	
4	106966	1/4-28 x 2 1/4 BH C/S	
5	483041	FLIP-UP ARM CAM ZP/P5	
6	106935	1/4-28 NYLOCK NUT THIN	
7	561004	1" SADDLE 1/4" HOLE	
8	106944	1/4 AN WASHER FLAT THIN	
9	560599	8 x .705 x .200 FOAM GRIP	
10	561000-QU	3/4" INSERT PLUG .083	
12	267350	1/4" WASHER INT LOCK ZN	
1-11A,12	880268	ZP FLIP-UP ARM ASSEMBLY SEAT DEPTH 10"-13"	RIGHT
1-11A,12	880269	ZP FLIP-UP ARM ASSEMBLY SEAT DEPTH 10"-13"	LEFT
1-11A,12	900446	FLIP-UP ARM ASSEMBLY 10"-13" ZP	PAIR
1-11B,12	880270	FLIP-UP ARM ASSEMBLY SEAT DEPTH 14"-16" ZP/P5	RIGHT
1-11B,12	880271	FLIP-UP ARM ASSEMBLY SEAT DEPTH 14"-16" ZP/P5	LEFT
1-11B,12	900448	FLIP-UP ARM ASSEMBLY	PAIR

## ADJUSTABLE LOCKING FLIP-UP ARMREST

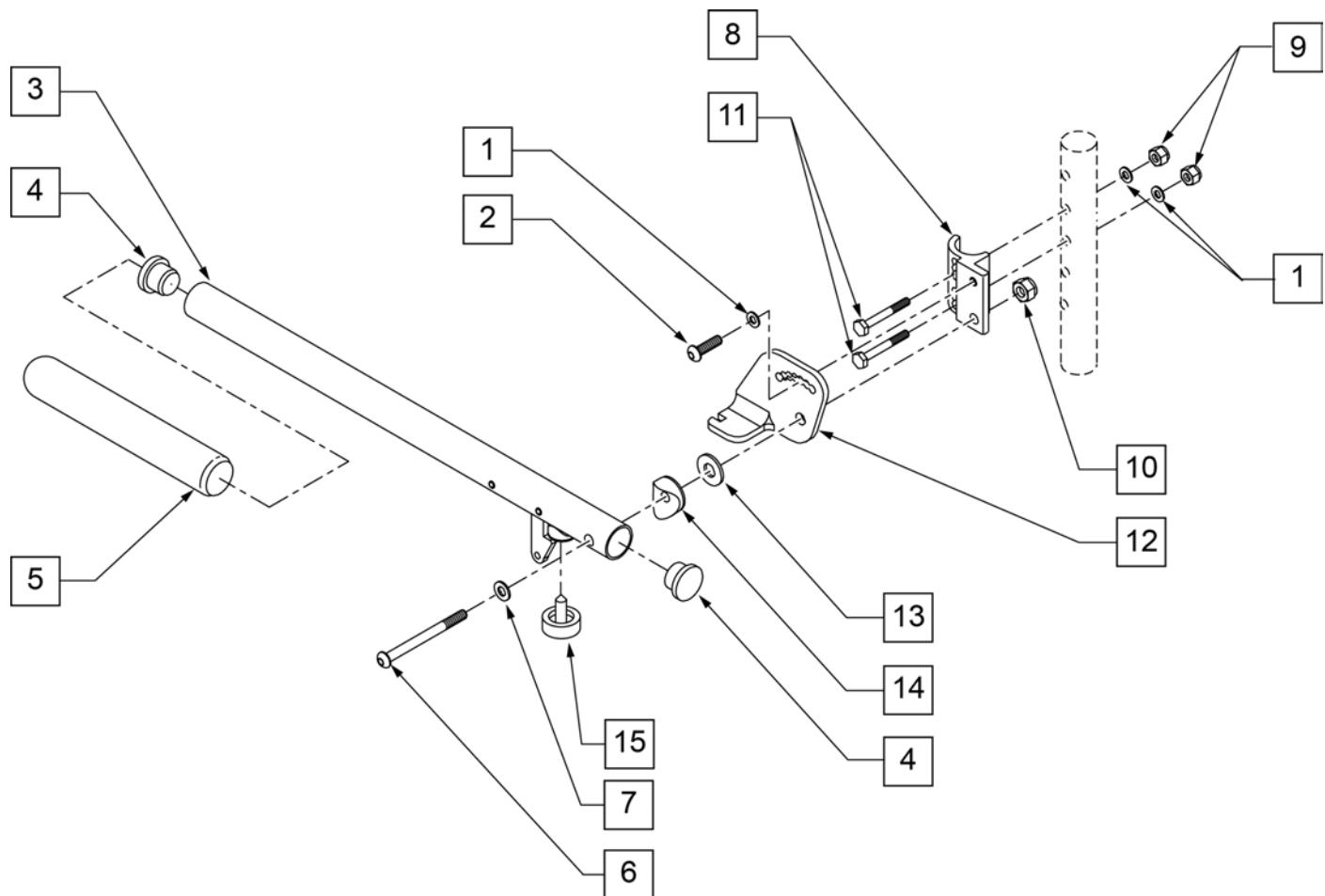


Pos.	Item Number	Description	Remarks
1	106943	1/4 AN WASHER FLAT	
2	107025	1/4-28 x 1/2 BH C/S	
4	561002	1" INSERT PLUG .083	
5	560640	9 x .93 x .155 FOAM GRIP	
6	106983	5/16-18 x 2 BH C/S SHT THREAD	
7	106947	5/16 WASHER FLAT THIN	
8	550168	ADJ LOCK FLIP-UP ARM MNT R	
8	550169	ADJ LOCK FLIP-UP ARM MNT L	
9	106935	1/4-28 NYLOCK NUT THIN	
10	202117	5/16-18 NYLOCK NUT THICK BLK	
11	106972	1/4-28 x 1 1/2 G8 HEX C/S	
12	550474	ADJ FLIP-BACK ARM BRACKET R	
12	550475	ADJ FLIP BACK ARM BRACKET L	
13	561016	1" WASHER 5/16" HOLE	
14	561006	1" SADDLE 5/16" HOLE	
15	560252	1" COVED STEM BUMPER	
3-5	880274	ADJ FLIP-BACK TUBE W/LATCH	
1,2,6,8-10	881272	ADJ LOCK F-UP ARMREST RETRO R	USE FOR ARMREST PRIOR TO 7/22/96
1,2,6,8-10	881273	ADJ LOCK F-UP ARMREST RETRO L	USE FOR ARMREST PRIOR TO 7/22/96
1-15	880272	ADJ FLIP BACK ARMREST R	
1-15	880273	ADJ FLIP BACK ARMREST L	
1-15	900449	ADJ LOCK FLIP-UP ARMREST ASSM	PAIR

# LENGTH ADJUSTABLE LOCKING FLIP-UP ARMREST

(09/2010)

**NOTE:** FOR ARMRESTS PRIOR TO 7/22/96 USE RETRO FIT KIT TO REPLACE ADJUSTABLE FLIP-UP ARM CLAMP.

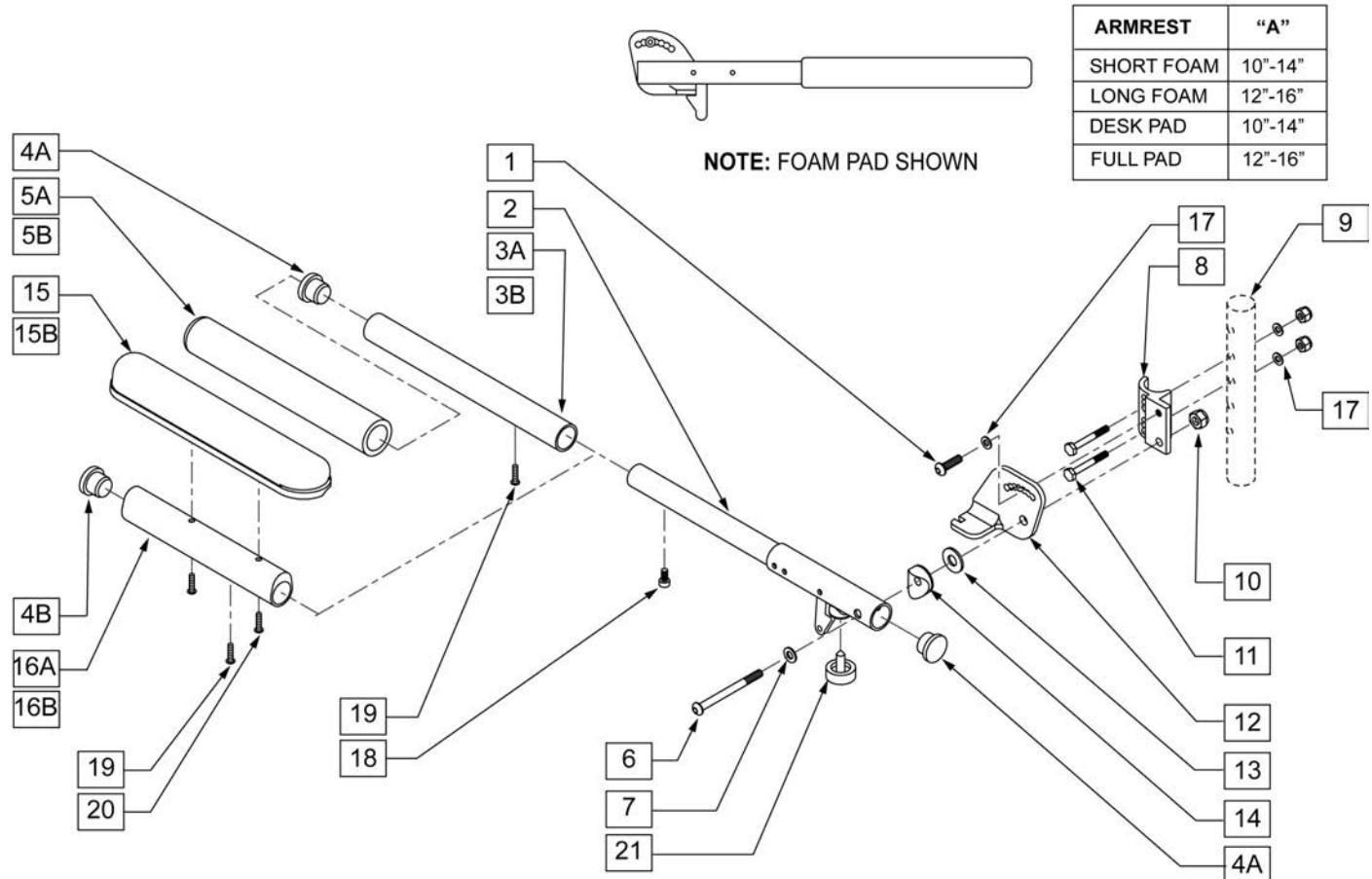

Pos.	Item Number	Description	Remarks
1	107025	BHCS 1/4-28 x 1/2 ROHS	
4A	561002	1" INSERT PLUG .083	
4B	561062	1" INSERT PLUG .065	
5A	560299	7" X .940 X .20 FOAM GRIP	SHORT PAD
5B	560640	9" X .93" X .155" FOAM GRIP	LONG PAD
6	106983	BHCS 5/16-18 x 2 SHLDR ROHS	
7	106947	WASHER FLAT THIN AN 5/16 ROHS	
8	550168	ADJUSTABLE LOCK FLIP-UP ARM MOUNT	RIGHT
8	550169	ADJUSTABLE LOCK FLIP-UP ARM MOUNT	LEFT
9	106935	NYLOCK NUT THIN 1/4 -28 ROHS	
10	202117	5/16-18 NYLOCK NUT THICK BLK	
11	106972	HHCS 1/4-28 x 1 1/2 ROHS	
12	550474	ADJUSTABLE FLIP-BACK ARM BRACKET	RIGHT
12	550475	ADJUSTABLE FLIP-BACK ARM BRACKET	LEFT
13	561016	1" WASHER 5/16" HOLE	
14	561006	1" SADDLE 5/16" HOLE	
15A	561024	ARMREST PAD 10" DESK	

Pos.	Item Number	Description	Remarks
15B	561031	ARMREST PAD 14" FULL	
17	106943	WASHER FLAT AN 1/4 ROHS	
18	107036	SHCS 8 -32 x 3/8 ROHS	
19	225110	10-32 X 1/2" SET SCREW	
20	107022	BHCS 10-32 X 5/8 ROHS	
21	560252	1" COVED STEM BUMPER	
3B, 4A, 5B, 19	882074	FOAM PAD LONG ADJUSTABLE LENGTH	LONG PAD ASSM ONLY
3A, 4A, 5A, 19	882075	FOAM PAD LONG ADJUSTABLE LENGTH	SHORT PAD ASSM ONLY
4B,15B,16B,19,20	882070	FULL PAD ADJUSTABLE LENGTH ASSM	RIGHT FULL PAD ASSM ONLY
4B,15B,16B,19,20	882071	FULL PAD ADJUSTABLE LENGTH ASSM	LEFT FULL PAD ASSM ONLY
4B,15A,16A,19,20	882072	DESK PAD ADJUSTABLE LENGTH ASSM	RIGHT DESK PAD ASSM ONLY
4B,15A,16A,19,20	882073	DESK PAD ADJUSTABLE LENGTH ASSM	LEFT DESK PAD ASSM ONLY
1,2,6-14,17,22	882088	ADJ LENGTH LOCKING FLIP-UP ASSM R	RIGHT ARM ASSM W/O PAD
1,2,6-14,17,22	882089	ADJ LENGTH LOCKING FLIP-UP ASSM R	LEFT ARM ASSM W/O PAD
1,2,3B,4A,5B,6-14,17,19 -21	900454	ADJ LENGTH LOCKING FLIP-UP FOAM LONG	PAIR - LONG FOAM ARM ASSM
1,2,3B,4A,5B,6-14,17,19 -21	881294	ADJ LENGTH LOCKING FLIP-UP FOAM LONG	RIGHT - LONG FOAM ARM ASSM
1,2,3B,4A,5B,6-14,17,19 -21	881295	ADJ LENGTH LOCKING FLIP-UP FOAM LONG	LEFT - LONG FOAM ARM ASSM
1,2,3A,4A,5A,6-14,17,19 -21	900455	ADJ LENGTH LOCKING FLIP-UP FOAM SHORT	PAIR - SHORT FOAM ARM ASSM
1,2,3A,4A,5A,6-14,17,19 -21	881296	ADJ LENGTH LOCKING FLIP-UP FOAM SHORT	RIGHT - SHORT FOAM ARM ASSM
1,2,3A,4A,5A,6-14,17,19 -21	881297	ADJ LENGTH LOCKING FLIP-UP FOAM SHORT	LEFT - SHORT FOAM ARM ASSM
1,2,4B,6-14,15B,16B,17-21	900452	ADJ LENGTH LOCKING FLIP-UP ASSM FULL	PAIR - FULL PAD ARM ASSM
1,2,4B,6-14,15B,16B,17-21	881290	ADJ LENGTH LOCKING FLIP-UP FULL PAD	RIGHT - FULL PAD ARM ASSM
1,2,4B,6-14,15B,16B,17-21	881291	ADJ LENGTH LOCKING FLIP-UP FULL PAD	LEFT - FULL PAD ARM ASSM
1,2,4B,6-14,15A,16A,17-21	900453	ADJ LENGTH LOCKING FLIP-UP ASSM DESK	PAIR - DESK PAD ARM ASSM
1,2,4B,6-14,15A,16A,17-21	881292	ADJ LENGTH LOCKING FLIP-UP DESK PAD	RIGHT - DESK PAD ARM ASSM
1,2,4B,6-14,15A,16A,17-21	881293	ADJ LENGTH LOCKING FLIP-UP DESK PAD	LEFT - DESK PAD ARM ASSM

**NOTE:** USED ON QUICKIE 2 CHAIRS W/ S/N PREFIX Q2V.


**NOTE:** USED ON QUICKIE 2 LITE CHAIRS.

**NOTE:** USED ON ZIPPIE TS CHAIRS W/ S/N PREFIX Z1A & Z1SE.

**NOTE:** USED ON ZIPPIE GS CHAIRS W/ S/N PREFIX ZGS & ZGSE.

**NOTE:** USED ON ZIPPIE 2 CHAIRS W/ S/N PREFIX Z2A.

**NOTE:** CLAMP MOUNTS TO 1" TUBING.

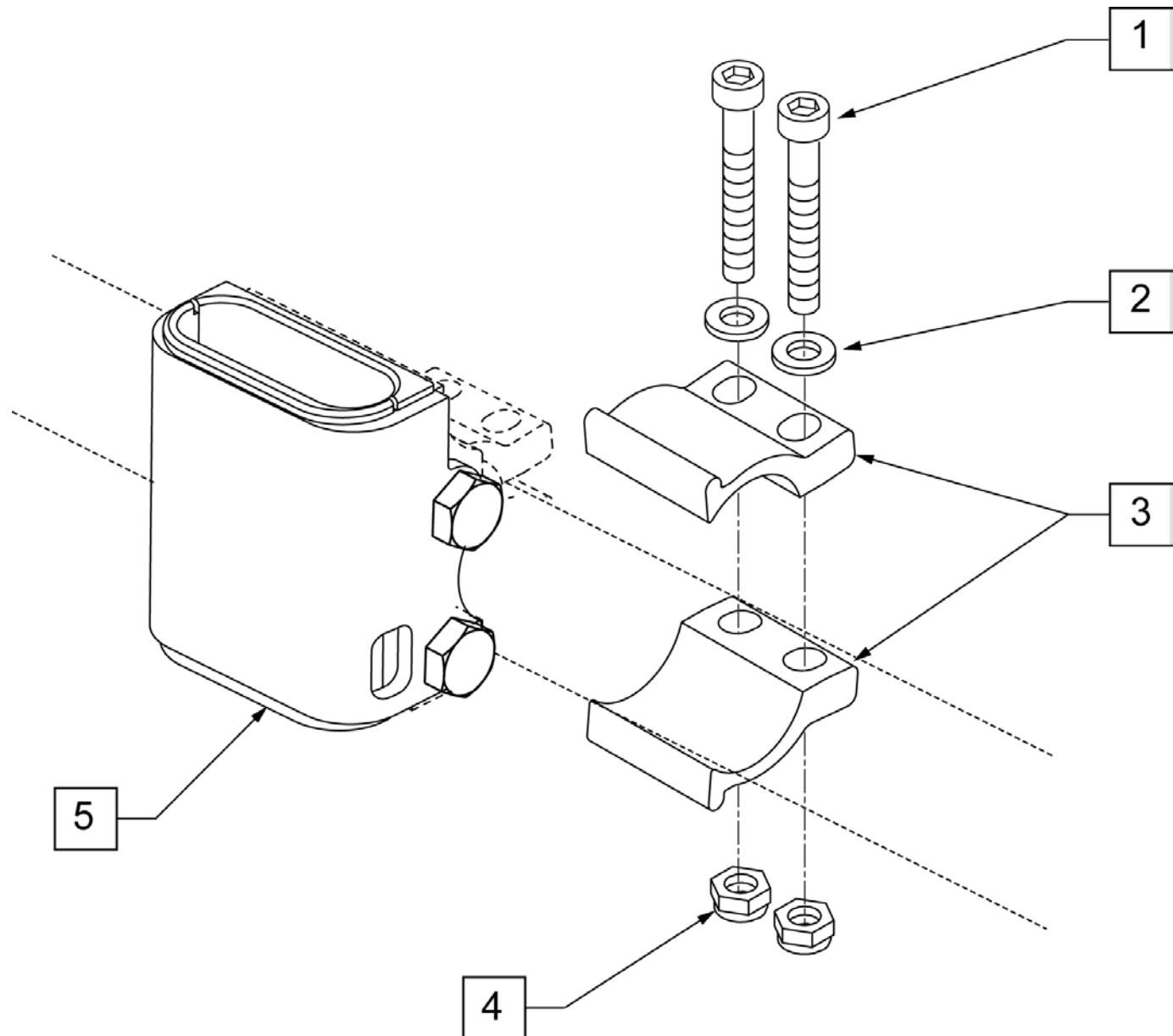


Pos.	Item Number	Description
1	106975	SHCS 1/4 -28 x 1 1/4 ROHS
2	106943	WASHER FLAT AN 1/4 ROHS
3	107123	CLAMP, RECEIVER, 1" OD
4	106935	NUT NYLOCK THIN 1/4 -28 ROHS
5	107124	RECEIVER, SPAH 1" OD
1-5	108101	SPAH RECEIVER W/ CLAMP KIT

**NOTE:** CLAMP MOUNTS TO 1" TUBING.

**NOTE:** USED ON ZIPPIE TS CHAIRS W/ S/N PREFIX Z1A & Z1SE.


**NOTE:** USED ON ZIPPIE GS CHAIRS W/ S/N PREFIX ZGSA & ZGSE.

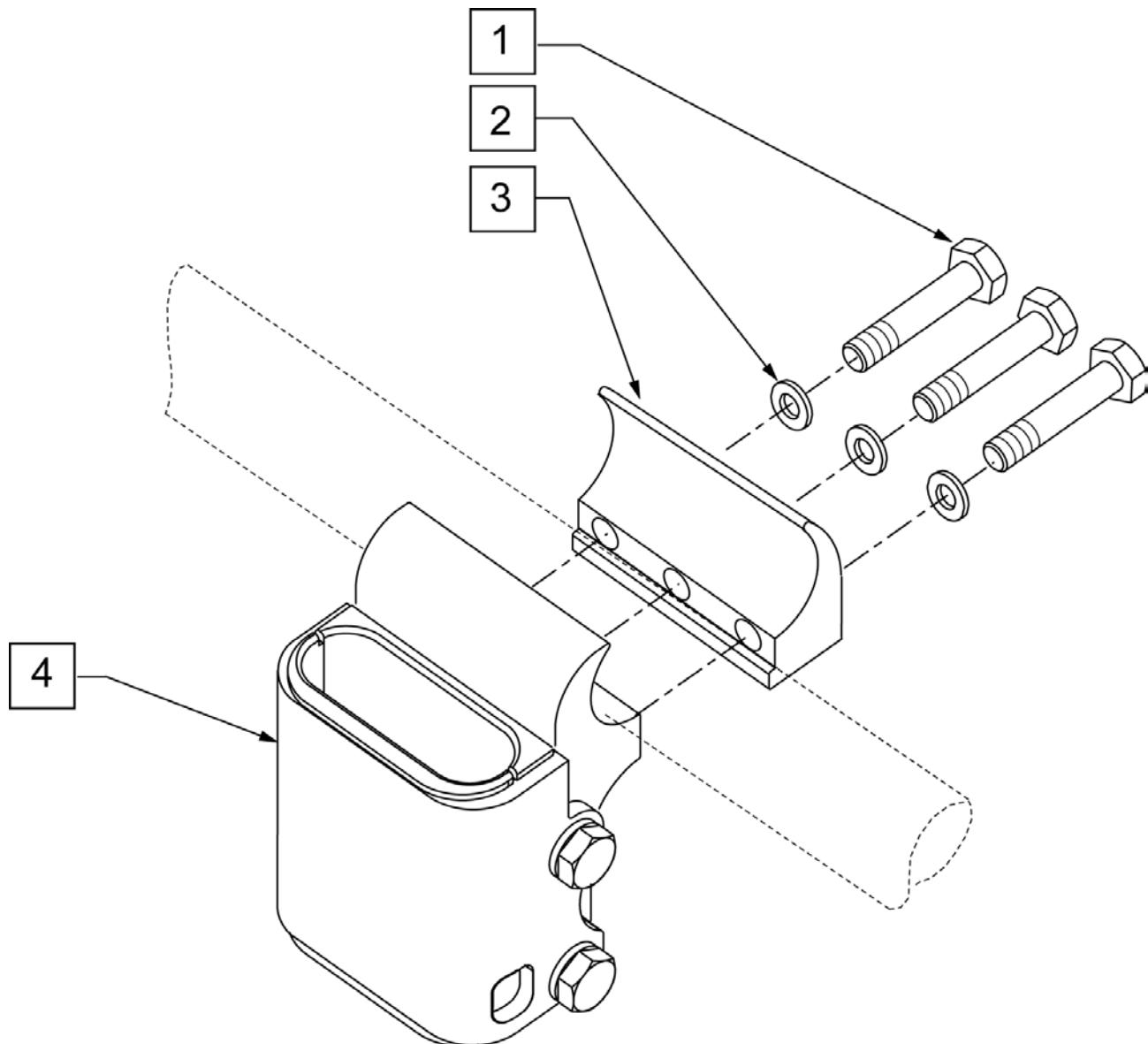


<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>
1	107360	HHCS 1/4 -28 x 1 1/4 PL ROHS
2	106943	WASHER FLAT AN 1/4 ROHS
3	550529	HEIGHT ADJ RECEIVER CLAMP ZIPPIE
4	112424	RECEIVER ARMREST LOW ZP
1-4	115650	SPHA REC LOW ZP W/ CLAMP KIT

**AUTO BUCKLE POSITIONING BELTS**

(04/2011)

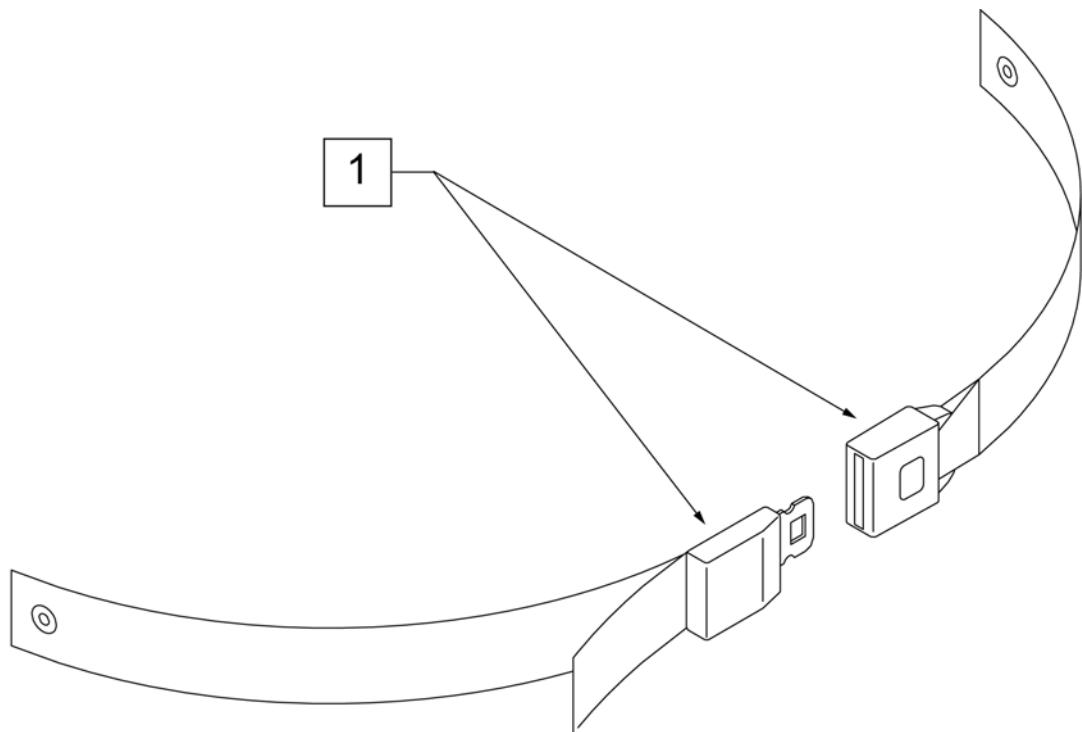


Pos.	Item Number	Description	Remarks
1	750276	1 1/2" POSITION BELT	AUTOSTYLE-64" Max length of belt.

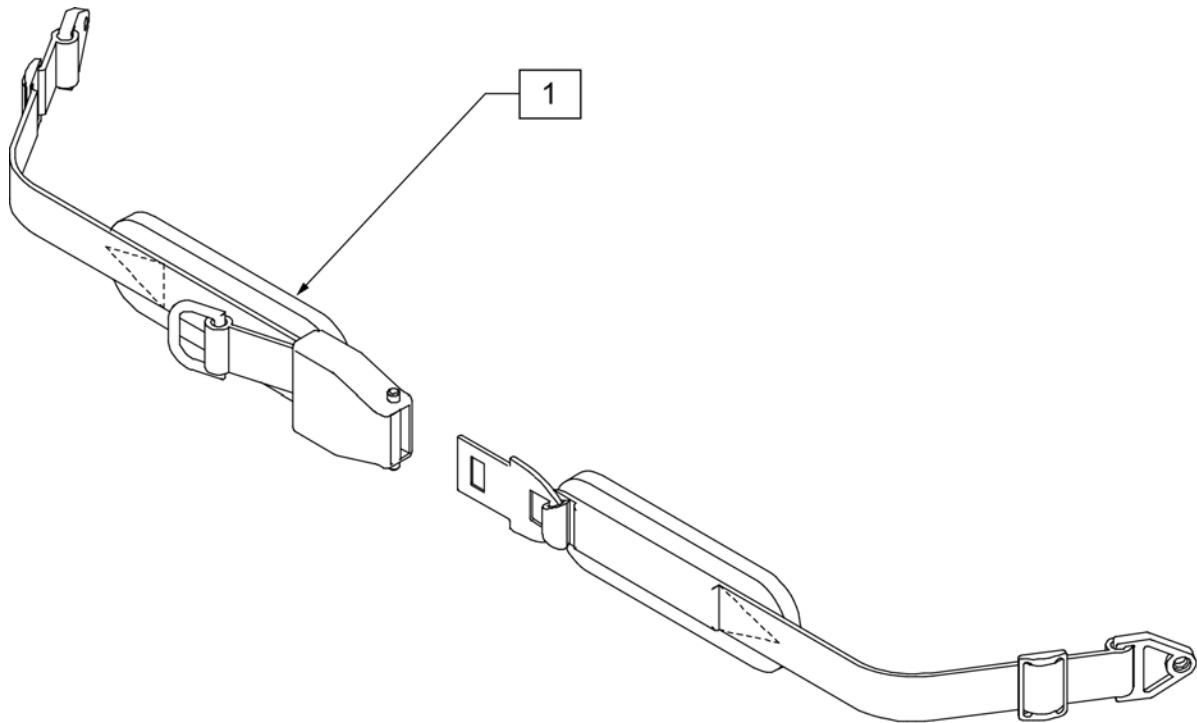

Pos.	Item Number	Description
1	450418-QU	BELT 2" AIRCRAFT BUCKLE
1	450423-QU	BELT 2" AIRCRAFT BUCKLE W/PAD
1	450424	BELT 2" AIRCRAFT BKL LONG W/PAD
1	450425	BELT 2" AIRCRAFT BKL LONG
1	450419-QU	BELT 2" AIRCRAFT BKL W/QUAD LP
1	468419	BELT 2" AIRCRT BKL W/QD LPCZ

PADDED DOUBLE PULL HIP BELT

[11/2011]

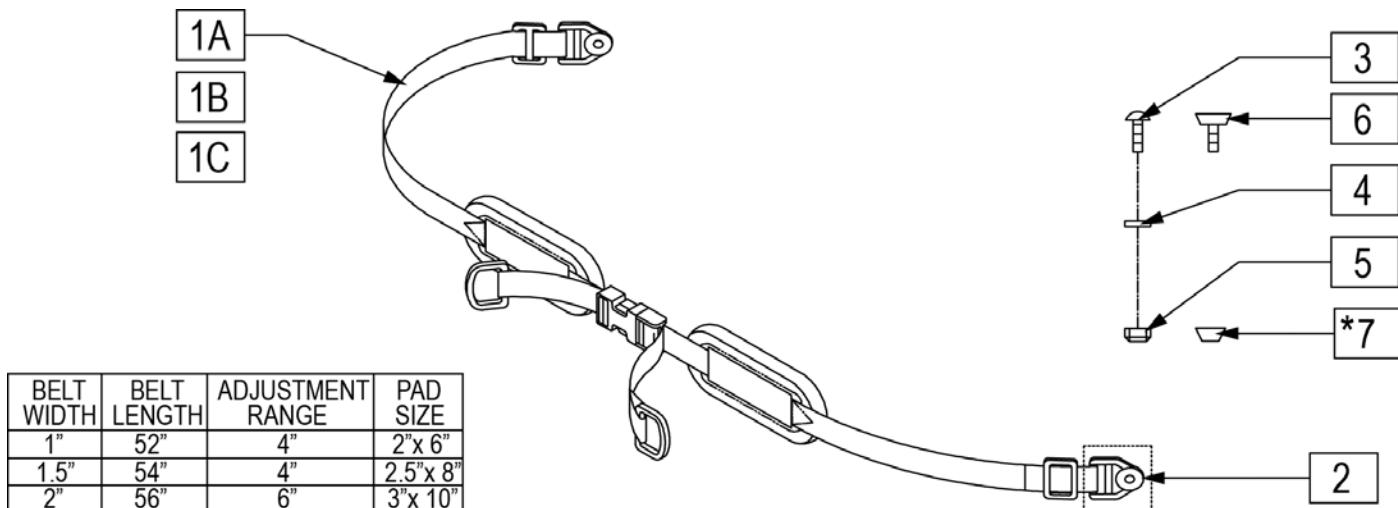


NOTE : ITEM 7 CHANNEL NUT IS USED WHEN ORDERING IN  
CONJUNCTION WITH CUSTOM CONVENTIONAL SEATING

Pos.	Item Number	Description
1A	450420-QU	BELT 1" DUAL ADJ. SQZ BKL PAD
1B	450421-QU	BELT 1.5" DUAL ADJ. SQZ BKL PAD
1C	450422-QU	BELT 2" DUAL ADJ. SQZ BKL PAD
2	451116-QU	PLASTIC WEBBING GROMMET
3	455036-QU	1/4-28 x 9/16 SPEC ROUND HD
4	106943	WASHER FLAT AN 1/4 ROHS
5	479020	CHAN.UNF NUT 1/4-28 W/SETSCRW
6	106935	NUT NYLOCK THIN 1/4-28 ROHS
7	481106-QU	1/4-28 CHANNELNUT
1A,2,4-6	476520	BELT 1" DUAL ADJ SQZ BKLPAD AC
1B,2,4-6	476521	BELT 1.5" DUAL ADJ SQZ BKLPAD AC
1C,2,4-6	476522	BELT 2" DUAL ADJ SQZ BKLPAD AC
1A-5	468420	BELT 1" DUAL ADJ.SQZ BKLPAD CZ
1B-5	468421	BELT 1.5" DUAL ADJ.SQZ BKLPAD CZ
1C-5	468422	BELT 2" DUAL ADJ.SQZ BKLPAD CZ


**POSITIONING BELT ATTACHMENT KIT**

Note: Positioning Belt Attachment Kit updated to Metric Fasteners April 2013. If in doubt order complete kit.

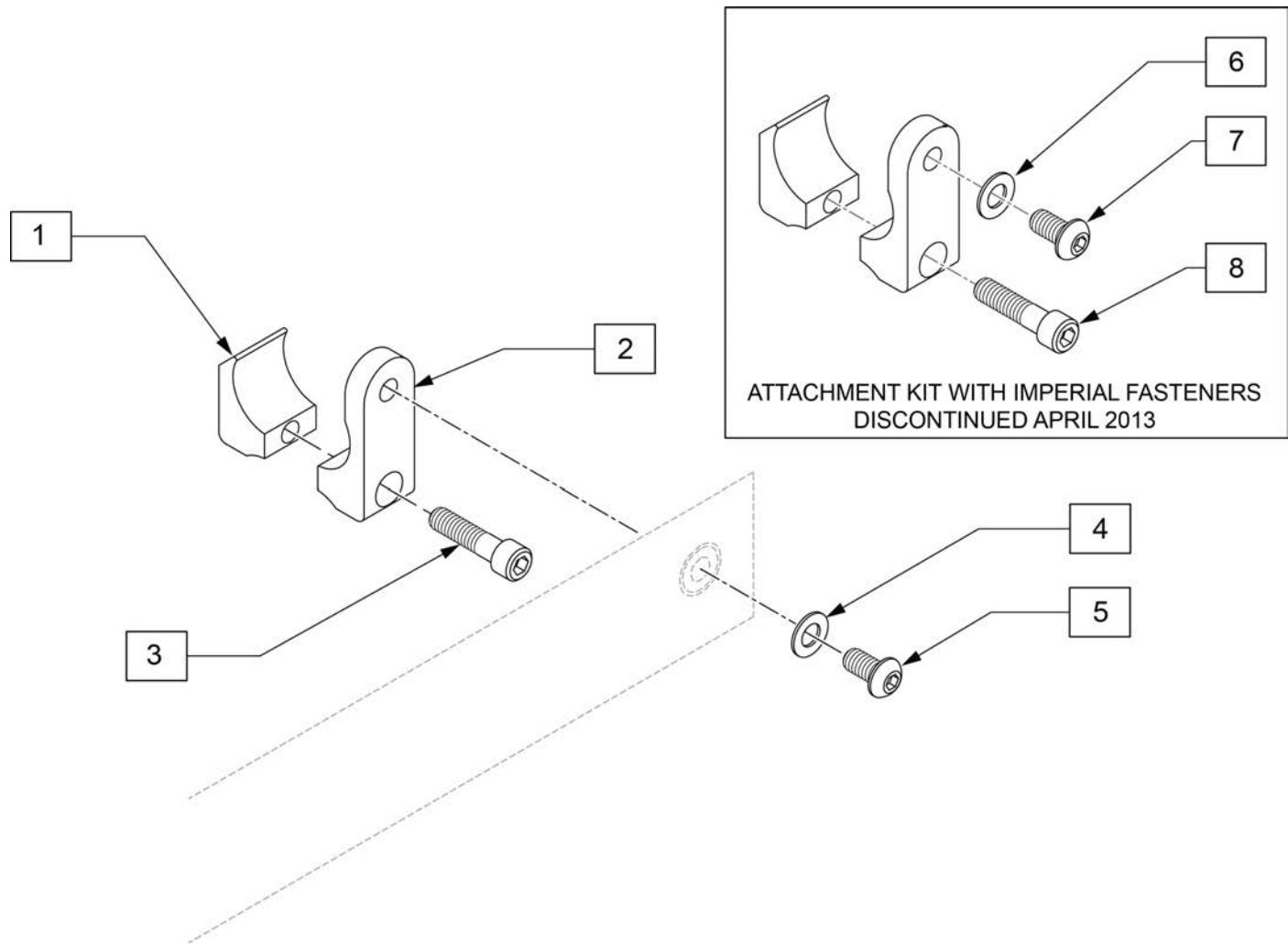


Pos.	Item Number	Description	Remarks
3	102263-003	SHCS M6 X 1 X 25 W/PL INORG	Metric
4	250104	WASHER FLAT M6	Metric
5	103185-005	BHCSM6X1X12FULLPL	Metric
1-5	120750	PSTN BELT CLAMP ASSM PR	Metric, Pair
6	106943	WASHER FLAT AN 1/4 ROHS	Imperial
7	226154	1/4-20 X 1/2 BH C/S W/PATCH	Imperial
8	221116	/4-20 X 1 SOC HD BLK	Imperial


**NOTE:** KIDS BACKPACKS DO NOT HAVE ADJUSTABLE STRAPS W/ BUCKLES.

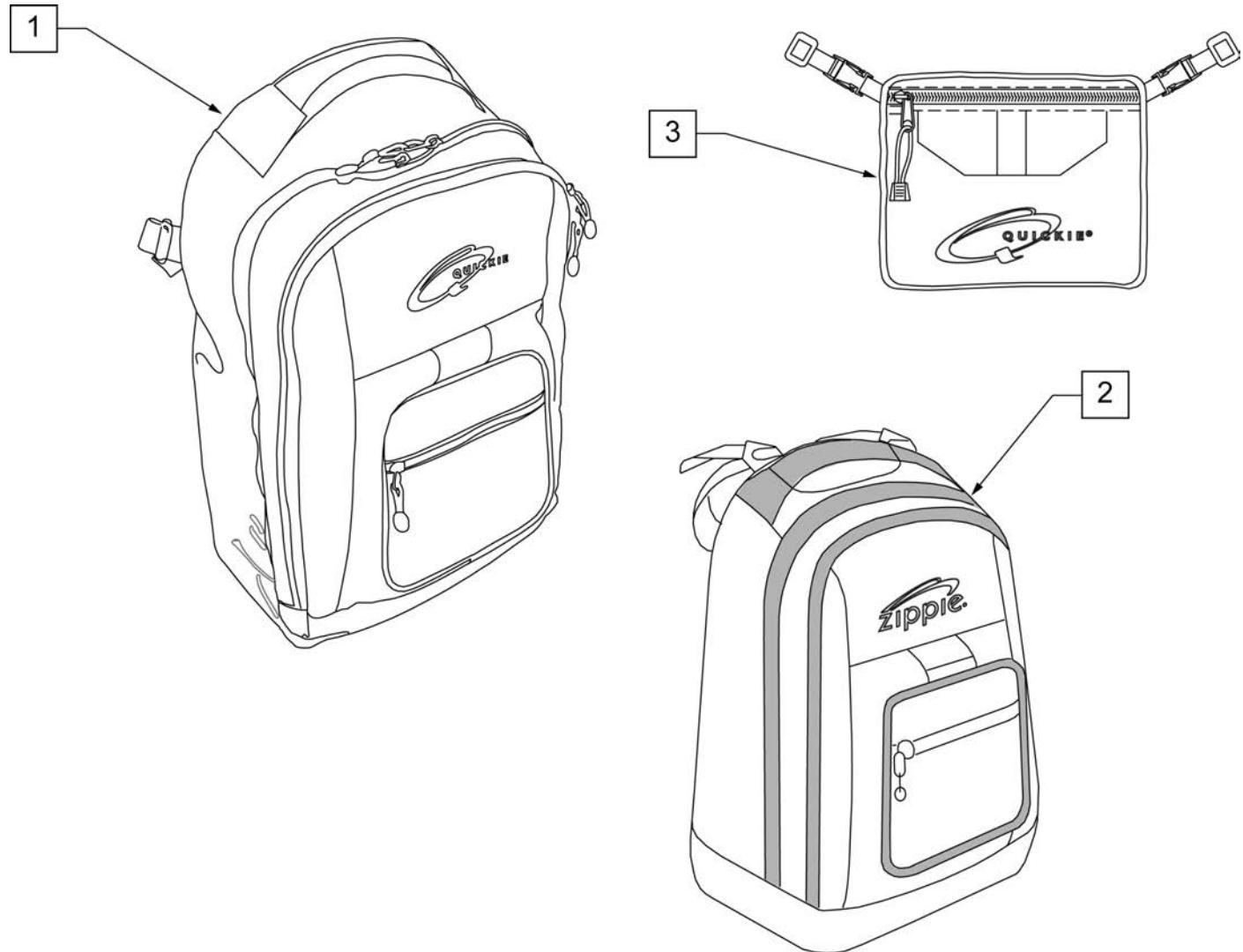

Pos.	Item Number	Description
1	750115	BACK PACK QUICKIE
2	104572	BACKPACK ZIPPIE
3	750307	SEAT POUCH QUICKIE

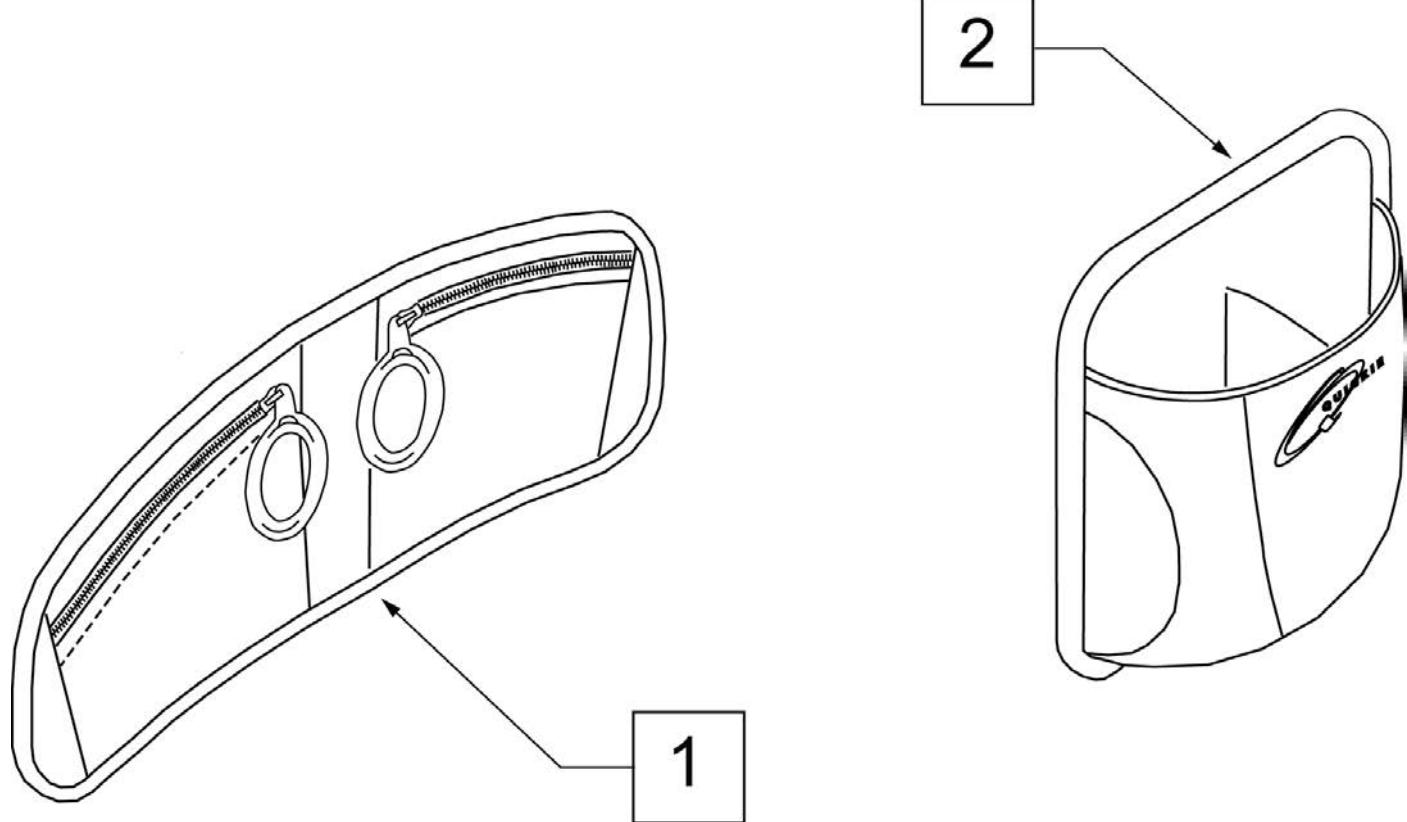


Pos.	Item Number	Description	Remarks
1	013477	LAP ORGANIZER	Size: 18.25"x 7.75"x .75"
2	013475	QUICKIE SIDE POCKET	Size: 6.5"x 6.5"x 2"

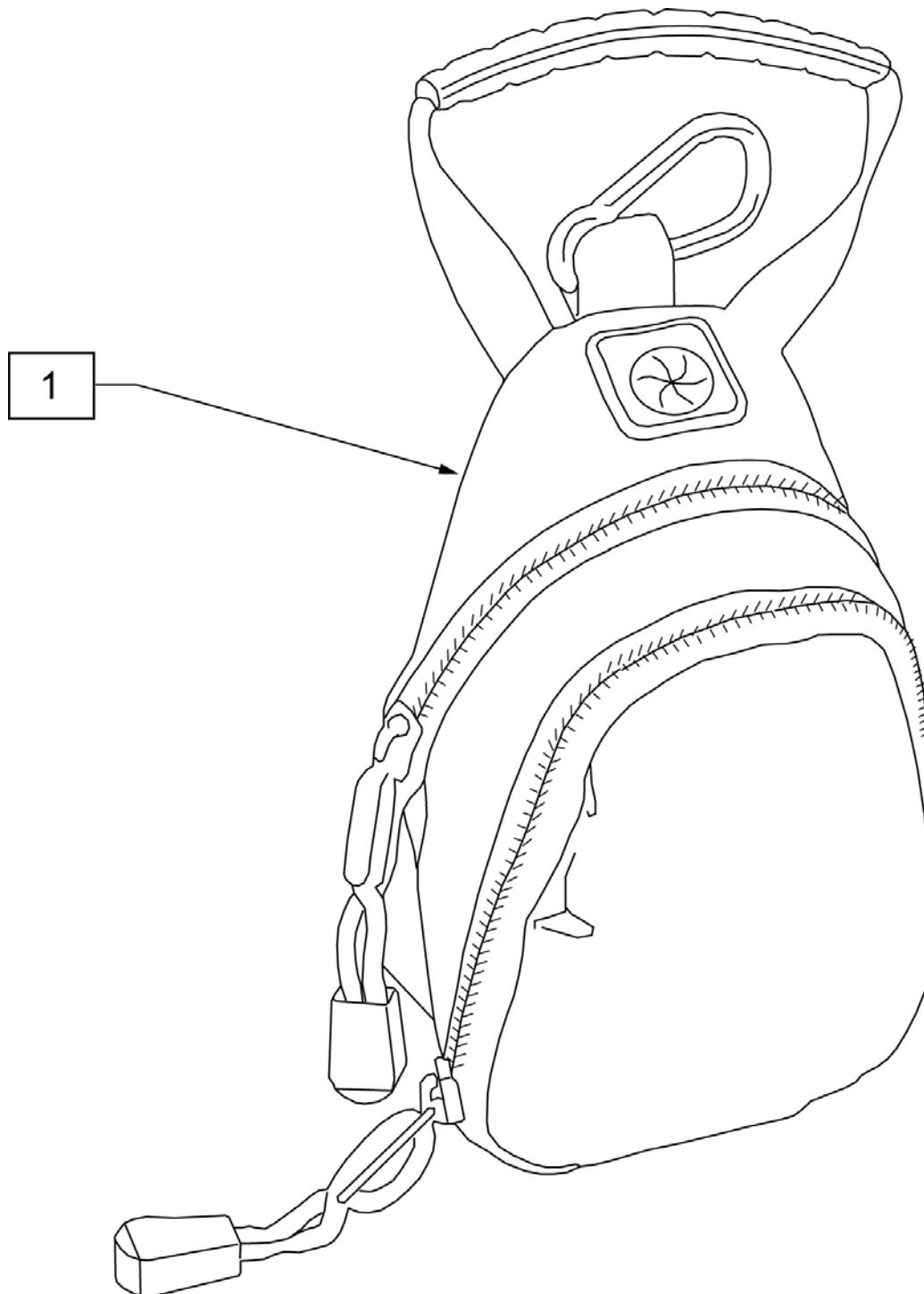


Pos.	Item Number	Description
1	109229	CELL PHONE PACK (ORANGE)

**NOTE:** WHEN ADDING A TRAY TO AN EXISTING CHAIR WITH DESK OR FULL LENGTH ARM PADS, ADD 1" TO CURRENT SEAT WIDTH TO ACHIEVE THE BEST TRAY SIZE.

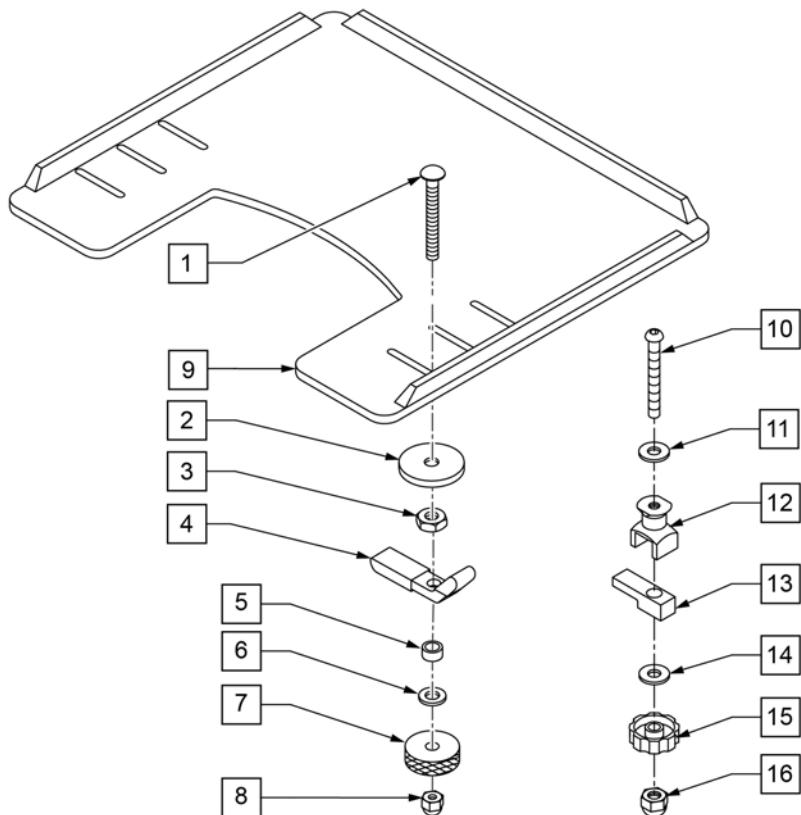**MEASURING GUIDE**


	Dim "A"	Dim "B"	Dim "C"	Dim "D"
X-Small	21	16	10	7
Small	21	18	12	7
Medium	24	22	14	9
Large	26	24	18	10
X-Large	28	24	19	10

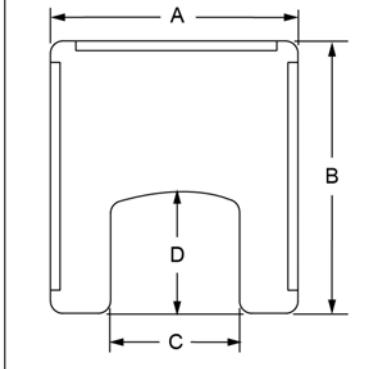


Pos.	Item Number	Description	Remarks
1	226199	5/16-18 x 3 1/2 CARR BOLT	
2	263112	5/16 WASHER FENDER	
3	202110	5/16-18 NUT THIN	
4	550400	TRAY CLAMP	
5	481113	SPACER 5/16 X .380	
6	263100	5/16 AN WASHER FLAT	
7	483026	KNURLED TRAY KNOB	
8	202112	5/16-18 ACORN NUT	
1-8	881051	TRAY HARDWARE KIT TUFFAK NEW STYLE	Pair
9A	562619	TRAY TOP EX-SM W/RAIL	
9B	562620	TRAY TOP SM W/RAIL	
9C	562621	TRAY TOP MED W/RAIL	
9D	562622	TRAY TOP LRG W/RAIL	
9E	562623	TRAY TOP X-LRG W/RAIL	
10-15	881053	TRAY HARDWARE ADJ ARM PWR (14219)	Pair
1-8,9A	902306	TRAY ASSEMBLY EX-SM TUFFAK (10 -12")	Fits 10"-12" wide chairs
1-8,9B	902303	TRAY ASSEMBLY SM TUFFAK (13 -14")	Fits 13"-14" wide chairs
1-8,9C	902304	TRAY ASSEMBLY MED TUFFAK (15 -17")	Fits 15"-17" wide chairs
1-8,9D	902305	TRAY ASSEMBLY LRG TUFFAK (18 -20")	Fits 18"-20" wide chairs
1-8,9E	902309	TRAY ASSEMBLY X-LRG TUFFAK (21 -22")	Fits 21"-22" wide chairs
9A,10-15	902330	TRAY ASSM X-SM TFK 10-12/CAM	Fits 10"-12" wide chairs

<b>Pos.</b>	<b>Item Number</b>	<b>Description</b>	<b>Remarks</b>
9B,10-15	902331	TRAY ASSM SM TUFFAK 13-14/CAM	Fits 13"-14" wide chairs
9C,10-15	902332	TRAY ASSM MED TUFFAK 15-17/CAM	Fits 15"-17" wide chairs
9D,10-15	902333	TRAY ASSM LRG TUFFAK 18-20/CAM	Fits 18"-20" wide chairs

## TRAY HARDWARE



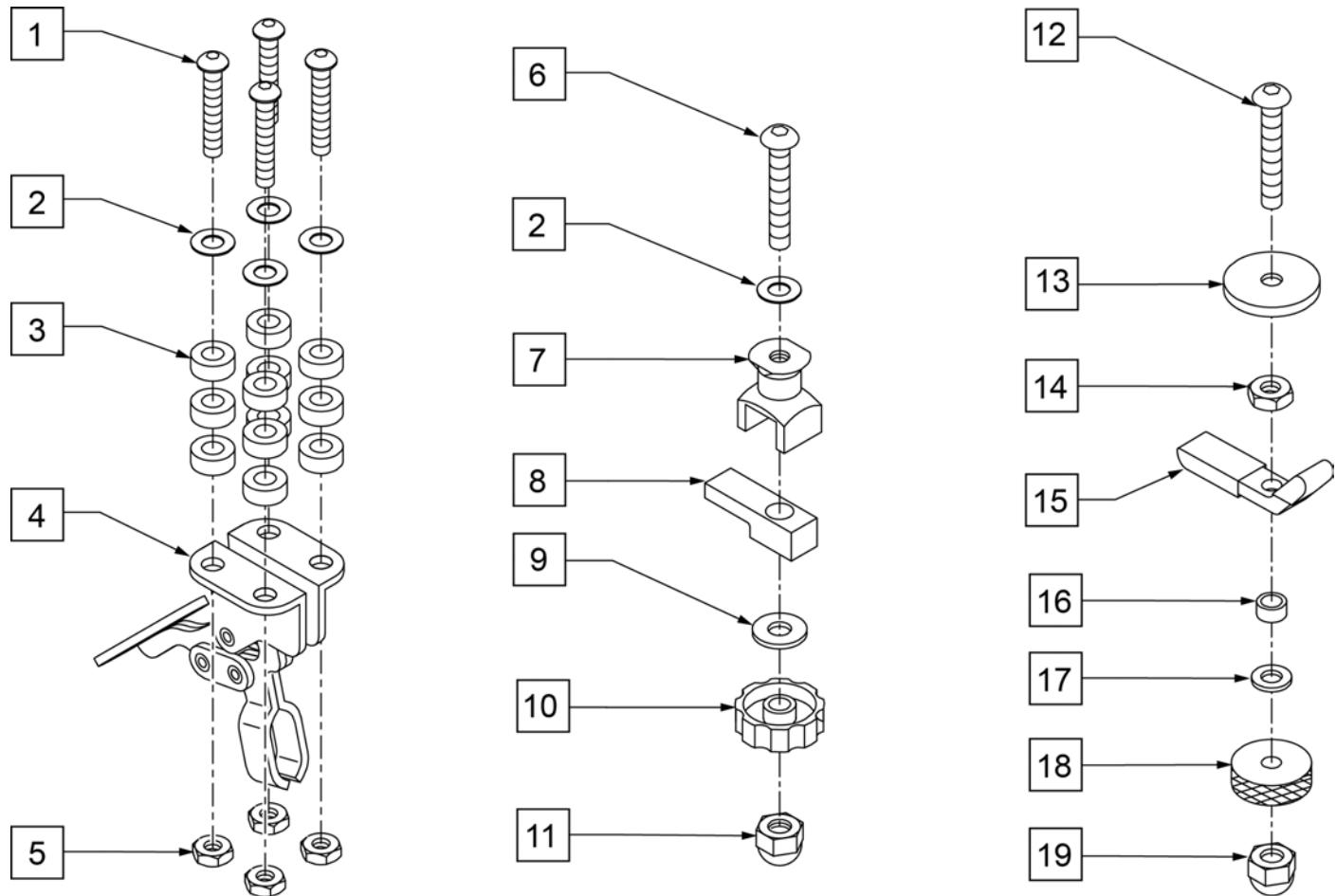


Pos.	Item Number	Description	Remarks
1	2249	5/16-18 x 1 1/2 BH C/S BLK	
2	2709	.625 x .135 x .031 NYLON WASHER	
3	2710	.609 x .250 x .250 NYLON SPACER	
5	202110	5/16-18" NUT THIN	
6	2021	5/16-18" x 2 3/4" BUTTON HD C/S	
7	14240	TRAY BUSHING	
8	14230	TRAY LOCK BAR	
9	2705	1/4" FLAT WASHER	
10	14220	TRAY KNOB	
11	202114	5/16-18" ACORN LOCK NUT	
12	226199	5/16-18" x 3 1/2" CARR BOLT	
13	263112	5/16 WASHER FENDER	
14	202110	5/16-18" NUT THIN	
15	550400	TRAY CLAMP	
16	481013	BEARING SPACER	
17	263100	5/16" AN WASHER FLAT	
18	483026	KNURLED TRAY KNOB	
19	202112	5/16-18" ACORN NUT	
1-5	4099	TOGGLE CLAMP - HIDEAWAY	
6-11	881053	TRAY HARDWARE ADJ ARM PWR (14219)	
12-19	881051	TRAY HDWR KIT TUFFAK NEW STYLE (9QH)	Pair

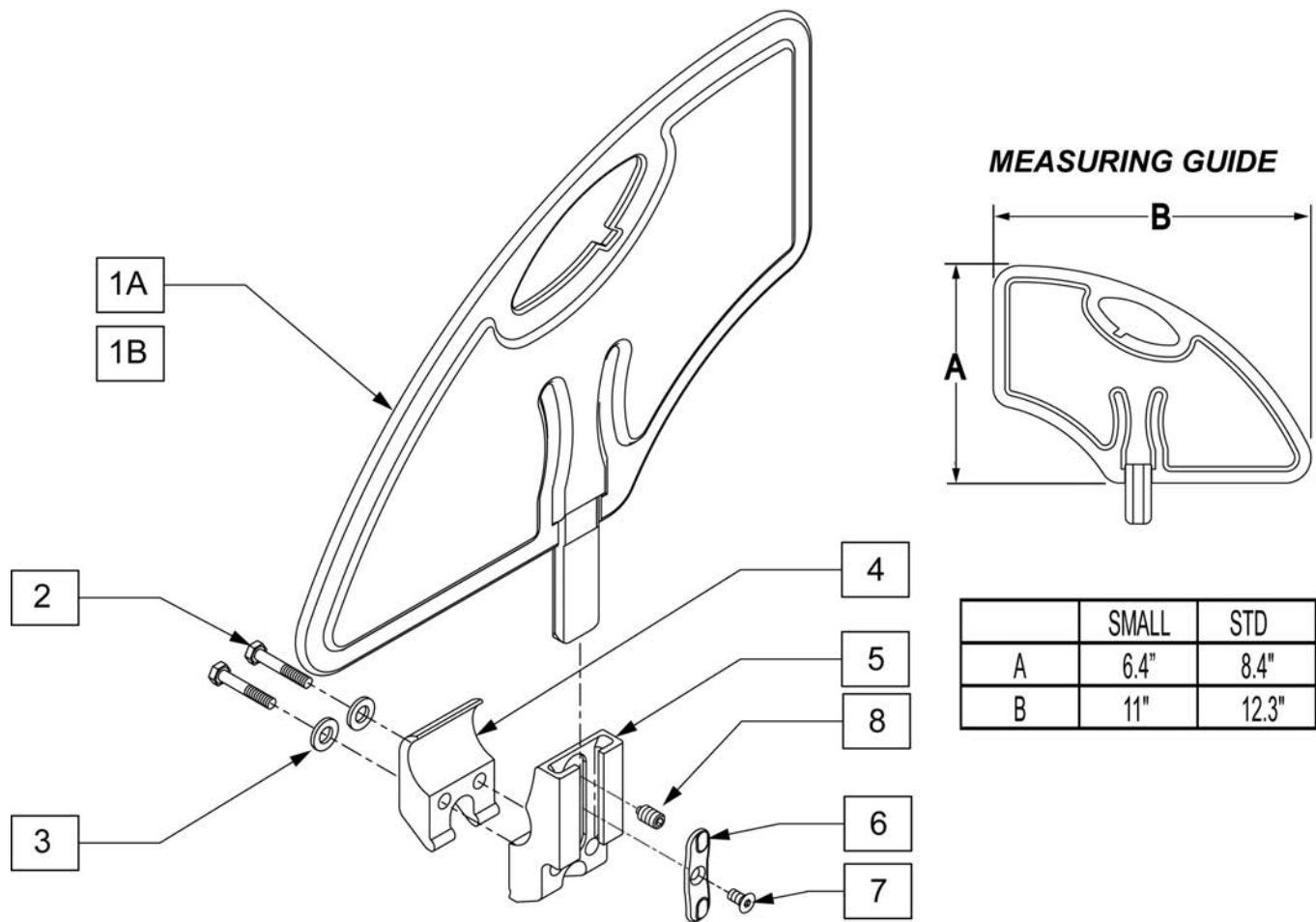

	SMALL	STD
A	6.4"	8.4"
B	11"	12.3"

Pos.	Item Number	Description	Remarks
1A	104164-001	SIDE GUARD ASSY STD	STANDARD
1B	104164-002	SIDE GUARD ASSY SMALL	SMALL
2	106970	HHCS 1/4-28 X 1-1/4 ROHS	
3	106943	WASHER FLAT AN 1/4 ROHS	
4	106714	REC SIDE GUARD CLAMP THREADED	
5	108780	RECEIVER SG PLASTIC INSERT	
6	107135	FHCS #6-32 X 1/4 BLK	
7	107136	SETSCREW 1/4-28 X 5/16 OVAL PT	
8	108781	RECIEVER SIDE GUARD ASSY	
2-8	115651	RECEIVER SIDE GUARD ASSY	
1A-8	115755	SIDE GUARD ASSY W/ CLAMP STD	EACH - STANDARD
1B-8	115756	SIDE GUARD ASSY W/ CLAMP SMALL	EACH - SMALL

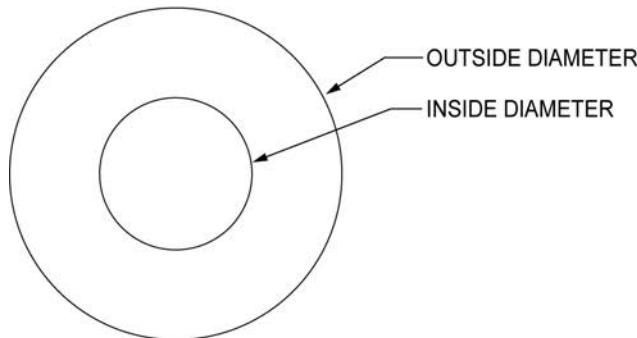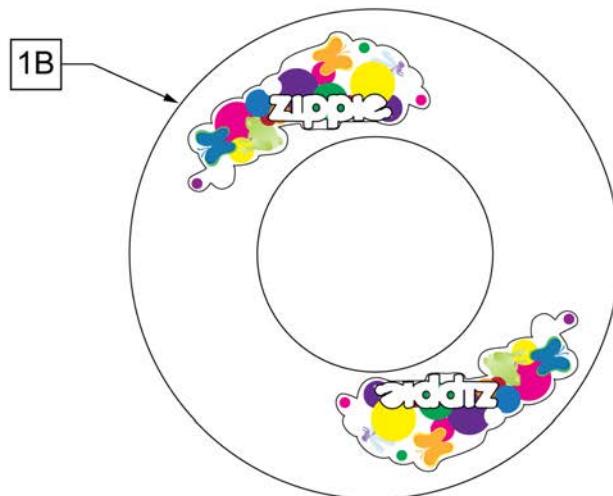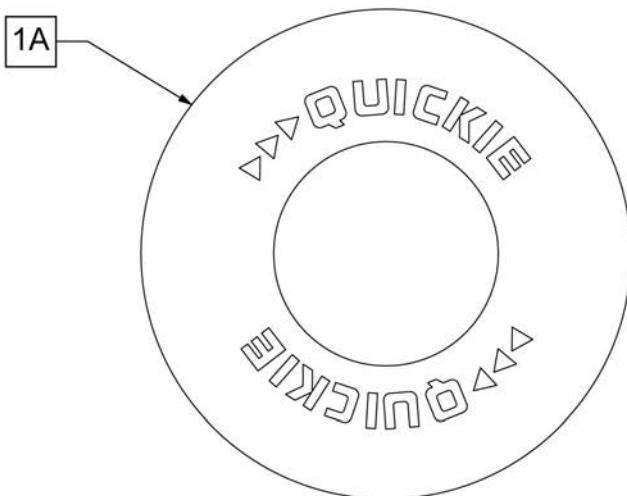


## SPOKE GUARD MEASURING GUIDE

PART NUMBER	INSIDE DIAMETER	OUTSIDE DIAMETER
902001	8 1/4	15 7/8
902011	8 1/4	18 1/8
902021	10 1/4	19 5/8
902025	10 1/4	20 3/8
103660-001	8 1/4	16
103660-002	8 1/4	18
103663-001	10 1/4	20 1/2
103663-002	10 1/4	23 1/2


Pos.	Item Number	Description	Remarks
>>> BLACK SPOKE GUARDS			
1A	902001	SPK GRDS 20" QUICKIE BLK	Pair-BLACK-FITS 20" WHEELS
1A	902011	SPK GRD 22" QUICKIE BLK	Pair-BLACK-FITS 22" WHEELS
1A	902021	SPK GRD 24" QUICKIE SCR MT BLK	Pair-BLACK-FITS 24" (540) MAG & SPOKE WHEELS
1A	902025	SPK GRD 24" QUICKIE TAB MT BLK	Pair-BLACK-FITS 25" (559) SPOKE WHEELS
>>> CLEAR SPOKE GUARDS			
1B	112644-001	SPOKE GUARD ZIPPY 20"	Pair-CLEAR-FITS 20" WHEELS
1B	112644-002	SPOKE GUARD ZIPPY 22"	Pair-CLEAR-FITS 22" WHEELS
1B	112644-003	SPOKE GUARDZIPPY 24" MAG	Pair-CLEAR-FITS 24" (540) MAG WHEELS
1B	112644-004	SPOKE GUARD ZIPPY 24" SPOKE	Pair-CLEAR-FITS 24" (540) SPOKE WHEELS

**IMPACT GUARD NEOPRENE & GEL VELCRO PADS**


(09/2009)

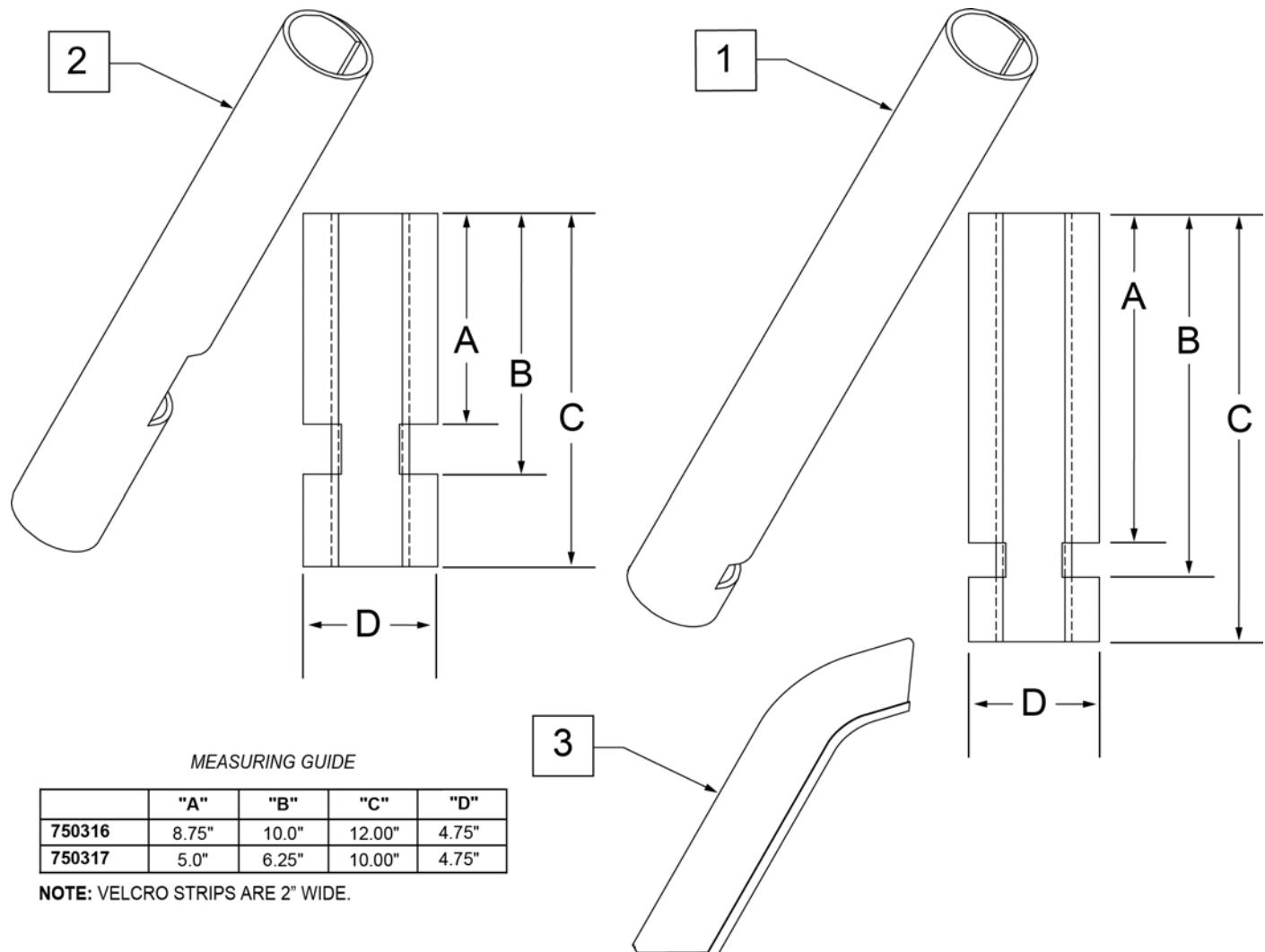


Pos.	Item Number	Description	Remarks
1	750316	IMPACT GUARDS NEOPRENE FIXED (PR)	PAIR-Fixed Front Frame
2	750317	IMPACT GUARDS NEOPRENE S/A	PAIR-S/A Footrest & Q7
3	109231	GEL VELCRO PAD HANGER/FRAME	PAIR- fits S/A & Fixed Front Frame

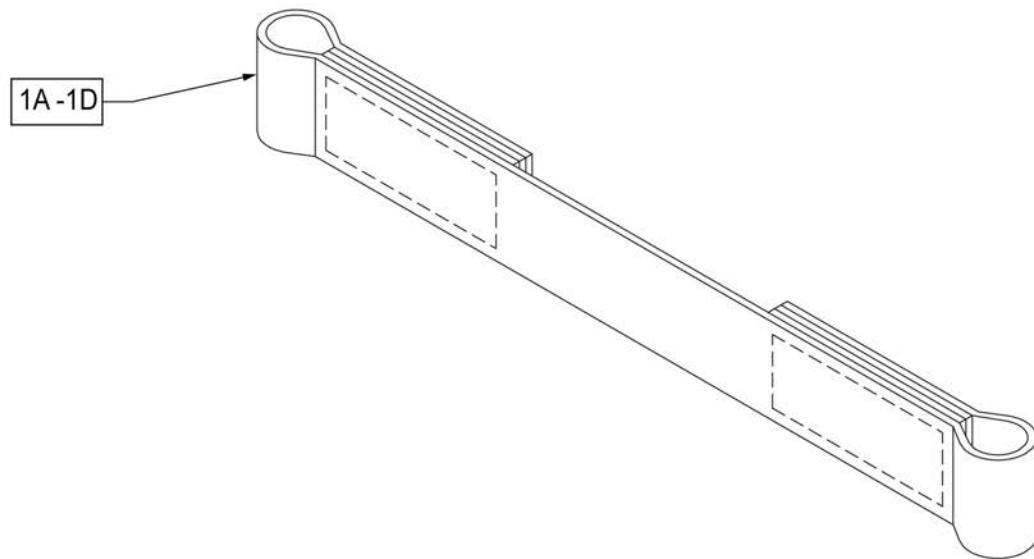

Pos.	Item Number	Description
1A	109356	LEG STRAP SOFT FEEL Q 11 -12"
1B	109357	LEG STRAP SOFT FEEL Q 13 -15"
1C	109366	LEG STRAP SOFT FEEL Q 16 -18"
1D	111487	LEG STRAP SOFT FEEL Q 19 -20"

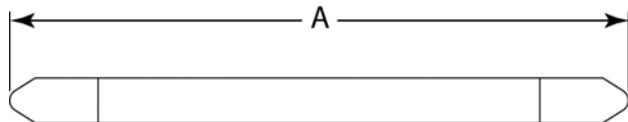
**MEASURING GUIDE**

DESCRIPTION	DIM "A"
LEG STRAP 11"-12" BLK	27 1/2
LEG STRAP 13"-15" BLK	32 1/2
LEG STRAP 16"-18" BLK	39 1/2
LEG STRAP 19"-20" BLK	45 1/2

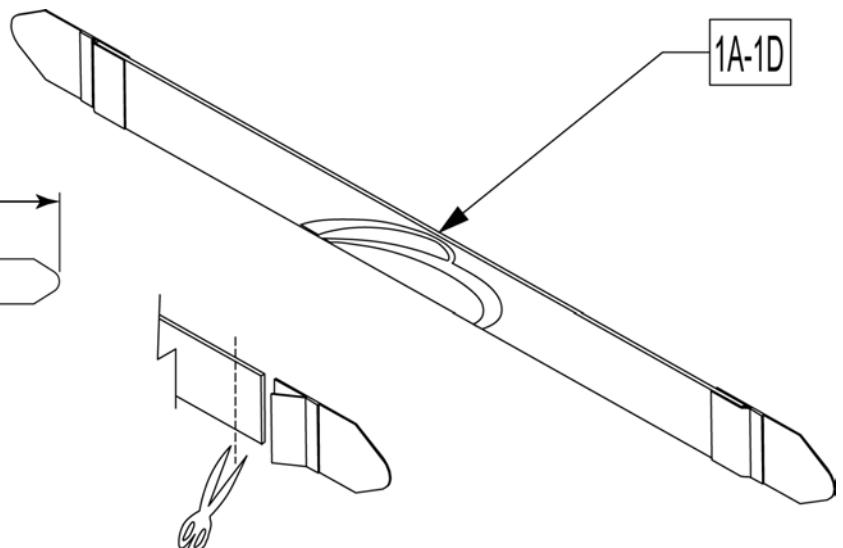

Leg Straps may be custom cut to a smaller size


Pos.	Item Number	Description
1A	113255	LEG STRAP 11"-12" BLK
1B	113256	LEG STRAP 13"-15" BLK
1C	113257	LEG STRAP 16"-18" BLK
1D	113258	LEG STRAP 19"-20" BLK

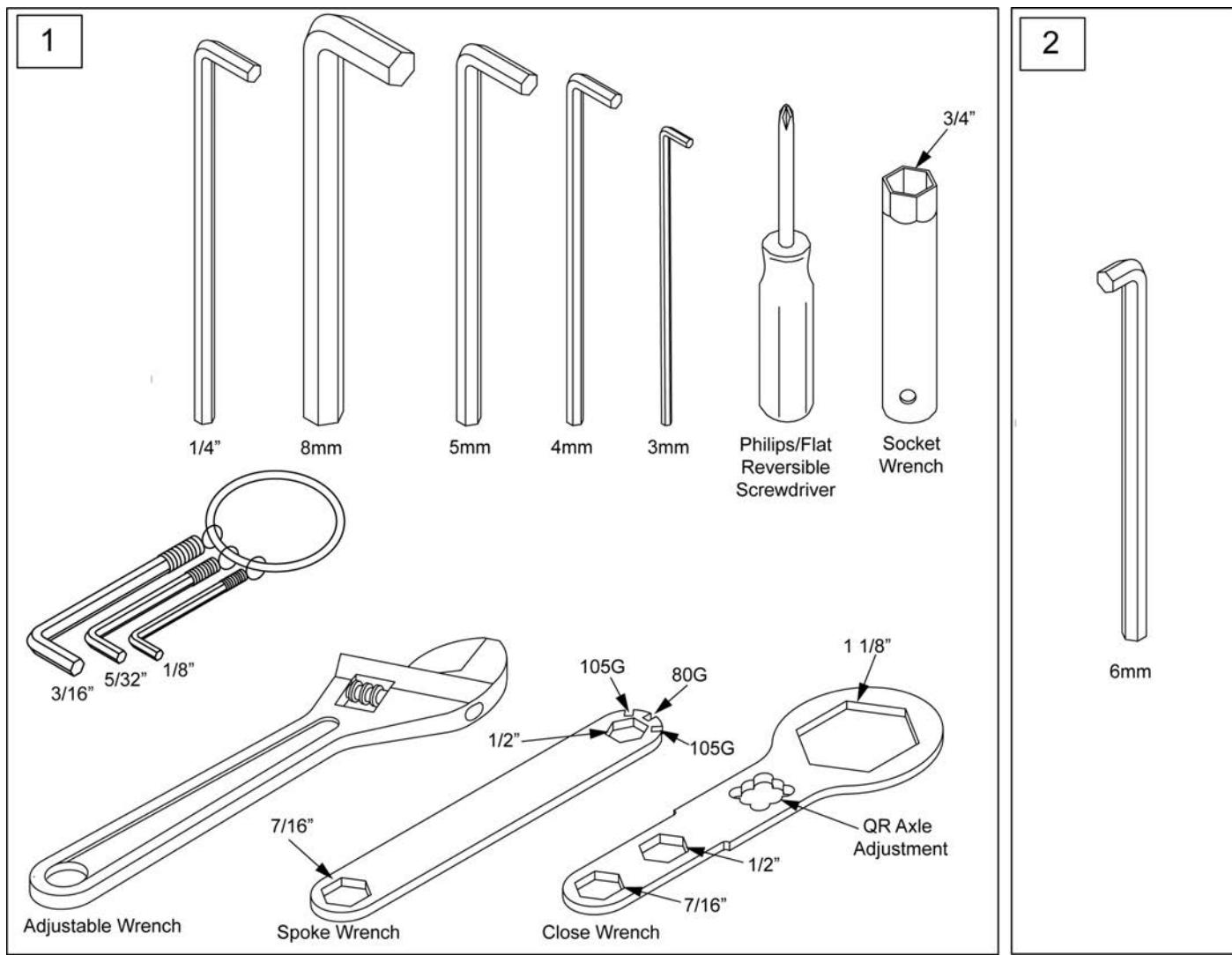


Pos.	Item Number	Description	Remarks
1	108314	QUICKIE COMBO TOOL KIT	Includes pouch
2	113403	6MM ALLEN WRENCH L, 7MM SHORT	caster spline chairs only

# ONE-ARM DRIVE RETRO KIT

[06/2011]

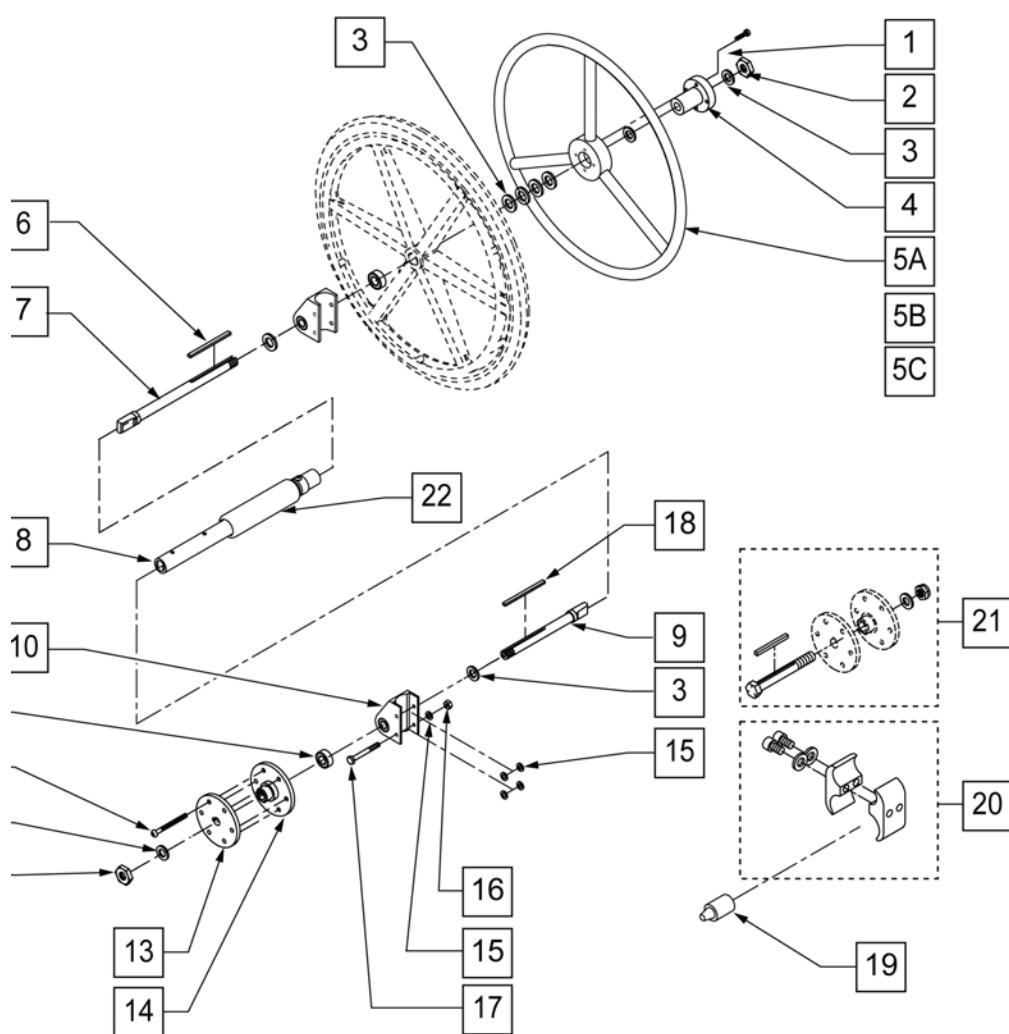


**MEASURING GUIDE**

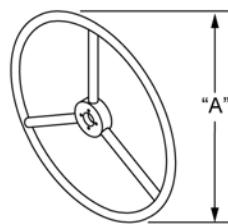


WHEEL SIZE	STD "A"	SMALL "A"
20"	16 1/2"	-
22"	18"	16 1/2"
24"	19 1/4"	18"

**UNLOCKED - OPEN**

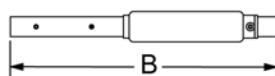

**LOCKED - CLOSED**

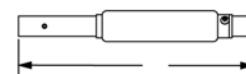


CHAIR WIDTH	"B"	"C"
13"	9 3/4"	8"
14"	10 3/4"	9"
15"	11 3/4"	10"
16"	12 3/4"	11"
17"	13 3/4"	12"
18"	14 3/4"	13"
19"	15 3/4"	14"
20"	16 3/4"	15"

Pos.	Item Number	Description	Remarks
1	106956	10-32 x 3/4 SOC C/S BLK	
2	106940	1/2-20 NYLOCK NUT	
3	106949	1/2 AN WASHER FLAT	
4	106691	1 AD HUB CENTER	
5A	529801	1 AD HR ONLY 20" STD/22"SM	
5B	529800	1 AD HR ONLY 22" STD/24"SM	
5C	529802	1 AD HR ONLY 24" STD	
6	341274	3/16 SQ x 1-1/2 KEY STL	
7	481131	1 AD HANDRIM DRIVE SHAFT	
8	101097-001	1 AD SHAFT ASSM 12"	
8	101097-002	1 AD SHAFT ASSM 13"	
8	101097-003	1 AD SHAFT ASSM 14"	
8	101097-004	1 AD SHAFT ASSM 15"	
8	101097-005	1 AD SHAFT ASSM 16"	
8	101097-006	1 AD SHAFT ASSM 17"	
8	101097-007	1 AD SHAFT ASSM 18"	
9	481132	1 AD WHEEL DRIVE SHAFT	
10	880834	1 AD AXLE PLATE W/BEARINGS LX	


Pos.	Item Number	Description	Remarks
11	483058	MAG AXLE SPACER W/HOLE 1 AD WHEEL SPACER	LEFT
12	107227	1/4-28 x 2 1/2 BH C/S BLK	
13	500630	1 AD WHEEL HUB W/O TAP	
14	500631	WHEEL HUB 1 ARM DRIVE W/TAP	
15	106944	1/4 AN WASHER FLAT THIN	
16	106934	1/4-28 NYLOCK NUT	
17	106973	1/4-28 x 2 G8 HEX C/S	
18	341277	3/16" SQ x 2-3/8" KEY	
19	482030	1 AD DRIVE RECEIVER	
20	107639	WHL LCK CLMP ASM	
21	330015	1AD ALIGNMENT TOOL ASSM	
22	101109	1 AD COVER ASSM	
23	483057	MAG AXLE SPACER W/HOLE 1 AD WHEEL SPACER	RIGHT
2,3,7,9-19	881081	1 AD HDWR KIT TS	
1-4,5A,6-20	116616	1AD 12 RTRO 20 STD/22SM HR TS	
1-4,5A,6-20	923780	1AD 13" RETRO 20 STD/22SM HR TS	
1-4,5A,6-20	923781	1AD 14" RETRO 20 STD/22SM HR TS	
1-4,5A,6-20	923782	1AD 15" RETRO 20 STD/22SM HR TS	
1-4,5A,6-20	923783	1AD 16" RETRO 20 STD/22SM HR TS	
1-4,5B,6-20	116617	1AD 12 RTRO 22 STD/24SM HR TS	
1-4,5B,6-20	923788	1AD 13" RETRO 22 STD/24SM HR TS	
1-4,5B,6-20	923789	1AD 14" RETRO 22 STD/24SM HR TS	
1-4,5B,6-20	923790	1AD 15" RETRO 22 STD/24SM HR TS	
1-4,5B,6-20	923791	1AD 16" RETRO 22 STD/24SM HR TS	
1-4,5C,6-20	116618	1AD 12 RTRO 24 STD HR TS	
1-4,5C,6-20	923794	1 AD 13" RETRO 24 STD HR TS	
1-4,5C,6-20	923795	1 AD 14" RETRO 24 STD HR TS	
1-4,5C,6-20	923796	1 AD 15" RETRO 24 STD HR TS	
1-4,5C,6-20	923797	1 AD 16" RETRO 24 STD HR TS	
1-4,5A,6-20	923850	1 AD 17" RETRO 20 STD/22 SM HR TS	RIGHT
1-4,5A,6-20	923853	1 AD 18" RETRO 20 STD/22 SM HR TS	LEFT
1-4,5B,6-20	923851	1 AD 17" RETRO 22 STD/24 SM HR TS	RIGHT
1-4,5B,6-20	923854	1 AD 18" RETRO 22 STD/24 SM HR TS	LEFT
1-4,5C,6-20	923852	1 AD RETRO 24 STD HR TS	RIGHT
1-4,5C,6-20	923855	1 AD RETRO 24 STD HR TS	LEFT

**TIRE PUMP**

(03/2011)

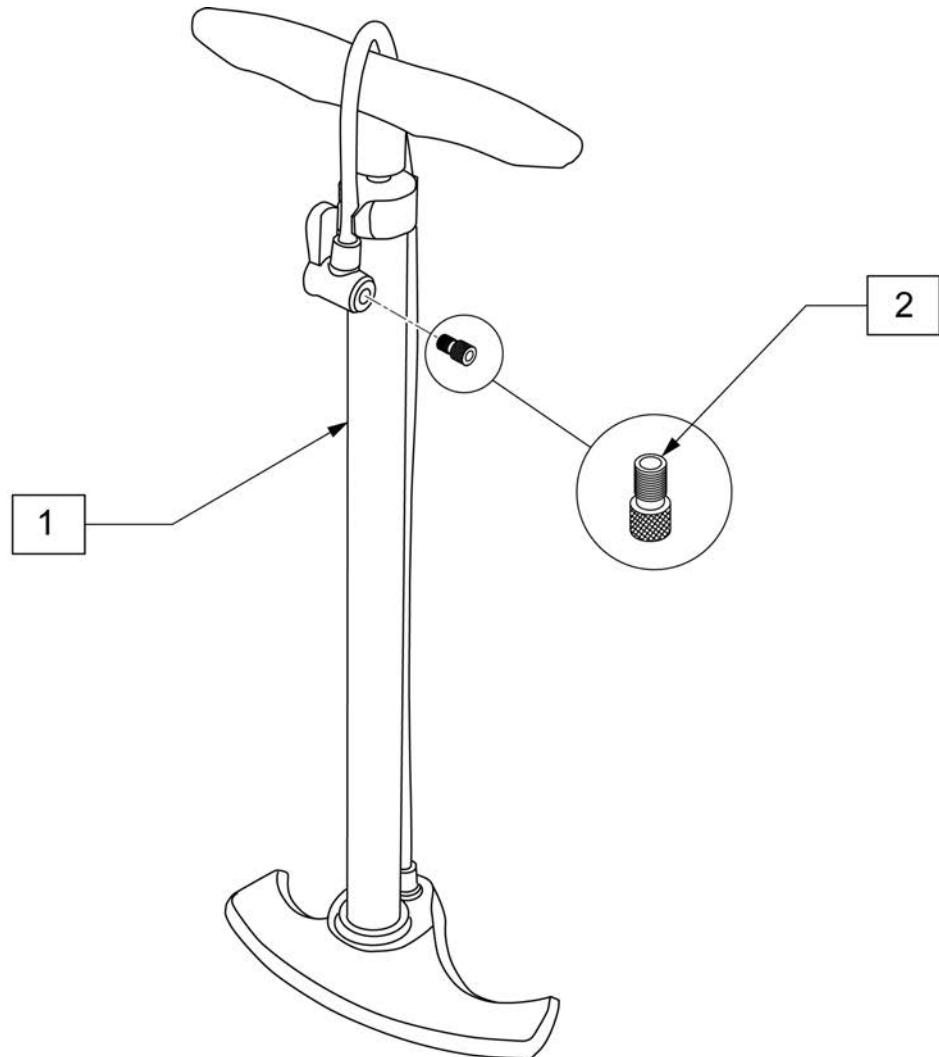

Pos.	Item Number	Description
1	389050	TIRE PUMP
2	381230	VALVE ADAPTER PRESTA

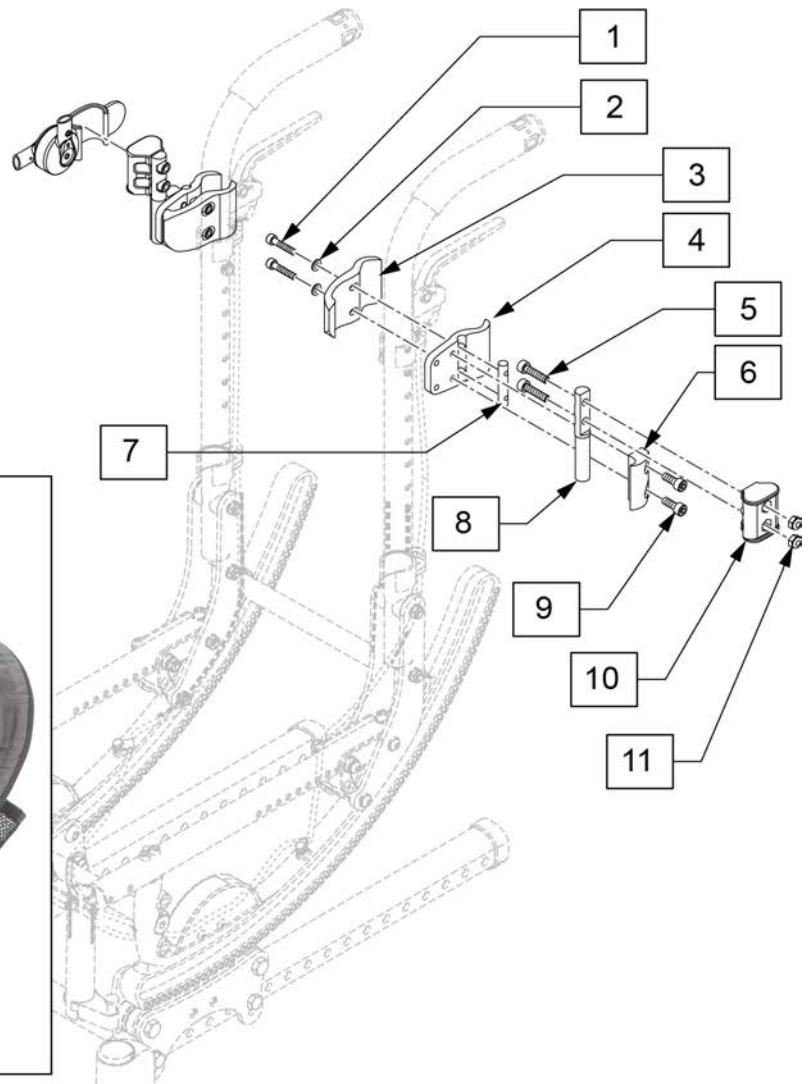

Pos.	Item Number	Description
1-11	125761	HARDWARE 10-14 WIDE RIGHT
1-11	125762	HARDWARE 10-14 WIDE LEFT
1-11	125763	HARDWARE 15-20 WIDE RIGHT
1-11	125764	HARDWARE 15-20 WIDE LEFT
12	126053	CANOPY BLACK 10-14 WIDE
12	126054	CANOPY BLACK 15-20 WIDE
1-12	125320	CANOPY ASSEMBLY 10-14 WIDE
1-12	125321	CANOPY ASSEMBLY 15-18 WIDE



# ZIPPIE® TS